SIEMENS

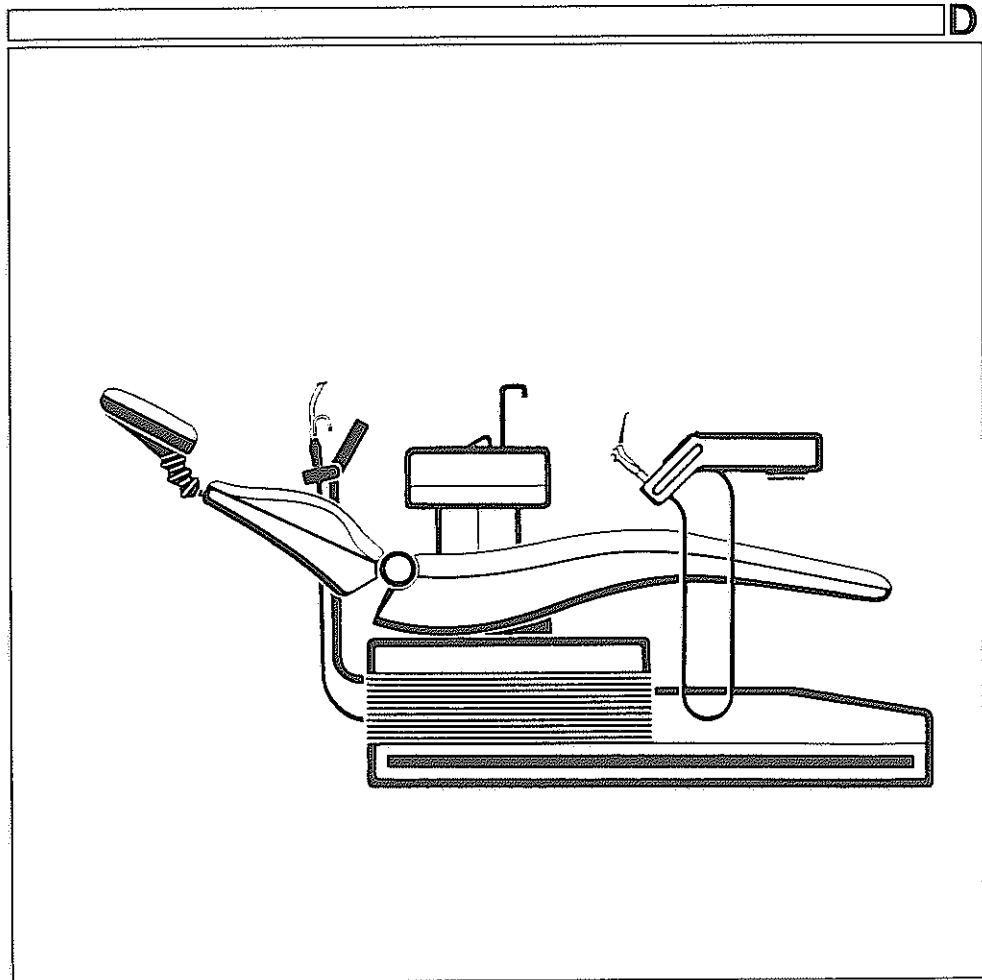
Neu ab:
New since: **07.1993**
Nouveau depuis:
Nuevo desde:

Sirona A/E

ZMD

1. UKT. 1996

**Montageanleitung
Mounting Instructions
Instructions de montage
Instrucciones de montaje**

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Water Quality

Calcareous deposits and corrosion residue in tap water can lead to the following problems:

- Premature clogging of the filter in the unit.
- Rapid clogging of the fine water paths and jets in the dental instruments.

For this reason we recommend:

- Install a water softener when the water hardness is 12° d.H. (German hardness = 2.15 mmol) or above. Set blending hardness to 8° d.H. (1.43 mmol).
- Connect conventional fine-filter. Minimum degree of fineness 20 µm (0.02 mm).
- The installation should be made in accordance with local code requirements.

Air Quality

The air for driving the turbines, cooling the drill drives and for the cooling spray must be oil-free, dry and hygienically impeccable.

Suction Pipe

In order to protect the customer's warranty rights on our suction machine a condensate separator K must be installed.

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Checking the Supply Leads

Unfold the Mounting Template.

Cut out the angle-shaped installation field.

Lay the template over lead ends on the site and check the distances of the installation field from the walls in accordance with the plan of the practice.

The site installation should be further checked for the following :

The top edges of the corner valves for air and water must not project more than 60 mm above the upper surface of the finished floor.

The suction and drain pipes must finish flush with the upper surface of the floor. (Permissible deviation + 5 mm). Internal diameter for both pipes: 36.5 mm Ø.

The power supply line must project at least 25 cm, all other electrical leads 50 cm.

Supply above the floor

The supply pipes with the corner valves are installed close to the installation field, at the wall.

Extend the pipes as far as the installation field in accordance with the requirements of the Mounting Template.

Where the pipes pass into the connection box, there is a shaped closure No. 29 41 961 (colour information is needed!).

If the chair should be installed on an installation field of SIRODONT 5/SIRO-FLEX 5, use the part set order no. 29 46 283.

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Installing the Upper Part of the Chair

NOTE:
The upper part may be mounted after
'Floor Fastening of the Chair'.

CAUTION: The backrest is only connec-
ted to the seat by a lead!

Fit upper part.
Screw on screws A and B.
The screw holes A are accessible from
the side, the screw holes B can be
reached through holes in the seat base.

Screw on the backrest.
Don't lift the chair at the backrest.

Secure the connecting cable.
Insert cover F.

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Floor Fastening of the Chair

- Remove the cover of the connection
box:
Take out 2 screws at the front side,
draw the cover 1 cm forwards, lift 3 cm
at the rear and the draw off fully for-
wards.
- Connect chair to power line, travel up-
wards, remove plug again.
- With built-in tank only:
Screw out the closing plug.
- Remove the cover moulding.
- Take off the front half of the skirting
frame:
First release screws A.
The front and side skirting frames are
joined by plug connections.
To separate, insert a screw driver at the
inside of the joints and lever the frames
apart.
Remove frames B, C and D.
- Free the bellows from the upper hold-
ing beading and lower down.
- Separate the chair from the pallet:
Take out screws E, F and G.

Tilt the chair and remove the frame
angle.

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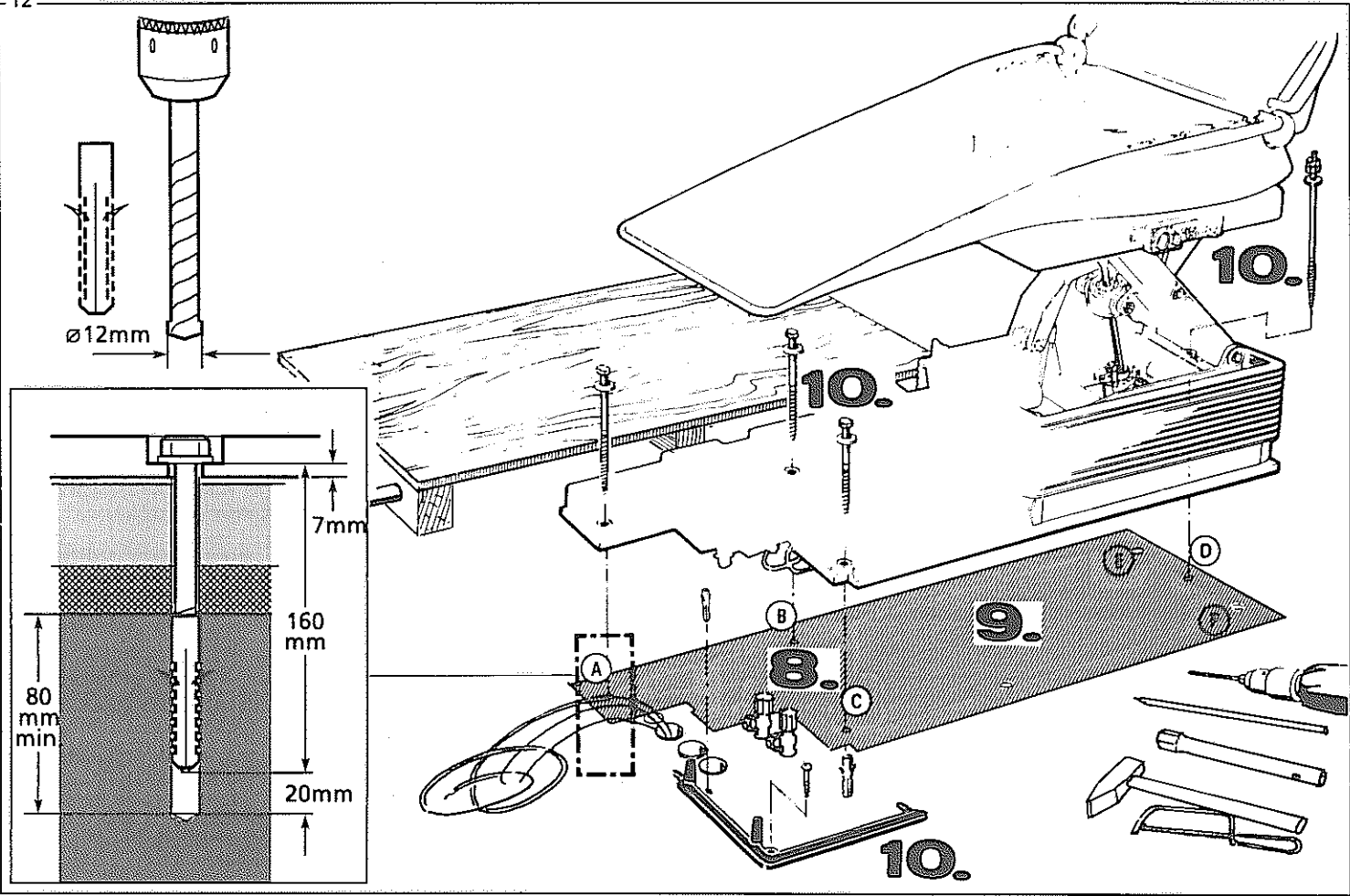
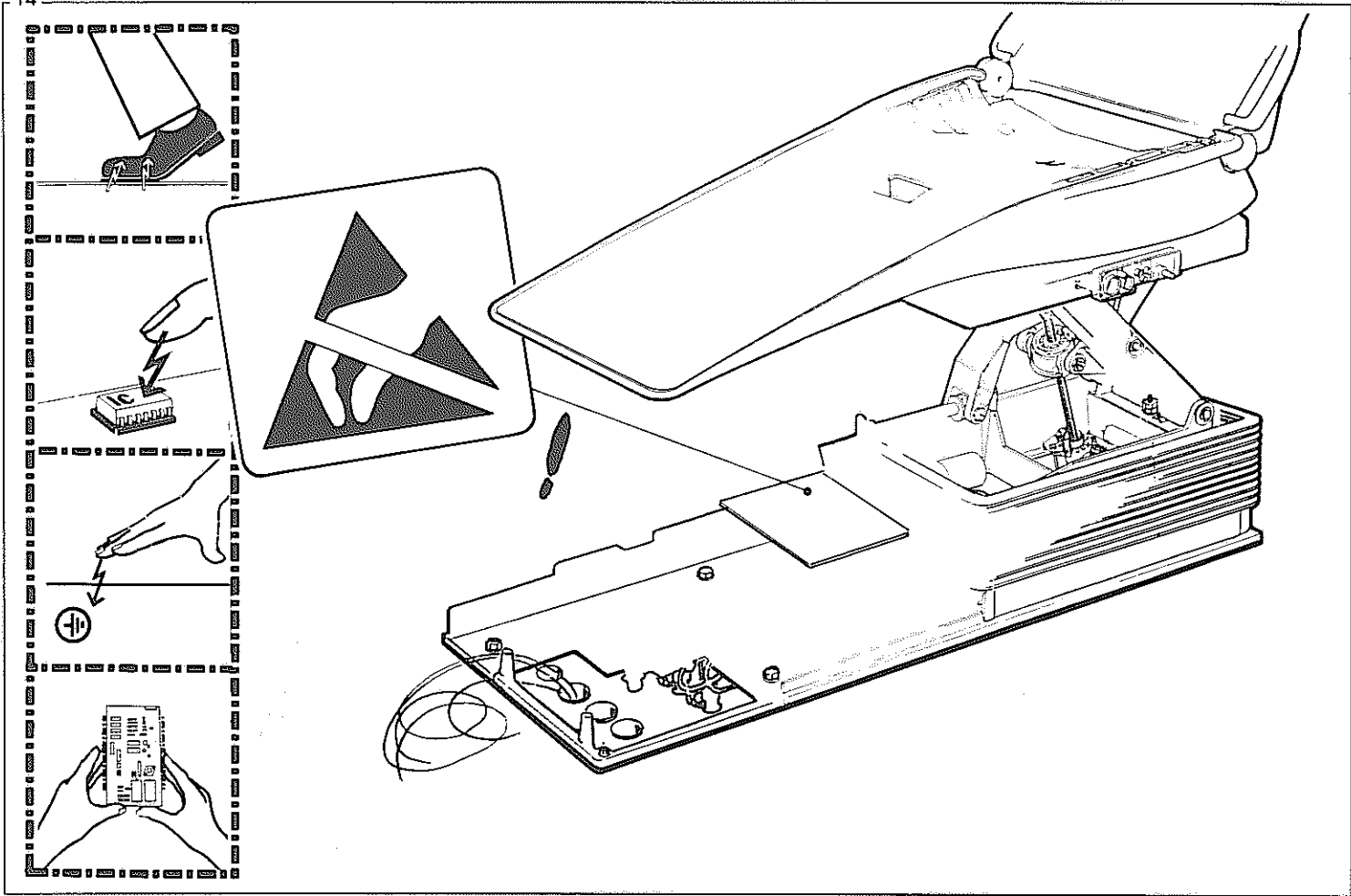
CAUTION Damage to PC's!

With the connecting box open, PC's are accessible which are fitted with highly efficient electronic components.

Electronic components are sensitive to electrostatic charges.

Static charges can arise, when people move.

Therefore, fundamentally:

- Avoid touching electronic components.
- As far as possible, only hold the PC's at the edge.
- Discharge your own electrostatic energy by touching an earthing point, before taking hold of PC's.

Endangered PC's are marked with adhesive label shown.

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Connecting Air, Water and Drain

Adapt the flex metal tubes between the corner valves on site and the unit corner valves.

(To make the connection clearer the supply unit is shown taken off).

Insert the endpieces until they make contact with the stop. Tighten the connections with the screws, until they seal.

To satisfy regulations for assistant's elements with amalgam centrifuges: Exchange endpiece A of the drainage hose (hose with smaller diameter) with siphon B.

If the insertion depth is insufficient, cut the drainage hose at a suitable point and insert siphon B with siphon connector C.

Check that the suction pipe is, in fact, connected as suction pipe!

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Electrical Connection

- Connect the power supply in place of the plug lead.
- Connect the potential equalization lead.
- Connect the call lead (bell) to X10.3 and X10.4.
- Connect the control lead of the suction machine to X10.47 and X10.49
- X10.1 and X10.2 are terminal points for button # (e. g. door opener). Only feasible if control of Sirolux S - Light Track is not desired, see "Function change, button #".

Connecting technique for X10:
"First press down then plug in" (see sectional image).

ATTENTION!

The control voltage connected to X10 can amount to 220 VAC, 6A.

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Adapting the Dentist and the Assistant Element

Dentist element A1/E1

Remove cover from supporting arm.

Mind two connectors A!

Slide the dentist element on the guide pins of the carriage and then push further onto the electrical and hydraulic-pneumatic connection elements.

Secure dentist element with a screw, washer and fan washer. Reattach cover.

Assistant element

Unscrew the rear panelling.

Slide the assistant element on the guide pins of the carriage and then push further onto the electrical and hydraulic-pneumatic connection elements.

Secure assistant element with a screw, washer and fan washer.

Plug the suction and saliva ejector hose onto the connection pieces.

Plug on the Sprayvit syringe hose and screw in place. Adapt the handpieces.

Record serial numbers in the Unit Passport.

Reattach cover.

Dentist element Sirona E4 in a cabinet see page 23.

Dentist element Sirona E5 cart-type see page 33

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Dentist Element Sirona E4 (Cabinet)

An installation tube with min. 29 mm internal diameter must be run from the connection box in the chair to the distributor box in the cabinet.

The hose set (3.5m long)

CAUTION! Protect both plugs against damages!

The end with the 37-pole plug X1 is needed on the chair side!

Draw hose set into the installation tube.

Drilling the mounting holes.

Position installation template in the cabinet according to the construction plans. Center-punch and drill mounting boreholes. Drilling depth: see above insert.

The dowels must be completely enclosed in supporting concrete!

Screed and sound insulation do not provide sufficient support.

Dowels are not used with wooden floors. A solid wooden beam must then be provided in the mounting area!

Distributor box

Unscrew cover.
Remove transport guard screw for foot adapter.

Bolt distributor box bottom and foot adapter to the floor.

As an exception it is possible to mount the adapter on the cabinet wall.

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Installing the Dentist Element

Position the reinforcing ring E at the end of the support arm.
Insert and secure support arm in the foot adapter.

ATTENTION:

When completed, the following must apply:

1. Borehole F points forwards
(Important for the stop)
2. The rigid tube must be inserted in the foot adapter until it makes contact with the stop.
3. The rigid tube is completely vertical.
(to prevent crooked mounting of dentist's element).

Secure and level the dentist's element with grubscrews 1 to 4 in the order shown above.
Tighten all grubscrews!

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In the Chair Connection Box

Air connection

A1: Connect blue hose Ø6 from the hose set to the reducing valve (see page 17, point 4).

A2: Connect red hose Ø3 from the hose set to the footswitch line.

A3: Connect blue hose Ø3 from the hose set to the footswitch line.

Water connection

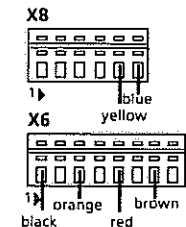
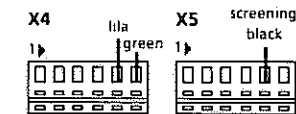
W1: Connect natural-colored hose Ø5 from the hose set to the reducing valve.

• With water purification system: see figure above.

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Insert plug X1 onto board P.

Connect single wires to terminal strips X4, X5, X6 and X8 on fuse board 0.

Connect the potential equalization lead.

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Insert plugs X1 and X4.
Connect single wires to X2.

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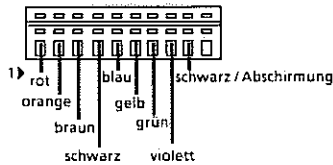
Insert plugs X5 and X9.
Connect single wires to X8.

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
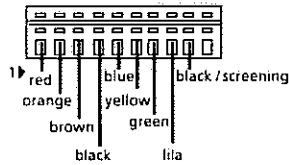
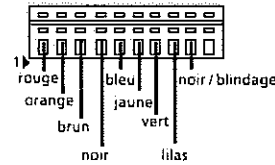
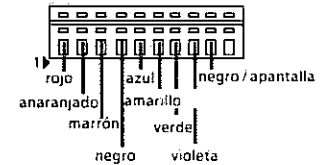
Connect potential equalization lead and ground.

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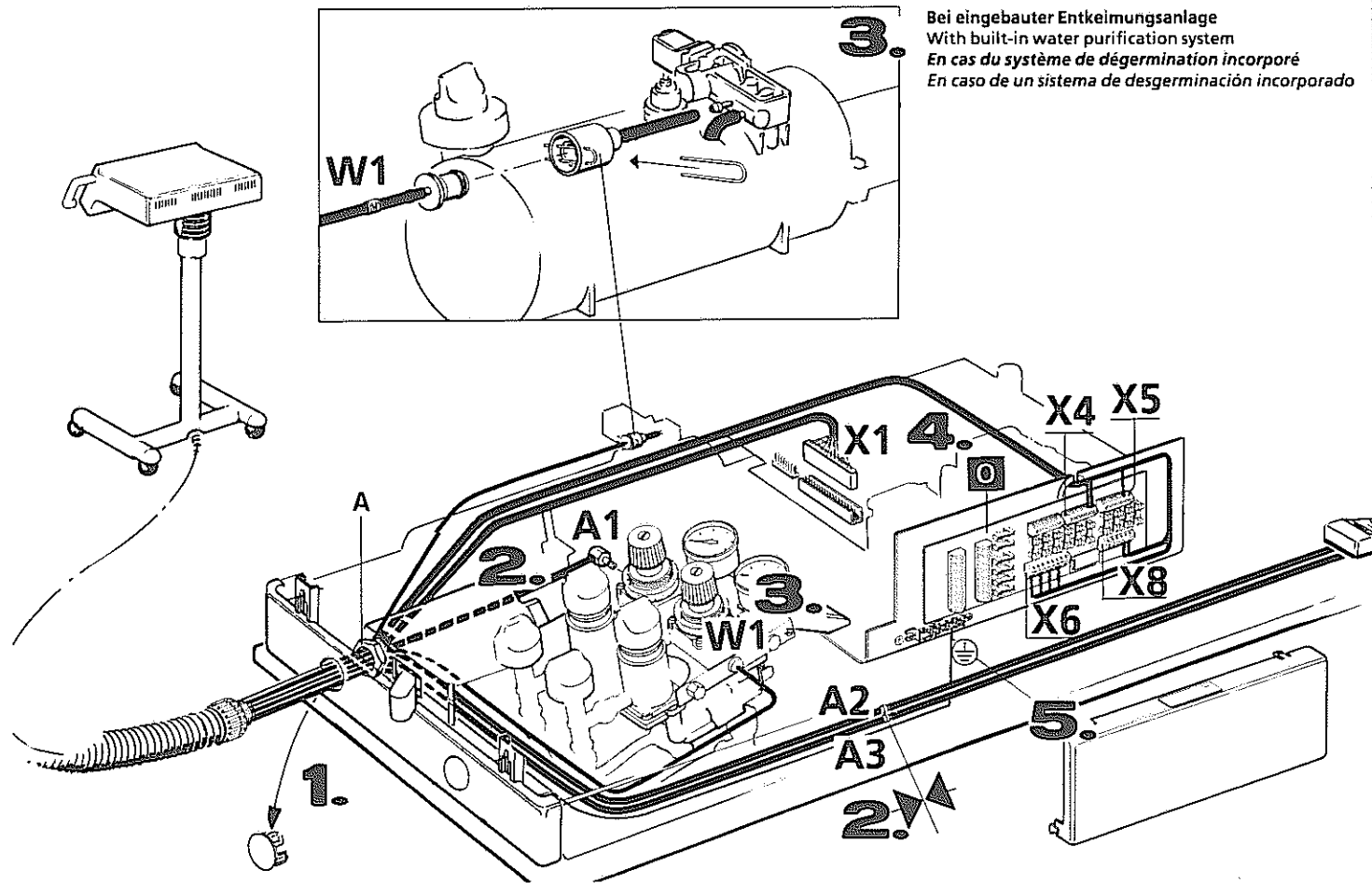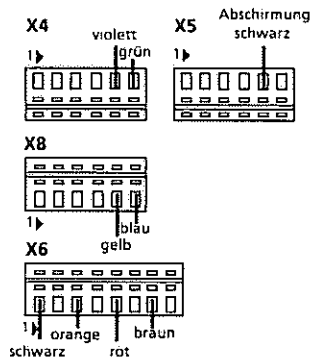
Dentist Element Sirona E5 (Cart Type)

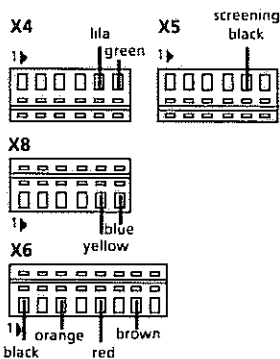
Remove cap.
Lead hose set through the cover and the nut (A).
Screw on the flexible hose.
Air connection
A1: Connect blue hose Ø6 from the hose set to the reducing valve (see page 17, point 4).
A2: Connect red hose Ø3 from the hose set to the footswitch line.
A3: Connect blue hose Ø3 from the hose set to the footswitch line.

Water connection

W1: Connect green hose from the hose set to the reducing.

Establish electrical connections.

Insert plug X1 onto board P.
Connect single wires to terminal strips X4, X5, X6 and X8 on fuse board O.

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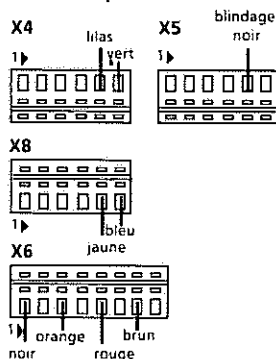
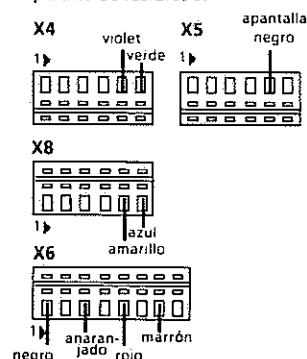
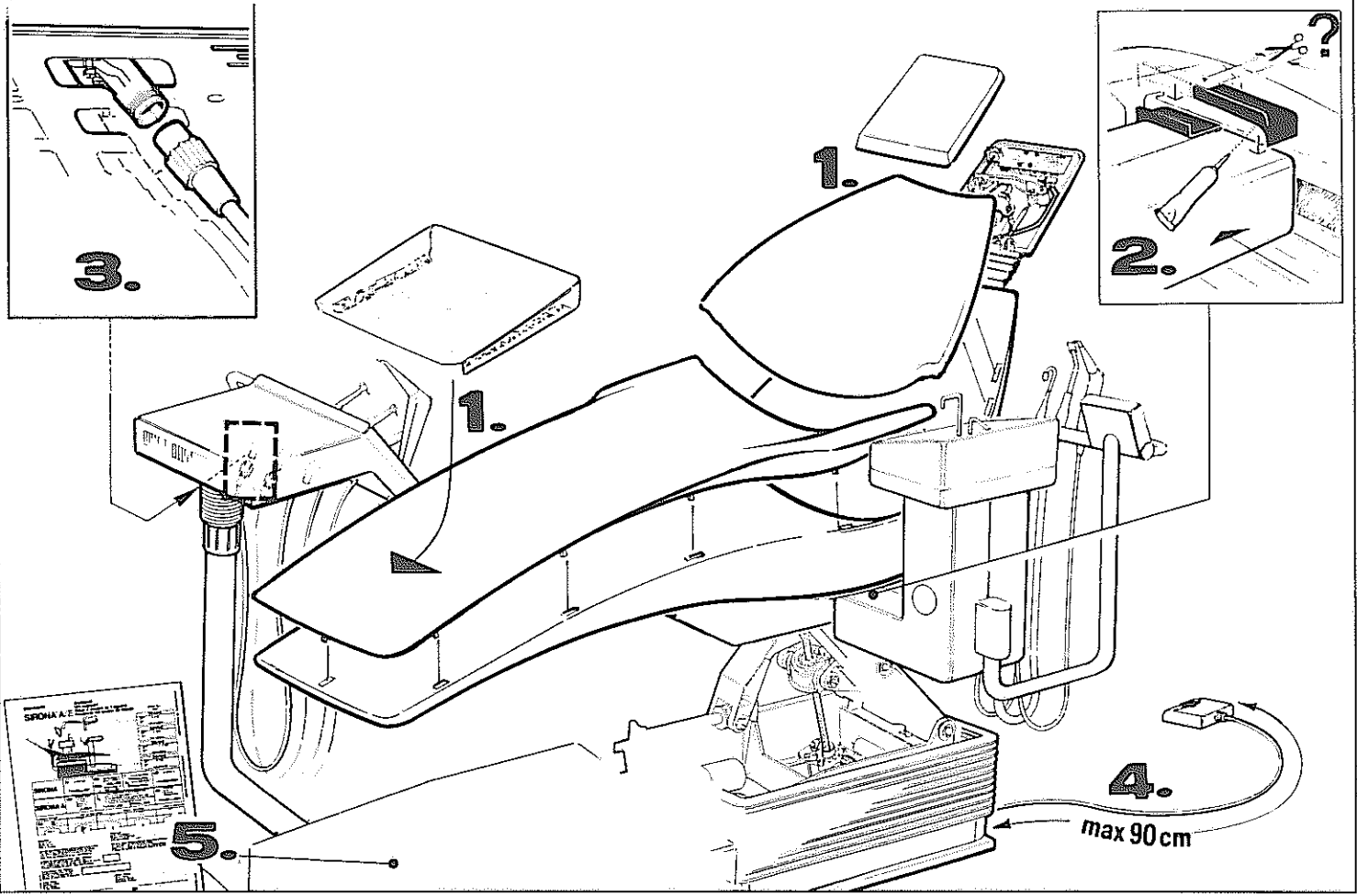
Concluding Work

Snap in upholstery.
Install protective lining.

Finger protector: (delivered with upper part). Glue both pieces flush onto the covers of the assistant's element.

Fit supply hoses and handpieces.
Each hose coupling has a torsion protection device on the inside.
Plug the supply hose on to the coupling stub, noting the torsion protection device and then screw on the capnut.

Connect footswitch.
The footswitch cable is fixed at a length which is normally adequate, but it can be extended up to approx. 90cm by drawing it out of the chair.
To allow this, loosen the cable clip near the cable exit.

Unit Passport
Enter the components of the dentist's element and fill in the remaining lines with the customer.

The **CUSTOMER'S COPY** is to be kept below the cover.
The **DEALER'S COPY** is for the customer index of the Technical Department.

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Function Change

Agree item 2 with the customer

Scaler or fourth drill drive? (Sirona E only)

At delivery, the fourth instrument holder is wired for a scaler Siroson S.

If a fourth drill drive (highspeed turbine without light only) is desired in this place, move switch S2 on PC board J to TUB.

To open dentist's unit: Loosen set screw and pull off grip. Unscrew two screws (A) and swing cover up.

Call button

for switching the SIROLUX unit lamp. On PC board O, disconnect wire L from X2.3 and connect to terminal strip X10.2.

Lay wire between X2.3 (PC board O) and terminal strip X10.1. Set switch S9 on PC board P in position 1 button function (see next page).

Motor S/SL/T1 Titan (Sirona E only)

At delivery, the physician's unit is wired for Sirona Motor S/SL/T1.

For operation of the former System 4000 move switch S3 on board J from S3.1 to S3.2.

Operating with System 4000 and Motor S/SL in one physician's unit switch S3 must be in position S3.1 (Mot S).

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Function Change (continuation)

agree with the customer

Chair program button 1 for motor-driven Sirolux 5 ceiling light track. A motor-driven ceiling light track can be moved into the parking position with the program button 1. This is expedient if the entry/exit position for the chair is programmed with this button. See table above for switch positions on PC boards PV and P.

Call button # for motor-driven light track.

Footswitch S1/S2 for motor-driven light track. The entry/exit position for the chair can be triggered with the footswitch when the ceiling track moves away. See table above for switch position on PC boards PV and P.

Call buttons # and bell for free choice functions
S9 in position 1: # button function
S9 in position 2: # switching function
S11 in position 1:
Bell button has button function
S11 in position 2:
Bell button has switching function.

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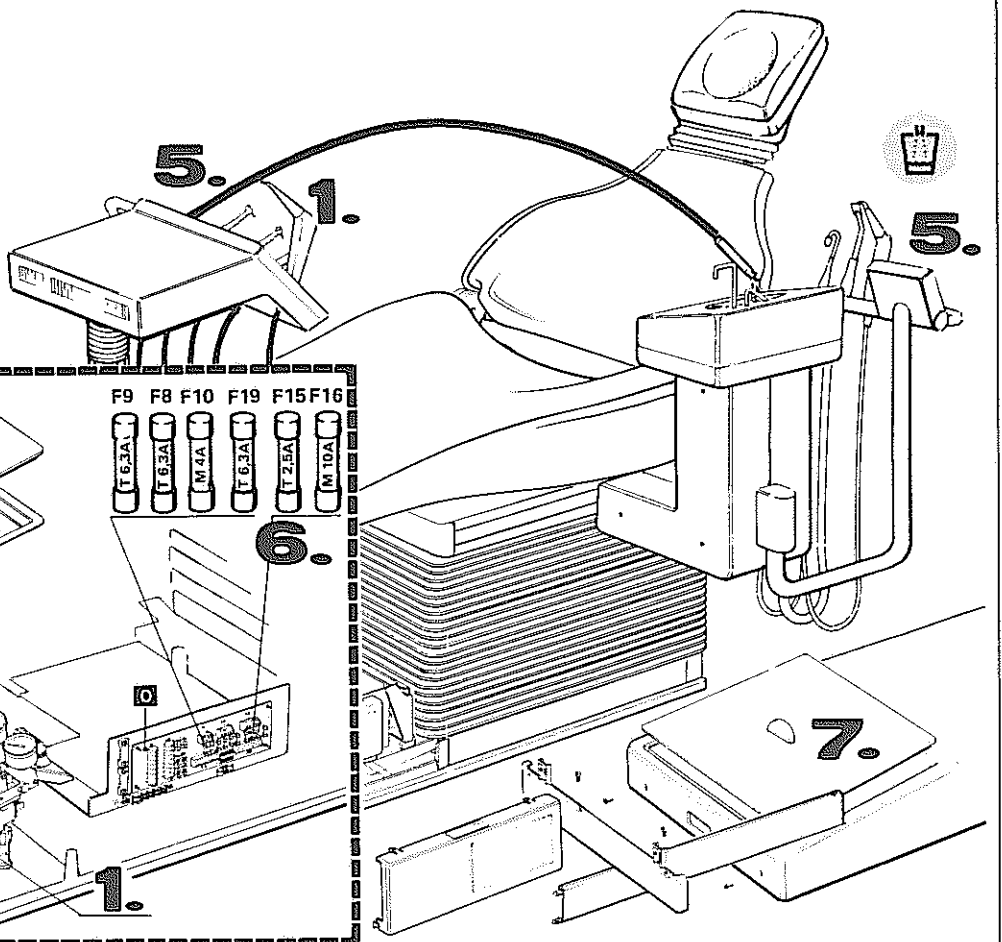
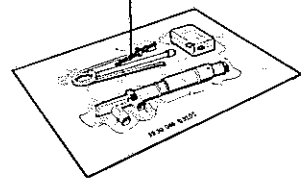
Starting-up the Treatment Unit

Close the filters A firmly and open the corner valves.

Switch on the main switch (Air and water pressure have already been set in the factory and are marked on the pressure gauges).

Check the air and water connections in the connection box for leaks.

With built-in water tank only. (NOT for water purification system!) Fill up 1.5 l water.

Screw in the closing plug. Operate the pneumatic switch AB3 for air pressure by positioning the cover. **CAUTION**

Never let the unit work without water in the tank and without positioned cover. The water heating elements in the unit can burn out!

Beginning from the right operate all treatment instruments and both syringes in sequences until a satisfactory spray is obtained (see Operating Instr.)

Connect a supply hose temporarily to the couplings of receptacles not used. Initiate mouth-rinsing glass filling. Fill two cups.

Insert the fuses for the heating systems into the fuse strip on PC Board O in accordance with the values given.

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Starting-up the Water Purification System

For the initial purification, the water system (except cuspidor flushing) must be disinfected by exposing it to DENTOSEPT P:

- Fill the tank with DENTOSEPT P and close it.
- Set switch B to position II "Sanitation" – an interval sound indicates that the switch B is in position II and the tank is filled. (When the tank is empty, a continuous sound becomes audible).
- Operate the pneumatic switch AB3 by placing on the cover – compressed air is directed into the tank.
- Operate all treatment instruments with maximum spray for approx. 60 sec. Make sure that the period will not be shorter than 60 sec!
- Operate the mouth-rinsing glass filling until the continuous sound becomes audible – the tank is now empty. The water system is filled with pure DENTOSEPT P – the purification begins.
- Turn off the main switch!
- Set switch B to position I. Position I is for normal operation with purification system.
- Fill the tank with DENTOSEPT PL.

FROM NOW ON, THE UNIT MUST NOT BE OPERATED FOR 24 HOURS!

The dentist performing the initial operation should follow the note in the Operating instructions.

For operation without purification system: Set switch B to position 0.

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Operating Air Pressures
Check and adjust

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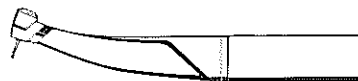
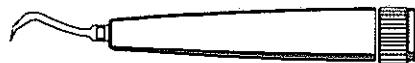
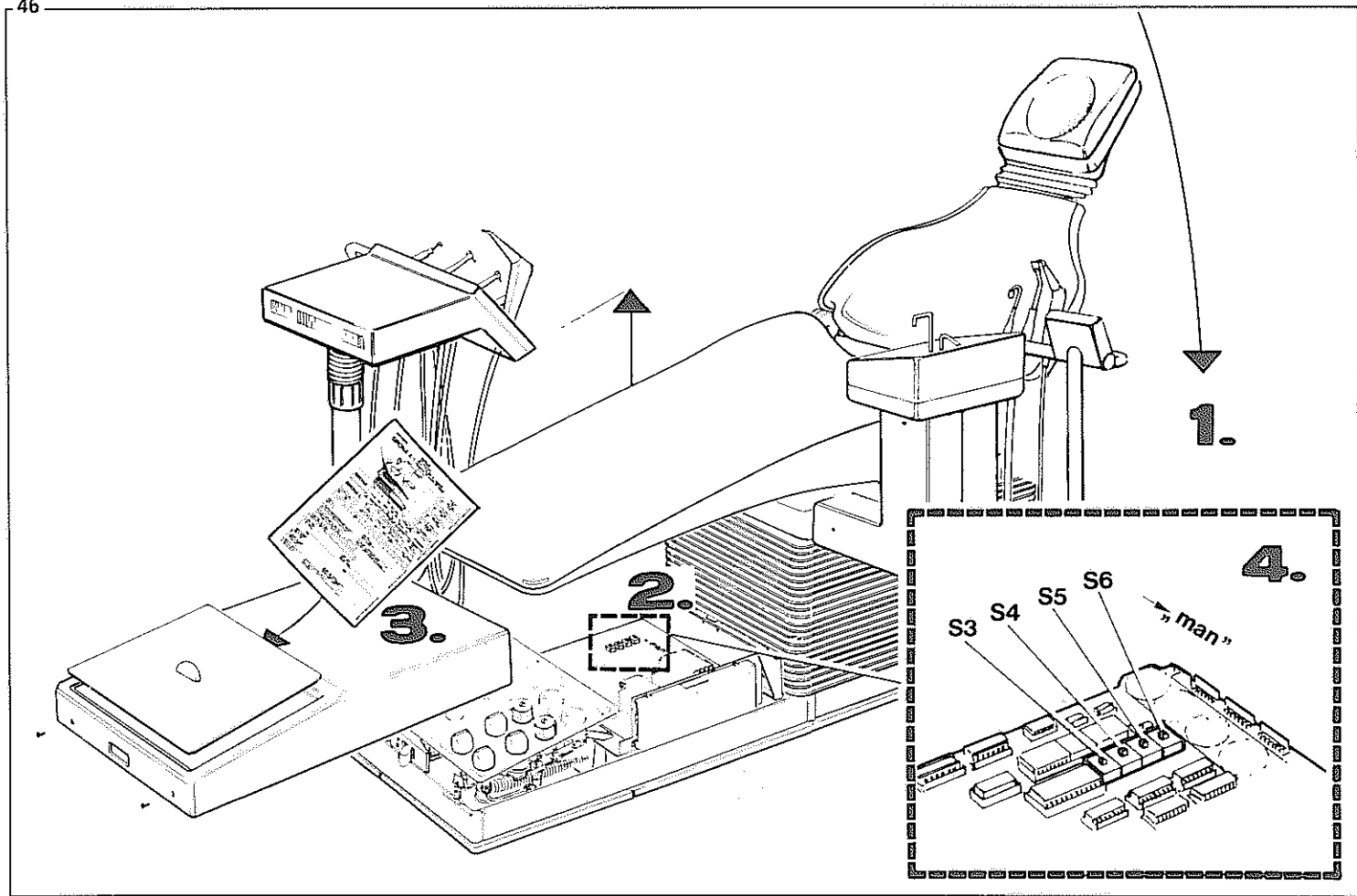
Adjust the pressure for non Siemens instruments at RD 31, 32, 33 or 34, following the indication of the manufacturer.

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The Customer Should be Informed, ...

... that before programming the treatment positions for the first time (1st treatment day) both the lower part and upper part of the chair should be travelled once into each of the end positions.
Thus, the program store receives the necessary base values for the initial programming.

... that, in case of a built-in water purification system, prior to initial operation of the unit through the dentist, and after the prescribed waiting time, the filled-in DENTOSEPT for the initial purification should be flushed out of the unit.
Show the dentist this important note in the Operating Instructions!

... that the Unit Passport is an assistance in case of disturbance and should be kept below the molder cover.

... that in case of any disturbance to the programmed treatment positions, the electronics can be transferred to "man" ual operation by switching over the switches S3, S4, S5, S6.
The chair can now be operated further by using the handswitched on the headrest, until the fault is corrected.

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