
Neu ab:
New since: 09.2008
Nouveau depuis:
Nuevo desde:

sirona.
The Dental Company

C5⁺

Schaltunterlagen
Wiring References
Schémas de circuits
Colección de esquemas

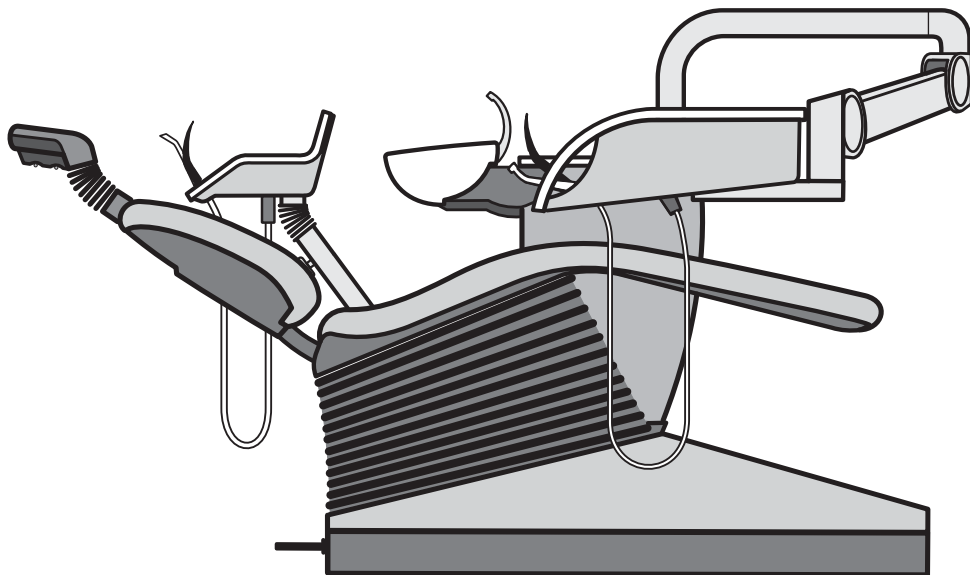! ACHTUNG

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! ATTENTION

Interference with electromedical devices by radio telephones:
To guarantee the operational safety of electromedical devices, the operation of mobile radio telephones in the medical practice or hospital area is prohibited.

! ATTENTION

When opening the equipment:
Please observe the safety measures for handling PC boards. Touch a ground point to remove any personal electrostatic charge before touching the components.

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Modification compared with last edition:
09.2007

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List of Contents Sheet

Electrical circuit diagram:
 Patient's Chair 1/1A
 Dentist Element 2
 Assistant Element, Water Unit..... 3
 Video application Dentist Element 4
 Video application Assistant Element 5/5A
 Legend 6A
 Hydraulic-pneumatic diagram:
 C5+ with disinfection system 7
 C5+ without disinfection system 7A
 Legend 8A
 CAN BUS, data communication 9
 RS232 Interface 10A

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Legend for electrical circuit diagram

ST Patient chair

F1	Power fuse
S1	Power switch
M1	Motor for lift
M2	Motor for backrest
M3	Motor for headrest tilt
M5	Motor for headrest up/down
MV1	Solenoid valve for water
MV35	Solenoid valve for cuspidor rinsing
R1	Transmitter pot. for lift
R2	Transmitter pot. for backrest
R3	Transmitter pot. for headrest tilt
S1	Limit switch lift up
S2	Limit switch lift down
S3	Limit switch backrest up
S4	Limit switch backrest down
S5	Limit switch headrest up
S6	Limit switch headrest down
SS1	Safety switch for backrest
SS2	Safety switch for base
SS3	Safety switch for headrest

CF Fuse board

CJ Control board ST

V401	LED shifting track/backward
V403	LED shifting track/forward
V405	LED coupling/shifting track
V439	LED headrest up
V440	LED headrest down
V441	LED headrest in
V442	LED headrest out
V443	LED M2 backrest up
V444	LED M2 backrest down
V445	LED M1 lift up
V446	LED M1 lift down
V447	LED limit switch
V448	LED programming mode
V449	LED switch activated
V450	LED emergency shutdown active
V451	LED on if a safety switch is active
V452	LED D-Error, a motor does not react
V453	LED +32V
V454	LED +5V
V455	LED +24V
V473	LED relay 115V
V474	LED Reset

CK Switch board for tilting part

S1-S8	Chair movements
-------	-----------------

CSF Footswitch

S2-S4, S6	Cursor
S5	Switch activate instruments
S7	Switch Chip blower
S8	Switch Spray

FT Foot button for Sirocam C in assistant element

KA Connection Kompakt

KS Foot control

S9	Switch chair up ↑
S10	Switch chair down ↓
S11	Switch back or headrest up ↑
S12	Switch back or headrest down ↓

MD Monitor

NM Power pack/Monitor

SL Wiring board

SR Relay board

K1	Relay for key
K2	Relay for key #
K3	Relay for suction machine

SVI

SVC Video adapter

V11	LED +5V
V97	LED +12V
V98	LED display off

T1 Transformer

AE Dentist's element

MV21.1-4	Solenoid valve for bur drive module
MV22	Solenoid valve f. ultrasonic handpiece water
MV23	Solenoid valve f. ultrasonic handpiece air
MV24	Solenoid valve for SPRAYVIT water
MV25	Solenoid valve for SPRAYVIT air
MV6S	Solenoid valve for brake

A7C Camera module

V1	LED +5V
V11	LED +5V
V14	LED +12V
S1	Selector switch: Instrument holder or external holder for camera

ADX Control module AE

V6	LED salt solution pump ON
V12	LED fuse salt solution pump defective
V27	LED Sprayvit water heater relay on
V30	LED Sprayvit air heater relay on
V41	LED reset
V49	LED MV22 US water
V57	LED claw-type relay K4 for motor voltage
V60	LED claw-type relay K3 for motor voltage
V63	LED claw-type relay K2 for motor voltage
V69	LED MV21.1 bur drive module 1
V74	LED MV21.2 bur drive module 2
V79	LED MV21.3 bur drive module 3
V84	LED MV23 US air
V89	LED MV24 Sprayvit water
V98	LED MV25 Sprayvit air
V110	LED MV21.4 bur drive module
V115	LED +24V
V116	LED +5V
V117	LED +35V
V134	LED +15V
V135	LED +8V
V136	LED +15V
V141	LED heater ON
V145/146	LED without function

AL Module for Electromotor

V30	LED reset
V31	LED motor runs (run)
V32	LED error (error)
V34	LED +32V
V35	LED +8V

APC5+ Control panel

V50	LED +5V
V51	LED reset
V52	LED service

AU Module US - Handpiece SIROSON

V1	LED flashes 5x: standby ok
----	----------------------------

H Water heating

F1	Thermal fuse
R1	Heater cartridge
TS1	Temperature sensor

IR4 Light barrier board AE

V1/11	Light barrier claw 1 SPRAYVIT
V2/12	Light barrier claw 2
V3/13	Light barrier claw 3
V4/14	Light barrier claw 4
V5/15	Light barrier claw 5 (on right)

KSP Salt solution pump

LCD Liquid crystal display

MI SATELEC MiniLED DC/DC module

ROEBI X-ray image viewer

SP SPRAYVIT

AR2	Air heater cartridge
WR2	Air heater cartridge
AR	Signal generator air
WR	Signal generator water

HE Assistant's element

MV31	Solenoid valve saliva ejector precontrol
MV32	Solenoid valve vacuum hose precontrol
MV36	Solenoid valve surgical cannula precontrol
MV38	Solenoid valve for SPRAYVIT water
MV39	Solenoid valve for SPRAYVIT air

HC7 Camera module HE

V1	LED +24V
V11	LED +5V
V14	LED +12V
V42	LED Camera on
V46	LED image stop

HC Control board HE

V7	LED +5V
V14	LED +24V
V22	LED reset
V26	LED Sprayvit light

HEP Control panel

HP Supply board HE

MI SATELEC MiniLED DC/DC module

PL POLYLUX

V	Fan
H	Lamp
V1	LED +5V supply voltage CPU
V39	LED +12V
V40	LED +5V operating voltage

RK Light barrier board HE

V1/11	Light barrier claw 4
V2/12	Light barrier claw 3
V3/13	Light barrier claw 2 SPRAYVIT
V4/14	Light barrier claw 1

SP SPRAYVIT

AR2	Air heater cartridge
WR2	Air heater cartridge
AR	Signal generator air
WR	Signal generator water

WE Water unit

MV43	Solenoid valve for cuspidor movement
SS1	Safety switch for cuspidor
SS2/SS3	Safety switch for HE
MV P1	Valve for driving air
MV P2	Valve for chip blower
MV P3	Valve for spray air

X-ray image viewer

H1/2	Lamps
------	-------

AP

M Pump for automatic separator

M	Motor for pump and centrifuge
---	-------------------------------

AW

S3 Amalgam separator

S3	Safety switch for amalgam separator
----	-------------------------------------

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DNA Dürr Wet Suction

MSBV	Mouth-rinsing basin valve
DS	Level switch

EAD Disinfection system with DENTOSEPT

DS31	Level switch in mixing tank
DS32	Level switch in tank
MV3	Solenoid valve for DENTOSEPT

H

F2 Water heating

F2	Thermal fuse
MV34	Solenoid valve for tumbler filling
R2	Heater cartridge
TS2	Temperature sensor

P

DS33/34 MV40 Pump

DS33/34	Protective gas contact for pump control
MV40	Solenoid valve for pump

SA

DS30 V1 MV33 Automatic separator

DS30	Level switch in separator tank
V1	Speed transmitter light barrier
MV33	Precontrol solenoid valve f. unit selector valve

SC

MV41 Flushing vessel for automatic separator

MV41	Solenoid valve for flushing system
------	------------------------------------

WM Water alarm

DN30	Pressure switch
MV37	Solenoid valve for drain flap

WE Control board WE

V40	LED for MV33
V41	LED for water heater for tumbler filling
V69	LED on=amalgam separator vessel empty, level switch DS30
V70	LED on=amalgam rotor closed, safety switch S3
V71	LED on=no water, pressure switch DN30
V72/73	LED for protective gas contact DS33/34 water pump limit switches
V74	LED on=tank filled, for level switch DS32
V75	LED without function
V76	LED on=tank filled, for level switch DS31
V77/78	LED without function
V79	LED for tumbler filling
V80	LED for centrifugal rinsing
V82	LED for vacuum active
V81/83	LED without function
V84	LED for heater
V85	LED for MV37
V86	LED for MV40
V87	LED for MV34
V88	LED for MV3
V89-91	LED without function
V92	LED for x-ray film viewer
V101	LED reset
V102	LED for amalgam motor
V103	LED +5V
V104	LED +35V
V105	LED +24V
V106	LED +15V
V107	LED +5Vref
V109	LED without function

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Legend for hydraulic-pneumatic circuit diagram

AK Connection box

ANB Connection block AK

AB Mechanical valve for air and water
 AD1/2 Pressure controller for air
 AD3 Pressure controller for water
 AF1 Filter for air
 AF2 Filter for water
 MA1/2 Pressure gauge for air
 MV1 Solenoid valve for water
 MV35 Solenoid valve for centrifugal rinsing

SDI Plug connection

ZG Assembly for additional units

WE Water unit

AC37 Check valve
 AO33 Acoustic damper
 RD35 Flow regulator
 V1-V3 Three-way/four-way manifold

AP Pump for automatic separator

AC31 Check valve
 M Motor for pump and centrifuge
 SN30 Siphon

AW Amalgam separator

FA Solids separator with switch S3
 SN31 Siphon

BK Solid matter collector

SN31 Siphon

DNA Dürr Wet Suction

MSBV Mouth-rinsing basin valve
 DS Level switch

EAD Disinf. system with DENTOSEPT

AF3 Filter for DENTOSEPT
 AO34 Acoustic damper
 DS31 Level switch in mixing tank
 DS32 Level switch in tank
 E Funnel with vent for DENTOSEPT
 MB Mixing tank
 MV3 Solenoid valve for DENTOSEPT
 SD36 Flow limiter
 T Tank for DENTOSEPT

H Water heater

MV34 Solenoid valve for tumbler filling
 R2 Heater with temperature sensor TS2 and overheating protection

P Pump

AC32/33 Check valve
 DS33/34 Protective gas contact for pump control
 EV30 Dewatering valve
 MV40 Solenoid valve for pump

SA Automatic separator

AF33 Filter
 BP Separator tank with light barrier V1
 BQ Unit selector valve
 DS30 Level switch in separator tank
 MV33 Precontrol solenoid valve f. unit selec. valve
 RD33 Flow regulator

SC Flushing vessel for automatic separator

MV41 Solenoid valve for flushing system
 SD41 Flow limiter

SWS Swivellable cuspidor

AC43 Check valve
 AE Nozzle for heater overflow
 AF4 Rough filter
 AH Cuspidor
 AT Tumbler filling pipe
 AO43 Acoustic damper
 BN Cuspidor flushing
 D43 Regulator
 MV43 Solenoid valve for cuspidor movement
 PZ43 Pneumatic cylinder
 STA Plug adapter for tumbler filling/hydrocolloid

VB Valve block

MV P1 Valve for driving air
 MV P2 Valve for chipblower
 MV P3 Valve for spray air

WM Water alarm

AO31 Acoustic damper
 DN30 Pressure switch
 MV37 Solenoid valve for drain flap
 PK Pneumatically operated drain flap

AE Dentist's element

M Motor SL
 T T1 CONTROL
 US SIROSON L ultrasonic handpiece
 AA1 Hose coupling for SPRAYVIT
 AA2 Hose coupling for bur drives
 AA10 Hose coupling for ultrasonic handpiece
 AD 4 Pressure controller for spray air
 AO6 Acoustic damper
 DM21 Pressure alarm, measuring air
 MV6 Solenoid valve for brake

AB Connection block AE

MV24 Solenoid valve for SPRAYVIT water
 MV25 Solenoid valve for SPRAYVIT air
 AO26-28 Acoustic damper

BS Bur drive module

AC Double check valve
 AC1 Check valve for bur drive
 AR4 Back suction device
 MV21.1-4 Solenoid valve bur drive module
 PV1 Pneumatic valve for driving air
 PV2 Pneumatic valve for spray
 PV3 Pneumatic valve for chip-blower
 PV5 Pneumatic valve for pressure gauge air
 RD3/5 Control valve for driving air

H Water heater

AF Water filter
 R1 Heater with temperature sensor TS1 and overheating protection

US Ultrasonic block

MV22 Solenoid valve for water US
 NV23 Solenoid valve for air US

VA Supply connection

HE Assistant's element

AA1 Fast-action coupling for SPRAYVIT
 AA3 Fast-action coupling for saliva ejector
 AA4 Fast-action coupling for vacuum hose
 AC34 Check valve for vacuum hose
 AC35 Check valve for saliva ejector
 AF5/7/8 Filter
 AJ1 Suction cannula
 AL1 Saliva ejector
 MV31 Precontrol valve for saliva ejector
 MV32 Precontrol valve for vacuum hose
 MV38 Solenoid valve for water SPRAYVIT
 MV39 Solenoid valve for air SPRAYVIT
 SV31 Control valve for saliva ejector
 SV32 Control valve for vacuum hose

CK Surgical cannula

AA7 Fast-action coupling for surgical cannula
 AC36 Check valve for surgical cannula
 AF5/9 Filter
 MV36 Precontrol valve for surgical cannula
 SV36 Control valve for surgical cannula

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RS232 Interface

Dental workstation → PC

An extended Mouse System Protocol is used to transmit defined conditions or commands from the dental workstation to a connected PC. For the camera and foot switch conditions in byte 1, the bits 0-2 are used to control the video application, to transmit additional commands, the bits 3-6 are used. The bytes 2-5 are 0 and are not used.

7	6	5	4	3	2	1	0
1				User command			
				Video function			

Byte 1 of the Mouse System

In each case only one user command or information to the video function can be transmitted in one data package. The bits of the other block are set to 0.

The following events and interface commands are defined:

The commands are generated in each case when the event is triggered.

Event	Remark	RS232 command 32 (1st byte of the 5- byte protocol)
Rhomb key activated		88H
Rhomb deactivated	only if switching mode	90H
Call key		98H
X-ray image viewer key		A0H
CCW rotation key (with instruments deposited or video camera removed)		A8H
Instrument light key (with instruments deposited or video camera removed)		B0H
Quick setting key 100 (with instruments deposited or video camera removed)	also be operated via FS	B8H
Quick setting key 75 (with instruments deposited or video camera removed)	cannot be operated via FS	C0H
Quick setting key 50 (with instruments deposited or video camera removed)	cannot be operated via FS	C8H
Quick setting key 25 (with instruments deposited or video camera removed)	cannot be operated via FS	D0H
Quick setting key 1 (with instruments deposited or video camera removed)	also be operated via FS	D8H
Forward cursor from chair key block	only be operated via FS	E0H
Forward cursor from instrument key block	only be operated via FS	E8H
Spray key Left discrete C+ FS button active with camera removed		F0H
Chip blower key Right discrete C+ FS button active with camera removed		F8H
Camera removed		81H
Camera deposited		83H
Foot switch active with camera removed		85H
Foot switch inactive with camera removed		81H
When video camera is assembled on HE side and when in program SIUCOM.exe mode: "Sirocam C direct" is selected:		
Press foot switch		E8H
Press button on camera		A0H

PC → Dental workstation

The RS232 configuration and service information can be output via PC with an 1-byte command.

Command	Action	Remark
0DH	Output of configuration and service information	

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Property of Sirona. All rights reserved.
© Eigentum von Sirona. Alle Rechte vorbehalten. © Property of Sirona. All rights reserved.
Property of Siraco. All rights reserved.
Sirona Dental Systems GmbH