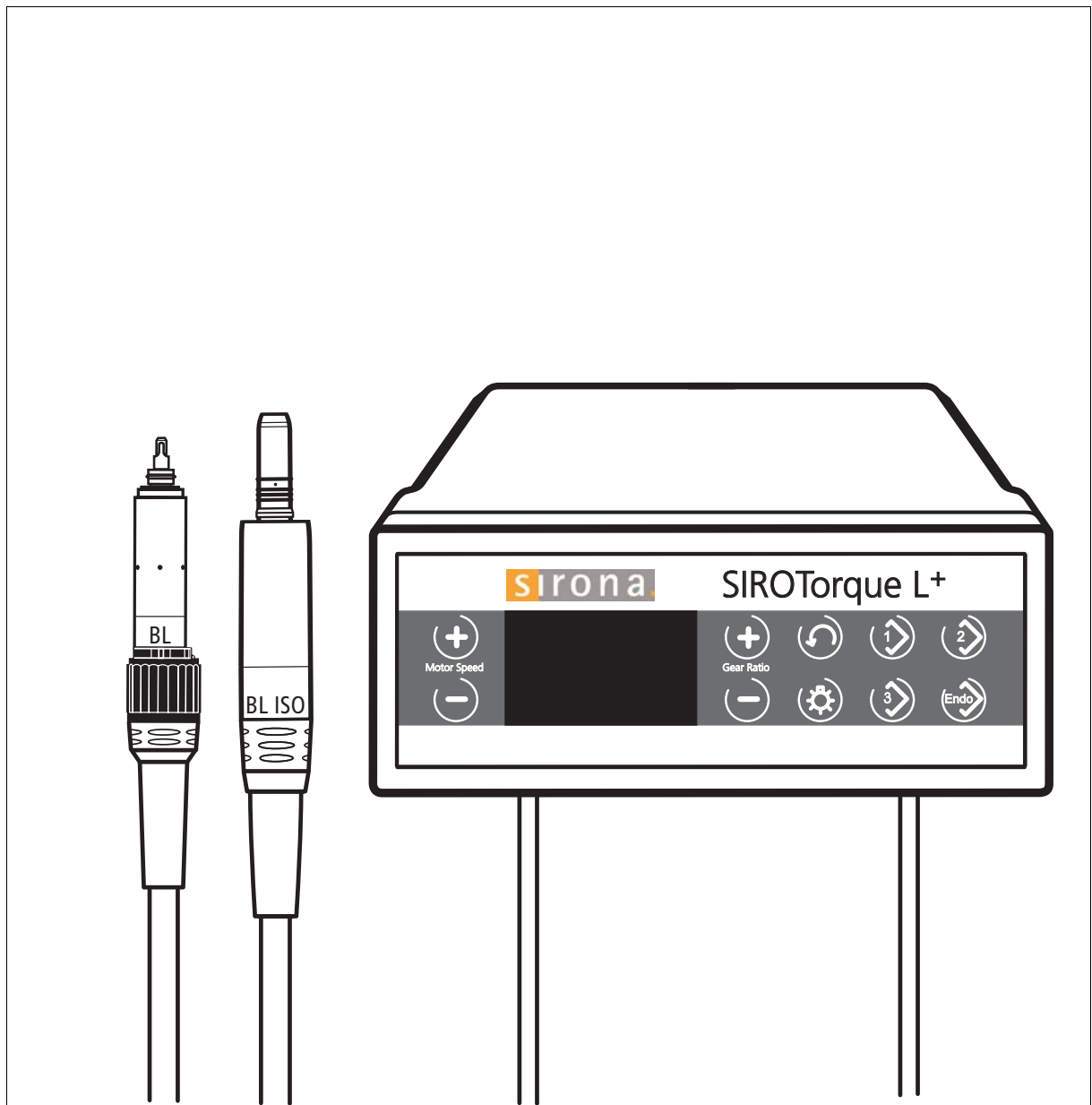
SIROTorque L⁺

Service Notes
Instruction de service

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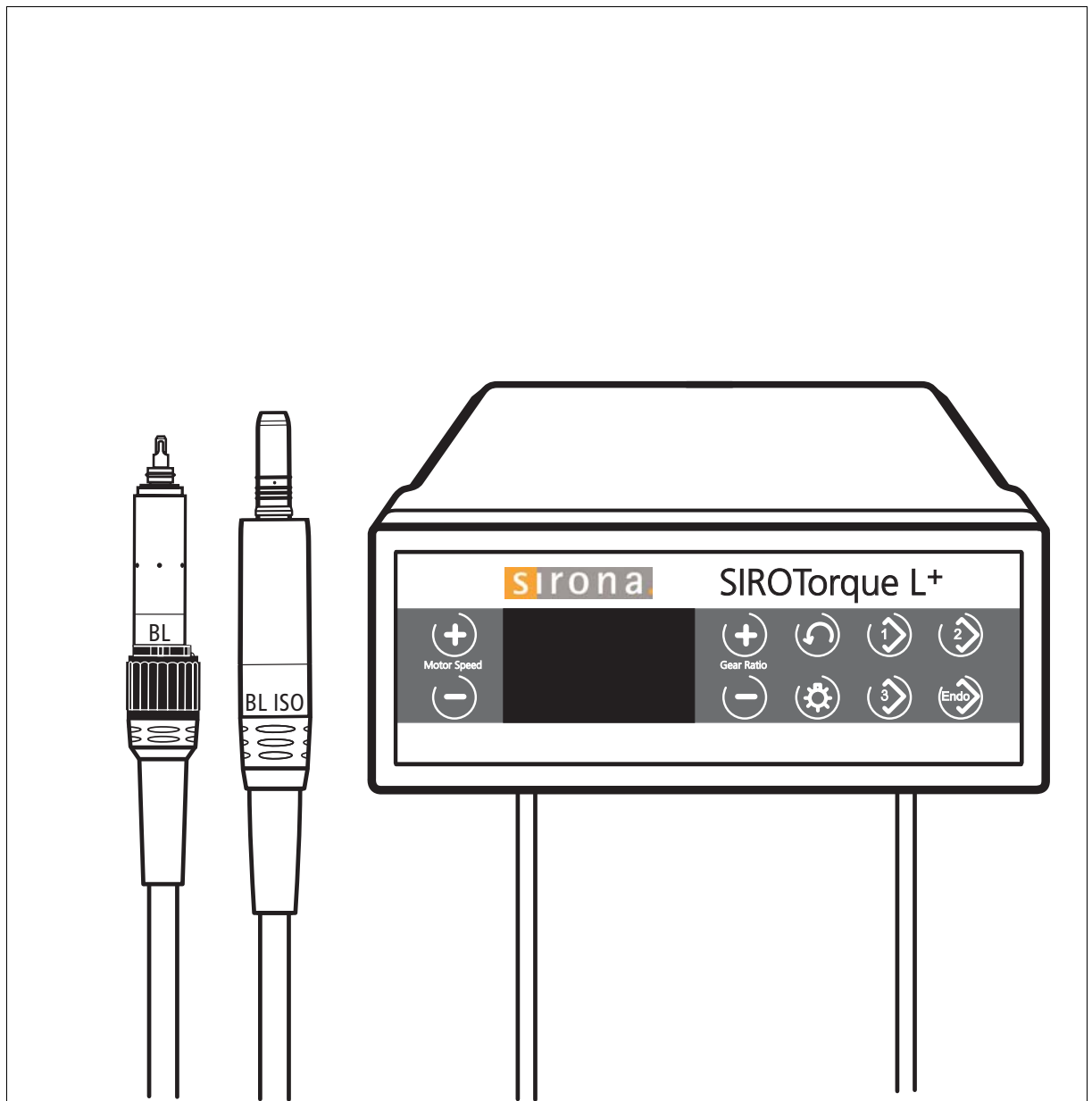
SIROTorque L⁺

Service Notes

English

General information

The savable settings described in these service instructions may be performed by a service engineer only.

No tools are necessary to perform the work.

Your SIROTorque L⁺ and Instruments Team


Contents

1

Savable settings	5
1.1 "Limit value adjustment" menu	5
1.2 „Bulb“ Instrument light - lamp voltage (light intensity)	6
1.3 "CU" limit of the maximum motor current (torque)	6
1.4 „P_LO“ Air pressure for the lower switching threshold on the foot switch	7
1.5 „P_HI“ Air pressure for the upper switching threshold on the foot switch	7
1.6 "FI_S" reset to factory settings	8

1 Savable settings

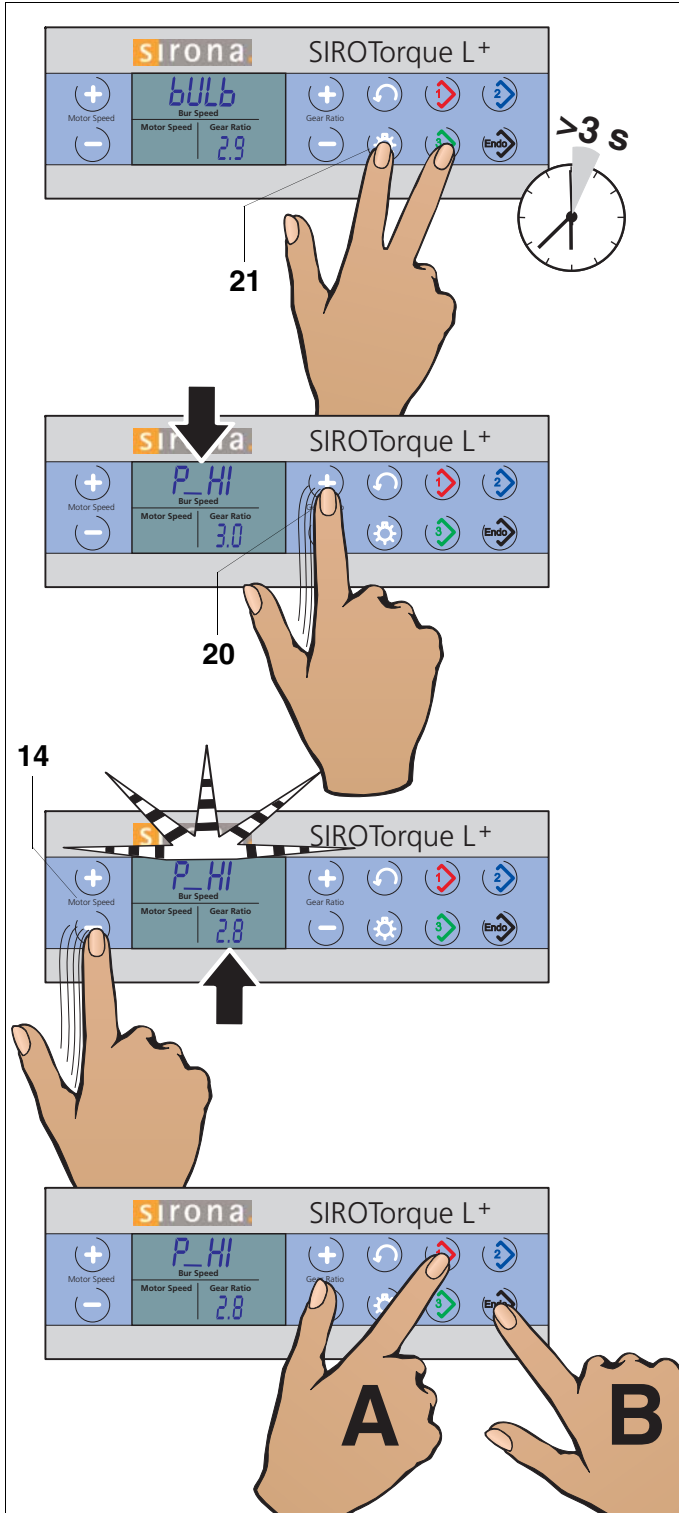1.1 "Limit value adjustment" menu

With an incoming air pressure of at least 3 bar you can work with the factory settings without having to adjust the limit values.
The limit value adjustments described below only show the different setting options.

Entering the "limit value adjustment" menu

Briefly press the **instrument light key (21)** and **save key 3** simultaneously.
The parameter *BULB* then appears on the "Bur Speed" display. At the same time, the value programmed for this parameter, *2.9*, appears on the gear ratio display. You are now in the "limit value adjustment" menu.

Parameter selection is performed using the +/- keys for the gear ratio (20).

The name of the relevant parameter is shown on the "Bur Speed" display field.

The **adjustment of the value** of the selected parameter within the corresponding limits is performed via the +/- keys for the motor speed (14). The value of the parameter is shown in the "Gear Ratio" display field. As soon as this value is changed, the name of the parameter starts flashing on the "Bur Speed" display.

Order of parameters with factory-programmed values:


- | | | |
|--|------|-----|
| 1 Lamp voltage (light intensity) | BULB | 2.9 |
| 2 Max. current consumption (torque) | CU | 6.5 |
| 3 Air pressure at which the motor starts running (lower switching threshold) | P_LO | 1.0 |
| 4 Air pressure for maximum speed (upper pressure switching threshold) | P_HI | 3.0 |
| 5 Reset to factory settings | FI_S | 0.0 |
| 6 Pressure sensor calibration 1 bar (no setting required) | P_P1 | 0.0 |
| 7 Pressure sensor calibration 3 bar (no setting required) | P_P3 | 0.0 |

Saving the settings and quitting the menu

A – Briefly press **save key 1** to save the new parameter. When it has been successfully saved, the parameter stops flashing.

B – Briefly press the **Endo save key** to quit the "limit value adjustment" programming mode for the system parameters.

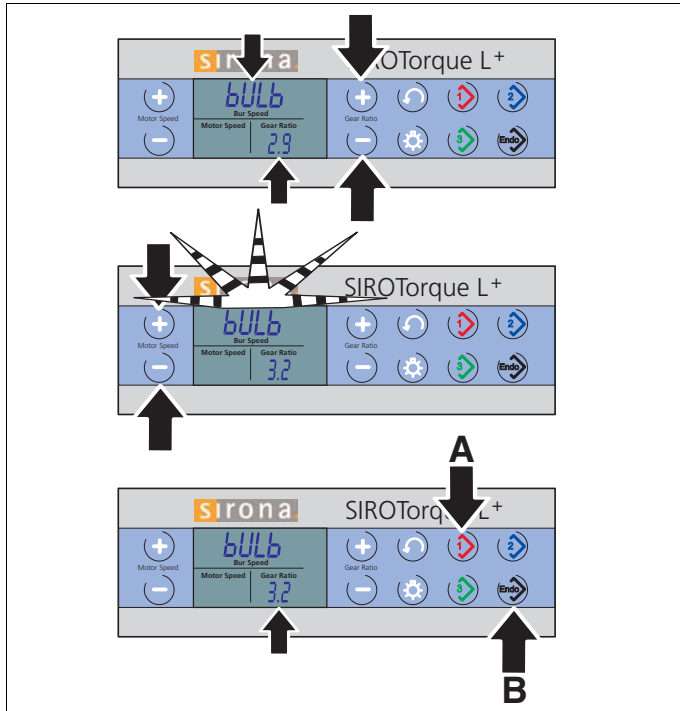1.2 „Bulb“ Instrument light - lamp voltage (light intensity)

If you feel that the cavity is over-illuminated or under-illuminated by the instrument light, you can adjust the lamp voltage.

To do this, select the "BULB" (instrument light) option from the "Limit value adjustment" menu on the "Bur Speed" display.

The "Gear Ratio" display shows a value of **2.9 (V)** (default setting).

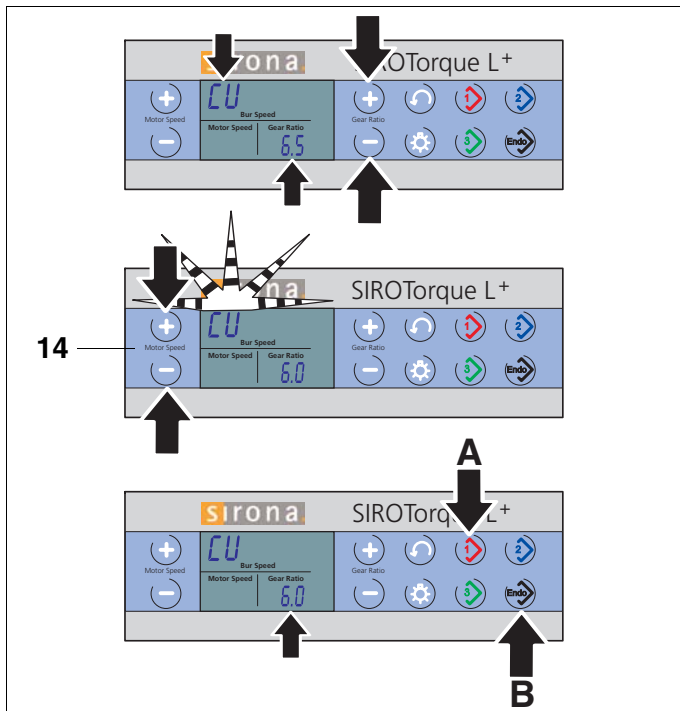
The value may be adjusted in a range between **2.5 V** and **3.5 V**.

To adjust the value, press the **- key (14) (darker)** or the **+ key (14) (brighter)** repeatedly; the step width is **0.1 V**. At the same time, "BULB" flashes.

A Briefly press **save key 1**, "BULB" stops flashing and the new setting is saved.

B Briefly press the **Endo save key** to quit the "limit value adjustment" programming mode.

1.3 "CU" limit of the maximum motor current (torque)

The torque of the electric motor can be changed as needed.

To do this, select the "CU" (max. motor current, torque) option from the "Limit value adjustment" menu on the "Bur Speed" display.

The "Gear Ratio" display shows a value of **6.5 (A)** (default setting).

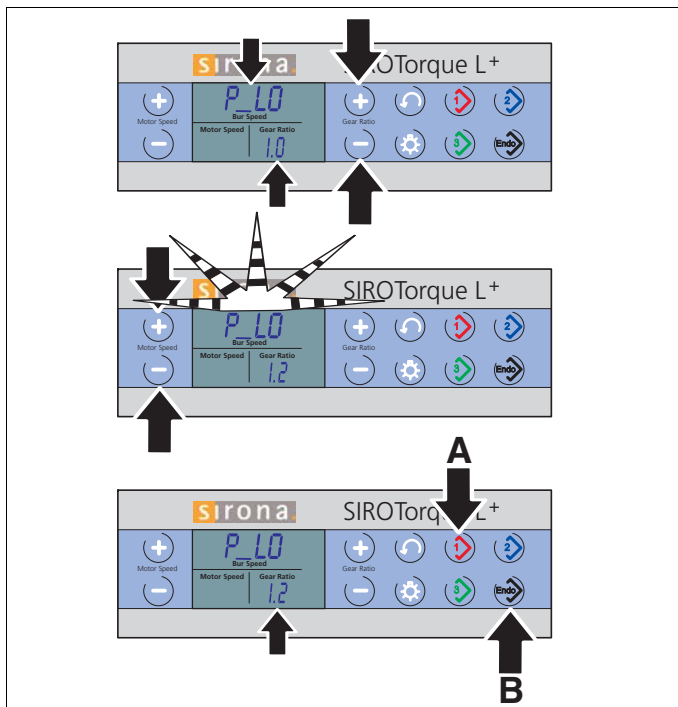
The value may be adjusted in a range between **0.5 A** and **7 A**.

To adjust the value, press the **- key (14) (lower)** or the **+ key (14) (higher)** repeatedly; the step width is **0.1 VA**. At the same time "CU" flashes.

A Briefly press **save key 1**, "CU" stops flashing and the new setting is saved.

B Briefly press the **Endo save key** to quit the "limit value adjustment" programming mode.

1.4 „P_LO“ Air pressure for the lower switching threshold on the foot switch

If the pneumatic speed controller foot switch reacts too sensitively for you at the switch-on point, you can delay the motor start until after a short pedal travel by increasing the switch-on air pressure.

To do this, select the "P_LO" (lower switching threshold of foot switch) option from the "Limit value adjustment" menu on the "Bur Speed" display.

The "Gear Ratio" display shows a value of 1.0 (bar) (default setting).

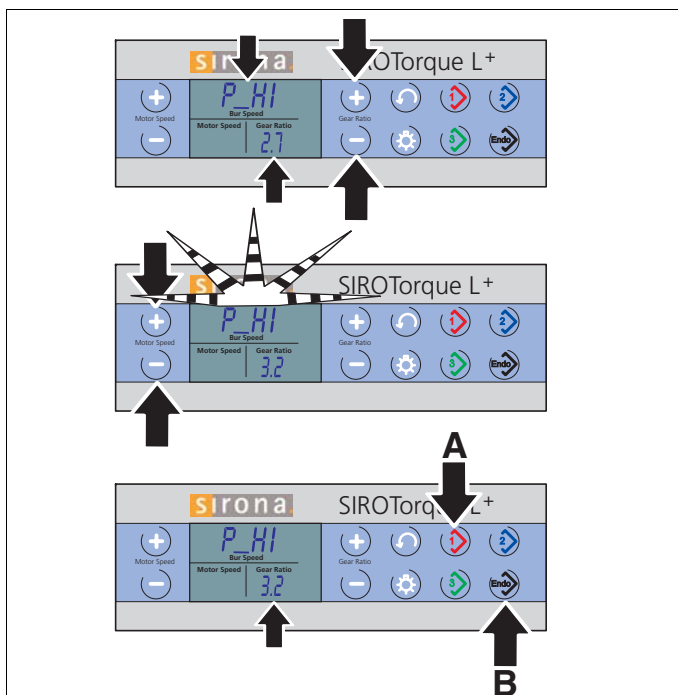
The value may be adjusted in a range between 7.0 psi (0.5 bar) and 29 psi (2.0 bar).

To adjust the value, press the – key (14) (**less pressure**) or + (14) (**more pressure**) repeatedly; the step width is 1.4 psi (0.1 bar). At the same time, "P_LO" flashes.

A Briefly press **save key 1**, "P_LO" stops flashing and the new setting is saved.

B Briefly press the **Endo save key** to quit the "limit value adjustment" programming mode.

1.5 „P_HI“ Air pressure for the upper switching threshold on the foot switch

To be able to fully use the control range of the pneumatic speed controller foot switch, the compressed air pressure in bar which is present at the air supply hose outlet should be set in this menu. If you want to reach the pre-set maximum speed already before having pushed the pedal fully down, you can reduce this pressure.

To do this, select the "P_HI" (upper switching threshold of foot switch) option from the "Limit value adjustment" menu on the "Bur Speed" display.

The "Gear Ratio" display shows a value of 2.7 (bar) (default setting).

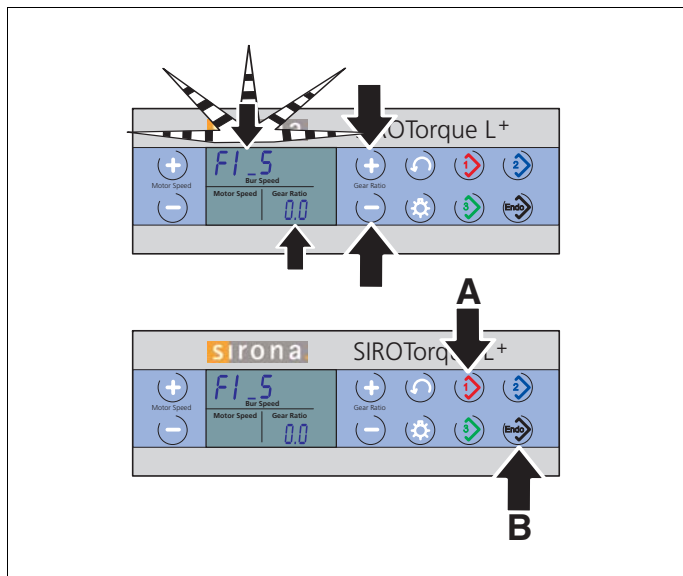
The value may be adjusted in a range between 39 psi (2.7 bar) and 58 psi (4.0 bar).

To adjust the value, press the – key (14) (**less pressure**) or + (14) (**more pressure**) repeatedly; the step width is 1.4 psi (0.1 bar). At the same time, "P_HI" flashes.

A Briefly press **save key 1**, "P_HI" stops flashing and the new setting is saved.

B Briefly press the **Endo save key** to quit the "limit value adjustment" programming mode.

1.6 "FI_S" reset to factory settings

If you are no longer satisfied with the modified settings, you can reset them to the factory settings.

To do this, select the "**FI_S**" (Reset to factory settings) option from the "**Limit value adjustment**" menu on the "Bur Speed" display.

The "Gear Ratio" display shows a value of **0.0**.

"**FI_S**" immediately starts to flash.

A Briefly press **save key 1**, "**FI_S**" stops flashing and the factory settings are restored.

B Briefly press the **Endo save key** to quit the "**limit value adjustment**" programming mode.

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH 2005
Sirona Dental Systems GmbH
Sirona Dental Systems LLC