# Softly

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# ENGLISH

## UNIVERSAL HIGH SPEED AMALGAMATOR FOR PRE-MEASURED AMALGAM CAPSULES

Thank you for choosing one of our products. We are at your entire disposal for any question you may have.

Please find below the instructions for use of your SOFTLY:

- After having connected the unit to the power supply main with earth (230V or 110V), switch on your SOFTLY by pushing the green switch on the back of the unit.
- Make your time mixing selection (up-down) following the instructions of the amalgam supplier.
- Insert the amalgam capsules into the fork placed under the front protection cover.
- Start the working cycle by pushing "start" ; you can interrupt it by pushing "stop".
- For the external cleaning of your SOFTLY, use a soft cloth moistened with water and soap.

### DISPOSAL AND RECYCLING

As electrical and electronic equipment, the device must be disposed of according to a specialized procedure for collection, pick-up and recycling or destruction (in particular on the European market, with reference to Directive 2002/96/EC of 27/01/2003).

When your device reaches the end of its life, we consequently recommend that you contact your dental equipment dealer (or, failing this, the nearest ACTEON GROUP office, the list of which is given in chapter 15), for information on how to proceed.

### LIABILITY

The manufacturer is not liable if:

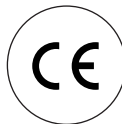
- the manufacturer's installation recommendations have not been followed (supply voltage, electromagnetic environment, etc.),
- repairs have been performed by persons not authorized by the manufacturer,
- the device has been used in an electrical installation which does not comply with current standards,
- the device has been used in a way which is not stipulated in this manual,
- accessories (tips, handpiece, pump, etc.) other than those supplied by SATELEC have been used,
- the instructions in this document have not been followed.

### Note:

The manufacturer reserves the right to modify the device and/or the user manual without notice.

### TECHNICAL DATA

- **Nominal voltage:**  
230V / 50Hz  
110V / 60Hz upon request
- **Absorbed power:** 50W
- **Fuse:** 500mA (230V); 1A (110 V)
- **Bipolar Switch with green light (ON)**
- **Mixing frequency:** 4000 vibrations/mn
- **Amplitude:** 25 mm
- **Functioning:** intermittent
- **Programmable mixing time:** 0-30 sec.
- **Dimensions:** 210 x 205 x 120 mm.
- **Weight:** 3.1kg

Manufactured in conformity with applicable regulations : De Götzen - Via Roma 45 - 21057 OLGiate OLONA (VARESE) - Italy

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