

ALIENS 3/32 - 9/64 - 3/8

## Diagnostic Menu

DSD-201 Endoscope Reprocessor Diagnostic Functions (ver. 5.00 and above)			
0	Deactivate all valves and motors	25	Test Keypad. (Hold * and press # to exit)
1	Activate detergent valve	26	Adjust LCD. (contrast push # to adjust)
2	Activate water valve	27	Adjust alarm volume. (2 to ↑ and 0 to ↓)
3	Activate alcohol pump	28	<b>Activate lidlock (Optional)</b>
4	Activate air system (compressor & valve)	29	<b>Deactivate lidlock (Optional)</b>
5	Activate air valve	30	Not used
6	Activate chamber valve	31	Calibrate basin temperature monitor
7	Activate disinfectant pump	32	Calibrate reservoir temperature monitor
8	Activate disinfectant valve	33	Calibrate A basin volume
9	Activate disinfectant overflow valve	34	Display all SSG/DSD Inputs
10	Activate disinfectant return valve	35	Set SSG chemical flow rates and LCG temp.
11	Activate drain valve	36	Set SSG PWM rates for chemical pumps
12	Activate alcohol valve	37	Test IIC/SSG
13	<b>Activate re-circulation pump (Optional)</b>	38	Calibrate SSG turbines
14	Activate water inlet valve	39	Set SSG water temperature and calibration point
15	Activate disinfectant inlet valve	40	"Fluid" and "Air" flow sense inhibit
16	<b>Activate re-circ. Inlet valve (Optional)</b>	41	Cover sense Enable
17	<b>Activate re-circ. Chamber valve (Optional)</b>	42	Basin level sense Enable
18	Activate valves incrementally	43	Reservoir high/low level sense Enable
19	Activate detergent pump	44	SSG sensor enable
20	Activate station A LED's	45	Sheath sensor enable
21	Activate station B LED's	46	Disinfectant warning inhibit (setup 7)
22	Activate system LED's	47	Alcohol sense enable
23	Display LCD test pattern	48	Detergent sense enable
24	Display counter operation (60 → 1)	49	Basin/reservoir temperature monitor Enable

Items in blue are contained within a 'VALVE CONTROL' setting. Diagnostics 0 will allow access.