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Instruction Manual

Manual de instrucciones

KARL STORZ AIDA® compact NEO

KARL STORZ AIDA® compact NEO

KARL STORZ AIDA® compact NEO

(

(

(

(

Inhalt	Contents	Contendio del manual
WillkommenIV	Welcome.....IV	Bienvenidos.....IV
Wichtiger HinweisIV	Important NoteIV	Important NoteIV
WarenzeichenIV	TrademarkIV	Marcas registradasIV
SchutzrechteIV	SchutzrechteIV	SchutzrechteIV
Software AIDA compact NEO.....V	AIDA compact NEO softwareV	Software AIDA compact NEO.....V
Auspacken 6	Unpacking 6	Desembalaje 6
Sicherheitsmaßnahmen am Aufstellort 6	Safety precautions at the site of installation..... 6	Medidas de seguridad en el lugar de emplazamiento..... 6
Sicherheitshinweise 7	Safety Instructions 7	Instrucciones de seguridad 7
Erklärung zu Warn- und Vorsichtshinweisen..... 7	Explanation of warnings and cautions..... 7	Declaración referente a las indicaciones de alarma y advertencia 7
Allgemeine Sicherheitshinweise 8	General safety information 8	Instrucciones generales de seguridad..... 8
Sicherheitshinweise AIDA® compact NEO 9	Safety instructions AIDA® compact NEO 9	Instrucciones de seguridadAIDA® compact NEO 9
Bestimmungsgemäße Verwendung 11	Intended use 11	Uso previsto 11
Qualifikation des Anwenders..... 11	User qualification 11	Cualificación del usuario 11
Sicherheitsmaßnahmen beim Einsatz des AIDA compact NEO Systems..... 12	Safety Precautions During System Use of AIDA compact NEO systems..... 12	Medidas de seguridad durante la utilización del sistema AIDA compact NEO..... 12
Sicherheitsmaßnahmen am Aufstellort 12	Safety Precautions at the Installation Location 12	Medidas de seguridad en el lugar de instalación 12
Grundlegende Software Bedienungshinweise..... 13	Fundamental software operating instructions..... 13	Instrucciones operativas básicas del software 13
Touch Screen Bedienung..... 13	Touch screen Operation..... 13	Manejo de la pantalla táctil..... 13
Software Einstellungsmöglichkeiten 14	Software setting options 14	Posibilidades de configuración del software..... 14

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1. Welcome

1.1 Important Note

Thank you for placing your trust in the name KARL STORZ. This product also reflects our utmost care and expertise. You have chosen an image and data archiving system based on modern and top-quality software by KARL STORZ.

This instruction manual will help you operate KARL STORZ AIDA® compact NEO correctly and to use its full potential. All necessary entries and commands are clearly explained. Please read the instruction manual carefully and keep within easy reach of your PC for future reference.

NOTE: The configuration and installation of KARL STORZ AIDA® compact NEO is described in a separate instruction manual. Please read this carefully.

KARL STORZ AIDA® compact NEO is delivered as a package with coordinated hard- and software. Any modification of hard- or software unauthorized by KARL STORZ results in a loss of medical approval and warranty. This includes in particular the additional installation of third-party software of any type and the updating of software of the KARL STORZ AIDA® compact NEO system (e.g., Windows®, MS-Office).

1.2 Trademark

KARL STORZ AIDA® is a registered trademark of KARL STORZ.

Windows® is a registered trademark of Microsoft Corporation.

AIDA: Advanced Image and Data Acquisition System

1.3 Property right

This product is protected in the USA by (at least one of the following) US Patent No(s) 5,788,688; 6,397,286; 6,484,221; 6,824,539.

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20 040402-20

20 040402-21 with communication

NOTE: The design and installation of KARL STORZ AIDA® compact NEO are described in the installation manual (96206171D). Please read it carefully.

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3. Unpacking

Check for missing items and evidence of damage during shipment. Should there be any grounds for complaint, notify the manufacturer or supplier immediately.

Keep the original packaging for later use. This may be useful if the unit has to be transported.

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
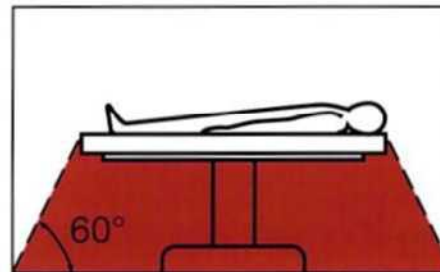
3.1 Safety precautions at the site of installation

The unit may only be used in medical rooms whose electrical systems have been installed in accordance with applicable national regulations.

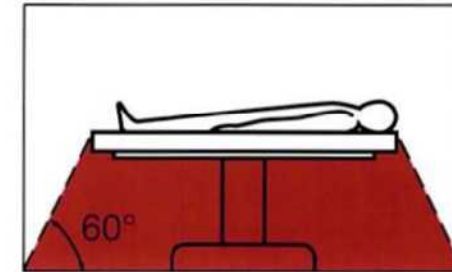It is not intended for use in hazardous zones. This means, for example, that when using easily combustible and explosive inhalation anesthetics or mixtures thereof, the unit must not be operated inside the hazard zone shown in the diagram. This also applies for easily combustible and explosive chemicals, e.g. skin disinfectants and fast-acting surface disinfectants.

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4. Safety Instructions

Safety instructions are measures intended to protect the user and patients from the risks which could arise in conjunction with use of the equipment cart.

4.1 Explanation of warnings and cautions

Please read this manual carefully and follow its instructions precisely. The words Warning, Caution and Note convey special meanings. Wherever they are used in this manual, the accompanying text should be carefully reviewed to ensure the safe and effective operation of the device. To make the words Warning and Caution stand out more clearly, they are accompanied by a pictogram.

WARNING: A Warning indicates that the personal safety of the patient or physician may be involved.

CAUTION: A Caution indicates that particular maintenance procedures or safety precautions must be followed to avoid possible damage to the device.

NOTE: A Note indicates special information about operating the product or clarifies important information

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WARNING: Read this instruction manual thoroughly and be familiar with its contents prior to using this equipment. Read the following safety instructions carefully to avoid putting your patients, personnel or yourself at risk.

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4.2 General safety information

WARNING: Combinations of medical devices are only assured to be safe if: they are identified as such in the respective instruction manuals or the intended purpose and interface specifications of the devices used in combination permit this.

WARNING: Always unplug the unit from the power supply before carrying out any maintenance work.

WARNING: Danger of an electrical shock. Do not open unit! Arrange for service work to be carried out by the manufacturer or by personnel authorized by the manufacturer only. Removal of covers by unauthorized personnel will void the equipment's warranty.

CAUTION: The device may only be operated at the voltage stated on the rating plate.

CAUTION: It is essential to ensure that no liquids can penetrate the housing. Do not store liquids on or above the unit.

NOTE: Any damage to the unit resulting from incorrect operation is not covered by the manufacturer's warranty.

NOTE: After repair work to the unit, the default factory settings may be activated for safety reasons. In this case, it will be necessary to reprogram all user-specific values.

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4.3 Safety instructions AIDA® compact NEO

-  **WARNUNG:** The electric installation of the operating room must comply with applicable IEC standards.
-  **WARNUNG:** The instructions for use and interface specifications for medical products and/or system components used in combination with the AIDA system must be observed precisely.
-  **WARNUNG:** Safety regarding the use of a combination of medical products is assured only if
- specified as such in the respective instruction manual or
 - intended use and the interface specifications of the products used in combination permit this (cf. IEC 60601-1-1).
-  **WARNUNG:** Accessory equipment connected to the analog and digital interfaces (signal inputs and outputs) must be set up and tested according to IEC 60601-1 for medical electrical equipment and additionally to IEC 60950 for data processing equipment. Any person who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore responsible for ensuring that the system complies with the requirements of the system standard IEC 60601-1-1. In doubt, consult the technical service department or your local representative.
-  **WARNUNG:** The KARL STORZ AIDA® compact NEO System is only reliably grounded when connected to a properly installed, shockproof power outlet. The plug and cord should be routinely inspected. Do not use if damaged.
-  **WARNUNG:** Keep the KARL STORZ AIDA® compact NEO System out of the reach of patients.
-  **WARNUNG:** Do NOT use the KARL STORZ AIDA® compact NEO System in the presence of inflammable anesthetic gas.
-  **WARNUNG:** Always disconnect power supply before carrying out any maintenance work on the KARL STORZ AIDA® compact NEO System control computer.

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 **WARNING:** Do not open the control computer of the KARL STORZ AIDA® compact NEO System! Risk of electric shock. Servicing should only be carried out by the manufacturer or personnel authorized by the manufacturer. Any opening of the control computer by unauthorized personnel shall cause the warranty to be terminated.

 **CAUTION:** Please inspect the control computer regularly for mechanical defects. Report any damage to KARL STORZ.

 **CAUTION:** Only change operating system settings authorized by KARL STORZ. Incorrect operating system settings can cause the KARL STORZ AIDA® compact NEO software to malfunction (including loss of data).

 **CAUTION:** Regular back-ups must be performed to safeguard data. KARL STORZ is not liable for loss of data. The user is solely responsible for data security.

 **CAUTION:** The control computer of the KARL STORZ AIDA® compact NEO System may only be operated at the voltage indicated on the manufacturer's identification plate.

NOTE: The manufacturer's identification plate can be found on the base of the housing.

NOTE: Any damage to the AIDA system resulting from operating errors is not covered by the manufacturer's warranty.

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4.4 Intended use

The KARL STORZ AIDA® compact NEO system (Advanced Image and Data Acquisition System) is a PC-based documentation system for still images, video and audio sequences. AIDA software may only be used for documentation purposes. The documented images are not approved for diagnosis.

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4.5 User qualification

The KARL STORZ AIDA® compact NEO system may only be used by physicians or medical assistants with the relevant qualifications and authorization and who have been instructed in the use of the system. The user must observe all applicable national data protection laws regarding patient data. In public hospitals, the data protection officer must be notified.

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4.6 Safety Precautions During System Use of AIDA compact NEO systems

The user is to ensure that the system is functioning properly before use. If a KARL STORZ AIDA® compact NEO system is used during the procedure, the patient must be treated and kept under observation with the usual care. This includes constant monitoring of the procedure, the patient's vital signs and anesthesia. A procedure may only be performed if visual observation of the unit's function is ensured.

4.7 Safety Precautions at the Installation Location

The KARL STORZ AIDA® compact NEO system may only be used in medical rooms whose electrical systems are installed in compliance with applicable national regulations. The system is not intended for use in areas where there is a risk of explosion. For example: In areas where easily combustible and explosive inhalation anesthetic agents or mixtures are used, the system should not be operated in the demarcated danger zone. This also applies to easily combustible and explosive chemicals, e.g. skin disinfectants and surface disinfectants. The control computer of KARL STORZ AIDA® compact NEO System is equipped with a connector for attaching a potential equalizer.

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5. Fundamental software operating instructions**5.1 Touch screen Operation**

If the program is operated on a regular basis, KARL STORZ recommends the use of a touchsensitive screen (touch screen).

The on-screen keyboard for touch screen operation can always be displayed by touching any input field on the screen. The input field is active when it is shown against an orange background.

The AIDA compact NEO software can be operated in a number of different ways. Operation is possible using the:

- Touch screen
- Mouse
- Keyboard

The terminology in this instruction manual has thus been intentionally selected as follows:

①	Touch
②	Double-touch

This means you can choose whether to use the mouse, the keyboard or your fingertip for touch screen operation.

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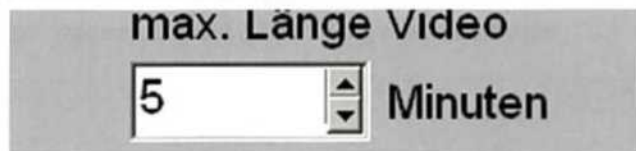
5.2 Software setting options

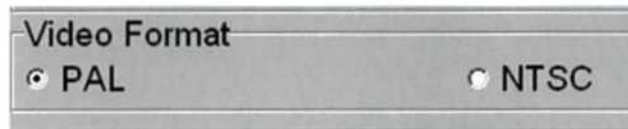
Selection field: If you touch this field, a list opens. One item at a time can be selected from this list.


Check button: You can activate and deactivate the property offered by this option. You can also activate several options which are adjacent.

Option button: You can activate and deactivate the property offered by this option. Other options which are adjacent cannot be simultaneously activated. Only one option can be activated at a time. As soon as you activate another, the option which was previously activated will be automatically deactivated again.

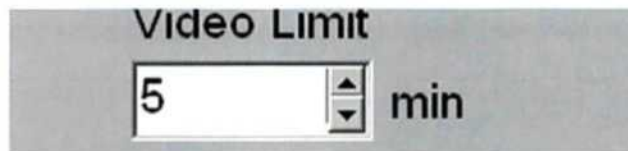Spin field (increment and decrement field): If you touch this field directly, you can enter a numerical value. If you touch the plus or minus sign or the up/down arrows, you can increase/decrease the numerical value by one each time.

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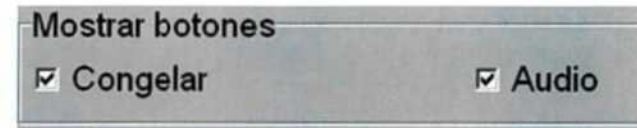
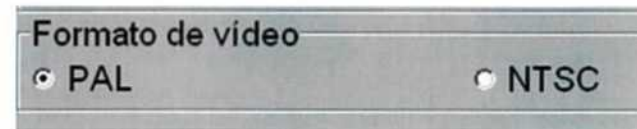
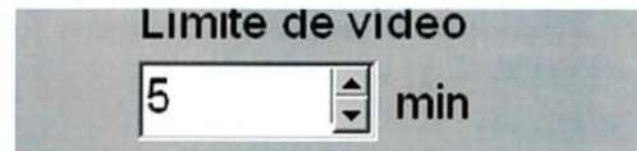
Fundamental software operating instructions

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aktiv

D:\EXPORT

Input field: Here you can enter any text you want.

Header 2

Path field: If you touch the button next to this field, an Explorer window opens.

Here you can navigate to a file or a folder as usual in Explorer and select this file/folder to specify a path.

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6. Overview of user interface

The user interface is subdivided into three main areas:

- | | |
|---|--|
| ① | Information area |
| ② | Application window (application header, general functions, content area) |
| ③ | Navigation area (main menu, workflow bar) |

6.1 Information area

The information area displays the date and time in every application, as well as information and warning messages in the event of device alarms.

NOTE: Always take note of warning messages from connected devices. The warnings displayed by the system merely serve as additional information and may sometimes be incomplete.

NOTE: Further details about these information and warning messages can be found in the system instruction manual 96206177D.

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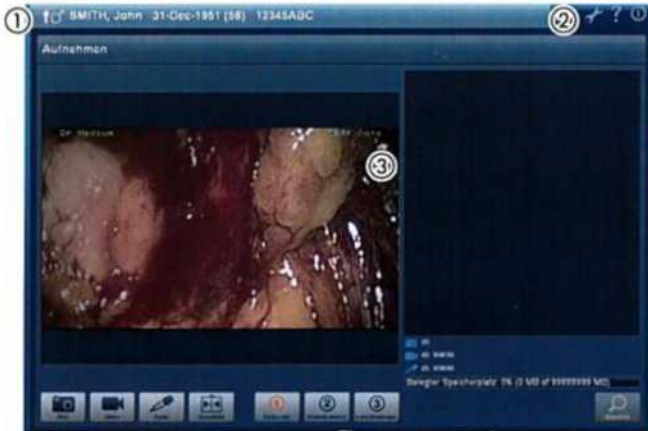
6.2 Application window

The application window is divided into three sub-areas:

- ① Application header
- ② General functions
- ③ Content area

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6.2.1 Application header

- ① Select Patient/Show Patient Data

You can see the key patient data in the application header. Double-touch the patient's name here to access the "Patient Data" menu.

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6.2.2 General functions

- ③ Settings
- ④ Additional information
- ⑤ Log Out

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6.2.3 Content area

The content area varies depending on the menu called up.

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6.3 Navigation area

The navigation bar is subdivided into two areas:

- ① Main menu
- ② Workflow bar

6.3.1 Main menu

The main menu contains buttons to switch between the 4 systems CHEX, SCB, AV and AIDA. Whenever each system is activated, its button is increased in size to highlight it.

NOTE: There is a separate instruction manual for each of the individual systems. The Article number for the individual instruction manuals are shown in the table below.

①	Button: Chex	
②	Button: SCB	96206149D
③	Button: AV	96206176D
④	Button: AIDA	

NOTE: Whenever a system is activated, its button is shown in the main menu increased in size. If an application is not available, its button appears in gray.

- ① Application inactive - Normal display for button
- ② Application not available - Button shown in gray
- ③ Application active - Button shown increased in size

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6.3.2 Workflow bar

In AIDA compact NEO mode, the workflow bar contains the 4 buttons Patient, Record, Reference and Finish. You can use the workflow bar to navigate between the 4 menu items of the AIDA system.

①	Button: Patient
②	Button: Record
③	Button: Reference
④	Button: Finish

NOTE: The content area varies depending on which of the 4 buttons you have activated in the workflow bar. The section "Operation of software's describes the 4 possible content areas.

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7. Starting/exiting AIDA compact NEO

You can start and exit AIDA compact NEO via the software or the ON/OFF switch at the unit.

➤ Starting AIDA compact NEO

1. Actuate the "ON/OFF switch" at the unit.

☛ The following start screen will appear:

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➤ Exiting AIDA compact NEO

1. To exit the system, click on the "Exit" button in "General functions" on the top right.

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8. Settings

The most important settings of KARL STORZ AIDA compact NEO are made in the "Settings" menu.

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» Bringing up the Settings menu

1. Click on the "wrench icon" in the header.

☛ Opening the "Settings" screen requires a password.

2. Enter the password 0000 and click OK to confirm.

☛ The "Settings" screen then opens. The "Global Settings" tab is preselected.

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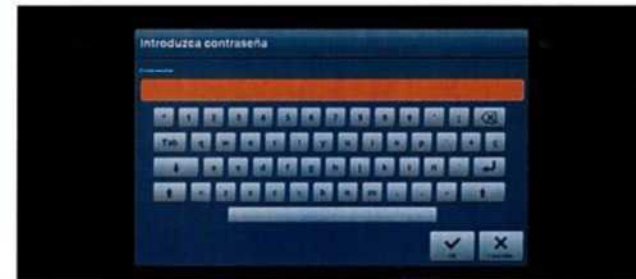
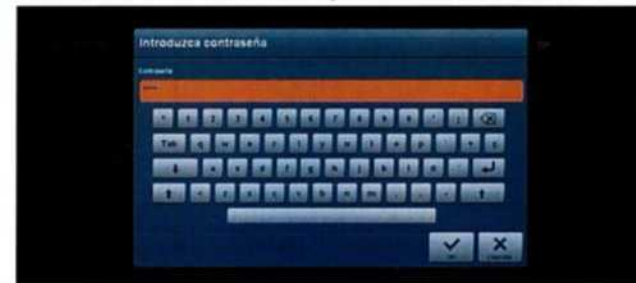
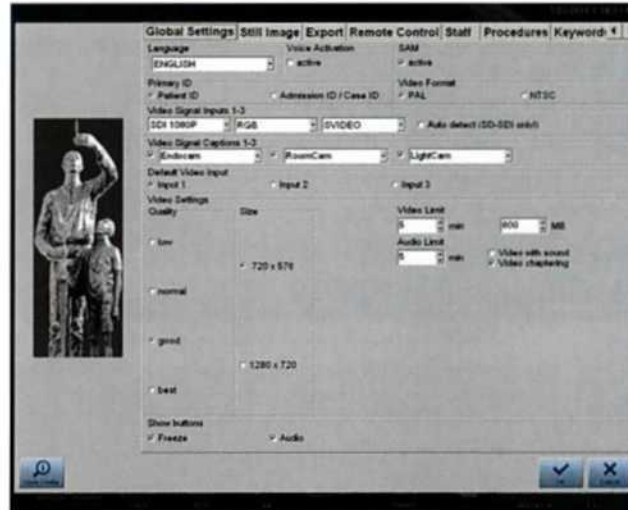
The following tabs are available:

- Global Settings
- Still Image
- Export
- Remote Control
- Staff
- Procedures
- Keywords
- Serial Numbers
- Communication (described in Installation Instructions 96206171D)

The following pages describe the settings you can make under each tab.

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8.1 General information on the "Settings" interface

Actuating this button allows settings to be changed under the "Communication" tab. This feature is described in the Installation Instructions 96206171D rather than in these instructions for use.

Actuating this button confirms the changed settings and takes you out of the Settings menu. You are returned to the "software".

NOTE: To ensure that the settings are saved correctly, restart KARL STORZ AIDA compact NEO after every change in settings.

Actuating this button cancels any changes in settings and takes you out of the "Settings" menu. You are returned to the "software".

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
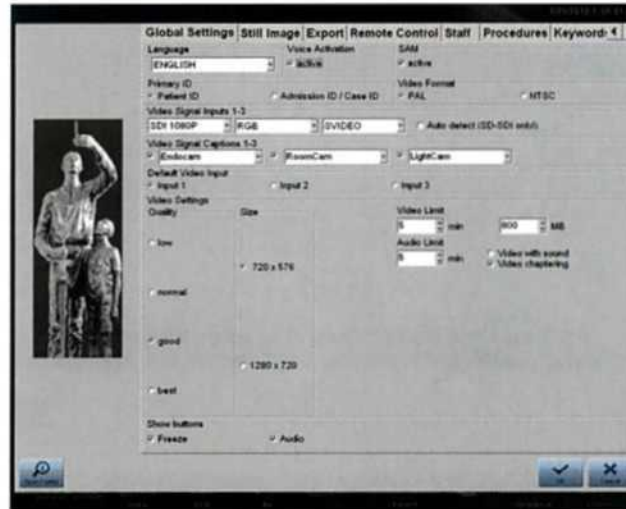
8.2 Global Settings

The "Global Settings" tab is used for:

- Selecting a language
- Enabling/disabling voice activation
- Enabling/disabling SAM
- Selecting the primary ID
- Selecting the video format
- Configuring video signals 1-3
- Enabling/disabling auto detect at SDI input
- Configuring, enabling/disabling video signal captions 1-3
- Selecting the default video input
- Changing video settings (Quality, Size, Video Limit)
- Configuring audio to video
- Selecting buttons to be displayed

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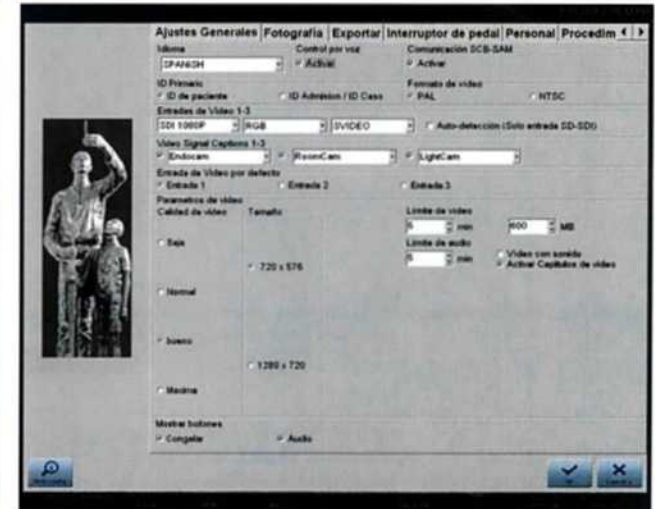
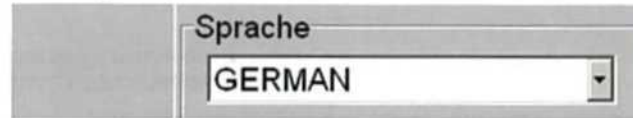
- Chinese
- Danish
- English
- French
- German
- Italian
- Japanese
- Polish
- Portuguese
- Russian
- Spanish
- Swedish

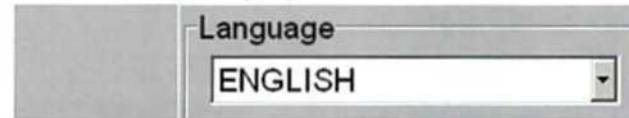
8.2.1 Selecting a language

Language: Clicking on the "Language" selection field opens the language list (see below) from which you can select the desired language.

- Chinese
- Danish
- English
- French
- German
- Italian
- Japanese
- Polish
- Portuguese
- Russian
- Spanish
- Swedish

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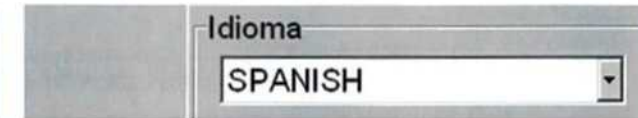
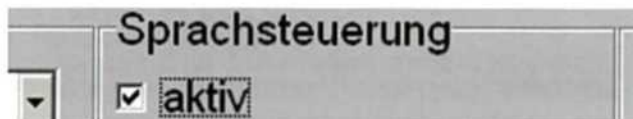
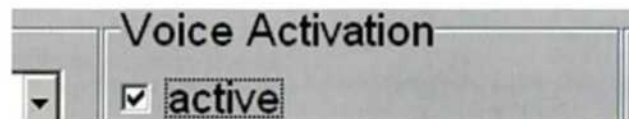
8.2.2 Enabling/disabling voice activation

Voice Activation: Checking the box under "Voice Activation" allows you to control the software with voice commands using the predefined set of commands.

NOTE: If your selected language is German, the German set of commands is available for voice activation. For all other languages, the English set of commands is active.

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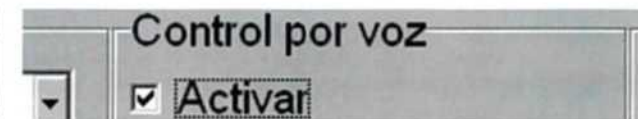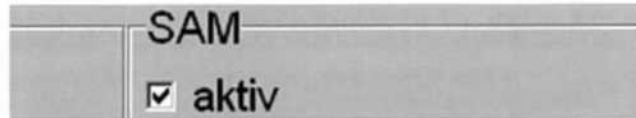
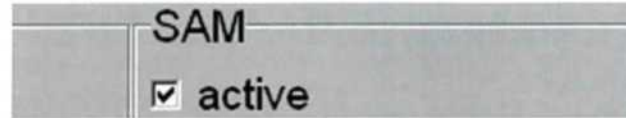
8.2.3 Enabling/disabling SAM

SAM: Checking the box under "SAM" allows AIDA compact NEO to communicate with other OR1 systems.

NOTE: To permit communication between AIDA compact NEO and the OR1 system, SAM must be correctly configured and enabled (see Installation Instructions 96206171D).

8.2.4 Selecting the primary ID

Patient ID, Admission ID / Case ID: Using the option buttons "Patient ID" and "Admission ID / Case ID", you can choose which number to use as the primary ID.

NOTE: When using DICOM or HL7, the primary ID for a dataset must be indicated.

8.2.5 Selecting the video format

PAL, NTSC: You can select the video file format using the option buttons "PAL" and "NTSC".

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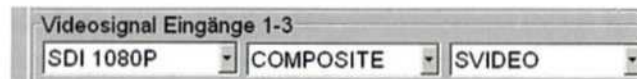
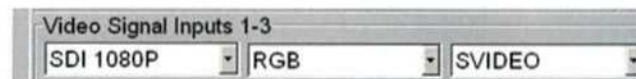
8.2.6 Configuring video signals 1-3

Video signals 1-3: Clicking one of the "Video Signal Inputs 1-3" selection fields opens the list of device configuration files (DCF) (see below), from which you can select a device configuration file (DCF).

The following device configuration files (DCF) are available:

- COMPOSITE
- DVI 1080I
- DVI 1080P
- RGB
- SDI 1080I
- SDI 1080P
- SDI
- SVIDEO

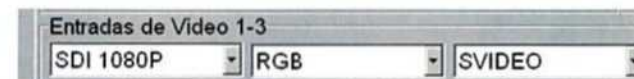
8.2.7 Auto detect at the SDI input

Auto detect (SD-SDI only!)

Auto detect SD-SDI only: If you check the "Auto detect (SD-SDI only)" box, SD signals are automatically detected at the SDI input.

NOTE: Autodetection of SD signals only works with IMAGE1™ camera systems.

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8.2.8 Configuring, enabling/disabling video signal captions 1-3

Video Signal Captions 1-3 (configuring): Clicking on one of the "Video Signal Captions 1-3" selection fields opens a list; here, you can select an entry or enter your own text.

NOTE: This text is then displayed in the button on the user interface ("Capture" content area).

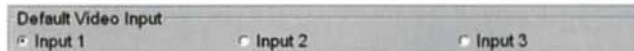
Video Signal Captions 1-3 (enabling/disabling): Checking one of the "Video Signal Captions 1-3" boxes displays the camera input on the user interface ("Capture" content area) as a button.

NOTE: Only the activated camera inputs are displayed on the user interface.

NOTE: The inputs that receive a signal must be activated.

NOTE: If only one input is activated, no button is shown on the user interface of the program.

8.2.9 Selecting the default video input

Input 1, Input 2, Input 3: Using the option buttons "Input 1", "Input 2", and "Input 3", you can select a default video input.

NOTE: The selected default video input is displayed as a default at bootup.

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Video Limit
5 min 600 MB

Audio Limit
5 min Video with sound
 Video chaptering

Video Limit: Using the "Video Limit" up/down button, you can select the video length in minutes.

NOTE: If the video limit is set to zero, the recording continues until you stop it.

Audio limit: Using the "Audio Limit" up/down button, you can select the length of the audio comment in minutes.

Video Limit [MB]: Using the "Video Limit [MB]" up/down button, you can select the length of the video in MB.

NOTE: If the video limit is set to zero, the recording continues until you stop it.

Video with sound: Checking the "Video with sound" box records the video with sound.

Video chaptering: Checking the "Video chaptering" box automatically creates a new video file when the defined file limit (video limit [MB]) has been reached.

NOTE: The recorded video file (video chapter) is divided into individual chapters based on the selected file size.

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8.2.11 Selecting buttons to be displayed

Freeze: Checking the "Freeze" box shows the "Freeze" button on the user interface.

Audio: Checking the "Audio" box shows the "Audio" button on the user interface.

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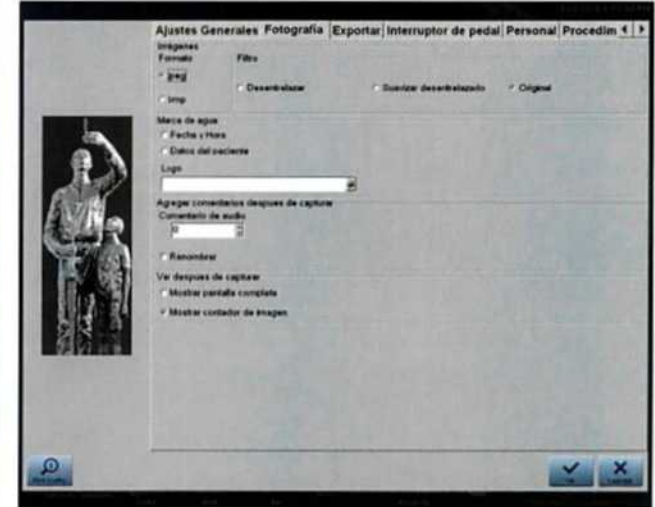
8.3 Still Image

The "Still Image" tab is used for:

- Selecting the format
- Selecting filters
- Enabling watermarks
(selecting the type of watermark)
- Selecting the length of audio comment after capture
- Adding comments after capture
- Viewing after capture

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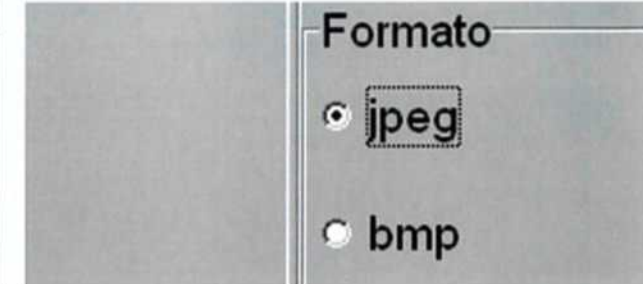
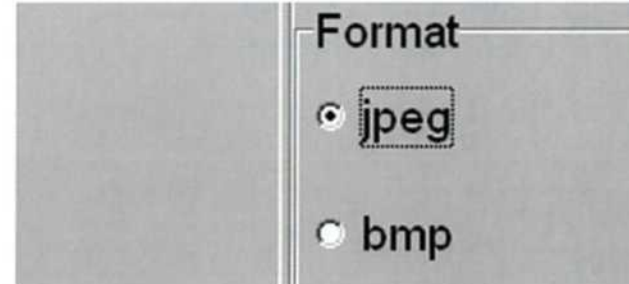
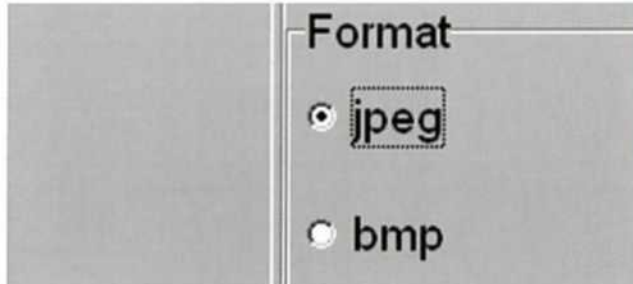
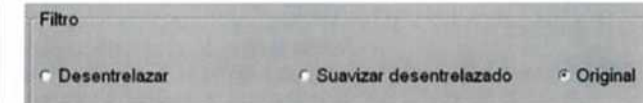
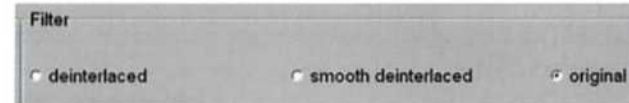
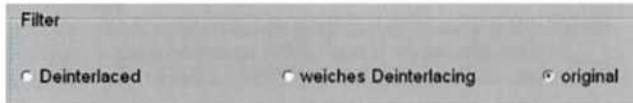
jpeg, bmp: Using the "jpeg" and "bmp" option buttons, you can select a format for the recorded images.

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Deinterlaced, smooth deinterlaced, original: Using the "deinterlaced", "smooth deinterlaced" and "original" option buttons, you can select a conversion filter for the recorded images.

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8.3.3 Enabling watermarks (selecting the type of watermark)

NOTE: The information selected for the watermark is placed in the corners of the image.

Date and time: Checking the "Date and Time" box displays the date and time the image was recorded as a watermark.

Patient data: Checking the "Patient Data" box displays the patient data as a watermark.

Logo: If you enter the path to a desired logo file using the path field, the selected logo is displayed as a watermark.

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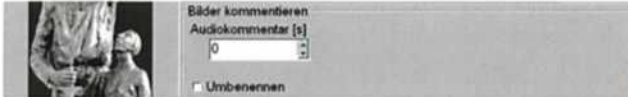
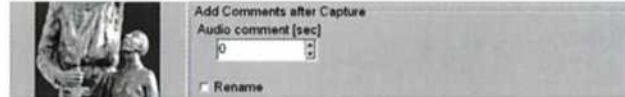
8.3.4 Adding comments after capture


Audio comment: Using the "Audio comment" up/down button, you can set the length of the audio comment in seconds.

NOTE: An audio comment of the selected length is then recorded after each image capture.

Rename: Checking the "Rename" box displays the dialog for renaming the audio comment after each image capture.

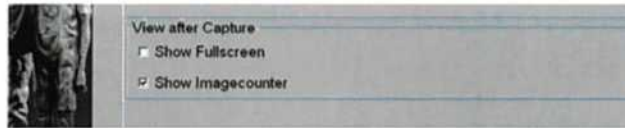8.3.5 Viewing after Capture

Show Fullscreen: Checking the "Show Fullscreen" box displays the full screen after each image capture.

Show Imagecounter: Checking the "Show Imagecounter" box displays the image counter after each image capture.

NOTE: Only one of the two boxes can be checked at a time.

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8.4.1 Selecting the Maximum Storage Space per Procedure

650 MB CD, 700 MB CD, 4,7 GB DVD, user defined: You can select the maximum storage space per procedure using the "650 MB CD", "700 MB CD", "4,7 GB DVD", and "user defined" option buttons.

Export to Directory: Identify the directory to which to export by entering a path to a desired storage location in the "Export to Directory" path field.

Active: Checking the "Active" box activates export to that directory.

NOTE: This setting is independent of the setting for burning to CD/DVD and printing to a printer.

NOTE: Network drives, USB drives, and FTP servers can be selected as paths.

CAUTION: Please note that AIDA compact NEO is not an archiving system and that KARL STORZ does not accept responsibility for data stored on the local hard drive.

NOTE: The FTP function can only be used in conjunction with a Western European font (non-Unicode).

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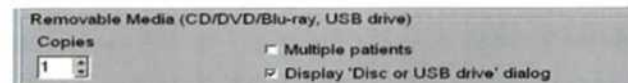
8.4.3 Enabling XML viewer

XML viewer: Checking the "XML viewer" box renders datasets burned onto CD/DVD viewable on the Internet as xml files using a style sheet.

8.4.4 Setting up storage to removable media (CD, DVD, Blu-ray, USB)

Copies: Using the "Copies" up/down button, you can select the number of CDs or DVDs to be burned.

Multiple patients to storage medium: If you check the "Multiple patients" box, several procedures are stored on one CD or DVD.

NOTE: At the end of each procedure, the data are then stored as buffered procedures; you can edit this memory after every procedure and select which of the procedures in the memory to store to the CD/DVD.

Display 'Disc or USB drive' dialog: If the "Display 'Disc or USB drive' dialog" box is checked, you are asked after every procedure to choose between storing the patient data to a USB stick or burning it to a CD/DVD.

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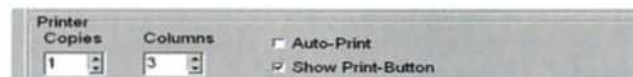
8.4.5 Configuring the printer

Here, you can configure the printer and print-outs.

Copies: Using the "Copies" up/down button, you can select the number of automatically printed reports.

Columns: Using the "Columns" up/down button, you can select the number of images placed side-by-side.

Auto-Print: Checking the "Auto-Print" box automatically generates a print-out after every procedure.

NOTE: You can select the number of printouts to be automatically generated under Copies.

Show Print Button: If the "Show Print Button" box is checked, a button with which you can manually initiate the printing of reports appears in the editing screen.

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8.4.6 Selecting Logo/Header1/Header2 for print-out

Here, you can identify the logo and various recurring headers for the printed reports.

Logo: If you enter the path to your desired logo file using the "Logo" path field, the selected logo is printed on the print-out.

NOTE: Only bmp files are displayed and printed.

NOTE: The logo is printed in the right upper corner of the report.

NOTE: The selected file is displayed as a preview. Double-clicking on the preview allows you to select a new logo as well.

Header 1: If you click on the input field "Header 1" and enter a text, this text is printed on the reports as Header 1.

Header 2: If you click on the input field "Header 2" and enter a text, this text is printed on the reports as Header 2.

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
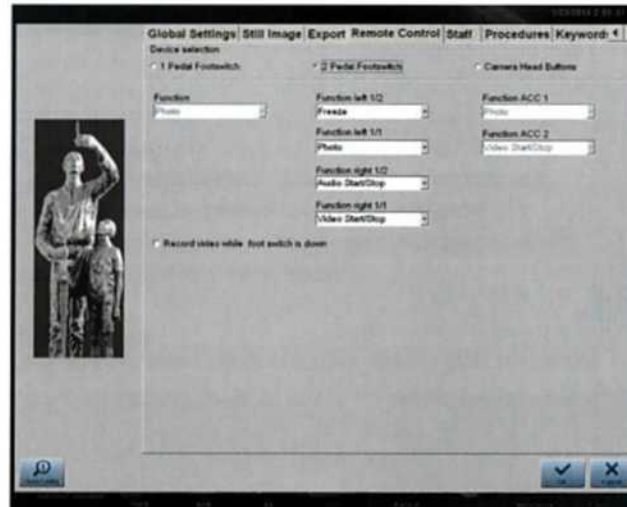
8.5 Remote Control

The "Remote Control" tab used for:

- Selecting device (1 Pedal Footswitch, 2 Pedal Footswitch, Camera Head Buttons)
- Assigning functions to the 1-pedal footswitch
- Assigning functions to the 2-pedal footswitch
- Assigning functions to the camera head buttons

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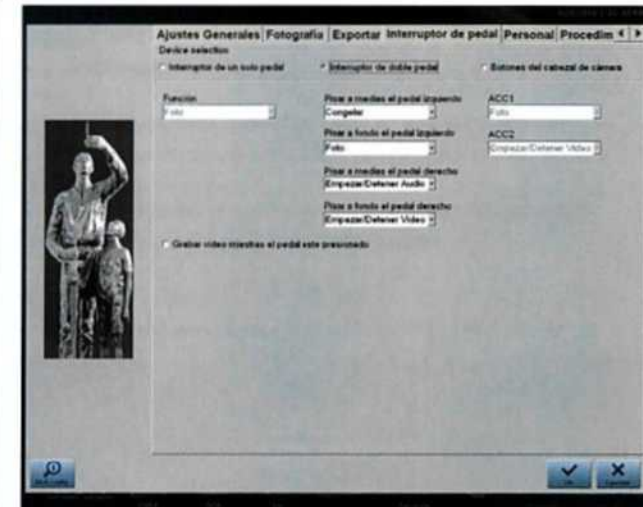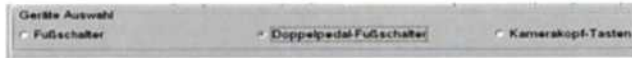
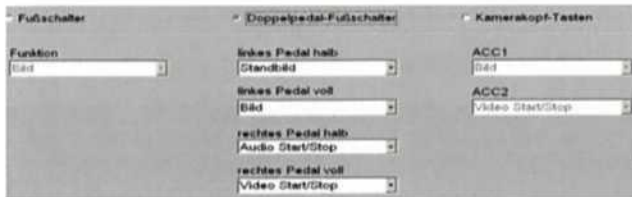
8.5.1 Device selection (1 Pedal Footswitch, 2 Pedal Footswitch, Camera Head Buttons)

Here, you can select the device with which you want to control specific functions.

1 Pedal Footswitch, 2 Pedal Footswitch, Camera Head Buttons: You can select the employed device using the "1 Pedal Footswitch", "2 Pedal Footswitch", and "Camera Head Buttons" option buttons.

8.5.2 Assigning functions to the device

Here, you can select the functions you want to control with the selected device.

The following functions are available:

- **Photo:** Stores the displayed live image as a freeze frame
- **Freeze:** Switches between freeze and live image
- **Video Start/Stop:** Starts and stops the video recording
- **Audio Start/Stop:** Starts and stops the audio recording
- **Next video input:** Switches to the next signal input of the grabber card
- **None:** No function
- **Switch monitor input:** Switches between two video inputs in KARL STORZ 19" and 23" monitors

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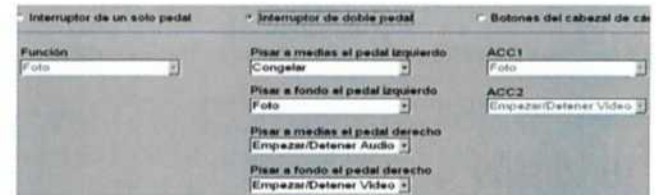
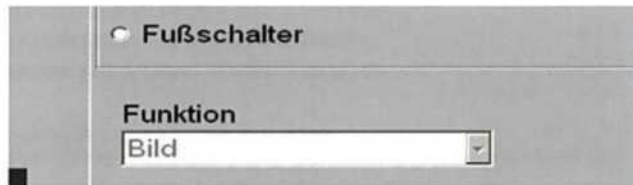
NOTE: When using the 2-pedal footswitch and the camera head buttons, the camera head button ACC1 corresponds to the function of the completely depressed left footswitch pedal. The function of the camera head button ACC2 corresponds to the function of the completely depressed right footswitch pedal. Therefore, you must connect the ACC1 output of the camera module with the ACC input 2 at the AIDA PC (on the back). Connect the ACC2 output of the camera module with the ACC input 3 at the AIDA PC.

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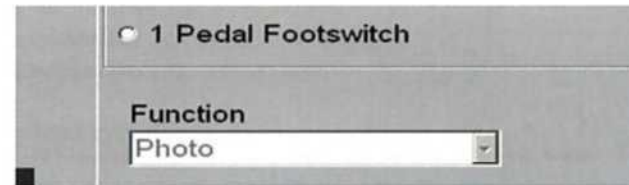
8.5.3 Assigning functions to the 1-pedal footswitch

The selection field is used to select the desired function of the 1-pedal footswitch.

Function: Select function

Record video while foot switch is down: If the "Record video while foot switch is down" box is checked, video recording continues as long as the pedal of the footswitch is pushed down.

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8.5.5 Assigning functions to the camera head buttons

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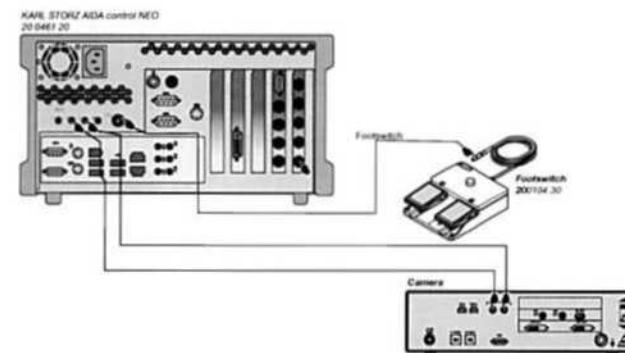
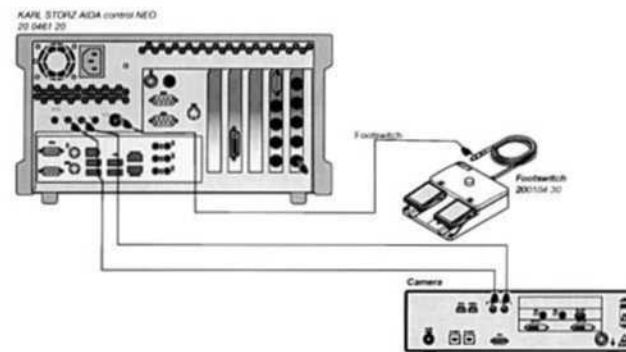
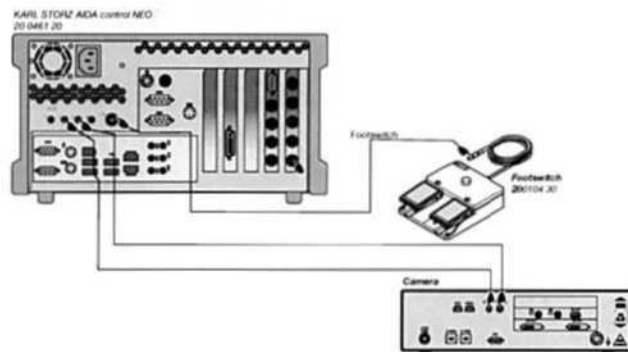
NOTE: Connect the ACC1 output of the camera module with the ACC input 2 at the AIDA PC (on the back). Connect the ACC2 output of the camera module with the ACC input 3 of the AIDA PC (on the back).

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Function for camera head buttons: The selection field is used to select the desired functions of the camera head buttons.

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
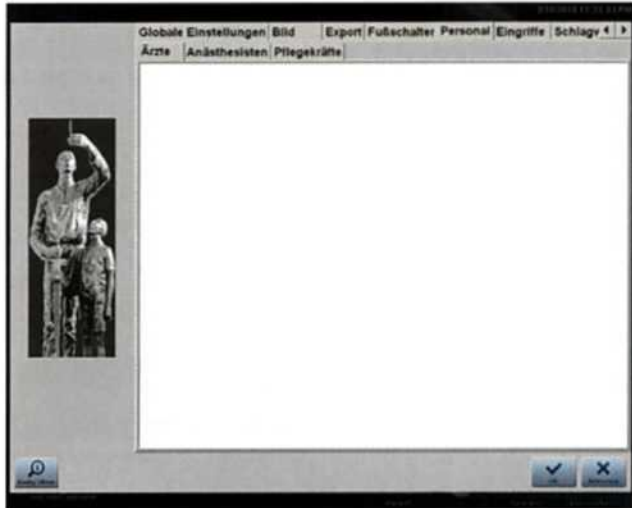
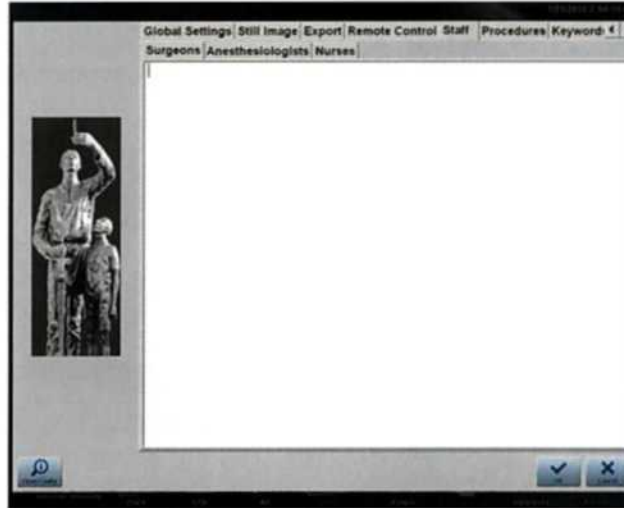
8.6 Staff

The "Staff" tab is used for:

- Creating a surgeon list
- Creating an anesthesiologist list
- Creating a nurse list

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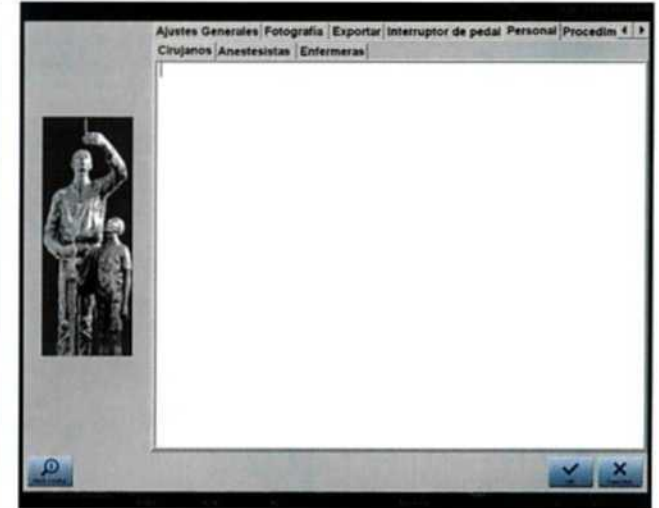Surgeons: On this screen, you can create a list of surgeons. The white area is a text editor which you can use to create a list of surgeons. Each line corresponds to a list entry.

8.6.2 Creating an anesthesiologist list

Surgeons	Anesthesiologists	Nurses
	Dr. Bloggs	

Anesthesiologists: On this screen, you can create a list of anesthesiologists. The white area is a text editor which you can use to create a list of anesthesiologists. Each line corresponds to a list entry.

8.6.3 Creating a nurse list

Surgeons	Anesthesiologists	Nurses
		Jane Doe

Nurses: On this screen, you can create a list of nurses. The white area is a text editor which you can use to create a list of nurses. Each line corresponds to a list entry.

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
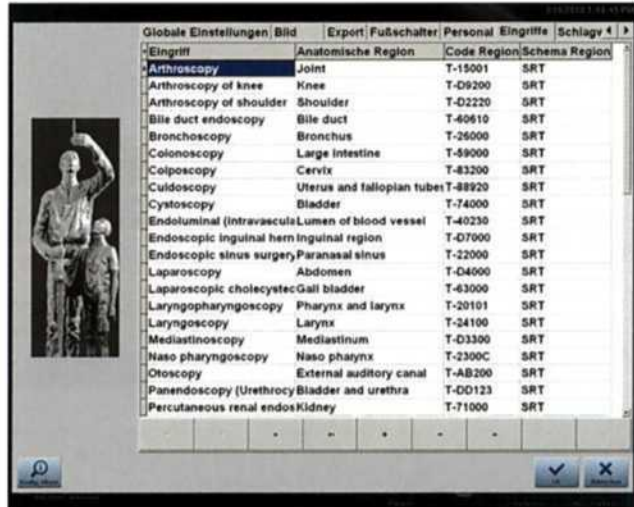
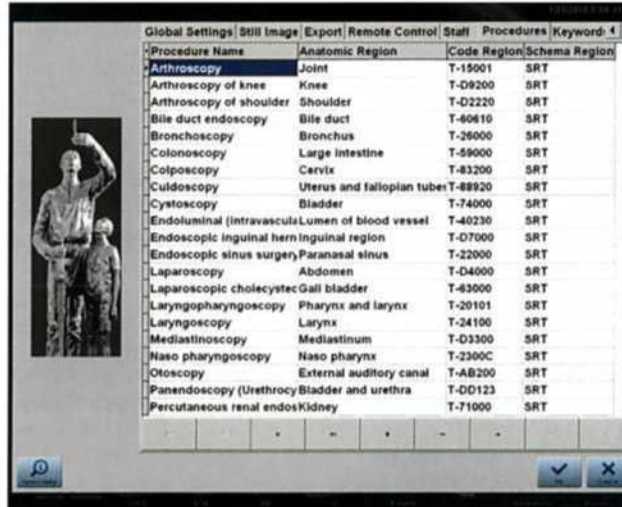
8.7 Procedures

The "Procedures" tab is used to do the following:

- Creating a procedure list with procedure / anatomic region / code region / schema region

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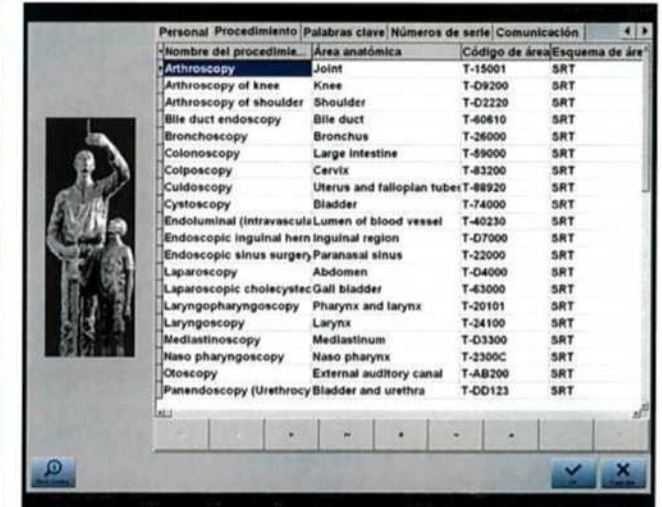8.7.1 Creating a procedure list

Procedure Name	Anatomic Region	Code Region	Schema Region
Arthroscopy	Joint	T-15001	SRT
Arthroscopy of knee	Knee	T-D9200	SRT
Arthroscopy of shoulder	Shoulder	T-D2220	SRT
Bile duct endoscopy	Bile duct	T-60610	SRT
Bronchoscopy	Bronchus	T-26000	SRT
Colonoscopy	Large intestine	T-59000	SRT
Colposcopy	Cervix	T-83200	SRT
Culdoscopy	Uterus and falloplan tubes	T-88920	SRT
Cystoscopy	Bladder	T-74000	SRT

Procedure: Using this screen, you can specify the procedure name in the first column. Each line corresponds to a list entry.

Anatomic region: Using this screen, you can specify the anatomic region in the second column. Each line corresponds to a list entry.

Code Region: Using this screen, you can specify the code region in the third column. Each line corresponds to a list entry.

Schema Region: Using this screen, you can specify the schema region in the fourth column. Each line corresponds to a list entry.

NOTE: You can enter procedures without indicating the anatomic region.

NOTE: If you use DICOM, please use only the anatomic regions defined in the DICOM standard.

NOTE: If you use DICOM, you must complete all four columns for each entry.

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Nombre del procedimie...	Área anatómica	Código de área	Esquema de área
Arthroscopy	Joint	T-15001	SRT
Arthroscopy of knee	Knee	T-D9200	SRT
Arthroscopy of shoulder	Shoulder	T-D2220	SRT
Bile duct endoscopy	Bile duct	T-60610	SRT
Bronchoscopy	Bronchus	T-26000	SRT
Colonoscopy	Large intestine	T-59000	SRT
Colposcopy	Cervix	T-83200	SRT
Culdoscopy	Uterus and falloplan tubes	T-88920	SRT
Cystoscopy	Bladder	T-74000	SRT

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
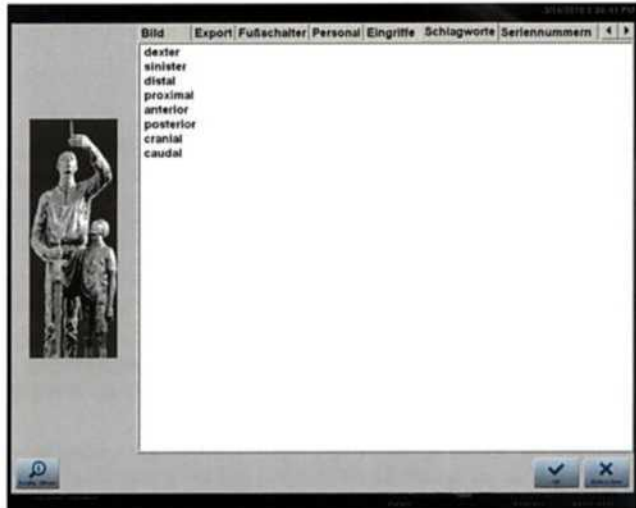
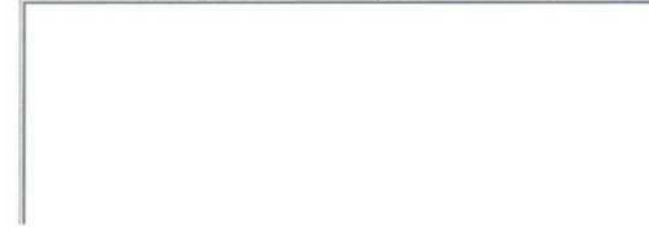
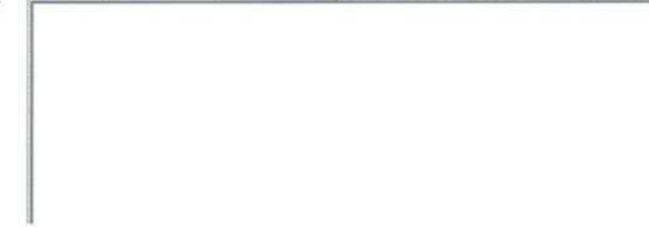
Keywords: Using this screen, you can create a list of keywords. The white area is a text editor which you can use to create a list of keywords. Each line corresponds to a list entry.

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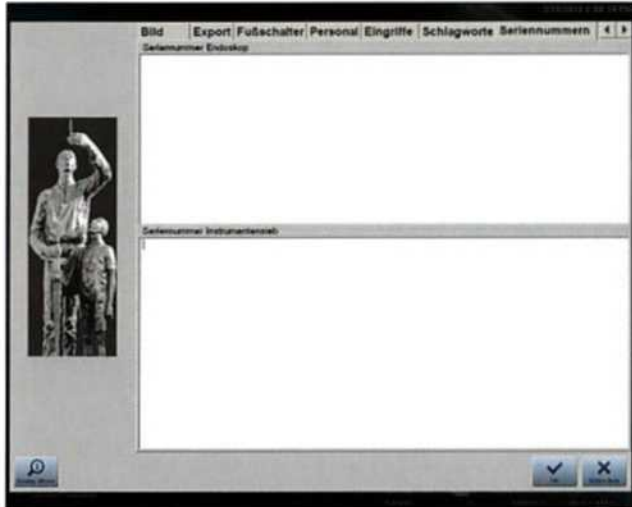
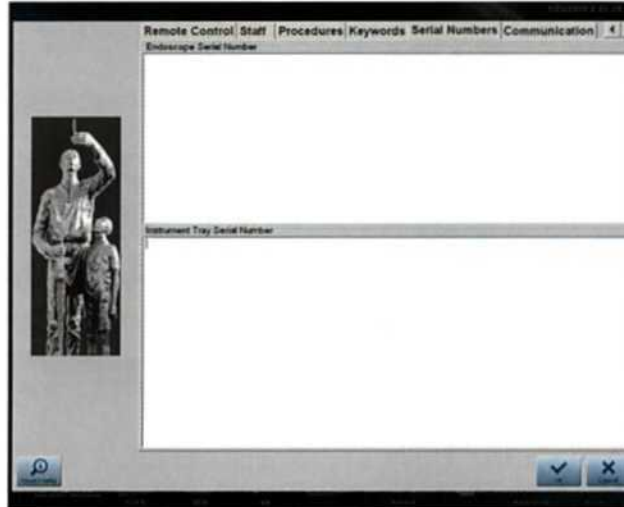
8.9 Serial numbers

The "Serial Numbers" tab is used for:

- Creating a list of endoscope serial numbers
- Creating a list of instrument tray serial numbers

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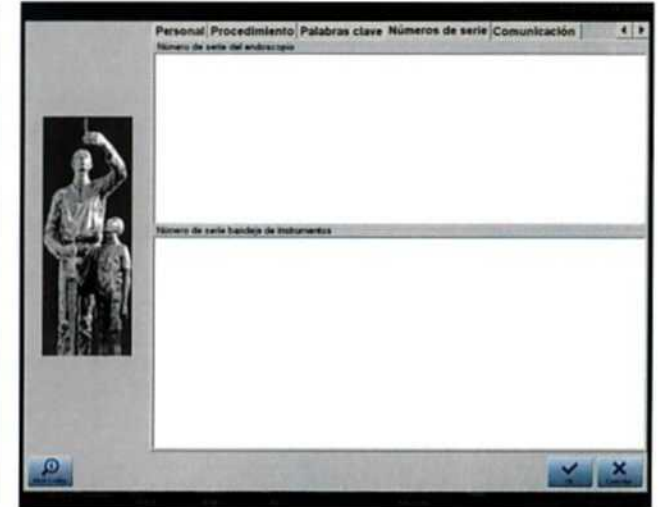8.9.1 Creating a list of endoscope serial numbers

Endoscope Serial Number
SER12345IAL

Endoscope Serial Number: Using this screen, you can create a list of endoscope serial numbers. The white area is a text editor which you can use to create a list of endoscope serial numbers. Each line corresponds to a list entry.

NOTE: When performing procedures, you can then select the serial number of the endoscope used.

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8.9.2 Liste Seriennummer Instrumentensieb anlegen

Seriennummer Instrumentensieb
Tray00001
Tray00002
Tray00031
Tray00104

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8.9.2 Creating a list of instrument tray serial numbers

Instrument Tray Serial Number
Tray00001
Tray00002
Tray00031
Tray00104

Instrument tray serial number: Using this screen, you can create a list of instrument tray serial numbers. The white area is a text editor which you can use to create a list of instrument tray serial numbers. Each line corresponds to a list entry.

NOTE: When performing procedures, you can then select the serial number of the instrument tray used.

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9. Operating the AIDA compact NEO software

The text below gives you step-by-step instructions on operating the AIDA compact NEO software.

NOTE: Correct configuration ensures trouble-free operation and maximum benefit of the software. Please also see the "Settings" chapter of this instruction manual and the Installation Instructions 96206171D.

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The "Patient" section is organized into 3 major areas:

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1	Navigation bar: Clicking on "Patient & Procedure Data", "Extended Patient & Procedure Data", and "Instrument Data" allows you to switch between the three associated input masks.
2	Input mask
3	On-screen keyboard

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NOTE: You can enter data using the on-screen keyboard or an external connected keyboard. Or you can select predefined entries using a selection field.

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In the "Patient" section, you can enter all important data on a patient. In particular, this includes the following data:

- Entering Patient and Procedure Data
- Entering Extended Patient and Procedure Data
- Entering Instrument Data

You can access the "Patient" section by clicking on the "Patient" button in the workflow bar or by clicking on the field [Select Patient] on top of the screen.

NOTE: You can enter data using the on-screen keyboard or an external connected keyboard. In some cases, you can select predefined entries using a selection field.

9.1.1 On-screen keyboard

The on-screen keyboard is set up like a standard PC keyboard and is used in the same way.

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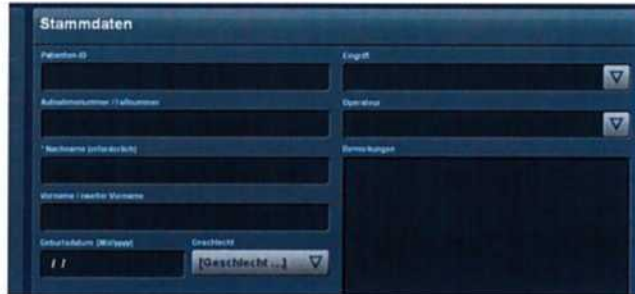
9.1.2 Entering patient and procedure data

Here, you can enter patient and procedure data. The patient and procedure data include the following information:

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Patient ID: Enter the patient ID into the entry field.

NOTE: The patient ID must be unique if it was selected as the primary ID (see Selecting the primary ID, p. 26).

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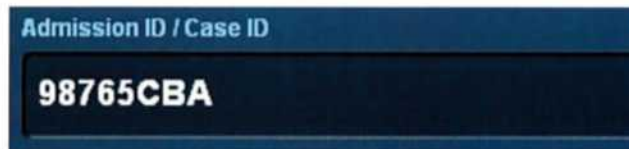
Admission ID / Case ID: Enter the Admission ID / Case ID: into the entry field.

NOTE: The Admission ID / Case ID must be unique if it was selected as the primary ID (see Selecting the primary ID, p. 26).

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* Last Name (required)

Smith

Last Name (required): Enter the patient's last name in the entry field.

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First Name / Middle Name

John

First Name / Middle Name: Enter the patient's first name in the entry field.

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12/31/1951

Date of Birth: Enter the patient's date of birth into the entry field.

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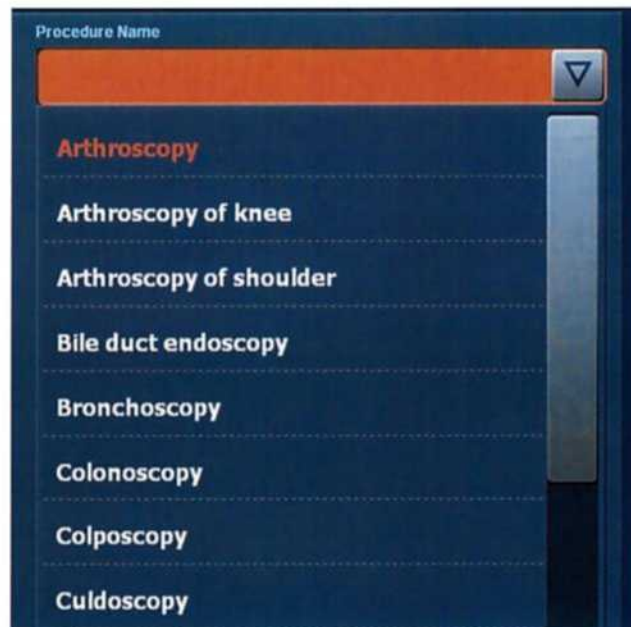
Select Sex: Clicking on the "Select Sex" selection field opens the "Select Sex" list. Click on the appropriate list entry.

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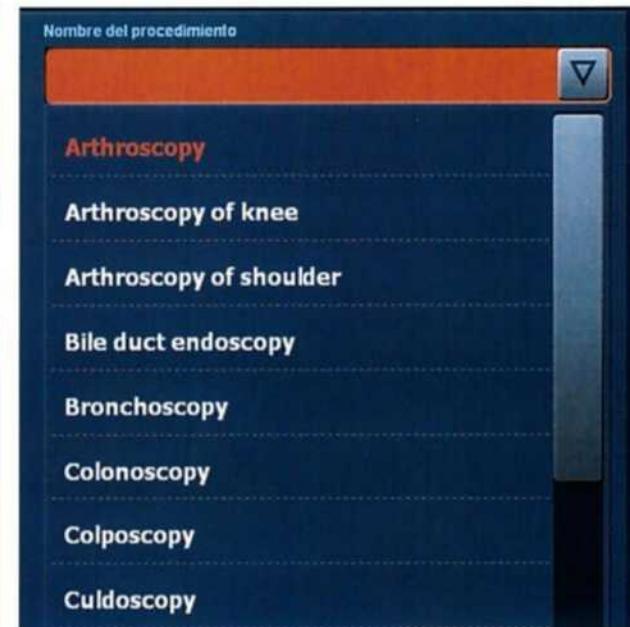
Procedure: Clicking on the "Procedure Name" selection field opens a list from which you can select the type of procedure.

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Performing Physician (Surgeon)

Dr. Dupont ▼

Performing Physician: Clicking on the "Performing Physician" selection field opens a list from which you can select the respective surgeon.

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Comments

Comments: You can enter additional comments here.

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
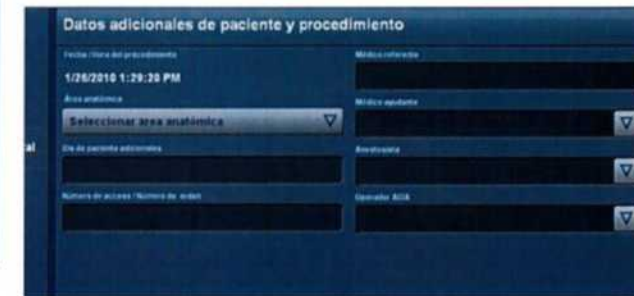
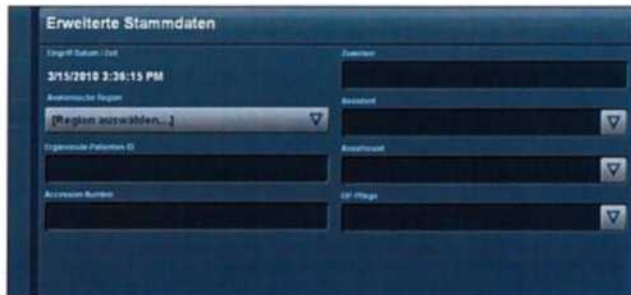
Here, you can enter extended patient and procedure data. The extended patient and procedure data include the following information:

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
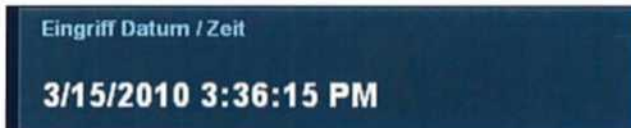
Procedure Date/Time: The date is copied from the information in "Patient and Procedure Data". The time is supplied by the system.

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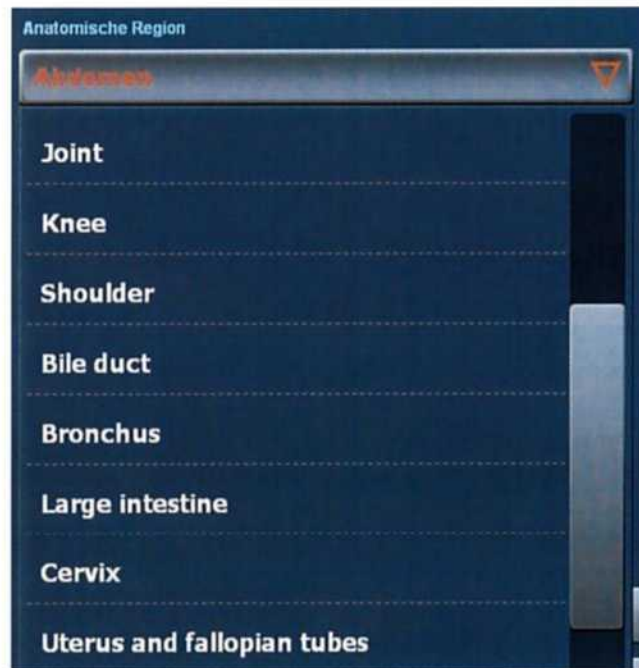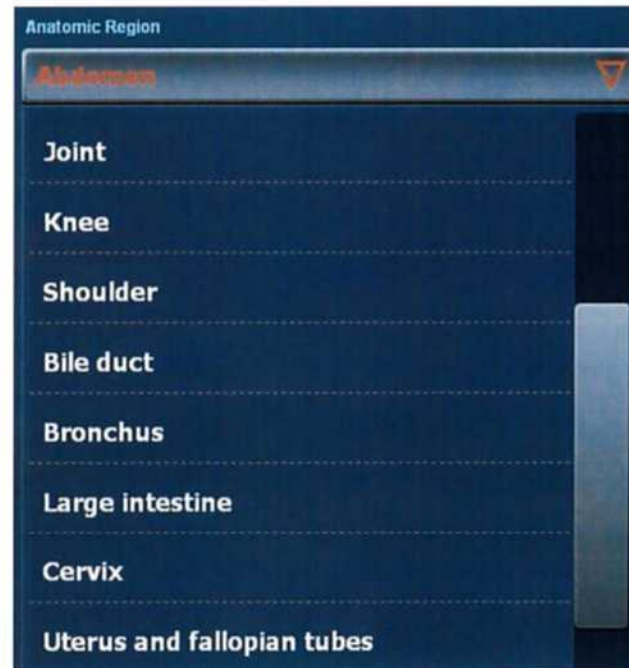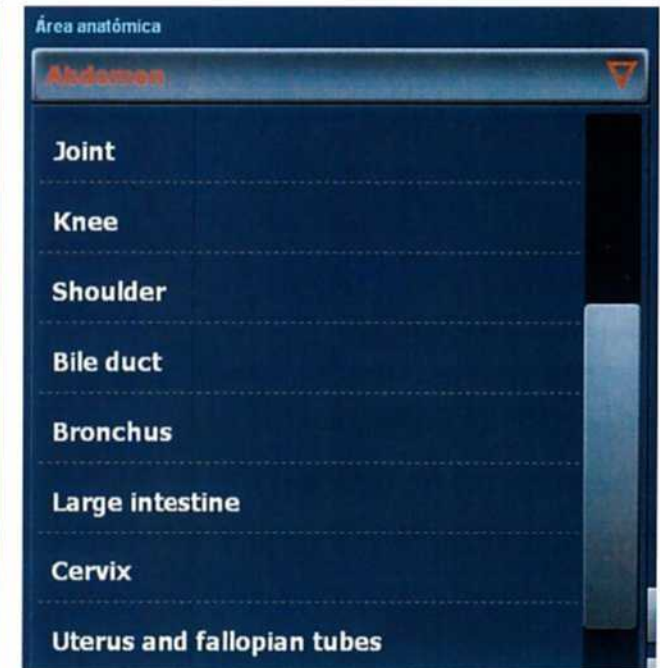
Anatomic Region: Clicking on the "Anatomic Region" selection field opens a list from which you can select the respective anatomic region.

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Other Patient IDs: Enter other patient IDs into the entry field.

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Accession Number / Order Number

ACC12345

Accession Number: Enter the accession number into the entry field.

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Referring Physician

Dr. Kowalski

Referring Physician: Enter the referring physician into the entry field.

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Dr. Kowalski

Dr. Iwanow

Assistant Physician: Clicking on the "Assistant Physician" selection field opens a list from which you can select the respective assistant.

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Anesthesiologist

Dr. Bloggs

Anesthesiologist: Clicking on the "Anesthesiologist" selection field opens a list from which you can select the respective anesthesiologist.

Operator

Jane Doe

Operator: Clicking on the "Operator" selection field opens a list from which you can select the respective OR nurses.

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
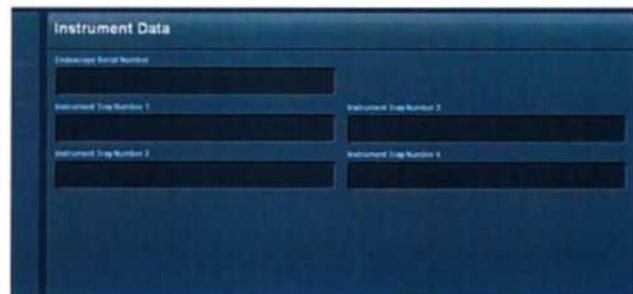
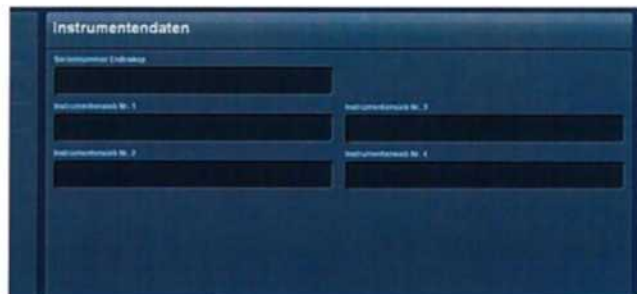
You can enter instrument data here. The instrument data include the following information:

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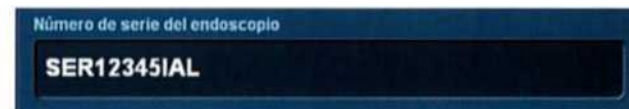
Endoscope Serial Number: Enter the endoscope's serial number in the entry field.

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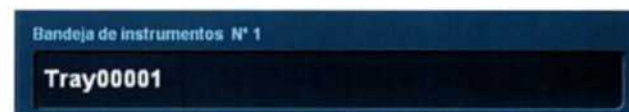Instrument Tray Number 1: Enter the number of the endoscope's instrument tray no.1.

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Instrument Tray Number 2: Enter the number of the endoscope's instrument tray no.2.

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Instrument Tray Number 3

Tray00031

Instrument Tray Number 3: Enter the number of the endoscope's instrument tray no.3.

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Instrument Tray Number 4

Tray00104

Instrument Tray Number 4: Enter the number of the endoscope's instrument tray no.4.

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9.2 "Capture" section

In the workflow bar, click the "Capture" section to get to the "Capture" section. You also reach it when the AIDA compact NEO system is rebooted, since the "Capture" section is the first screen shown after boot up.

The "Capture" section is organized into 3 major areas:

1	Preview area
2	Tools for media editing and media creation
3	Inputs/sources for display in the preview section
4	Preview area: File storage (pictures, audio, video)

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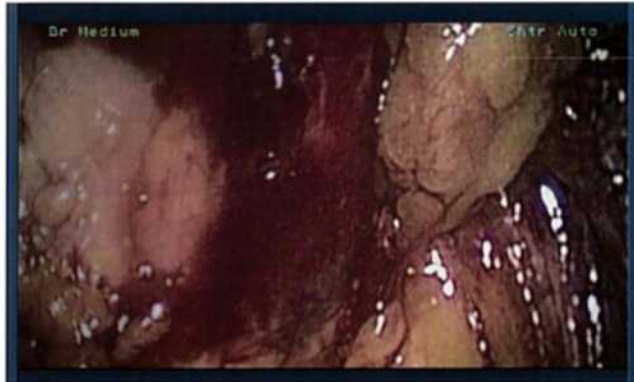
9.2.1 Preview area

This area shows the pictures and videos of the selected input.

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9.2.2 Tools for media editing and media creation

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Image: Clicking on the "Image" button takes a snapshot of the current live video that is shown in the preview area. This picture is stored as a file and is shown as an image icon in the preview area (file storage).

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Video: Clicking on the "Video" button creates a video recording of the current live video that is shown in the preview area. This video is stored as a file and is shown as a video icon in the preview area (file storage).

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Audio: Clicking on the "Audio" button records an audio sequence from the current live video that is shown in the preview area. This audio sequence is stored as a file and is shown as an audio sequence icon in the preview area (file storage).

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Freeze: Clicking on the "Freeze" button freezes the current live video that is shown in the preview area as a so-called freeze frame.

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Double-clicking on the Image icon the preview area (file storage) opens the following screen:

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There, you have the following options:

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- Editing image
- Deleting image
- Renaming image

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➤ **Editing image**

You can make the following settings through +/- controls:

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Operating the software

Contrast: The "Contrast" +/- control changes the contrast of the image.

☛ You can see the results in the preview.

Brightness: The "Brightness" +/- control changes the brightness of the image.

☛ You can see the results in the preview.

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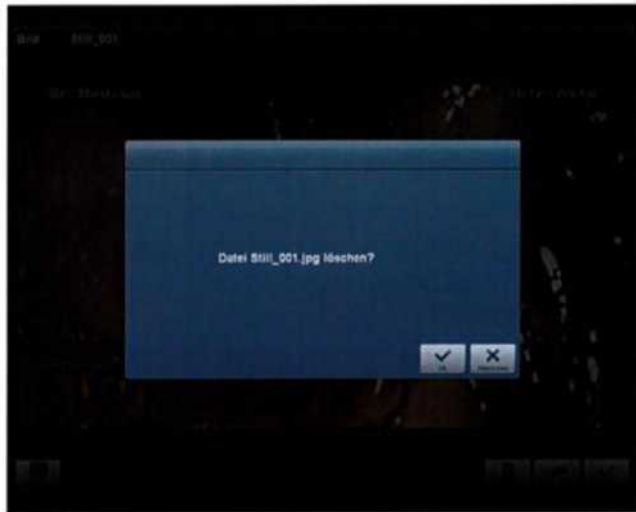
➤ Deleting image

Clicking on the "Delete" button brings up a confirmation dialog. Confirm that you wish to delete by pressing "OK" or cancel by pressing "Cancel."

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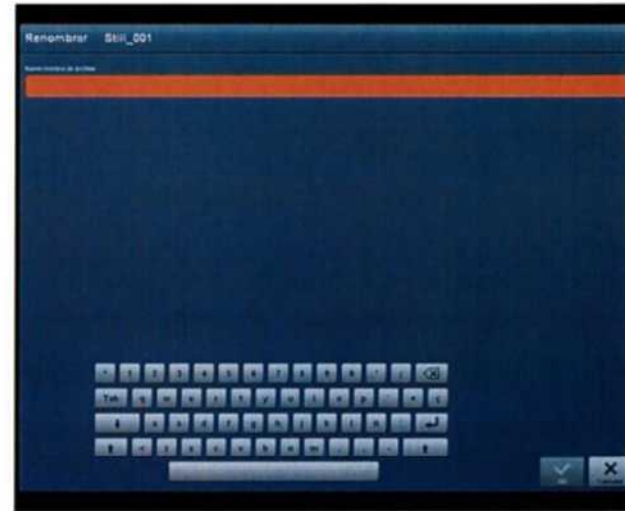
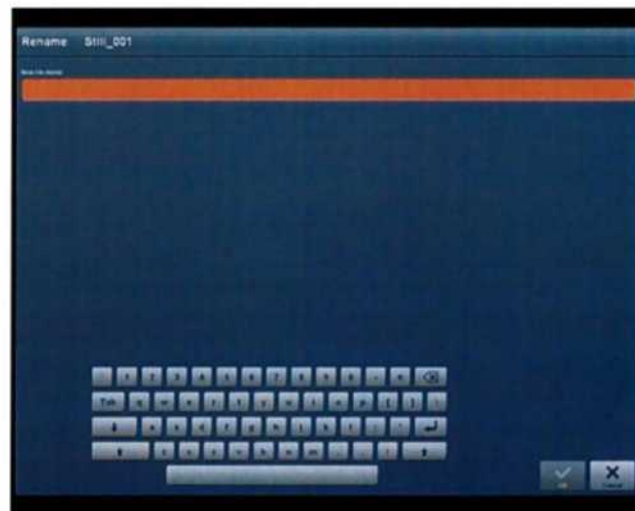
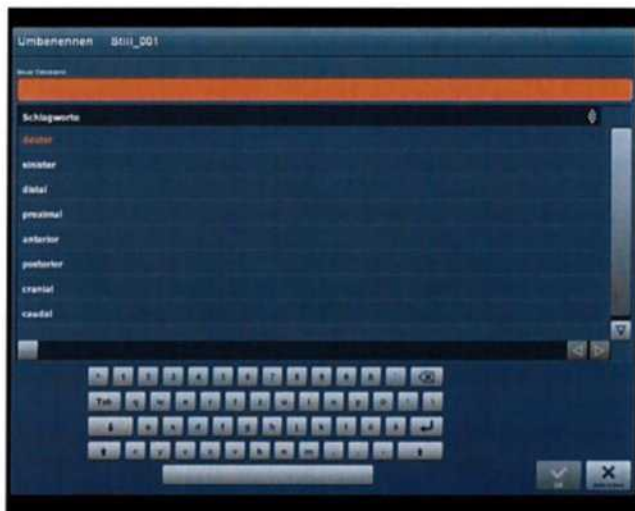
Clicking on the "Rename" button opens screen above. Enter a new name in the orange text field.

NOTE: Alternatively, you can select a name from the keywords list.

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Inputs: This area allows you to switch between different inputs (max. 3). The active input is shown in orange.

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9.3 "Reference" section

To reach the "Reference" section, click the "Reference" button in the workflow bar.

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
The "Reference" section is organized into 2 major areas:

1	Preview area
2	Tools

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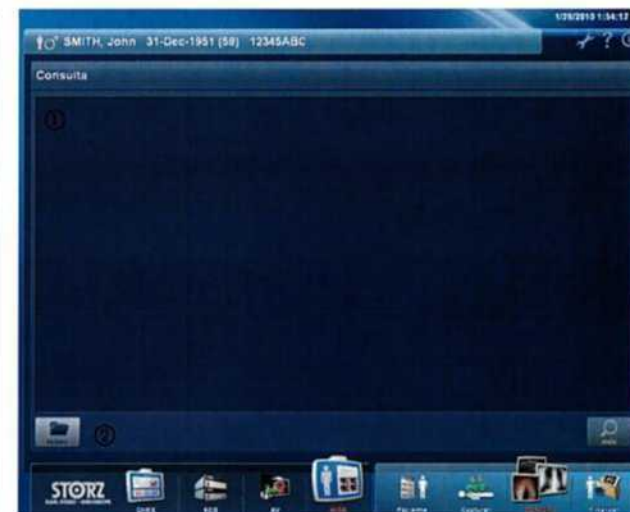In the "Reference" section, you have the following options:

- Opening reference file
- Viewing reference file

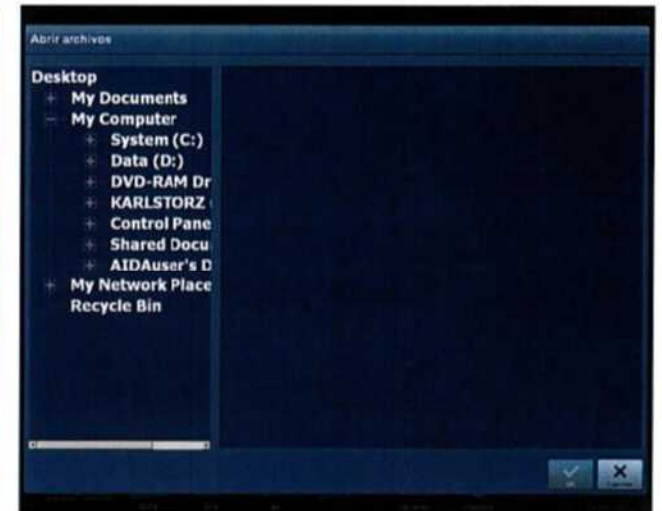
Clicking on the "Open" button opens a tree. You can then navigate to the desired file and open it.

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9.4 "Finish" section

Click the "Finish" button in the workflow bar to reach the "Finish" section.

The "Finish" section is organized into 2 major areas:

1	Finish procedure and save in pre-selected directory
2	Finish and select storage location

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➤ Storing on a network or on USB

Clicking on the "Save in" button opens the screen below.

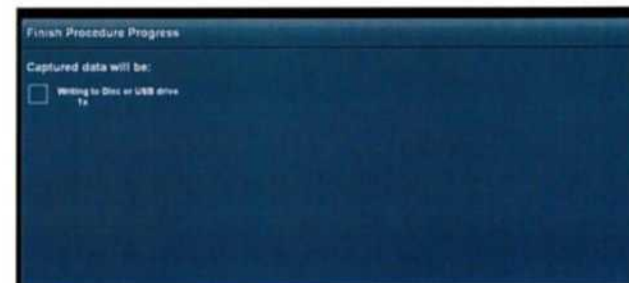
You can create a new folder by clicking on the "New Folder" button. The tree allows you to navigate to the place where you want to create the new folder.

There, you can navigate to the desired storage location.

Once you have selected the storage location, confirm by pressing OK.

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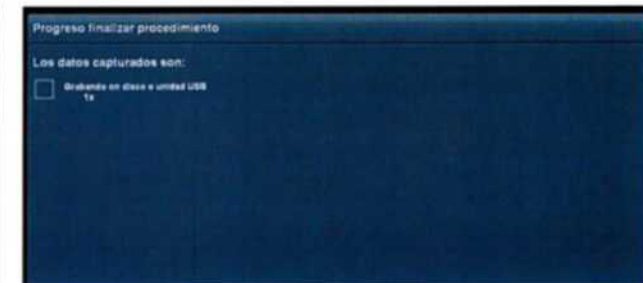
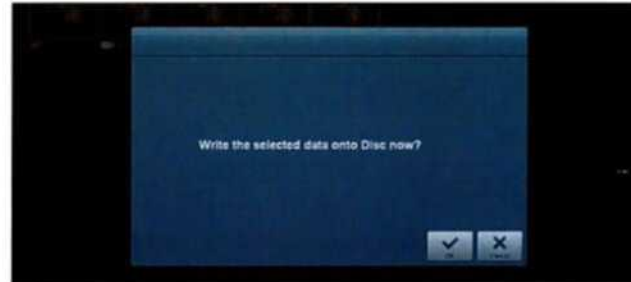
➤ **Burning disc**


Clicking on the "Disc" button opens the following screen:

Clicking on "OK" starts the burning process.

➤ **Printing the data**

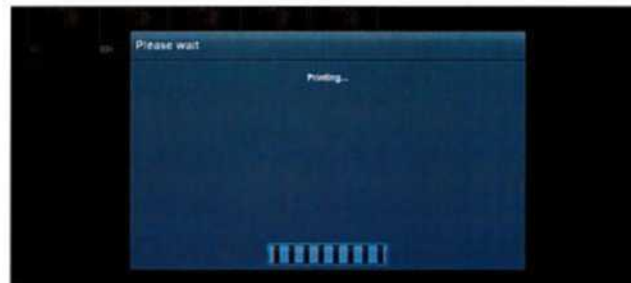Clicking on the "Print" button opens the following screen:

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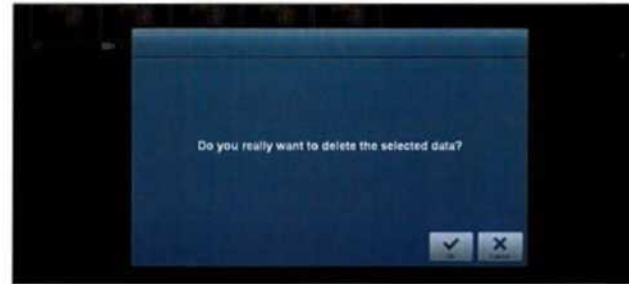
➤ Deleting the data

Clicking on the "Delete" button opens the following screen:

Clicking on the "OK" button deletes the selected data.

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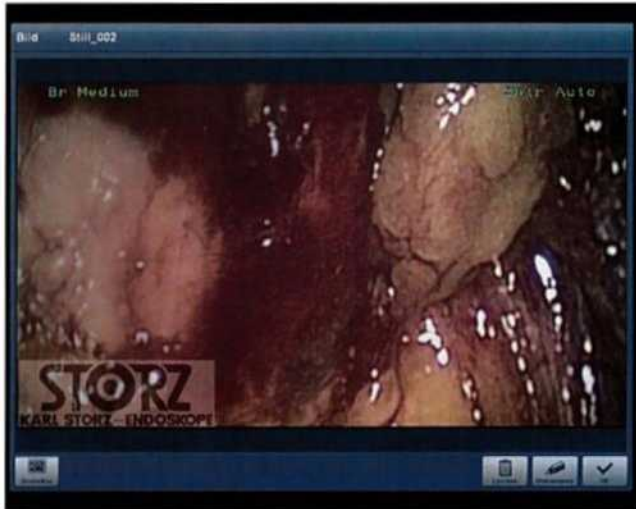
➤ **Viewing the data**

Clicking on the "View" button opens the following screen:

Here, you can view or edit the data (see "9.2.3 Image processing").

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According to Medical Device Directive (MDD):

These medical products belong to Class I
These medical products bear the CE mark in
accordance with MDD 93/42/EEC.

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According to IEC 60601-1, UL 60601-1, CAN/CSA C22.2 No. 601.1:

Type of protection against electric shocks:
Protection Class I

Nach IEC 60601-1-2:

Please read the Electromagnetic Compatibility
Information in the appendix.

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11. Technical documentation (AIDA control NEO)

On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified user personnel duly authorized by the manufacturer for their use in repairing those components of the unit that have been designated by their respective manufacturers as repairable. Supply of such technical documentation relating to the unit shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the unit. Explicitly exempted herefrom are those maintenance and repair operations described in this installation manual.

NOTE: We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified user personnel duly authorized by the manufacturer for their use in repairing those components of the unit that have been designated by their respective manufacturers as repairable. Supply of such technical documentation relating to the unit shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the unit. Explicitly exempted herefrom are those maintenance and repair operations described in this installation manual.

NOTA: We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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12 Maintenance

12.1 Cleaning, disinfection and care of the unit

CAUTION: Avoid allowing fluids to enter the housing of the AIDA compact NEO system components.

The exterior of the AIDA compact NEO system components must be cleaned with a disposable cloth moistened with disinfectant (no solutions predominantly containing alcohol). Observe all instruction manuals included with the system, e.g., for the monitors.

12.2 Maintenance and Repair

Defective components of the AIDA compact NEO system must be serviced and repaired exclusively by persons authorized by us; all repair work must employ genuine KARL STORZ parts only.

KARL STORZ recommends that all equipment be checked and inspected once a year by KARL STORZ, or by an authorized agent. All services such as modifications, repairs, calibrations, and/or readjustments may only be performed by KARL STORZ or by an authorized agent.

WARNING: Repairs may only be performed by qualified technicians trained in electrical or electronic engineering, in compliance with the relevant occupational, safety and accident prevention regulations. Always unplug the equipment before performing any repairs. Safety Testing based on VDE0751 and IEC 60601-1, whichever may apply, must be performed after servicing has been completed.

CAUTION: By making the enclosed technical information available, KARL STORZ does not authorize any service or repair by unauthorized service personnel. Tampering with the instruments or equipment, or unauthorized service or repair of the unit nullifies and voids the warranty.

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12.2.1 Repair program

Usually to bridge the repair period, you will receive a unit on loan, which you will then return to KARL STORZ as soon as you receive the repaired unit.

In Germany, you can refer the repairs directly to:

KARL STORZ GmbH & Co. KG
Dept. Repair service
Dr. Karl-Storz-Str. 34
78532 TUTTLINGEN

In other countries please contact your local KARL STORZ branch or authorized dealer.

12.2.2 Limitation of liability

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of the AIDA compact NEO system only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- all electrical installations at the location of use meet applicable national and local electrical codes, and
- the units have been used in accordance with their instruction manuals at all time.

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12.3 Manufacturer's warranty (AIDA control NEO)

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In so doing we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Please fill out the warranty card and return it as

soon as possible to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 TUTTLINGEN/GERMANY

If the equipment is opened or any repairs or modifications to the components of the AIDA control are performed by unauthorized persons, we accept no liability for its operational safety. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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13. Accessories

NOTE: Recommended accessories and spare parts are listed in the catalog.

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14. Appendix

14.1 Software: Right of Use

1. With KARL STORZ products containing software components, the customer acquires a non-exclusive, and within the terms and conditions of this agreement, limited license for the programmed logic, computer programs and/or software delivered by KARL STORZ and included in KARL STORZ products ("Software").
2. This license agreement is only applicable to the delivered form and only to the use of products sold to the customer in accordance with the accompanying KARL STORZ instruction manual. Utilization of the software with other products acquired from KARL STORZ or a third party is prohibited unless expressly permitted.
3. The customer may transfer this licence to another user provided the customer reveals the recipient's name to KARL STORZ and the recipient agrees to the terms of this software license agreement. In the case of a transfer, the customer's license is terminated.
4. KARL STORZ reserves all copyrights to the software as well as modifications, enhancements and updates. In particular, the customer does not have the right to have the software duplicated, copied, reverse engineered, decompiled or disassembled or modified in any way. The customer has no right to create derivatives of the software and may not copy, decipher or re-create the source code of the software. § 69g paragraph 2 of German copyright law will remain unaffected.
5. Any unauthorized modifications or enhancement of the software by the customer is considered to be a violation of this agreement. In the event of such modifications or enhancements, KARL STORZ shall immediately acquire all ownership and copyrights of any such modifications or enhancements. As a precautionary measure, the customer shall transfer an exclusive, gratis, and unlimited licence to use such modifications or enhancements in advance to KARL STORZ.

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6. KARL STORZ does not recommend the use of this software with other software products not supplied by KARL STORZ. KARL STORZ is not liable for any damage caused by the use of software or products not supplied or recommended by KARL STORZ.
7. In the event of a violation of the terms of this agreement by the customer, or its agents, employees or representatives, this software license shall be terminated immediately without notice of termination or further actions by KARL STORZ or any other party.

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WARRANTY

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

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To be completed by the supplier/importer:

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Tipo del aparato: Número de serie:

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Firma/Fecha:

To be filled out by instrument owner:
Return address/company stamp:

Field of application:

Type of instrument: Serial no.:

Purchase date:

Signature/Date:

Please attach
sufficient
postage

REPLY CARD

KARL STORZ GmbH & Co. KG
Postfach 230
78503 TUTTLINGEN
GERMANY