# STORZ

## KARL STORZ — ENDOSKOPE

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**Important information  
for users of KARL STORZ  
instruments**

Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, these products are the result of years of experience and great care in manufacture. You and your organization have decided in favor of modern, high-quality items of equipment from KARL STORZ.

This manual contains information for personnel authorized to prepare instruments. The instruments and accessories manufactured by KARL STORZ are, as a rule, sensitive precision instruments which require particular care during preparation. To prevent damage during preparation, support is provided here for cleaning, disinfection, care, assembly and sterilization of the KARL STORZ endoscopic instruments.

In the appendix you will find chemicals recommended for preparation and an overview of sealing caps used for the instruments.

A list of KARL STORZ approved products for preparation can be found on the Internet at „[www.karlstorz.com](http://www.karlstorz.com)“.

**Warning:** KARL STORZ instruments are provided unsterilized, and must be cleaned, disinfected and/or sterilized prior to the initial use and before each subsequent use.

KARL STORZ is constantly working on the further development of all products. Please appreciate that changes to the scope of supply in form, equipment and technology are possible for this reason. Therefore, no claims may be deduced from the information, figures and descriptions in this manual.

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**Important information for users of KARL STORZ units and instruments.....U2**

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**About this manual ..... 3**  
Handling these instructions..... 3  
Symbols and hazard information..... 3  
Overview of important points ..... 4

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**General information about cleaning, disinfection and sterilization ..... 4**

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Overview of important points ..... 4  
Brief introduction to the material..... 5  
New instruments ..... 6  
Water and steam quality..... 7  
Preparation of thermostable accessories..... 8  
Preparation methods..... 9  
Cleaning and disinfection methods..... 9  
Sterilization methods..... 9

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Disassembly and preparation for cleaning ..... 10  
Preparation during and after surgery..... 10  
Manual cleaning and disinfection..... 11  
Ultrasonic cleaning..... 14  
Machine cleaning and disinfection..... 15  
Assembly, testing and care ..... 17  
Care Products ..... 18

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Selection of appropriate sterilization methods ..... 19  
Steam sterilization ..... 20  
Validated steam sterilization methods ..... 21  
Vacuum-/Pre-vacuum ..... 22  
Fractionated fore-vacuum method ..... 22  
Gravitation method ..... 23  
Flash sterilization ..... 24  
Gas sterilization ..... 25  
Ethylene oxide (EtO) gas sterilization ..... 25  
Plasma sterilization ..... 26  
Chemical sterilization..... 28  
Storage..... 30

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**Using this manual**

This manual is not an instruction manual, but instead provides general information regarding cleaning, disinfection, maintenance and sterilization of endoscopic instrument sets from KARL STORZ. Particularly sensitive product groups, such as telescopes or fiberscopes, have their own manuals. Separate instruction manuals and service manuals are available for units. Other general regulations (Normal use, User qualification, Safety precautions when using instruments, etc.) must be heeded.

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**Symbols and hazard information**

Please read this manual and follow its instructions carefully. The words Warning, Caution and Note convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of this product. To make the words Warning and Caution stand out more clearly, they are accompanied by a pictogram.

**Warning:** A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result in injury to the patient or physician.

**Caution:** A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the instrument.

**Note:** A Note indicates special information about operating the instrument or clarifies important information.

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**Overview of important points**

- Follow instructions exactly.
- Clean brand new instruments first before sterilization.
- Prepare used instruments immediately.
- Take instruments apart before preparation, open joints.
- The manufacturer's specifications for concentration and exposure time must be followed under all circumstances during manual cleaning.
- For machine decontamination, only use approved methods for endoscopic instrument sets and telescopes.
- Use accessories which are suitable for cleaning.
- Do not overload ultrasonic installations and washers to prevent washing and acoustic shadows.
- Never use metal brushes or metal scouring pads for manual cleaning.
- Use completely demineralized water for rinsing after cleaning.
- Remove the instruments as soon as the RDG program has finished (Risk of corrosion!)
- Test the complete instrument for proper operation, removing damaged instruments.
- For every preparation, oil all jointed instruments with the instrument oil (Cat. no. 27656).
- Sterilization is no substitute for cleanliness! Correct sterilization can only be performed on cleaned, disinfected instruments!
- Sterilization must proceed according to recognized and validated methods.
- The use of methods and cleaning agents not listed in this manual can cause damage to the instrument. Such damages are not covered by the conditions of the manufacturer's warranty.
- The application of methods and cleaning agents not listed in this manual may lead to damage on the instrument. Such damages are not included in the warranty.

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**Brief introduction to the material**

High elasticity, toughness, rigidity, good cutting behavior and, at the same time, high wear resistance as well as the best possible corrosion resistance are frequently the requirements of instruments for endoscopic surgery. These are met by the use of metallic materials and special material combinations. Additional material groups, such as plastics for example, are used for endoscopic instruments and telescopes. These material combinations require careful preparation. This is especially true in regard to the selection of applied methods, chemicals and water quality.

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**Steel**

Stainless steel is corrosion resistant due to a natural passive layer which forms in the air. If this passive layer is destroyed by certain substances (solid, liquid or gaseous), there is no longer any corrosion resistance. The stainless steels may also corrode, particularly if they are in contact with blood or tissue residues. Therefore make certain, not only for hygienic reasons, that organic remnants have been completely removed.

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**Nonferrous metals and light alloys**

Surface-treated nonferrous alloys, such as nickel and chrome plated brass and light alloys (anodized aluminum) are also used. At first glance, the coated brass parts cannot be differentiated from stainless steel. Aluminum alloys are not always recognized as such, and may easily be mistaken for plastics. They are especially sensitive to alkaline and acidic cleansers during machine preparation.

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**Titanium**

Titanium alloys are among the highest-quality materials. They have high mechanical characteristics with light weight and extremely good corrosion resistance. For this reason, they present no problems during preparation.

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**Plastics**

All plastics processed by KARL STORZ for reusable instruments are steam sterilizable at 134 °C. Only high quality plastics are able to follow this criterion, because a normal plastic is primarily damaged by water intake in steam.

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**New instruments**

**Warning:** Instruments are delivered unsterilized and must therefore be cleaned, disinfected and, if necessary, sterilized before using for the first time.

Brand new instruments should be unpacked after delivery. However, check all instruments for transport damages and report these immediately. Included with every shipment is a description of the procedures which may be chosen in case of transport damages.

Every instrument passes through an extensive cleaning and final check at the end of the manufacturing process. However, soiling is possible during transportation and storage. Therefore the instruments must go through the complete processing procedure before their first use.

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(mg/l entspricht ppm)

### Water and steam quality

In producing cleaning and disinfectant solutions, as well as for rinsing, water of at least drinking water quality must be used. However, we point out that even water of drinking water quality may have too high a concentration of substances with a damaging effect on the instruments. Corrosion, spotting and discolorations may then occur. It is therefore strongly recommended that demineralized water be used. Only then can a long service life for the instrument be guaranteed.

The steam for sterilization must also satisfy the requirements of the standard EN 285 in order to prevent damage and discolorations.

Please determine the water quality available in your building with the maintenance department, since resulting damage may often only be recognized years later and then no longer be repaired. KARL STORZ will gladly help with any questions in this regard.

### Water quality required according to EN 285 (Maximum limits)

Total water hardness	5 °d (0.9 mmol/l)
Chlorides	100 mg/l
Silicic acid	15 mg/l
Iron	0.05 mg/l
Manganese	0.05 mg/l
Copper	0.05 mg/l

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**Cleaning, disinfection and sterilization of thermostable accessories**

**Warning:** The listed cleaning, disinfection and sterilization methods alone are not sufficient to guarantee the required level of sterility for medical devices. This can only be achieved if the recognized and validated preparation methods are followed.

Instrument sets and accessories for endoscopic surgery are, as a rule, sensitive precision instruments which require particular care during preparation. Only trained personnel is qualified to perform the disassembly, cleaning, disinfection, assembly and sterilization in such a way that perfect functioning of the instruments is maintained.

Some instruments, such as those used in minimally invasive surgery (MIS), which are exposed to gas overpressure intraoperatively, are more heavily soiled. For this reason, these instruments have an irrigation channel or were designed as take-apart.

The recommended cleaning instrument sets and sterilization containers may be found in the catalog in the chapter »Recommended accessories for cleaning, disinfection, care and sterilization«.

**Warning:** When cleaning contaminated instruments, the guidelines of the Employer's Liability Insurance Association and equivalent organizations must always be observed. Aerosols should be removed using a suction device.

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### Preparation methods

Listed here are the standard methods for cleaning, disinfection and sterilization. As far as possible, there should be machine decontamination followed by steam sterilization at 134 °C. These steps are only possible using thermostable instrument sets. There are numerous variations on the methods, which is why you must be certain that they have been tested (validated) for their effectiveness.

### Cleaning and disinfection methods

- Manually using chemical cleaning and disinfection solutions.
- Ultrasonic cleaning.
- By machine, using an automatic cleaning ..... and disinfection unit.

### Sterilization methods

- Steam sterilization.
- Gas sterilization (formaldehyde, ethylene oxide).
- Plasma sterilization (e.g. STERRAD®).
- Chemical sterilization (e.g. STERIS®).

**Caution:** Please consult the water and steam quality section in order to prevent damage to the instrument set.

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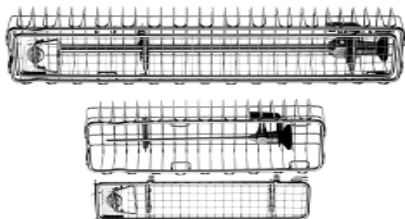
**Disassembly and preparation for  
cleaning**

Preparation should be begun as soon as possible, as incrustations can cause irreparable damage and make cleaning more difficult.

To prevent damage during transport and stor-

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Pointed instruments must be covered with a protective cap.

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**Preparation during and after  
surgery**

Already in the operating room, the following preparatory measures should be taken:

- Rinse out hoses and instruments with irrigation channels using sterile rinsing solution when there is heavy soiling.
- Remove heavy soiling immediately. If necessary, use a disposable cloth soaked in sterile rinsing solution.
- Coagulation instruments should be wiped off with a moist compress on the coagulation surfaces between uses during surgery. Incrustations may be dissolved by briefly immersing in a 3% H<sub>2</sub>O<sub>2</sub> solution (hydrogen peroxide solution). Before reusing, rinse off with sterile water and wipe off again.
- Instruments should be dismantled directly after surgery and rinsed out with water. If this is not possible, the instruments should be transported assembled to the cleaning location (dry disposal).

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*Prompt preparation is essential to prevent contaminants from coagulating or drying out on the instruments.*

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### Manual cleaning and disinfection

**Caution:** Total immersion in cleaning and disinfection solution is not possible for the following instrument groups:

- Telescopes and lenses with drive
- IMPERATOR drill handpieces
- Prismatic light deflectors
- Consult the respective instructions for use of motors and their handpieces.

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For cleaning and disinfecting, the products especially recommended for endoscopes and approved by KARL STORZ are suitable (see Appendix).

**Note:** When agents other than those named are used, KARL STORZ assumes no liability for any damage which may occur.

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**Caution:** Instruments must never be placed in physiological saline solution, contact leads to pitting and corrosion.

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**Caution:** Only EPA registered hard surface disinfectant solutions capable of high level of disinfection should be used. Solutions with a strongly acidic or alkaline pH are not recommended. Follow solution manufacturer instructions, paying close attention to proper concentration and exposure time. Prolonged exposure and incorrect concentration may result in damage. Observe the microbiological range of action of the chemicals used.

The efficiency of this stage of the preparation cannot be guaranteed in cases of contamination or when the solution used has been left for too long. Cleaning and/or disinfectant solutions should thus always be used fresh and in

When immersing instruments in cleaning and disinfectant solutions, make certain that air can escape from the instrument channels so that all surfaces are completely wetted. To do this, it is best to immerse the instruments at an angle until no more air bubbles can be seen. Always make certain during cleaning that the procedure particularly also includes all instrument channels (sheath lumen, stopcocks, valves, etc.).

Use items from the chapter »Recommended accessories for cleaning, disinfection, care and sterilization« or from the catalog chapter »Cleaning and care of the instrument«.

**Caution:** Improper cleaning tools (e.g. overlarge brushes, metal brushes) can cause damage. Use no scouring agents or steel wool.

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For preparation, multi-part instruments must be disassembled, seals and protective caps removed. Surfaces, inner cavities and jaws must be thoroughly cleaned and rinsed with a suitable brush, sponge or cleaning gun. Clean small parts carefully.

Using the cleaning gun and accessories (Art. no. 27660), instrument channels may be rinsed out especially well.

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On LUER connections, set the stopcock to open (stopcock in direction of flow) and rinse out. Disassemble the stopcock only after rinsing out. Make certain that rinsing continues until clear water exits the front end. Make certain the instrument channels are patent.

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After cleaning, rinse thoroughly with water. We recommend using demineralized water.

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When finished with cleaning work, disinfect the cleaning instruments, such as the brushes, and free them from residues. Do not reuse damaged brushes.

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**Manual disinfection**

Immerse all parts in a disinfectant solution. The chemical manufacturer's specification on the concentration and exposure time must be observed.

Instruments which cannot be dismantled should be attached to an irrigation port and rinsed through with sufficient disinfectant solution. Ensure that there is sufficient flow to the distal end!

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Then rinse through several times with microbiologically clean/sterile water to remove all traces of chemical residue.

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**Drying**

Dry the surfaces and all openings, channels and lumina thoroughly, preferably with sterile compressed air. The cleaning pistol with accessories is particularly suitable for this purpose (Art. no. 27660).

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**Ultrasonic cleaning**

The ultrasonic bath (at 35 kHz) is suited for an especially thorough, mild cleaning of heavily soiled and sensitive parts. The cleaning effect of the ultrasonics also reaches virtually inaccessible instrument channels. For this reason, all not dismountable scissors and forceps with and without irrigation channels, rail-sheath instruments, aspirators, circular punches, coagulation instruments and micro-instruments must be cleaned using ultrasonics. The ultrasonics have no cleaning effect on soft plastic parts since the waves are damped.

The ultrasound bath is suitable for both the manual and machine preparation of medical devices. The duration of manual ultrasonic use should only be as long as dirt is removed, though do not exceed five minutes generally. Prolonged ultrasonic application increases the temperature of the bath and reduces the cleaning effect.

Small parts, such as screws and nuts, which are not tightly fastened, may come lose due to the ultrasonic effect. Please check for completeness.

If a cleaning agent is added to ultrasonic baths (use no chlorine-containing agents), especially thorough rinsing is required afterward.

**Caution:** Ultrasonic cleaning is not permitted for instruments containing optical components (telescopes, light carriers)!

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**Machine preparation**

Machine preparation is more intensive on the instrument than manual cleaning. The advantage of machine cleaning/disinfection lies in the reduction of risks for the personnel and the standardization of the preparation procedure.

Selection of rinsing programs must be decided with the washer manufacturer. Usually, special programs for endoscopic instrument sets (or even for telescopes) are available which are better suited for cleaning and material preservation than general programs.

For machine decontamination, make certain that

- rigid telescopes are not heated above 93°C.
- flexible and semiflexible telescopes as well as some miniature endoscopes are not heated above 65°C. This also applies for gas sterilization, transport and storage.

Further details can be found in the Telescopes manual.

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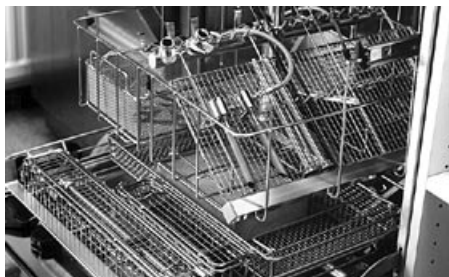
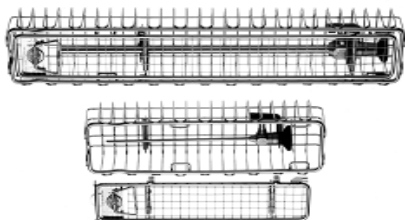
We recommend the use of demineralized water for all program steps. Also pay close attention to the choice of chemicals and their dosage. The pH values should preferably be in the neutral range. Do not use protective agents in the machine because deposits can form on the instruments, or plastics may be damaged by the protective agents. Specific manual care and lubrication of joints etc. is milder.

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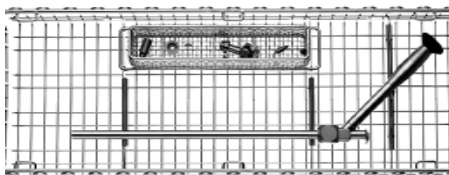
Due to the partially strong motion of the water, it is necessary that all parts be fixated in holders which prevent damage. For sufficient rinsing out of the instruments, it is necessary that instrument channels are correctly connected.

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For telescopes, the baskets type 39501 are best suited (see appendix). The washer manufacturers offer other holders for every type of instrument set. For the small parts (stopcocks, seals, etc.), small sieve holders are available.

*Fig: The first three figures in the left column provide an example of how loading should proceed. The last image shows how it should not be done.*

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For instruments with an LUER irrigation connection, good rinsing of the jaws is also provided through inside irrigation of the tube. Jointed instruments must be opened in such a way that good rinsing is achieved.

**Note:** The temperature of the inflowing water should not be above 45 °C, since higher temperatures cause coagulation of protein and therefore make problems for cleaning.

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**Assembly, testing and care of the instrument**

Before assembly, check all parts and surfaces for damage, complete cleanliness and dryness. If residue is still present after cleaning, prepare the parts again. Damaged or corroded instruments must be scrapped, or repaired/replaced (for spare part numbers, see catalog chapter 'Spare parts') or send them for repair ('Repair service').

If you require a test of the warranty claims, we ask that you send the affected instrument with exact information on the error to our repair service department (see section 'Repair service').

**Caution:** Even damages which are barely visible can impair perfect functioning (e.g. operating property), when it concerns high precision manufactured parts.

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After checking over the individual parts, multi-part instruments must be assembled. Before the final functional test, the instruments must be cooled down and oiled.

Deposits on metallic surfaces may be removed, if required, using the polishing wool (Art. no. 27658). The part must then go through the entire cleaning cycle again.

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Check plastic parts for discoloration, porosity and flexibility. If the parts show considerable deviations from their new condition, they must be exchanged. This particularly applies to seals. Ensure the patency of channels.

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Check jaws and joints especially carefully, as well as the insulation on instruments for HF application. The jaws on scissors should allow smooth cutting and forceps should have non-slip edges, aligned properly with one another, and which open and close completely.

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**Care products**

For preparation, use the items from the catalog chapter "Cleaning and care of the instrument" and the chapter "Recommended accessories for cleaning, care and sterilization" in this manual.

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Thinly coat sealing and contact surfaces, e.g. on stopcocks or plungers with special grease (Art. no. 27657). This thin layer of grease not only serves to lubricate the metal-to-metal surfaces in contact with one another, but also the seal on the stopcocks. Oil is not suitable as a sealant.

**⚠️ Warning:** Use only special silicone or oil-based lubricants, since sterilants cannot penetrate through every silicone or oil.

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For sterilization, the stopcocks must remain separated from the housing. Insert into the housing after sterilization.

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All other movable parts, such as joints on scissors and forceps, must be lubricated specifically with special instrument oil (Art. no 27656). Use only this oil since it is pharmacologically tested and its sterility during steam sterilization is guaranteed. When oiling, hold the jaws in such a way that oil may also flow into the sheath and coat the wire pull. Oil should not touch any other parts. Otherwise plastics could be damaged, though this only occurs in very exceptional cases.

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**Sterilization  
Selection of appropriate sterilization  
methods**

For almost all instruments, steam sterilization in the fractionated vacuum method at 134 °C, gas sterilization (formaldehyde, ethylene oxide), plasma sterilization and chemical sterilization (STERIS) are possible. Steam sterilization at 134 °C is usually the mildest and most inexpensive sterilization method. This also applies to rigid telescopes since the longer required time can have a negative effect on steam sterilization at 121 °C. Sterilization is always only possible on clean surfaces.

Steam sterilization is not permissible for the following instrument categories. Only methods with a maximum temperature of 65 °C, such as gas and plasma sterilization, may be used.

- Non-autoclavable telescopes
- Telescopes with eyepiece drive mechanism
- Fiberscopes/Semiflexible telescopes/Miniature Endoscopes
- Video cameras
- Prismatic light deflectors

Almost all instruments may be sterilized in assembled condition. Tighten screws and connections only slightly, so that they may later be easily loosened.

**Warning:** Before any disinfection/sterilization the instruments must be carefully cleaned and any organic material, blood and cleaning solution must be removed. The sterility of the surfaces can only be guaranteed on cleaned and disinfected instruments!

**Warning:** Remember that sterilization is only successful if it proceeds according to recognized and validated methods.

**Warning:** Any deviation from recommended disinfection methods must be validated by the user.

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 **Warning:** The recommended sterilization method is only valid with sterilization equipment that is properly maintained and calibrated.

 **Caution:** Make certain all parts are securely positioned during sterilization. To do this, articles from the chapter "Spare parts, recommended accessories" or the catalog chapters "Cleaning and care" may be useful. Packaging should conform to EN 868.

 **Caution:** Please consult the Water and steam quality section in order to prevent damage to the instrument set.

 **Caution:** Hot-air sterilization is not permissible. Since varying metals and plastics are used for almost all instruments, damage could occur e.g. due to differing expansion of the materials used.

**Note:** If deactivation of Creutzfeldt-Jakob-disease (CJD) is necessary, the sterilization method recommended by WHO (WHO/CDS/PH/92.104) may be used (18 minutes at 134 °C). Extension of the sterilization time up to one hour at 134 °C is possible in special cases.

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### Steam sterilization

For steam sterilization (with saturated steam at 134 °C), the methods in accordance with EN 285 or DIN 58946 Part 1, Number 3.25.2 should be used. For instruments with a lumen, fore-vacuum procedures must be used. Preferable are fractionated pre-vacuum methods. The steam must fulfil the requirements in EN 285, Appendix B, to prevent discolorations and spots. Cloths used for packing the instruments should contain no detergent residue since this also causes discolorations.

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Clean, disinfect and dry the instruments. Sealing and contact surfaces, such as on stopcocks or plungers, must be thinly coated with special grease (Art. no. 27657) before sterilization. These parts must remain disassembled for sterilization and only inserted in the housing after sterilization.

Through heating and cooling during sterilization, tensions could occur in instruments with a closed catch lock which can cause a reduction in clamping force or cracks in the joints. For this reason (and for reasons of hygienic safety), such instruments may only be sterilized while opened. After sterilization, they must be cooled uniformly at room temperature.

These methods are also permitted for telescopes labeled with "AUTOCLAV". For the sterilization of telescopes, please refer to the Telescopes manual.

**Warning:** Long lumens, splits and channels as well as assembled instruments can only be sterilized without restriction using the fractionated fore-vacuum method.

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### Validated steam sterilization methods

Fractionated vacuum methods in accordance with EN 285, as well as methods in accordance with DIN 58946 Part 1, Clauses 3.25.1.1 and 3.25.2.1 have been validated.

To achieve the desired sterility assurance level (SAL) of  $10^6$ , KARL STORZ recommends the named steam, gas and plasma methods.

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**Vacuum-/Pre-vacuum**

The pre-vacuum method consists of four phases.

- **Conditioning phase**  
The conditioning phase removes air from the chamber by pulling a vacuum and then warms the instruments by injecting steam.
- **Sterilization phase**  
Sterilization takes place at a temperature of 134 °C (+ 3 °C) and a pressure (absolute) of 300 kPa (3 bar/43.5 psi) over a period of 5 minutes (minimum).
- **Evacuation phase**  
The steam is released.
- **Drying phase**  
Drying takes place under renewed application of a vacuum over a period of approx. 5-20 minutes.

**Validated parameters:**

Temperature:	270 to 272 °F; 134 °C (+ 3 °C)
Pressure (p <sub>absolute</sub> ):	300 kPa (3 bar, 43.5 psi)
Exposure time:	5 minutes

**Fractionated pre-vacuum procedure**

The fractionated prevacuum procedure consists of four phases.

- **Conditioning phase**  
The conditioning phase removes air from the chamber by pulling a vacuum up to four times. Steam is then blown in.
- **Sterilization phase**  
Sterilization takes place at a temperature of 134 °C (+3 °C) and a pressure (absolute) of 300 kPa (3 bar/43.5 psi) over a period of 5 minutes (minimum time).
- **Evacuation phase**  
The steam is released.
- **Drying phase**  
Drying takes place under renewed application of a vacuum over a period of approx. 5-20 minutes.

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**Gravitation method**

**!** *Warning: This method is only suitable for smooth and accessible surfaces, not for instruments with channels, gaps or stopcocks.*

The gravitation method is also made up of 4 phases:

- **Conditioning phase**  
Steam is pressed into the chamber and heats the instruments.
- **Sterilization phase**  
Sterilization takes place at a temperature of 121°C and a pressure (absolute) of 2 bar over a period of 30 - 45 minutes. Sterilization may also take place at 134 °C and 3 bar, however the time required is 5 - 10 minutes.
- **Evacuation phase**  
Steam is released through the outlet.
- **Drying phase**  
Drying takes place under ambient air pressure by heating through the heating jacket of the autoclave.

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### Flash sterilization

**Warning:** This method is only suitable for smooth and accessible surfaces, not for instruments with channels, gaps or stopcocks.

**Warning:** Flash sterilization is a method which, from the perspective of hygiene, must only be used in exceptional cases.

**Warning:** This procedure is permitted if it is certain that the telescope will not be subjected to any thermal shock.

**Note:** If the telescope is cooled down too quickly due to the room temperature being lower when the telescope is removed from the unit directly after autoclaving, the ambient air enclosed in the telescope will cause condensation. This misting will mean the telescope cannot be used until the condensation has been absorbed by the air.

Flash sterilization may take place either in a fore-vacuum or gravitation sterilization unit. flash sterilization in a fore-vacuum unit takes place without a conditioning or drying phase. Flash sterilization in a gravitation sterilization unit takes place without a drying phase.

We have validated flash sterilization with the use of a gravitation sterilization unit under the following conditions:

Temperature:	134 °C (+3 °C)
Pressure (p <sub>b</sub> ):	300 kPa (3 bar, 43,5 psi)
Exposure time:	10 minutes

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**Gas sterilization/plasma sterilization**

Sterilization using formaldehyde or ethylene oxide and plasma sterilization are gentle and therefore recommended methods of sterilization for the instruments.

Immerse the instruments in suitable vessels and vacuum seal them in suitable plastic film.

**Caution:** During sterilization the instruments must not have any direct contact with metal.

**Warning:** Plasma sterilization is not possible on greased or oiled surfaces.

**Note:** Observe national laws and regulations for the gas sterilization method.

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For gas sterilization with ethylene oxide, the following airing times must be observed for instruments due to gas absorption of the materials: When airing in the airing cabinet, the temperature should be maintained between 45 °C and 55 °C over a period of twelve hours.

	at 20 °C	at 42 °C	at 55 °C
Telescopes	24 hrs	18 hrs	12 hrs
Rubber items	7 days	4 days	3 days
Plastics/ Fiberscopes	5 days	2 days	30 hrs

(minimum time in simple transparent packaging and an application time on the patient of more than 30 minutes).

Maximum acceptable levels (per 21 CFR 812.100; Federal Register, Vol. 43, No. 122, June 23, 1978) of residues following EtO sterilization are as follows:

Ethylene oxide:	25 ppm* / 250 ppm**
Ethylene chlorohydrine:	25 ppm* / 250 ppm**
Ethylene glycol:	250 ppm* / 5000 ppm**

\*\* for instruments in contact with skin and mucous membrane,  
\* for instruments in contact with blood.

The instruments can be used immediately after sterilization if sterilized using formaldehyde or plasma sterilization. Extra airing time is not necessary.

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**Plasma sterilization**

Plasma sterilization (STERRAD®) is a special method which is particularly suited for thermolabile items. In plasma sterilization, changes to the coloration of the surfaces of aluminum alloys could occur, though this does not influence the function of the instruments. It must be pointed out in this context that anodized aluminum parts are not always recognizable as such.

Fig: Decolorized aluminium on a camera

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 **Warning:** Sterility is only assured if the manufacturer's instructions for the STERRAD® procedure are followed.

 **Warning:** All deviations from the sterilization parameters recommended for the STERRAD® system must be validated by the user.

**Note:** Instruments which KARL STORZ has classified as compatible with the STERRAD® sterilization procedure, have been validated with at least a hundred STERRAD® cycles.

**Note:** Observe the requirements of the sterilizer manufacturer when packaging. The use of telescope holders 39501A to U is possible without impairing the results of sterilization.

No outgassing time is prescribed for plasma sterilization.

Following Product categories have been tested using the STERRAD® method. They are suitable for sterilization without significant influence on the material or function.

- Rigid telescopes (except telescopes with eyepiece drive mechanism)
- Flexible, semiflexible fiberscopes and miniature endoscopes
- Video cameras<sup>1</sup> (except Endovision XL)
- Ureteral light probe
- Fiber light cable
- HF cable
- Insulated and noninsulated Take-apart and  
CLICK<sup>®</sup>liné instruments
- Insulated surgical instruments
- Non-insulated surgical instruments
- Trocars / sheaths / cannulas / obturators /  
Working elements<sup>1</sup>

<sup>1</sup> KARL STORZ video cameras manufactured after September 5, 1996, are compatible to STERRAD®. Cameras manufactured before this date may be refitted. Please contact the KARL STORZ service department for details regarding refitting.

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Please remember that there are limits regarding the size of the lumen and material in regard to instruments which may be sterilized using STERRAD®. In this regard, observe the operating instructions for the unit or contact the units manufacturer so that sterility for all parts can be ensured.

KARL STORZ products that cannot be sterilized using STERRAD:

- Telescopes with eyepiece mechanism
- Endovision XL video camera
- Reusable silicone tubing sets, e. g. for the Endoflator, Endomat, Hydromat
- Special connecting cord, Art. no. 27080 KA

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**Chemical sterilization**

Chemical sterilization (STERIS™) is a special method which is particularly suited for thermolabile goods such as fiberscopes. Except for telescopes with eyepiece drive mechanism, all KARL STORZ products are suitable for STERIS™ sterilization. Color changes to the surface of aluminum alloys could occur during sterilization, which do not however affect the function of the instrument. It must be pointed out in this context that anodized aluminum parts are not always recognizable as such.

**Warning:** Sterility is only guaranteed if the manufacturer's instructions for the STERIS System have been followed. This also applies to use of the sterilization containers.

**Warning:** Chemical sterilization is not possible on greased or oiled surfaces.

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**Warning:** Parts with a lumen must be prepared together with the STERIS Quick Connect kit. All parts must be thoroughly cleaned before sterilization using the STERIS PROCESS™, which also applies to the other sterilization methods.

The following product categories have material compatibility and have been validated with the STERIS PROCESS™:

- Rigid telescopes without working channel, without eyepiece drive
- Semiflexible telescopes without working channel
- Flexible hysteroscopes\* and cystoscopes\* for adults
- Flexible ureteroscopes\*, pediatric cystoscopes\* and choledochoscopes\*, manufactured after July 1, 1997
- Immersible video cameras
- Illuminated ureteral probes
- Light cable
- Calcutript connecting cable

\* Flexible endoscopes are only compatible and validated if they are sterilized using the STERIS Quick Connect Kit in the STERIS container for flexible endoscopes

KARL STORZ product categories which have material compatibility, but which have not been validated in reference to sterility\*:

- Rigid and semiflexible telescopes with working channel
- Flexible bronchoscopes
- Flexible gastroscopes
- Working elements, proctoscopes and resectoscopes
- Trocars / sheaths
- Reusable tube sets
- Insulated and noninsulated Take-apart and CLICK'liné instruments
- Insulated and noninsulated forceps, scissors, punches, and suture instruments
- Instruments for coagulation, irrigation and suction

\* The sterility of parts, described as having material compatibility but not having been validated, cannot be guaranteed using the STERIS PROCESS. However, these parts may be steam sterilized, gas sterilized or plasma sterilized.

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### Error/damage detection

For material damages, a change in procedure must be introduced in collaboration with the suppliers of the cleaning, sterilization and steam supply systems, the manufacturers of the chemical agents, and the in-house maintenance department. Only then can a long service life for the endoscopic instruments be guaranteed. Cloths, used for packing the instruments, must not contain any chemical residues, for this may cause discolorations.

Water spots should be avoided by using demineralized water and steam in accordance with EN 285. KARL STORZ will gladly help you with any questions.

**Caution:** Please refer to the water and steam quality section.

**Caution:** Constantly switching between various preparation methods is especially hard on materials and must therefore be avoided. A one-time change of method, due to procurement of new cleaning equipment for example, is safe.

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### Corrosion

Corrosion effects always occur due to procedures in the preparation process which usually can be improved through simple measures. Often, foreign components in the steam or contact with other instruments made of non-stainless steel or particles from them are the cause. Therefore, before testing the steam quality, the remaining instrument set should be looked over.

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Pitting is a type of corrosion which also affects stainless steels. It is primarily caused by the action of chlorides which are contained in water and blood.

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Other halogen ions also have the same effect, and even nonferrous metals, such as copper and aluminum alloys, could be affected by pitting usually due to electrochemical causes. Pitting becomes apparent by the occurrence of holes spreading on the surface of the instrument. The occurrence of pitting is best prevented by immediate cleaning after use and by subsequent rinsing with demineralized water.

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*Stress cracking corrosion* may generally only occur on stainless instrument steels. To prevent a decrease in clamping force or even stress cracking, instruments with an locking mechanism should only be sterilized open or in the first locking position. Chloride ions in water also lead to stress cracking corrosion.

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*Fretting and crevice corrosion* may occur in narrow joint crevices due to chemical or mechanical destruction of the natural passive layer of the stainless steel in conjunction with moisture. It can be prevented by oiling joints or other movable parts.

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### Discolorations

Discolorations are also a form of corrosion if they do not originate from residues that are actually present. In particular, black discoloration is a phenomenon which in rare cases can affect the entire instrument set. It affects both stainless chrome and stainless chrome-nickel steels as well as nickel-plated and silver-plated materials. The water or steam is not always the cause. Frequently the discoloring reagents originate from other parts, e.g. cloths containing detergent residues from dry cleaning.

The following are possible causes:

1. Sulfur from packagings for steam sterilization (as indicator), rubber materials or oxygen corrosion agents in boiler feed water. This is the case with nickel-plated and silver-plated surfaces.
2. Phosphoric acid from acidic cleaning or neutralization agents used for cleaning by machine. In the event of malfunctions or incorrect metering during machine cleaning, these products reach the instruments and react with the surface to form black iron-phosphate layers on chrome steels during steam sterilization. Citric acid in cleaning agents and disinfectants causes a black layer which cannot be removed even with special chemicals, but only by mechanical cleaning.
3. It may be the natural passive layer which protects all stainless chrome and chrome-nickel steels. If this layer increases to an exceptional degree over a period of time, the instrument becomes increasingly black. The corrosion resistance of the surface is then actually increased. This layer also cannot be removed even with special chemicals, but only by mechanical cleaning.

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**Deplating**

Highly alkaline or acid cleaning agents cause deplating of chrome-plated brass surfaces in the long term when machine cleaning is used. If the metering for the machine is not set correctly and checked, deplating can occur after a period of between two months and three years. Since this effect is only recognizable once all the chrome-plated brass parts have been damaged, it is important to plan the choice of machine program and chemicals very carefully.

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**Seals**

Although high quality plastics are used almost exclusively in endoscopic instrument sets, these are also subject to a natural aging process which is accelerated by temperatures above 80 °C, expansion during storage as well as by light (UV rays). Usually, discoloration, brittleness or hardening occurs. Penetration of liquids or gases may sometimes lead to irreversible damage due to the occurrence of swelling.

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**Spare parts**

If spare parts are required, you will find a spare parts list in the catalog, chapter "Spare parts", with article numbers for the instruments in ascending order. You can then find the catalog number for the required spare part on this list. If you are not certain, for example, which seal is correct, please compare your damaged seal to the actual size figure in the illustrated spare parts list.

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**Removing deposits on metallic surfaces**

Deposits on the metal sheaths can be removed with DURAGLIT polishing wool (Art. No. 27658). Then the part must again go through a complete cleaning cycle.

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**Disposal**

Standard batteries and rechargeable battery inserts may contain toxic substances so dispose of in accordance with the environmental regulations.

To dispose of the Instrument set, no special measures are necessary.

National laws and regulations must be observed.

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**Directive compliance**

This medical product bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC. A code number after the CE mark indicates the responsible notified body.

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**Limitation of liability**

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the conditions that all assembly operations, system expansions, readjustments, modifications, or repairs to same have been performed by KARL STORZ, and that the instrument has been used in accordance with its operating instructions at all times.

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KARL STORZ GmbH & Co. KG  
Postfach 230  
D-78503 Tuttlingen

**Warranty**

For **two** years as of delivery to the end customer, we agree to replace the goods free of charge if proof can be provided of faulty materials or faulty workmanship. In doing so we cannot accept the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Please fill out the attached warranty card on the last page and return it as soon as possible to:

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Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall release us from any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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**Important information**

From a hygienic point of view and in order to prevent infection, telescopes, lenses and devices must be disinfected and, if necessary, sterilized before being sent in for repair.

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**Servicing and repair**

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ genuine KARL STORZ parts.

Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all warranty rights.

**Repair and replacement program**

KARL STORZ maintains a repair and replacement warehouse which is normally adequate to ensure prompt replacement of damaged telescopes and instruments. Under the repair and replacement plan, you receive an identical as-new instrument and are only charged the repair costs for the defective instrument.

For rigid telescopes you receive a warranty of 3 years, for fiberscopes and instruments 2 years.

For videoscopes and electronic units, individual repair is necessary.

To bridge the repair period, you will receive a unit on loan which you then return to KARL STORZ as soon as you receive the repaired unit.

In Germany you can refer repairs direct to

KARL STORZ GmbH & Co. KG  
Abt. Reparaturservice  
Dr. Karl-Storz-Straße 34  
D-78532 Tuttlingen

In other countries please contact your local KARL STORZ branch or authorized dealer.

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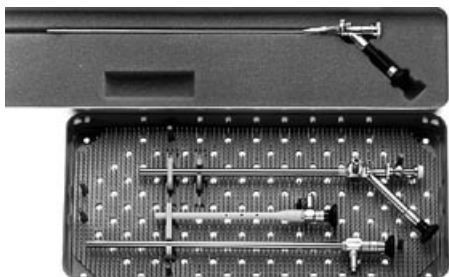
KARL STORZ product groups that are not sterilizable with the STERIS PROCESS™:

- Telescopes with eyepiece drive mechanism
- Motors and their handles (e.g. for shavers)
- IMPERATOR handpieces
- Prismatic light deflectors

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**Storage**

Instruments should always be stored in suitable containers (see Accessories) and must be absolutely dry.

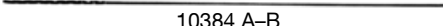
The place of storage must have normal temperature and air humidity values (30 - 50 % r.h.; 20°C).

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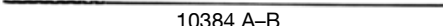
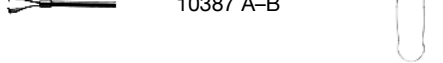
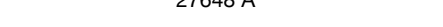
**Excerpt from the accessories  
for manual and machine preparation,  
including sterilization.**

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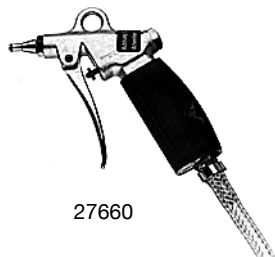
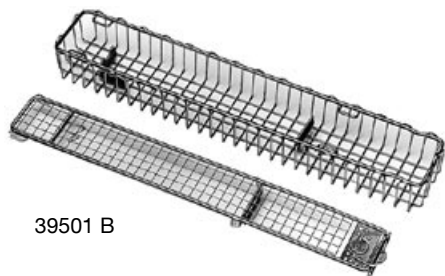
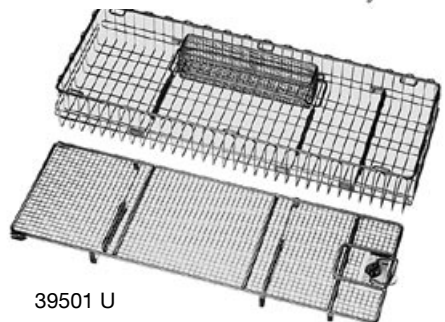
**Baskets for Sterilizing and storage  
of telescopes**

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**Plastic containers for sterilizing  
and storage**

**Note:** Fiberscopes stored in these containers can only be gas-sterilized.

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**Plastic containers for sterilizing  
and storage**

**Note:** The containers may be used for sterilization with  
vapor or gas.

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**Sterilizing Cases, Metal Cases for  
HOPKINS® Telescopes**

**Note:** The containers may be used for sterilization with vapor or gas.

Before sterilization the foam insert is to be removed and the container is to be laid out with drapes.

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**Sterilizing Cases, Metal Cases for Instrument Sets**

**Note:** Perforated for storage and sterilization with gas and vapor.  
Unperforated for storage.

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**Plastic Soaking/Storage System**

**Plastic bath for manual cleaning and  
disinfection**

**Note:**

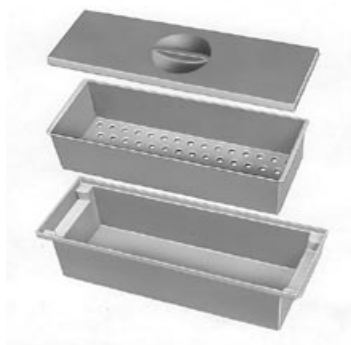
- Instruments must be disassembled and cleaned prior to disinfection. To disinfect, immerse instrument completely in solution.
- Follow manufacturer's instructions regarding time period for soaking and solution density.
- To wash out instruments, remove perforated storage tray from the disinfectant solution and rinse instruments thoroughly with sterile water in a second basin.
- Shake or tilt basin gently to remove air bubbles during disinfection and to insure complete neutralization.

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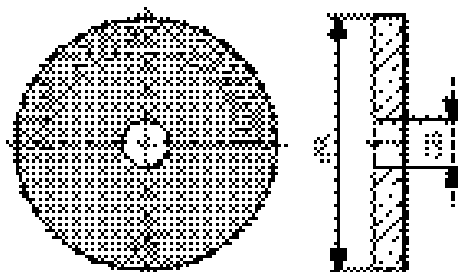
**Bag for storage  
for fixation to Mobile ENDO Cabinet 620 A  
or other storage cabinets.**

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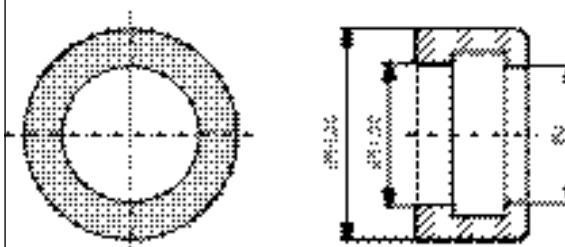
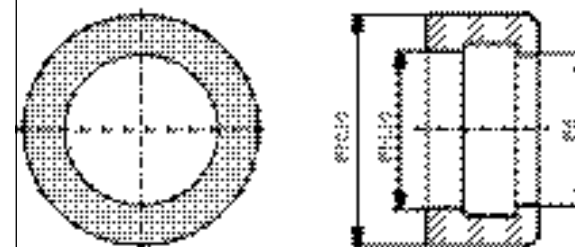
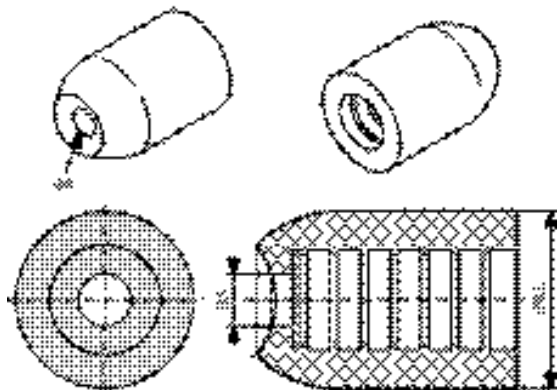
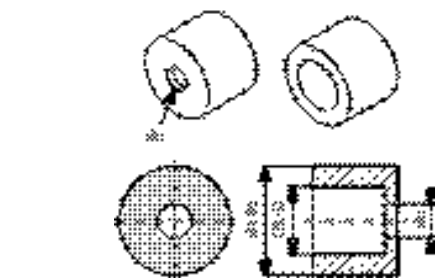
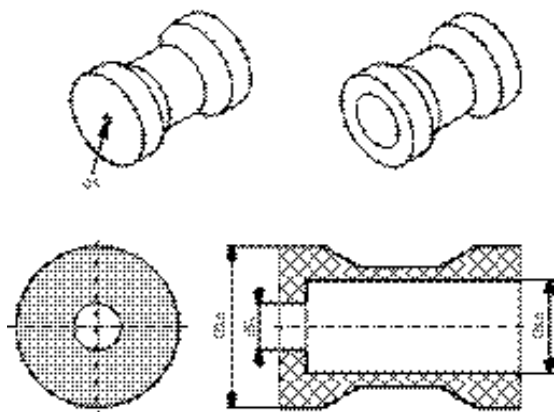
**Type**

- 1= Cleaner
- 2= Disinfectant
- 3= Suitable for ultrasound baths

**Note:** You will also find the current list of approved products on our Web site ([www.karlstorz.com](http://www.karlstorz.com)).

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**Excerpt from the list of chemicals for the preparation of telescopes and endoscopic instrument sets**

**Manual cleaning and disinfection/ultrasound**

The following instruments and telescopes produced by KARL STORZ must not be fully immersed in a liquid: HAMOU® telescopes with ocular focusing, magnifiers with focusing, motors and their handles, IMPERATOR drill handles, prismatic light deflectors. The following instruments produced by KARL STORZ must not be placed in an ultrasound cleaning bath:

rigid telescopes, flexible telescopes, other instruments with optical components made of glass.

When preparing and using the solutions, follow the solution manufacturer's instructions, paying close attention to proper dilution and recommended immersion times. Regarding the microbiological effectiveness please ask the manufacturer. The following cleaning and disinfectant solutions have been approved regarding material compatibility:

Ecolab GmbH & Co. OHG	Sekusept Plus	1, 2, 3
	Sekusept Pulver Classic	1, 2, 3
Esteer Pharma GmbH	Ultrademit AF	1, 2, 3
	Ultrasmit Aktiv	1, 2, 3
	Ultrasmitin Classic	1, 2, 3
	Afid	1, 2
Fresenius AG	Afid plus	2
	Sporcid FF	1, 2
Holifa Polska Sp. zo.o	Polsept Holifa	1, 2
	Letahdeido	2
Indeba Industria E Comercio LTDA	Darodor 4000 Liquido	1
	Darodor 9000	2
	Darodor Sinaldehyd 2000	2
Jose Collado S.A.	Aldasan 2000	1, 2
	Almyrol	1, 2
Lysoform Dr. Rosenann GmbH	Desoform	1, 2, 3
	Lysiformin 3000	1, 2
	MedDis	2
Medichem International	MediZyme	1
	Edisonite Super	1, 3
Merz Hygiene GmbH	Mucadont-IS	2, 3
	Mucadont-Zymaktiv	1, 3
	Mucocit-T neu	1, 2, 3
METASYS Medizintechnik GmbH (formerly LONZA)	ID 50	2
	A 20	1, 3, A
	Orochemie	2
Promagent AB	Wavacide	2
Schumacher, Dr. GmbH	Descoton Extra	1, 2
	Descoton forte	2
	Perfektan Endo	2, 3
	Perfektan Neu	1, 2
	Perfektan TB	1, 2, 3, A
	Gigasept	2, 3
Schülke & Mayr GmbH	Gigasept AF forte	1, 2
	Gigasept FF	1, 2
	Gigasept Instru AF	1, 2, 3, A
	Gigasept Med forte	1, 2
	Gigazyme	1, 3
	Lyselol V	2
Steris	EnzyCare 2	1, 3
	neodisher LM 2	1, 3
	neodisher mediclean	1
Weigert, Dr. GmbH & Co.	neodisher mediclean forte	1, 3
	neodisher medizym	1, 3
	neodisher SeptoClean	1, A
	neodisher SeptoPreClean	1, 2
	Aidal Plus	2
	Matrix	1

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**\* Caution:** The use of CAUSTIC SODA can cause changes to the surface of aluminium parts (including coated parts), plastics and soldered connections, and may reduce the service life of the instrument set.

For flexible endoscopes, only immerse the examination sheath, not the housing/handle part.

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**Type**

- 1= Cleaner
- 2= Disinfectant
- 3= Suitable for ultrasound baths
- 4= Neutralizer
- 5= Rinse aid

**Note:** You will also find the current list of approved products on our Web site ([www.karlstorz.com](http://www.karlstorz.com)).

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**Machine cleaning and disinfection**

The following instruments and telescopes produced by KARL STORZ must not be prepared in a cleaning and disinfection machine: HAMOU® telescopes with ocular focusing, magnifiers with focusing, motors and their handles, IMPERATOR drill handles, standard, tungsten carbide and diamond-tipped drills, prismatic light deflectors, and magnetic instrument rack.

Selection of a method for cleaning and disinfection must be discussed with the machine manufacturer and the manufacturer of the chemical reagents. Only special methods must be used which have been verified for this purpose. Regarding the microbiological effectiveness please ask the manufacturer. Thermal disinfection is preferable. The following agents for machine cleaning and disinfection have been approved regarding material compatibility:

	Thermosept alka clean	1
	Thermosept DK	2
Schülke & Mayr GmbH	Thermosept ED	2
	Thermosept ER	1
	Thermosept RKN-zym	1
Steris	Instru-Kienz	1, 3
	neodisher DuoClean	1, A
	neodisher FA	1
	neodisher FA forte	1, 3
Weigert, Dr. GmbH & Co.	neodisher Oxivario	1
	neodisher mediclean	1
	neodisher mediclean forte	1, 3
	neodisher mediklar	5
	neodisher medizym	1
	neodisher SeptoClean	1, 2, A
	neodisher Septo DA	2
	neodisher Septo DN	2

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Tipo del aparato: Número de serie:

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Firma/Fecha:

To be filled out by instrument owner:  
Return address/company stamp:

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# WARRANTY

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship.

In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

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To be completed by the supplier/importer:  
Company stamp/signature:

KARL STORZ GmbH & Co. KG  
Mittelstraße 8, 78532 Tuttlingen, Germany  
Postfach 230, 78503 Tuttlingen, Germany  
Phone: +49 (0)7461 708-0,  
Fax: +49 (0)7461 708-105  
E-Mail: info@karlstorz.de  
Web: www.karlstorz.com

KARL STORZ Endoskope Berlin GmbH  
Ohlauer Straße 43  
10999 Berlin, Germany  
Phone: +49 (0)30 30 69090  
Fax: +49 (0)30 30 19452

KARL STORZ Endoscopy Canada Ltd.  
2345 Argenta Road, Suite 100  
Mississauga, ON, L5N 8K4, Canada  
Phone: +1 905 816-8100, Fax: +1 905  
858-0933  
E-Mail: info@karlstorz.ca

KARL STORZ Endoscopy-America, Inc.  
600 Corporate Pointe  
Culver City, CA 90230-7600, USA  
Phone: +1 310 338-8100, +1 800 421-0837  
Fax: +1 310 410-5527  
E-Mail: info@kseaa.com

KARL STORZ Veterinary Endoscopy America,  
Inc.  
175 Cremona Drive  
Goleta, CA 93117, USA  
Phone: +1 805 968-7776, Fax: +1 805  
685-2588  
E-Mail: info@karlstorzvet.com

KARL STORZ Endoscopia Latino-America,  
Inc.  
815 N. W. 57th Avenue, Suite 480  
Miami, FL 33126-2042, USA  
Phone: +1 305 262-8980, Fax: +1 305  
262-8986  
E-Mail: info@ksela.com

KARL STORZ Endoscopia  
Miramar Trade Center  
Edificio Jerusalem, Oficina 308 A,  
La Habana, Cuba  
Phone: +53 72041097, Fax: +53 72041098

KARL STORZ Endoscopia México S.A. de C.V.  
Lago Constanza No 326  
Col. Chapultepec Morales  
D.F.C.P. 11520, México, Mexico  
Phone: +52 5552 5056 07, Fax: +52 55 5545  
0174

KARL STORZ Endoscopia Argentina S.A.  
Cerviño 4449 Piso 10°  
1425 Buenos Aires C. F., Argentina  
Phone: +54 11 4772 4545, Fax: +54 11 4772  
4433  
E-Mail: info@karlstorz.com.ar

KARL STORZ Endoskop Sverige AB  
Storsättragränd 14, 12739 Skärholmen,  
Sweden  
Postal address: Po Box 8013, 14108 Kungens  
Kurva, Sweden  
Phone: +46 8 50 56 480, Fax: +46 8 50 56  
4848  
E-Mail: kundservice@karl-storz.se

KARL STORZ Endoskop Danmark A/S  
Skovlytoften 33  
2840 Holte, Denmark  
Phone: +45 45162600, Fax: +45 45162609  
E-Mail: marketing@karlstorz.dk

KARL STORZ Endoscopy (UK) Ltd.  
392 Edinburgh Avenue, Slough  
Berkshire, SL1 4UF, Great Britain  
Phone: +44 1753 503500, Fax: +44 1753  
578124  
E-Mail: customerservice@karlstorz-uk.com

KARL STORZ Endoscopia Nederland B. V.  
Phone: +31 651 938 738, Fax: +31 135 302  
231

KARL STORZ Endoscopy Belgium N. V.  
Phone: +32 473 810 451, E-Mail: info@stoplep.  
be

KARL STORZ Endoscopia France S. A.  
12, rue Georges Guynemer  
Quartier de l'Europe  
78280 Guyancourt, France  
Phone: +33 1 30484200, Fax: +33 1 30484201  
E-Mail: marketing@karlstorz.fr

KARL STORZ Endoskop Austria GmbH  
Landstraßer Hauptstr. 146/11/18  
1030 Wien, Austria  
Phone: +43 1 71 56 0470, Fax: +43 1 71 56  
0479  
E-Mail: storz-austria@karlstorz.at

KARL STORZ Endoscopia Italia S. r. l.  
Via dell'Artigianato, 3  
37135 Verona, Italy  
Phone: +39 045 8222000, Fax: +39 045  
8222001  
E-Mail: info@karlstorz.it

KARL STORZ Endoscopia Ibérica S. A.  
Parque Empresarial San Fernando  
Edificio Francia – Planta Baja  
28830 Madrid, Spain  
Phone: +34 91 6771051, Fax: +34 91 6772981  
E-Mail: marketing@karlstorz.es

KARL STORZ Endoskope Greece Ltd.\*  
Ipsilantou Str. 32  
54248 Thessaloniki, Greece  
Phone: +30 2310 304868 Fax: +30 2310  
304862

\*Repair & Service Subsidiary

KARL STORZ Industrial\*\*  
Gedik Is Merkezi B Blok  
Kat 5, D 38-39, Bagdat Cad. No: 162  
Maltepe Istanbul, Turkey  
Phone: +90 216 442 9500, Fax: +90 216 442  
9030

\*\*Sales for Industrial Endoscopy

KARL STORZ Endoscopia ROMANIA srl  
Str. Prof. Dr. Anton Colorian, nr. 74, Sector 4  
041393 Bukarest, Romania  
Phone: +40 (0)31 4250800, Fax: +40 (0)31  
4250801  
E-Mail: office@karlstorz.ro

TOV KARL STORZ Ukraine  
18b Geroev Stalingrada avenu  
04210 Kiev, Ukraine  
Phone: +380 44 42668-14, -15, -19, -20  
Fax: +380 44 42668-14

OOO KARL STORZ Endoscopy – WOSTOK  
Derbenyevskaya nab. 7, building 4  
115114 Moscow, Russia  
Phone: +7 495 983 02 40, Fax: +7 495 983  
02 41  
E-Mail: kste-wostok@karlstorz.ru

KARL STORZ Endoskope  
Regional Center for Endoscopy S.A.L.  
St. Charles City Center, 5th Floor  
Phoenicia Street, Mina Elhosn  
2020 0908 Beirut, Lebanon  
Phone: +961 1 368181, Fax +961 1 365151

KARL STORZ Endoscopy South Africa, (Pty)  
Ltd.  
8th Floor Convention Tower  
Cnr Heerengracht & Coen Steytler, Foreshore  
Cape Town 8001, South Africa  
P.O.Box 6061, Roggebaai Cape Town,  
8012, South Africa  
Phone: +27 21 417 2600, Fax: +27 21 421  
5103  
E-Mail: info@karlstorz.co.za

TOO KARL STORZ ENDOSCOPY Kasachstan  
Khodjanova 17  
050060 Almaty, Kazakhstan  
Phone/Fax: +7 72 72 49 43 63, +7 72 72 49  
41 00  
e-mail: KarlStorzKazakhstan@itte.kz

KARL STORZ Endoscopy India Private Ltd.  
C-126, Okhla Industrial Area, Phase-1  
New Delhi 110020, India  
Phone: +91 11 26815445-51,  
Fax: +91 11 268129 86  
E-Mail: karlstorz@vsnl.com

KARL STORZ GmbH & Co. KG  
Resident Representative Office  
80/33 (44/19) Dang Van Ngu,  
F.10-Q. Phu Nhuan  
Ho Chi Minh City, Vietnam  
Phone: +848 991 8442, Fax: +848 844 0320

KARL STORZ Endoscopy (S) Pte. Ltd.  
#05 – 08 San Centre, 171 Chin Swee Road  
Singapore 0316, Singapore  
Phone: +65 6376-1066, Fax: +65 6376-1068  
E-Mail: karlstorz@pacific.net.sg

KARL STORZ Endoscopy Australia Pty. Ltd.  
174 Parramatta Rd.  
Camperdown, N.S.W. 2050, Australia  
Phone: +61 02 8594 9150, Fax: +61 02 8594  
9199  
E-Mail: info@karlstorz.au

KARL STORZ Endoscopy China Ltd.  
Hong Kong Representative Office  
Unit 1601, Chinachem Exchange Square  
1 Hoi Wan Street, Quarry Bay,  
Hong Kong, People's Republic of China  
Phone: +8 52 28 65 2411, Fax: +8 52 28 65  
4114  
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.  
Beijing Representative Office  
Room 610, China Life Tower  
No. 6, Chaowai Street  
Beijing, 100020, People's Republic of China  
Phone: +86 10 8525 3725, Fax:  
+86 10 8525 3728  
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.  
Unit 3901-3904, Tower 1 Grand Gateway  
No. 1 Hong Qiao Road  
Shanghai, 200030, People's Republic of China  
Phone: +86 21 6113-1188, Fax:  
+86 21 6113-1199  
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.  
Chengdu Representative Office  
F-5, 24/F., Chuanxing Mansion,  
No. 18 Renming Road South  
Chengdu, Sichuan, 610016, People's Rep.  
of China  
Phone: +86 28 8620-0175, Fax:  
+86 28 8620-0177  
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.  
Guangzhou Representative Office  
Room 1119-20, Dongshan Plaza  
69 Xianlie Road Middle, Dongshan District,  
Guangzhou, Guangdong, 510095,  
People's Rep. of China  
Phone: +86 20 8732-1281, Fax:  
+86 20 8732-1286  
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy Asia Marketing  
Pte Ltd  
3791 Jalan Bukit Merah  
06-11 e-Centre @ Redhill  
Singapore 159471, Singapore  
Tel. No. +65 63761066, Fax. No. +65  
63761068  
Email : infoasia@karlstorz.com.sg  
Email : serviceasia@karlstorz.com.sg

KARL STORZ Endoscopy Singapore Sales  
Pte Ltd  
3791 Jalan Bukit Merah  
10-18 e-Centre @ Redhill  
Singapore 159471  
Tel. No. +65 65325548, Fax No. +65  
65323832  
Email : infosp@karlstorz.com.sg  
Email : servicesgp@karlstorz.com.sg

KARL STORZ Endoscopy Japan K. K.  
Bois Hongo Building 6F1, 3-42-5 Hongo  
Bunkyo-ku, Tokyo 113-0033, Japan  
Phone: +81 3 5802-3966, Fax: +81 3  
5802-3988  
E-Mail: info@karlstorz.co.jp

www.karlstorz.com

**STORZ**  
KARL STORZ—ENDOSKOPE

**KARL STORZ GmbH & Co. KG**

Mittelstraße 8  
78532 Tuttlingen

Postfach 230  
78503 Tuttlingen  
Germany

Telefon: +49 (0)7461 708-0  
Telefax: +49 (0)7461 708-105  
E-Mail: [info@karlstorz.de](mailto:info@karlstorz.de)  
Web: [www.karlstorz.com](http://www.karlstorz.com)