STORZ

KARL STORZ — ENDOSKOPE

20092132

KARL STORZ *AIDA* INTERFACE II

20092132

KARL STORZ *AIDA* INTERFACE II

20092132

KARL STORZ *AIDA* INTERFACE II

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Instruction Manual

Manual de instrucciones

20092132

KARL STORZ *AIDA* INTERFACE II

20092132

KARL STORZ *AIDA* INTERFACE II

20092132

KARL STORZ *AIDA* INTERFACE II

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**Important information
for users of KARL STORZ
instruments**

Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern, high-quality item of equipment from KARL STORZ.

This instruction manual is intended to serve as an aid in the proper setup, installation, and operation of the interface unit **AIDA** INTERFACE II. All essential details of the equipment and all actions required on your part are clearly presented and explained. We thus ask that you read this manual carefully before proceeding to work with the equipment. Insert this manual in its protective wallet and keep it available for ready reference in a convenient and conspicuous location near the equipment.

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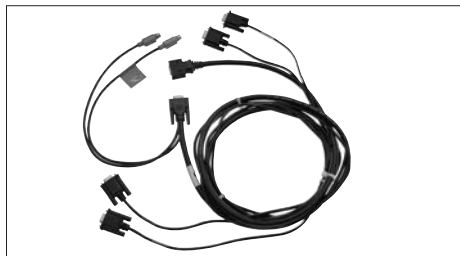
Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

KARL STORZ *AIDA* INTERFACE II 20092132: Grundausrüstung /Basic equipment /Equipo básico

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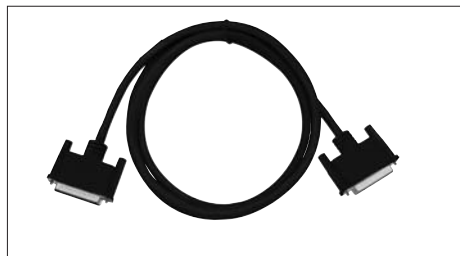
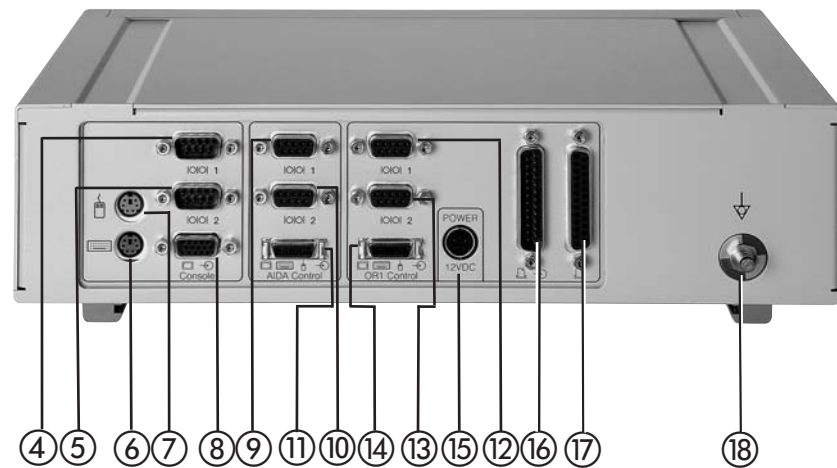
**Controls, displays,
connectors, and
their uses**

- ① Drawer for ready-reference guide
- ② SCB status display
- ③ AIDA status display
- ④ RS 232 connector (COM1) – Touch screen
- ⑤ RS-232 connector (COM2) – Touch screen
- ⑥ PC keyboard connector
- ⑦ PC mouse connector
- ⑧ VGA (RGB) connector – Touch screen

**PC connecting cable 20090374 from
KARL STORZ AIDA® control 20096020**

- ⑨ RS 232 connector (COM1)
- ⑩ RS 232 connector (COM2)
- ⑪ Keyboard + mouse + monitor connector (input).

**PC connecting cable 20090374 from
KARL STORZ OR1™ control 20097020-1**

- ⑫ RS 232 connector (COM1)
- ⑬ RS-232 connector (COM2)
- ⑭ Keyboard + mouse + monitor connector (input).

- ⑮ Socket for external AC adapter 20090585(EU), 20090586(UK), 20090587(US)
- ⑯ Connector for parallel port connecting cable 20090475 from KARL STORZ OR1™ control 20097020-1 (printer input)
- ⑰ Printer output
- ⑱ Potential equalization connector
- ⑲ External AC adapter with mains plug (connection to ⑮)

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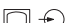
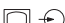
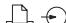
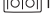
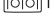
Photographs of the unit	.IV
Controls, displays, connectors, and their uses	.V
Symbols employed	.VI
General information	
Description of the unit	.2
Safety instructions	
Warnings and cautions	.3
Intended use	.5
User qualification	.5
Safety precautions at the site of installation	.6
Safety precautions when operating the unit	.6
Installation and operating instructions	
Unpacking the equipment	.7
Basic equipment	.7
Installing and connecting up the unit	.7
Connecting the KARL STORZ AIDA® control and OR1™ control	.9
Operating the KARL STORZ AIDA® computer	.11
Configuring the KARL STORZ AIDA® control – network	.12
Configuring the KARL STORZ AIDA® control – combinations 1 and 2	.13
Setting up - touch screen monitor	.15
Installation of the driver	.16
Spec. settings in conn. with AIDA Interface	.23
Calibrating the touch screen monitor	.27
Maintenance	
Cleaning and care of the unit	.35
Maintenance	.36
Servicing and repair	.36
Repair program	.37
Limitation of liability	.38
Manufacturer's warranty	.38
Technical description	
Troubleshooting	.39
Technical data	.41
Technical documentation	.42
Block diagram	.43
Spare parts, accessories	
Spare parts/accessories	.44
Electromagnetic Compatibility (EMC) Information	.45

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Description of the unit

The KARL STORZ interface unit **AIDA** INTERFACE II 20092132 enables the SCB and AIDA systems to share input/output devices (LCD monitor, touch screen, keyboard and mouse).

For this, the interface unit is connected between KARL STORZ OR1™ control (SCB control computer), KARL STORZ AIDA® control (AIDA control computer) and the input/output devices.

Generally speaking, it is of course possible to operate the SCB (R-UI) and AIDA system with separate input/output devices. If, however, only one ‚console station‘ (keyboard etc.) is to be used, the interface unit **AIDA** INTERFACE II 20092132 must be used.

System requirements

- KARL STORZ OR1™ control with integrated network card and appropriate software version (see the Instruction Manual KARL STORZ-SCB® System)
- KARL STORZ AIDA® control with integrated network card

Certain settings are also required for the KARL STORZ AIDA® control (incl. for the network).

A second touch screen can also be connected to the OR1 control or AIDA control in the combination of the **AIDA** Interface II 20092132 with a KARL STORZ splitter W20419 (OR1-RGB Interface).

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Warnings and cautions

Please read this manual and follow its instructions carefully. The words **Warning**, **Caution**, and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of this product. To make the words **Warning** and **Caution** stand out more clearly, they are accompanied by a pictogram.

Warning: A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result in injury to the patient or physician.

Caution: A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the product.

Note: A Note indicates special information about operating the product, or clarifies important information.

Warning: Read this instruction manual thoroughly and be familiar with its contents prior to using this equipment. Before using the unit, read the following safety instructions carefully to avoid putting your patients, personnel or yourself at risk.

Warning: The unit may be used together with the medical devices OR1™ control and AIDA control in accordance with IEC 60601-1-1. Please observe the instruction manuals of the OR1™ control and AIDA control, in particular all warnings and safety measures.

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Warning: The electrical installations of the operating room in which the equipment is connected and operated must comply with the applicable IEC requirements.

Warning: Keep the unit out of the reach of patients.

Warning: The instruction manuals and interface specifications for medical devices and/or system components used in combination must be observed precisely.

Warning: Combinations of medical devices are only assured to be safe if

- they are identified as such in the respective instruction manuals or
- the intended purpose and interface specifications of the devices used in combination permit this (cp. IEC 60601-1-1).

Warning: Test this equipment prior to each surgical procedure to ensure that it functions correctly.

Warning: Risk of explosion if used in the presence of flammable anesthetics.

Warning: To avoid the risk of electrical shock, do not open the unit. Refer servicing to the manufacturer or to personnel authorized by the manufacturer. Removal of covers by unauthorized personnel will void the equipment's warranty.

Caution: Liquid must never be allowed to enter the unit. Do not store liquids on or above the unit.

Note: Any damage to the unit resulting from incorrect operation is not covered by the manufacturer's warranty.

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Before using the unit it is imperative that you be acquainted with how the unit operates and is controlled.

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Intended use

The KARL STORZ interface unit **AIDA** INTERFACE II 20092132 enables the SCB and AIDA systems to share input/output devices (LCD monitor, touch screen, keyboard and mouse). For this, the interface unit is connected between the KARL STORZ OR1™ control, the KARL STORZ AIDA® control and the input/output devices.

The use of this device in any way other than indicated above is not permissible for reasons of safety.

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Unauthorized conversions or modifications to the unit are not allowed for safety reasons.

The **AIDA** INTERFACE II may be operated only with accessories which have been designated by KARL STORZ as suitable for the unit.

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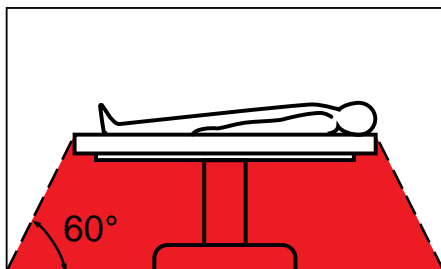
User qualification

The **AIDA** INTERFACE II may only be used in conjunction with the KARL STORZ SCB and AIDA systems by physicians and medical assistants with the appropriate specialized qualifications and who have been instructed in the use of the systems.

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Safety precautions at the site of installation

The unit may only be used in medical facilities where the electrical systems comply with the applicable national regulations.

It is not intended for use in hazardous zones. This means, for example, that when using easily combustible and explosive inhalation anesthetics or mixtures thereof, the unit must not be operated inside the demarcated hazard zone. This also applies for easily combustible and hazardous chemicals, e.g., skin disinfectants and rapid surface disinfectants.

The unit is equipped with a connector for attaching a ground line. It should be connected up in accordance with national regulations.

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Safety precautions when operating the unit

It is the user's responsibility to make sure the unit is safe and operates properly before use.

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Unpacking the equipment

Carefully remove the interface unit **AIDA** INTERFACE II and accessories from their packaging.

Check for missing items and evidence of shipping damage.

File any complaints with the manufacturer or supplier immediately.

If possible, retain the original packing materials for later use; these can come in handy if the unit has to be transported.

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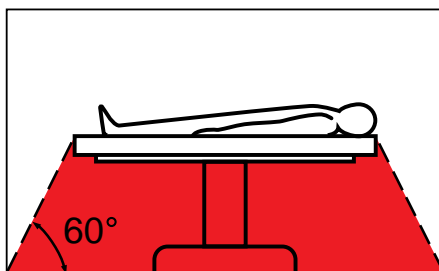
Installing and connecting up the unit

Note: The interface unit **AIDA** INTERFACE II and attached accessories may only be used in medical facilities where the electrical systems comply with the applicable national regulations.

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! Warning: The unit is not intended for use in hazardous zones. Do not operate the unit within the hazard zone shown in the diagram while explosive anesthetic gases are in use.

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The unit is equipped with a connector ⑱ for attaching a ground line. The unit's ground line should be installed by a qualified electrician.

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Connect the AC adapter ⑲ to the socket ⑮ and the mains power supply.

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Warning: Only connect or disconnect the adapter and the mains power supply outside areas where the danger of explosion exists.

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For the further ‚connections‘ from the interface device **AIDA** INTERFACE II to the KARL STORZ AIDA® control and OR1™ control please see the connection diagrams on pages 9 and 10.

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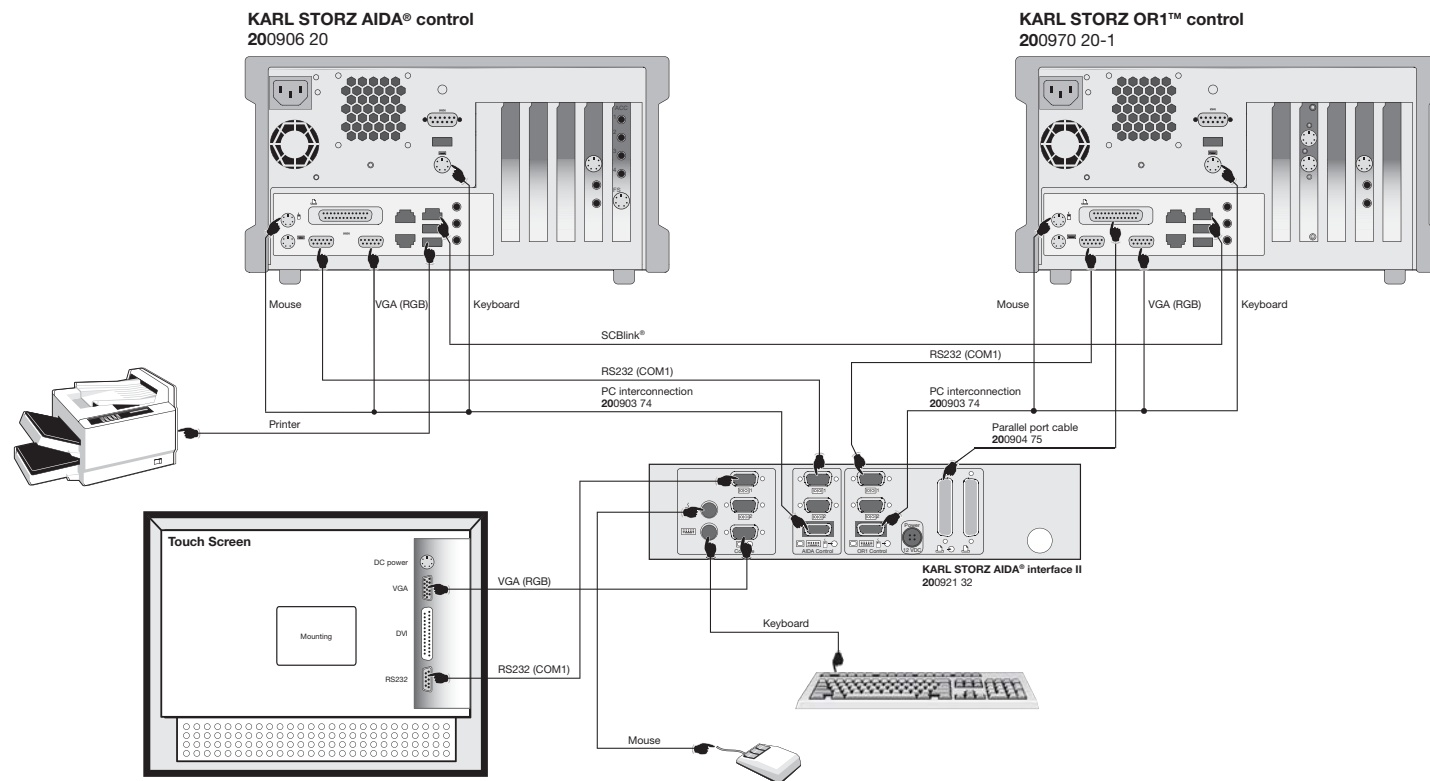
Note: In accordance with IEC 60601-1, the printer must be disconnected from the power supply via an isolation transformer.

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Connecting the KARL STORZ AIDA® control and OR1™ control

Connect the interface box, the two control computers and the input/output devices with the enclosed cables, as shown in the diagram below.

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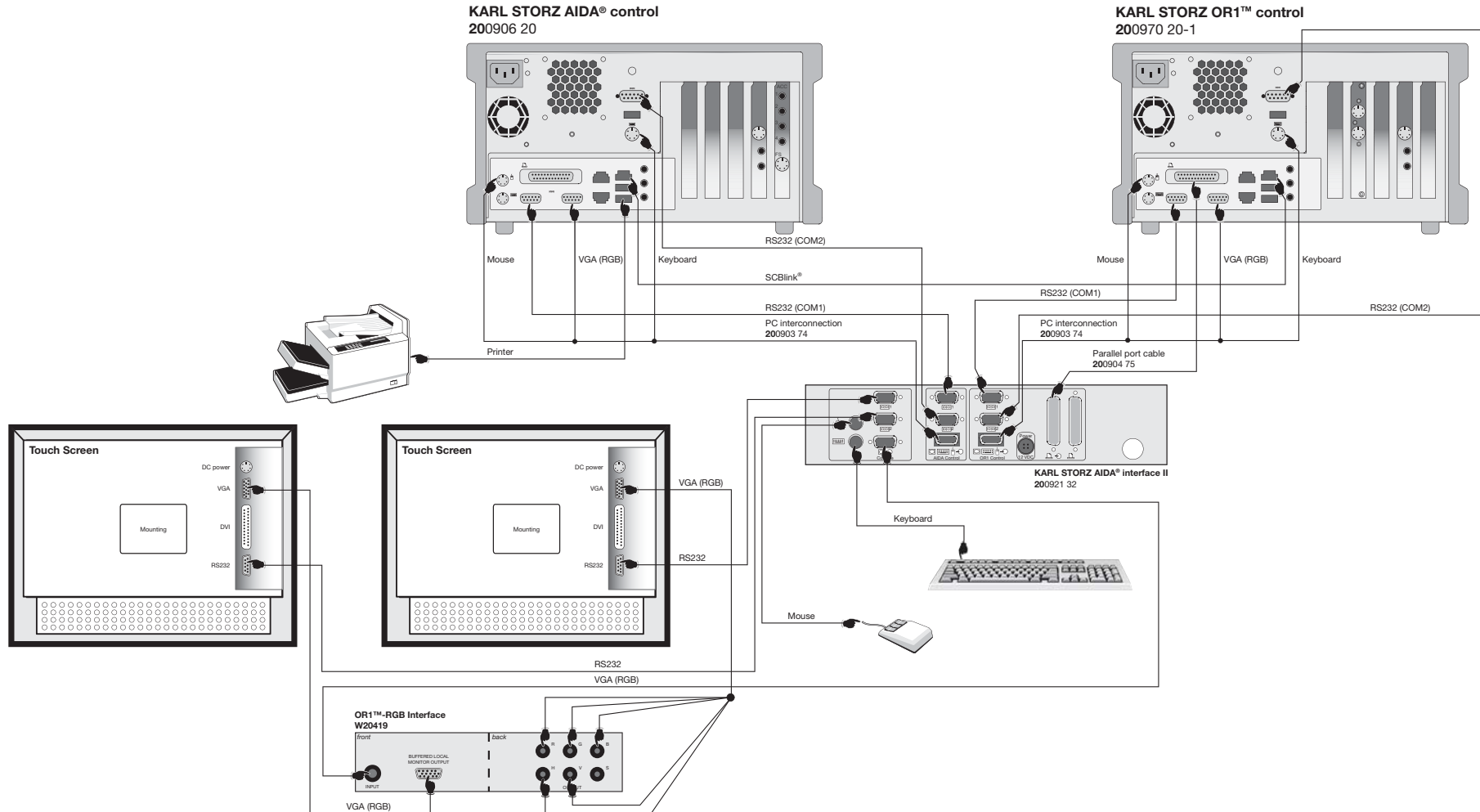
**Operating the
KARL STORZ AIDA® control**

The connectable KARL STORZ AIDA® control can be equipped with various software. Depending on the combination with OR1™ control/R-UI versions, various interface programs can be used.

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Note: If there are two network cards in the KARL STORZ AIDA® control, please carry out the settings on the network card which is connected to the OR1™ control. For safety reasons, we recommend deactivating the ‘second’ network card for the duration of the configuration described below.

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Note: For all the combinations shown in the table, the same basic requirements apply for setting the network and these are described once here. In the sections on the individual combinations only the specific settings are described.

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Configure the KARL STORZ AIDA® computer's network connections

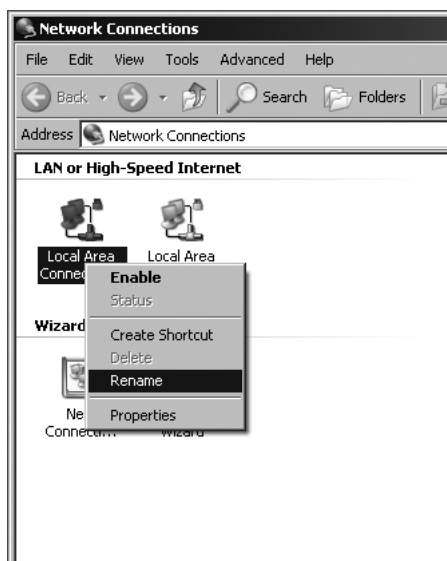
Check that you are registered as a Windows® administrator for the next steps.

When registering in Windows, enter **Administrator** as your user name and **Admin** as the password.

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Note: Carry out these steps on the KARL STORZ AIDA® control.

Call up the 'Local Area Connection Status' window (Start > Settings > Network Connections). Right-click on one of the local area connections and select 'Rename'. Rename this connection to 'SCB' and confirm with Enter.

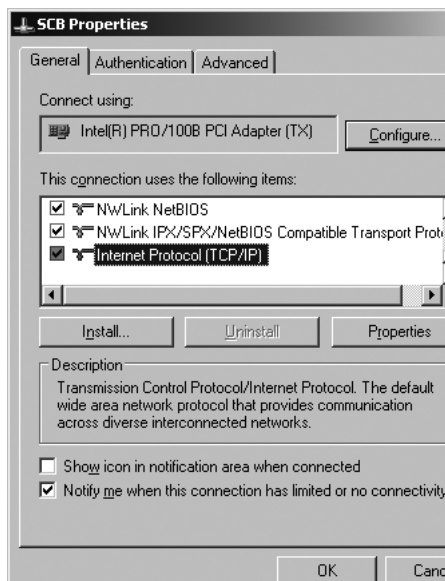
Note: If you have several LAN connections to choose from, please make sure that you choose the LAN connection that is used for the SCBlink® connection between the KARL STORZ AIDA® and the SCB system (see page 9).

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Double clicking the network connection ‚SCB‘ will establish the network connection. Ensure that the cross over cable (SCB link cable) coming from the OR1™ control leads to the AIDA control and is correctly plugged into the network port. Double click a second time on the network connection ‚SCB‘. This will show the window ‚SCB Status‘. Scroll through the list and double click on ‚Internet Protocol (TCP/IP)‘.

Choose ‚Use the following IP address‘ in the window ‚Internet Protocol (TCP/IP) Properties‘ and enter

IP Address: 192 . 168 . 1 . 10

Subnet mask: 255 . 255 . 255 . 0

Confirm all windows with OK and close them.

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- SAM.exe
- FTP.exe

SAM.exe

Note: The file is located in the KARL STORZ folder on the c: drive (c:\program files\karl storz\aida2\).

- Using Windows Explorer, open the Autostart folder ‘C:\Documents and Settings\All Users\Start Menu\Programs\Startup’. Press the right mouse button. Select New and then Shortcut. Click on Browse and locate the folder ‘c:\program files\karl storz\aida2.’ Select SAM.EXE and click OK. Then click on Next and subsequently Finish.

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Installation and operating instructions

Config. AIDA control - combination 2

FTP.exe

Note: The file is located in the KARL STORZ folder on the c: drive (c:\program files\karl storz\aida2\).

- Add a shortcut to FTP.exe to the Autostart folder (same procedure as for SAM.exe – see previous section).

- Configure SAM as described in the AIDA 2.x installation instructions.
- Start the AIDA 2.x application, log into AIDA 2.x as administrator and set the checkmark ‚SCB task bar‘ in the tab ‚Properties‘.

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Configuring the KARL STORZ AIDA® control – combination 2

SAM.exe

- Add a shortcut to SAM.exe to the Autostart folder (same procedure as for ‘combination 1’ – see previous page).
- Start AIDA compact II, open the settings screen with CTRL+ALT+S and activate ‘SAM’ in the tab ‘General Settings’. Confirm the changes and restart AIDA compact II to activate the changes.

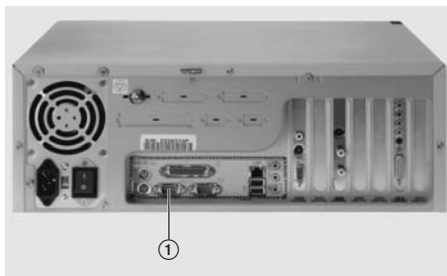
- Configure SAM as described in the AIDA installation instructions for SCB and AIDA compact II.

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1. Setting up the system

Note: Ensure that your computer and the touch screen monitor are both switched off.

Insert the serial cable into the **COM1** ① port on your computer.

Modelos 200903 01, 200903 02, 200903 03, 200903 04, 200904 01, 200904 02, 200904 03 200904 04 y 200904 05/06

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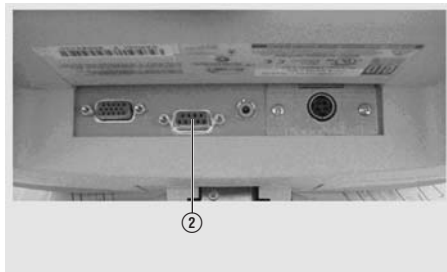
Models 200903 01 and 200903 02

Connect the KARL STORZ touch screen monitor to your computer by inserting the serial cable into the touch screen monitor port **RS232** ②.

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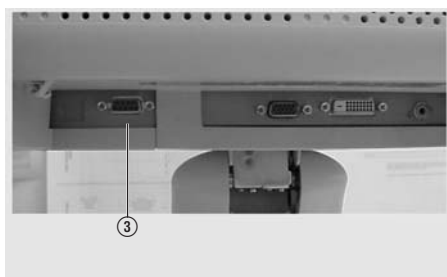
Models 200903 03 and 200903 04

Connect the KARL STORZ touch screen monitor to your computer by inserting the serial cable into the touch screen monitor port **RS232** ③.

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Models 200904 01, 200904 02, 200904 03, 200904 04 and 200904 05/06

Connect the KARL STORZ touch screen monitor to your computer by inserting the serial cable into the touch screen monitor port **RS232** ④.

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2. Installing the driver

When using AIDA or AIDA compact with the KARL STORZ touch screen monitor model **200903 01, 200903 02, 200903 03, 200903 04, 200904 01, 200904 02, 200904 03, 200904 04** or **200904 05/06** a driver must be installed additionally.

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Al utilizar AIDA o AIDA compact con el monitor de pantalla táctil KARL STORZ, modelo **200903 01, 200903 02, 200903 03, 200903 04, 200904 01, 200904 02, 200904 03, 200904 04** o **200904 05/06**, debe instalarse adicionalmente un controlador.

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Exit AIDA or AIDA compact if this application is running on your computer.

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Caution: When working with computers, it is always possible for problems to occur which result in a loss of data. For this reason, a complete data backup is recommended before starting any work.

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Driver version V3.3.0

Check that you are registered as a Windows® administrator for the next steps. When registering in Windows, enter **Administrator** as your user name and **Admin** as the password.

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Either insert the driver CD enclosed with the monitor into the CD-ROM or DVD drive, finding the driver in the directory ...\\drivers\\ELOW_330, or use the setup program in the directory C:\\drivers\\ELOW_330\\SetupELO or C:\\drivers\\ELOW_330\\SW600390.

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Double click on the file **SetupELO.exe**. The window **Elo MonitorMouse for Windows XP Setup** opens.

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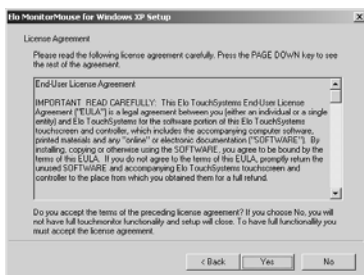
Click on the **Next** button in the **Elo MonitorMouse for Windows XP Setup** window.

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
You accept the license conditions by clicking on the **Yes** button.

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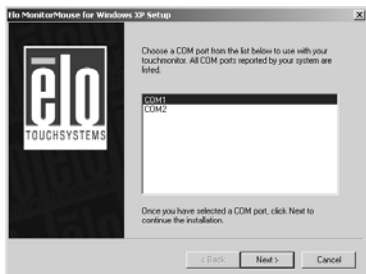
Select the **COM1** port and click on the **Next** button.

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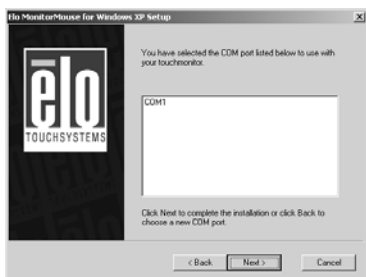Continue installation by clicking on the **Yes** button.

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Conclude the installation procedure by clicking on the **Finish** button.

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Confirm the prompt asking whether you wish to reboot your computer by clicking on the **Yes** button. The KARL STORZ touch screen monitor driver is installed.

Note: If the touch function of your touch screen monitor does not work, the touch screen monitor serial cable must be inserted into the COM1 port of your computer, also see the pictures on page 11 or 12.

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Select **Start/Settings/Control Panel**.

The System Settings window will open.

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Double click on **EloTouchscreen**.

The **Elo Touchscreen Properties** window will open.

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Confirm the settings by clicking on the **OK** button.

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Driver version V4.7.0

Check that you are registered as a Windows® administrator for the next steps.

Call up the following directory:
C:\Drivers\ELO

As an alternative, you can find the necessary data at **E:\ELO_2Touch** on the enclosed driver CD in your CD-ROM drive.

Note: E:\ corresponds to the drive letter of the CD-ROM drive (this depends on your computer configuration).

Driver version V4.7.0

Check that you are registered as a Windows® administrator for the next steps.

Call up the following directory:
C:\Drivers\ELO

As an alternative, you can find the necessary data at **E:\ELO_2Touch** on the enclosed driver CD in your CD-ROM drive.

Note: E:\ corresponds to the drive letter of the CD-ROM drive (this depends on your computer configuration).

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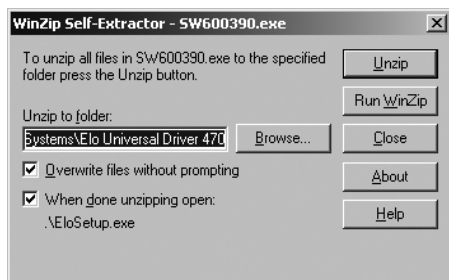
Double click on the file **SW600390**. The **WinZip Self-Extractor** window will open.

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In the **WinZip Self-Extractor - SW600390.exe** window, click on the **Unzip** button.

Confirm the window which opens after extraction by clicking on the **OK** button.

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In the **Elo Touchscreen Setup (Version 4.7.0)** window, select the language you require and then click on the **Next** button.

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Activate the first option **Install Serial Touchscreen Drivers**, to select the drivers for the serial port. Click on the **Next** button.

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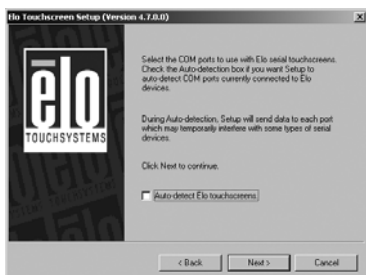
Accept the license conditions by clicking on the **Yes** button.

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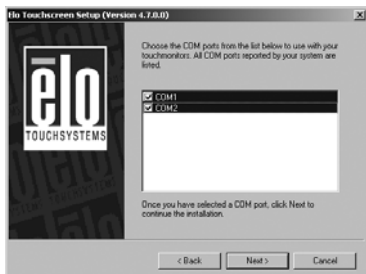
If you have only connected one touch screen, activate the **COM1** option.

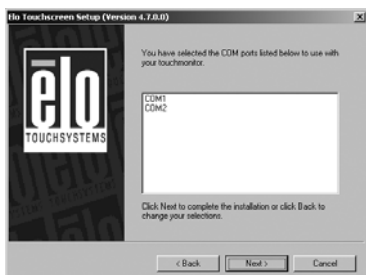
If you have connected two touch screens, activate the **COM1** and **COM2** options.

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Click on the **Next** button.

The installation procedure starts.

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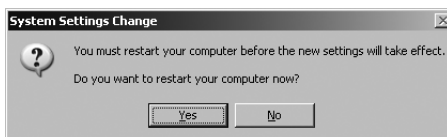
After the procedure has been completed, confirm the installation by clicking on the **Finish** button.

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Confirm the prompt asking whether you wish to reboot your computer by clicking on the **Yes** button. The KARL STORZ touch screen monitor driver is now installed.

Note: If the touch function of your touch screen monitor does not work, check whether the serial cable of the touch screen monitor is inserted in the corresponding port (COM1 oder COM2) on your computer.

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3. Special settings in connection with AIDA Interface

Driver version V3.3.0 with AIDA Interface Unit
Model 20091132

Note: In order to use the touch screen monitor together with the AIDA interface unit, it is necessary to edit the registration file. This step should only be performed by experienced computer users. We also recommend backing up the registration file beforehand.

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
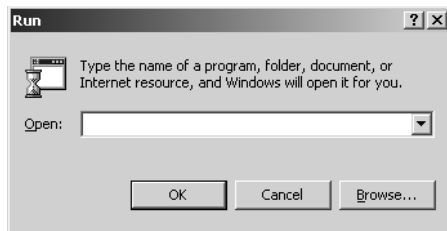
Select **Start/Run**. The **Run** window opens.

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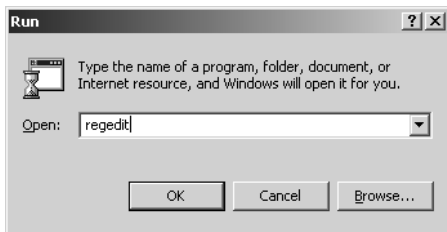
Enter *regedit* in the **Open:** line and click on the **OK** button.

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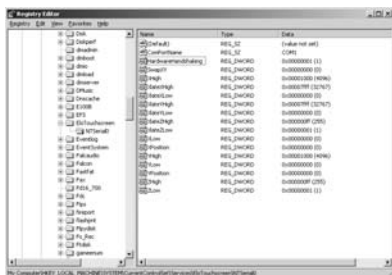
Select the folder **HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Services\EloTouchscreen\NTSerial0** in the **Registry Editor** window.

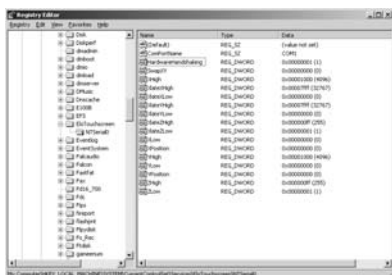
Seleccione en la ventana **Editor de registraci3n** la carpeta **HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Services\EloTouchscreen\NTSerial0**.

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Double click on the entry **HardwareHandShaking**.

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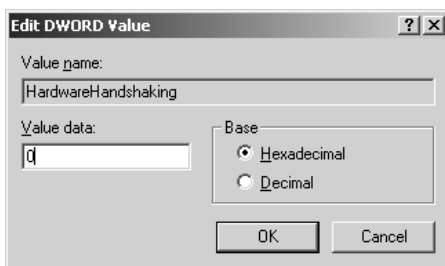Set the value to **0** and confirm this setting by clicking on the **OK** button in the **Edit DWORD Value** window.

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Exit the **Registry Editor** and reboot your computer.

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**Driver version V4.7.0 with AIDA Interface
Unit II Model 20092132**

Note: To be able to use the touch screen monitor together with the AIDA Interface unit, it is necessary to edit the registration file. This step should only be performed by experienced computer users. We also recommend backing up the registration file beforehand.

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
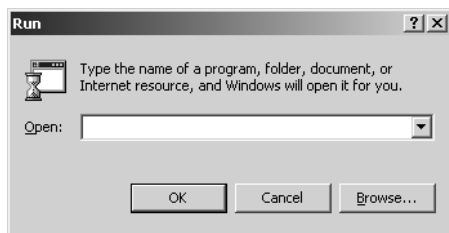
Select **Start/Run**. The **Run** window will open.

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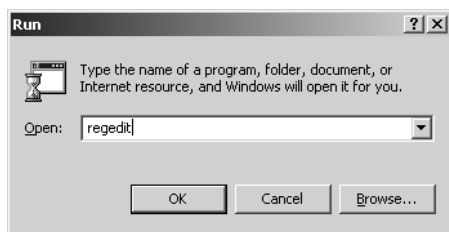
In the **Open:** field, enter *regedit* and click on the **OK** button.

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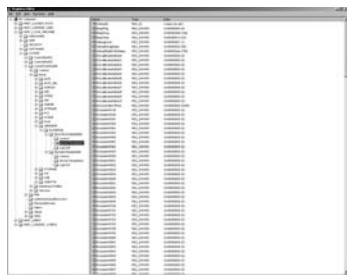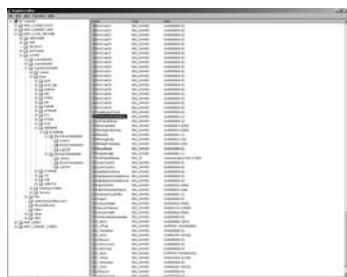
*: A separate directory (ID number) is created for each connected touch screen monitor when the driver is installed.

If you have connected two touch screen monitors, carry out the two following steps for each of the ID numbers separately.

Seleccione en la ventana **Editor del registro** la carpeta **HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Enum\SERENUM\ELOSERIAL*\Device Parameters**

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
Double click on the entry **HardwareHandshaking**.

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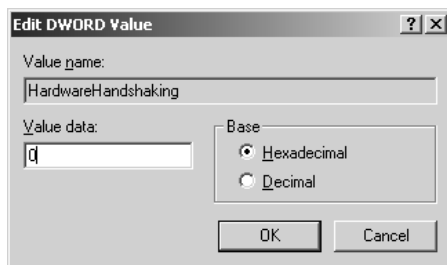In the **Edit DWORD Value** window, set the value to **0** and confirm this setting by clicking on the **OK** button.

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4. Calibrating the touch screen monitor

Driver version V3.3.0

To ensure that the touch screen monitor reacts precisely at the point/button which is touched, it must be calibrated after the driver is installed.

Note: Carry out calibration with the instrument (special pen or your fingertip) which will be used later.

You can only carry out calibration if you are registered as a Windows® administrator. As an alternative, the administrator can give you permission to carry out calibration in the registration file, as described in the following section.

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Allocation of user rights in the registration file

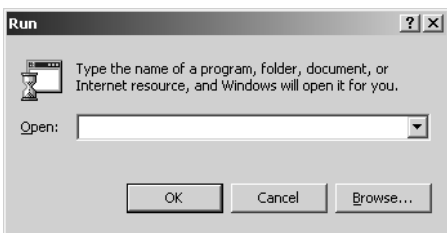
Note: Any editing of the registration file should only be performed by experienced computer users. We also recommend backing up the registration file beforehand.

Check that you are registered as a Windows® administrator for the next steps.

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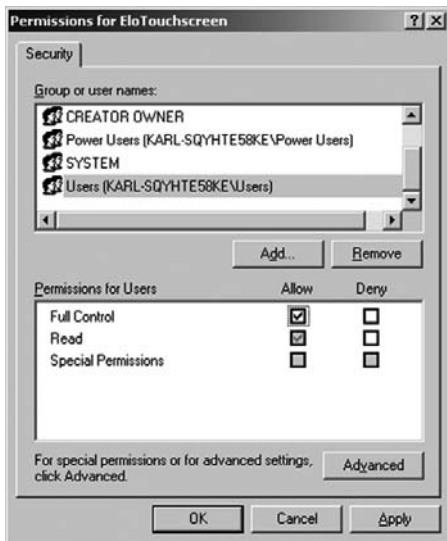
In the **Registry Editor** window, select the folder **HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Services\EloTouchscreen**.

Seleccione en la ventana **Editor del registro** la carpeta **HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Services\EloTouchscreen**.

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Click on the **EloTouchscreen** entry and select **Permissions...** in the context menu.

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In the **Permissions for Elo Touchscreen** window, select the entry **Users** under **Group or user names**.

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Tick the **Allow** column in the **Full Control** line under **Permissions for Users**.

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Confirm the setting by clicking on the **OK** button.

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Click on the entry **Schemes** and select **Permissions...** in the context menu.

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In the **Permissions for Schemes** window, select the entry **Users** under **Group or user names**.

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Close the Registry Editor and reboot the system.

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Calibration

After rebooting your computer, restart the monitor settings via **Start/Settings/Control Panel/Elo Touchscreen**.

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Click on the **Align** button. The window for calibrating the monitor opens.

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Touch the three points which appear one after the other on the screen using the pen enclosed with the touch screen monitor or your finger.

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Confirm the settings by clicking on the **Yes** button in the **Check Video Alignment** window.

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Driver version V4.7.0

To ensure that the touch screen monitor reacts precisely at the point/button which is touched it must be calibrated after the driver is installed.

Note: Carry out calibration with the special pen or your fingertip, whichever will be used later.

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Note: If you have installed two touch screens, they must be calibrated consecutively.

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Confirm the settings by clicking on the green tick button in the **Check Video Alignment** window.

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Cleaning and care of the unit

Warning: Always unplug the computers before cleaning!

Caution: It is essential to prevent any penetration of liquid into the housing.

The exterior of the unit can be surface disinfected with a disposable cloth soaked in disinfectant (preferably alcohol-free).

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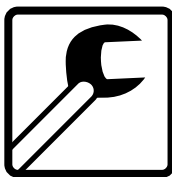
Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, thus enhancing the instrument's reliability and extending its useful operating life.

Maintenance services can be obtained from your local representative or from the manufacturer.

Regardless of the accident prevention regulations or testing intervals for medical devices prescribed in different countries, we recommend a functional or safety test of the unit at least once a year.

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Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original KARL STORZ parts only.

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Disposal

This unit has been marked in accordance with the European Directive 2002/96/EC on waste electrical and electronic equipment (WEEE).

At the end of its useful operating life, dispose of the unit as electronic scrap.

Please ask either KARL STORZ GmbH & Co. KG, a KARL STORZ subsidiary or your specialist dealer for information on your local collection point.

Within the scope of application of this Directive, KARL STORZ GmbH & Co. KG is responsible for the proper disposal of this unit.

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Repair program

Usually to bridge the repair period, you will receive a unit on loan which you then return to KARL STORZ as soon as you receive the repaired unit.

In Germany you can refer repairs direct to

KARL STORZ GmbH & Co. KG

Abt. Reparaturservice

Dr. Karl-Storz-Str. 34

78532 Tuttlingen.

In other countries please contact your local KARL STORZ branch or authorized dealer.

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Important information

In order to prevent infection, it is strictly forbidden to ship contaminated devices. All devices must be decontaminated on site to avoid contact and aerogenous infections (among staff). We reserve the right to return contaminated instruments/equipment to the sender.

Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all guarantee rights.

KARL STORZ gives no guarantee on the correct functioning of equipment or instruments which have been repaired by unauthorized third parties.

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Limitation of liability

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- the electrical installation of the location in which the device is connected and operated conforms to the applicable statutes and standards, and
- the unit has been used in accordance with its operating instructions at all times.

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Manufacturer's warranty

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In so doing we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply. Please fill out the attached warranty card on the last page and return it as soon as possible to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Germany

Note: The unit must always be sent to the following address (also during the warranty period; where applicable, with warranty card):
KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Str. 34
78532 Tuttlingen/Germany

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Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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Troubleshooting

Warning: Always unplug AIDA control and OR1™ control before carrying out any maintenance work on the unit!

Symptom:

- Monitor stays dark.

Possible causes:

- OR1™ control or AIDA control is not running.
- AC adapter is not connected
- Connection to the control computer interrupted.

Remedy:

- Check PC interconnection cable between **AIDA** INTERFACE II and OR1™ control and AIDA control.
- Check monitor connections ⑧, ⑩, ⑭ (see pages A 1/2).

Symptom:

- AIDA does not register on the SCB system.

Possible causes:

- AIDA control is not running.
- The network connection between OR1™ control and AIDA control interrupted.
- KARL STORZ Application Manager (SAM.exe) has either not been started or is incorrectly configured.

Remedy:

- Check whether AIDA control is running.
- Check the network connection.
- Check whether SAM has been started and is correctly configured.

Note: To check these points, the AIDAInterface program (-> Maintenance Manager -> Start other KARL STORZ programs -> AIDA Interface II) can be started on OR1™ control. Provided that the console switchover functions, you can change to AIDA control and check the above points.

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Symptom:
The error message, Network resource 192.168.1.10/SCB not available, appears on 'OR1™ control'.
Possible causes:
FTP server not started or wrongly configured.
Remedy:
Check whether the FTP server has been started and whether it is correctly configured.

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Symptom:
AIDA registers on the SCB system, but the console switchover to AIDA does not function, or the console is always switched to AIDA or SCB.
Possible causes:
Connection error. Keyboard cable is wrongly inserted in the AIDA control/OR1™ control.
Remedy:

- Check the AIDA control/OR1™ control connection to **AIDA** INTERFACE II via the PC connecting cable (see p. 9).
- Check the OR1™ control connection to **AIDA** INTERFACE II via the parallel port connecting cable.

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Symptom:
Keyboard/mouse does not function.
Possible causes:

- Keyboard/mouse wrongly connected.
- Cabling wrong.

Remedy:
Check the connection of the PC connecting cable to the sockets ⑥, ⑦, ⑪, ⑭ (see p. 9).

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Standard compliance (for 20092132)

According to EN 60950:

Type of protection against electric shocks:
Protection Class III

According to IEC 60601-1-2:

Please read the Electromagnetic Compatibility
Information in the appendix (p. 25-38).

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Directive compliance (for 20092132)

This product bears the CE sign in accordance
with RICHTLINIE 2006/95/EC.

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Technical documentation

On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified user personnel duly authorized by the manufacturer for their use in repairing those components of the unit that have been designated by their respective manufacturers as repairable.

Supply of such technical documentation relating to the unit shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the unit.

Explicitly exempted herefrom are those maintenance and repair operations described in this manual.

We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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Appendix**Electromagnetic Compatibility
(EMC) Information**

WARNING: Medical electrical equipment needs special precautions regarding Electromagnetic Compatibility (EMC). Since the AIDA Interface II model 20092132 is also operated in the immediate vicinity of medical devices, conformity with EN/EC 60601-1-2 has also been checked in this respect. Observe the EMC instructions in this appendix during installation and commissioning.

The **AIDA** INTERFACE II model 20092132 corresponds to EN/IEC 60601-1-2 :2001 [CISPR 11 Class B] and therefore meets the EMC requirements of the Medical Device Directive (MDD) 93/42/EEC.

These limits are designed to provide reasonable protection against the typical electromagnetic interference to be expected in a medical environment. The **AIDA** INTERFACE II model 20092132 is a Group 1 unit (as per CISPR 11). Group 1 contains all the 'equipment and systems which generate or use RF energy only for their internal functioning'.

NOTE: The tables and guidelines that are included in this appendix provide information to the customer or user that is essential in determining the suitability of the equipment or system for the electromagnetic environment of use, and in managing the electromagnetic environment of use to permit the equipment or system to perform its intended use without disturbing other equipment and systems or non-medical electrical equipment. If this equipment does cause harmful interference with other devices, the user is encouraged to try to correct the interference by one or more of the following measures:

- adjust direction or move to another location
- increase the separation between the equipment
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.

If you have any further questions, please contact your local representative or our service department.

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WARNING: The **AIDA** INTERFACE II model 20092132 should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

WARNING: The use of portable and mobile RF equipment may have an impact on this and other pieces of medical equipment.

WARNING: The use of accessories and cables other than those specified in the KARL STORZ instruction manual may result in increased emissions or decreased immunity of the **AIDA** INTERFACE II model 20092132.

The accessories and cables listed below have been shown to comply with the requirements of EN/IEC 60601-1-2. When using accessories and cables other than those specified here, it is the responsibility of the user to ensure that they comply with EN/IEC 60601-1-2.

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Table 200				
Accessories and cables which have been shown to comply with EN/IEC 60601-1-2:				
Type	Shielded	Length [m]	Ferrite	Used for
PA	No	> 3	No	Potential equalization
RS232	Yes	1.8	No	Touch Screen
Monitor	Yes	1.8	No	Monitor
Mouse/Keyboard	No	1.8	No	Mouse/Keyboard
20 0903 74	Yes	1.8	No	PC interconnection cable from KARL STORZ AIDA [®] control
20 0903 74	Yes	1.8	No	PC interconnection cable from KARL STORZ OR1 [™] control
20 0904 75	Yes	1.8	No	Parallel port cable from KARL STORZ OR1 [™] control (printer input)

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Table 201		
Guidance and manufacturer's declaration – electromagnetic emissions		
<p>AIDA INTERFACE II model 200921 32 is intended for use in the electromagnetic environment specified below. The customer or user of the AIDA INTERFACE II model 200921 32 should ensure that it is used in such an environment.</p>		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	AIDA INTERFACE II model 200921 32 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The AIDA INTERFACE II model 200921 32 is suitable for use in all establishments (e.g. hospitals and doctors' offices) including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

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Table 202 Guidance and manufacturer's declaration – electromagnetic immunity			
AIDA INTERFACE II model 200921 32 is intended for use in the electromagnetic environment specified below. The customer or user of the AIDA INTERFACE II model 200921 32 should ensure that it is used in such an environment.			
Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	Complies ± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Complies ± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Complies ± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T * (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	Complies <5% U_T * (>95% dip in U_T) for 0.5 cycle Complies 40% U_T (60% dip in U_T) for 5 cycles Complies 70% U_T (30% dip in U_T) for 25 cycles Complies <5% U_T (>95% dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment or system requires continued operation during power mains interruptions, it is recommended that the equipment or system be powered from an uninterruptible power supply.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	Complies 3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
* Note: U_T is the a.c. mains voltage prior to application of the test level.			

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Table 204

**Guidance and manufacturer's declaration – electromagnetic immunity –
for equipment and systems that are not life-supporting**

AIDA INTERFACE II model **200921 32** is intended for use in the electromagnetic environment specified below.
The customer or user of the AIDA INTERFACE II model **200921 32** should ensure that it is used in such an environment.

Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 V_{rms} 150 kHz to < 80 MHz</p> <p>3 V/m 80 MHz to 2.5 GHz</p>	<p>3 V_{rms}</p> <p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the AIDA INTERFACE model 200911 32, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance:</p> $d = [3.5/3]\sqrt{P} \text{ 150 kHz to } < 80 \text{ MHz}$ $d = [3.5/3]\sqrt{P} \text{ 80 MHz to } < 800 \text{ MHz}$ $d = [7/3]\sqrt{P} \text{ 800 MHz to } 2.5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts [W] according to the transmitter manufacturer and d is the recommended separation in meters [m]. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b. Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection by structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio [cellular/cordless] telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the unit is used exceeds the applicable RF compliance level above, the unit should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the unit.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

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Table 206

Recommended separation distances between portable and mobile RF communications equipment and the AIDA INTERFACE II model 200921 32

The AIDA INTERFACE II model 200921 32 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the unit can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the unit as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter [W]	Separation distance d [m] according to frequency of transmitter		
	150 kHz to < 80 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	80 MHz to < 800 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{3} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance **d** in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where **P** is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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WARRANTY

For a period of **two** years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship.

In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Germany

To be completed by the supplier/importer:
Company stamp/signature:

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Tipo del aparato: Número de serie: _____

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Firma/Fecha: _____

To be filled out by instrument owner:
Return address/company stamp:

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KARL STORZ Endoscopy South Africa, (Pty) Ltd.
8th Floor Convention Tower
Cnr Heerengracht & Coen Steytler, Foreshore
Cape Town 8001, South Africa
P.O.Box 6061, Roggebaai Cape Town, 8012, South Africa
Phone: +27 21 417 2600, Fax: +27 21 421 5103
E-Mail: info@karlstorz.co.za

TOO KARL STORZ ENDOSCOPY Kasachstan
Khodjanova 17
050060 Almaty, Kazakhstan
Phone/Fax: +7 72 72 49 43 63, +7 72 72 49 41 00
e-mail: KarlStorzKazakhstan@itite.kz

KARL STORZ Endoscopy India Private Ltd.
C-126, Okhla Industrial Area, Phase-1
New Delhi 110020, India
Phone: +91 11 26815445-51, Fax: +91 11 268129 86
E-Mail: karlstorz@vsnl.com

KARL STORZ GmbH & Co. KG
Resident Representative Office
80/33 (44/19) Dang Van Ngu,
F.10-Q. Phu Nhuan
Ho Chi Minh City, Vietnam
Phone: +848 991 8442, Fax: +848 844 0320

KARL STORZ Endoscopy (S) Pte. Ltd.
#05 – 08 San Centre, 171 Chin Swee Road
Singapore 0316, Singapore
Phone: +65 6376-1066, Fax: +65 6376-1068
E-Mail: karlstorz@pacific.net.sg

KARL STORZ Endoscopy Australia Pty. Ltd.
174 Parramatta Rd.
Camperdown, N.S.W. 2050, Australia
Phone: +61 02 8594 9150, Fax: +61 02 8594 9199
E-Mail: info@karlstorz.au

KARL STORZ Endoscopy China Ltd.
Hong Kong Representative Office
Unit 1601, Chinachem Exchange Square
1 Hoi Wan Street, Quarry Bay, Hong Kong,
People's Republic of China
Phone: +8 52 28 65 2411, Fax: +8 52 28 65 4114
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Beijing Representative Office
Room 610, China Life Tower
No. 6, Chaowai Street
Beijing, 100020, People's Republic of China
Phone: +86 10 8525 3725, Fax: +86 10 8525 3728
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Unit 3901-3904, Tower 1 Grand Gateway
No. 1 Hong Qiao Road
Shanghai, 200030, People's Republic of China
Phone: +86 21 6113-1188, Fax: +86 21 6113-1199
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Chengdu Representative Office
F-5, 24/F., Chuanxing Mansion,
No. 18 Renming Road South
Chengdu, Sichuan, 610016, People's Rep. of China
Phone: +86 28 8620-0175, Fax: +86 28 8620-0177
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Guangzhou Representative Office
Room 1119-20, Dongshan Plaza
69 Xianlie Road Middle, Dongshan District,
Kuangzhou, Guangdong, 510095, People's Rep. of China
Phone: +86 20 8732-1281, Fax: +86 20 8732-1286
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy Japan K. K.
Bois Hongo Building 6F1, 3-42-5 Hongo
Bunkyo-ku, Tokyo 113-0033, Japan
Phone: +81 3 5802-3966, Fax: +81 3 5802-3988
E-Mail: info@karlstorz.co.jp

www.karlstorz.com

STORZ
KARL STORZ—ENDOSKOPE

KARL STORZ GmbH & Co. KG

Mittelstraße 8
78532 Tuttlingen

Postfach 230
78503 Tuttlingen
Germany

Telefon: +49 (0)7461 708-0
Telefax: +49 (0)7461 708-105
E-Mail: info@karlstorz.de
Web: www.karlstorz.com