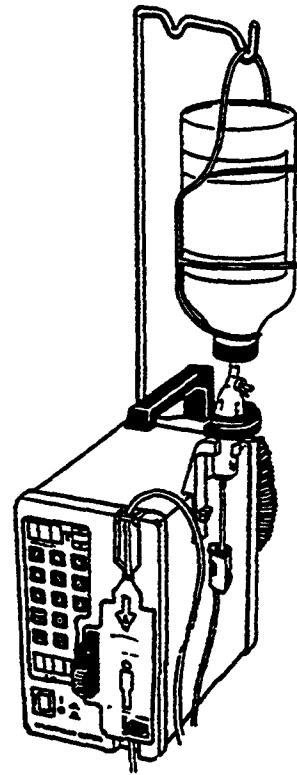
# Service-Manual

## INFUSOMAT<sup>®</sup>-secura [999ml/h]

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## **General Information**

Please pay attention to the following information:

### **Servicing**

This service manual is only intended for your information. The possession of this manual does not authorize servicing.

Servicing may only be performed if:

- you are trained on the respective unit by B. Braun,
- registered for update service,
- possess the necessary test equipments and tools and
- fulfill the following personal prerequisites (education, knowledge)

### **Technical Safety Inspections**

B. Braun also recommends attending a training or at least perform the safety inspection according to the current manual version, because:

- the technical safety inspection demands observing the procedure information in this service manual,
- the service manual also represents a reference for measurements,
- the service program must be activated depending on the unit types. This can lead to dangerous unit conditions if handled improperly, besides a special service adapter is required to activate the service program.

### **Current Service Manual**

This service manual represents the current status at the time of issue and is subjected to technical amendments, especially software. The change status of the service manual is indicated by an index number on the front page.

### **Update Service**

The possession of this service manual does not automatically include an update service.

Registration for update service by:

- attending a B. Braun Melsungen technical training or
- written order to the B. Braun sales department (against costs).

### **Exclusion of Liability**

B. Braun Melsungen AG excludes all liabilities for personal, property or other injuries, which are caused by

- using a false service manual or a service manual which does not correspond to the current edition, during maintenance, repair and servicing of the unit,
- service technicians not registered for update service,
- service technicians not attending a technical training by B. Braun Melsungen AG for the respective unit.

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The INFUSOMAT®secura has been designed in accordance with the latest technology and conforms to International Standard IEC 601-1.

The INFUSOMAT®secura is designed for long-term infusion of large volumes with high accuracy.

The INFUSOMAT®secura is suitable for intravenous/intraarteriel infusions and for blood transfusions.

Opening the pump door activates a mechanical safety clamp on the infusion tube to interrupt flow.

The electronics circuits are designed as a redundant system including two different, independent operating microprocessors controlling all functions. Malfunctions will be detected immediately and create a safe condition.

The INFUSOMAT®secura air monitoring system is based on ultrasonic. Therefore the functioning of the air monitoring system is not impaired by non-transparent solution (e.g. fat) or non-transparent tubing.

A connection of INFUSOMAT®secura to staff call systems is possible.

The INFUSOMAT®secura has integrated accumulators (NiCad) and can be operated independent from mains. In case of power failure the INFUSOMAT®secura automatically switches to battery operation.

Connected to mains the accumulator will be charged independent whether the unit is switched on or off.

The INFUSOMAT®secura is equipped with an interface (RS 232 C) and prepared for connections to external computers.

**Safety checks must be performed at 12-monthly intervals in accordance with § 11 of the Provision Regulating the Safety of Medico-Technical Equipment (Medizin-Geräte-Verordnung – MedGV).**

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### Caution: Follow the instructions for use (operator's manual)

- The INFUSOMAT®securā meets the safety requirements of the International Standard IEC 601-1 for electromedical equipment.

The INFUSOMAT®securā has passed the "Bauartzulassung" (government test procedure) as required in MedGV (Provision of regulating the safety of medico-electrical equipment).

- The INFUSOMAT®securā may only be operated in medically used rooms where the installation is in compliance to VDE 0107 respectively to National Requirements (e. g. IEC-Requirements).
- The INFUSOMAT®securā may not be operated in presence of flammable anaesthetics.
- The INFUSOMAT®securā may only be operated with "Original-INFUSOMAT-Disposables". The "Original-INFUSOMAT-Disposables" are intended for single use and have to be changed at least after 24 hours.

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- Interconnection of the INFUSOMAT®secura to computers.

Within the area the MedGV is operative combination of INFUSOMAT®secura with devices via the interface must be approved by the appropriate authority. The owner of that combination has to apply for a special regulation according to § 8, 1 MedGV. Particular documentation on request.

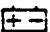
- The manufacturer considers himself responsible for the effects on safety, reliability and performance of the equipment only if:  
assembly operations, extensions readjustments, modifications or repairs are carried out by persons authorized by him and the electrical installation of the relevant room complies with IEC/VDE requirements, and the equipment is used in accordance with the instructions for use.
- Note according to MedGV  
The INFUSOMAT®secura may only be operated with accessories, spare parts and disposables which are proofed in view of safety by that appropriate authority which is allowed to proof the usable device.

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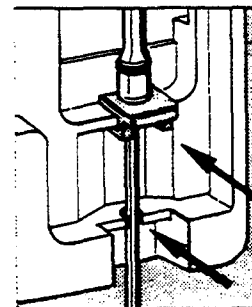
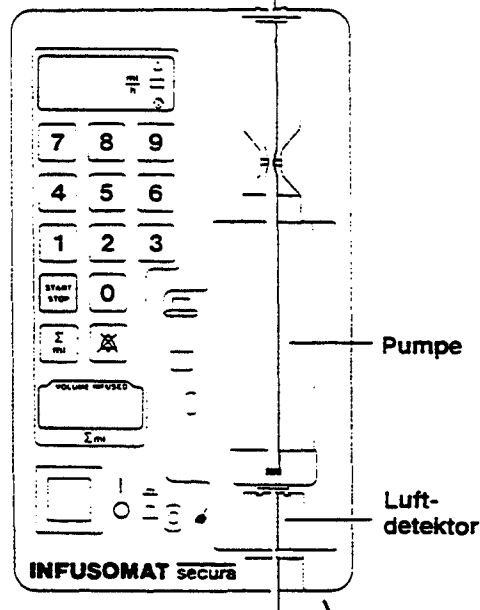
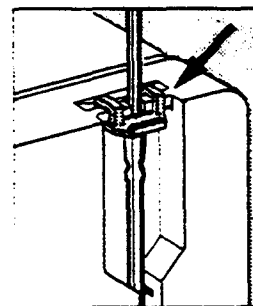
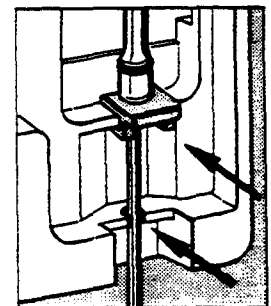
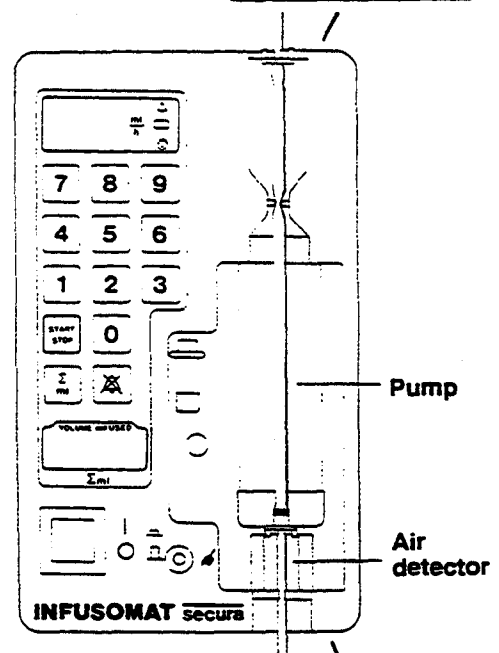
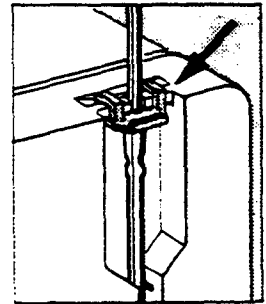
## INFUSOMAT<sup>®</sup> secura

### Short Operation Instruction

(For complete informations please refer to the Operating Instructions Manual)

1. Fill and deaerate the infusion giving set in the usual manner. **(Use only approved infusion-sets).** Close roller clamp and make connection to the patient.
2. Place flow detector on drop chamber.
3. Install giving set in accordance with illustrations. **Attention:** The reducer adapter must be positioned exactly. Close the pump door and open the roller clamp.
4. Turn on Mains switch. (There will be about a 5 second delay while the internal safety Monitor program activates.)
5. Check battery condition. Battery test push button  is located on the back panel. Yellow Monitor lamp  will light: Capacity min. 30 minutes
6. Select dosage rate 1–999 ml/h.
7. Select dosage to be given (Button  $\Sigma$  ml/0–9999 ml).
8. Start equipment with “Start” button. (Pump runs when the yellow operating lamp  illuminates.)
9. To stop equipment (e.g.: to change bottles) push “Stop” button.
10. **Alarm tone muting**  
The alarm tone can be muted for 2 minutes by pushing button 
11. **Taking equipment out of operation**  
Turn off Mains switch. (If possible keep Mains cable plugged in, so that the battery is recharged to full capacity.)


### Alarm Indicators

- |  |  |
|--|--|
| A40 = Flow sensor<br>(i. e.: Bottle empty) | AAA = Flow sensor still on<br>+ handle cradle or   |
| A50 = Air in the system                    |  Equipment defect |
- occlusion pressure 150 kPa (1125 mmHg)

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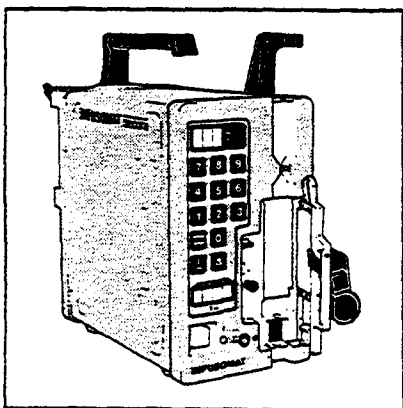
Unpack the unit and check for completeness or damage.

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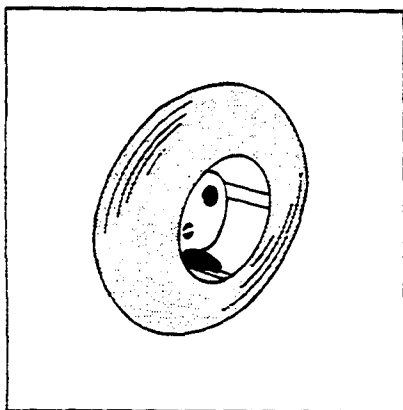### Important!

In case of damage do not start operations. Inform the authorized service station.

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
The electrical facilities in the room where the unit is to be operated shall comply with the national standards.

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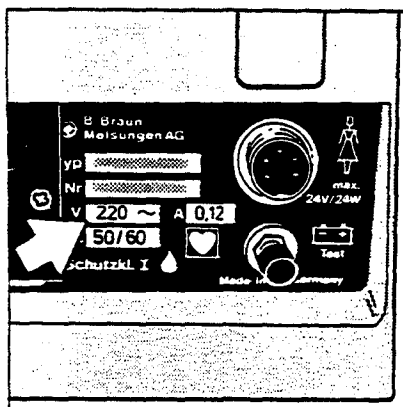The unit shall not be used in zones of risk, i.e. areas of explosive atmosphere.

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
The mains voltage available shall correspond to the mains voltage stated on the name plate.

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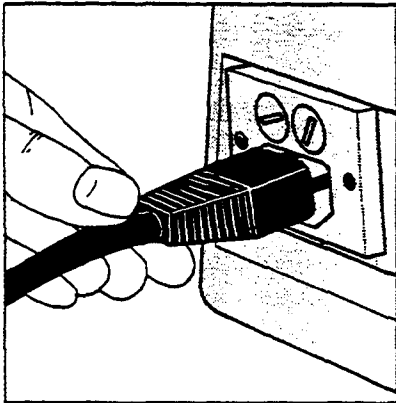
### Mains Connection

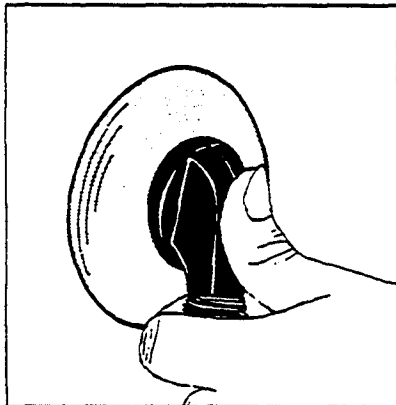Make mains connection by means of mains cable: Insert appliance plug into appliance socket pushing firmly until it is right in.

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
Insert mains plug into mains socket.

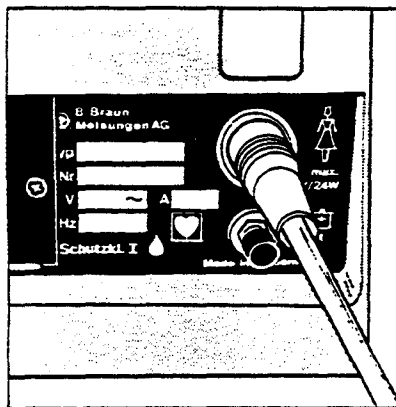Charging/mains control indicator lights up.

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
### Connection to staff call system:

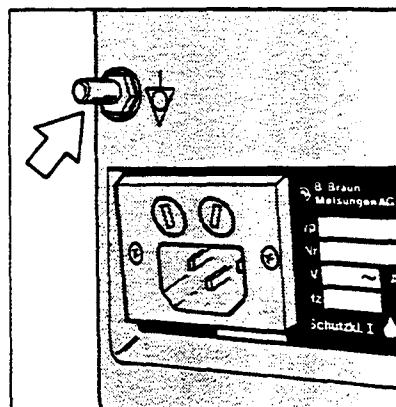Insert special cable (optional accessory) into connector.

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
CF-application: Connect potential equalization

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
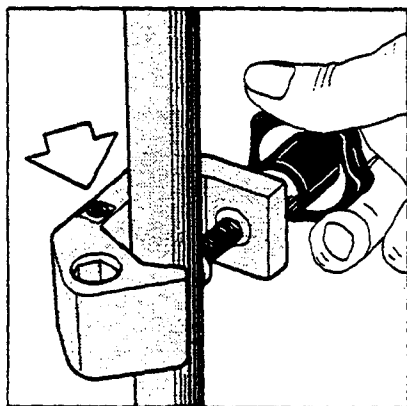
### Mounting on infusion pole

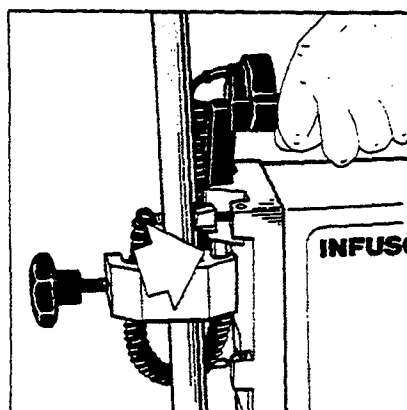Tighten universal clamp (with letters showing in upward direction) on infusion pole.

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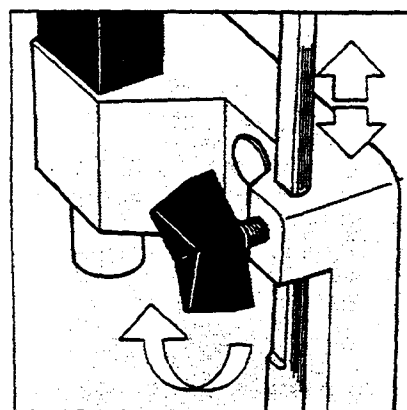Insert unit into hexagonal guide. During transportation universal clamp may remain at unit without any risk of losing it.

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
### • Operation as desk instrument and when moving patients:

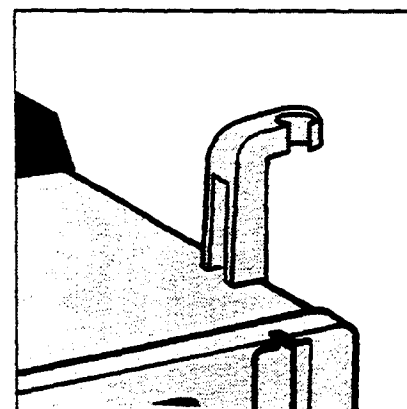Attach short support adjusting it to fit the size of bottle used.

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
Drip chamber support prevents angular positions and movement of the drip chamber during transportation.

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
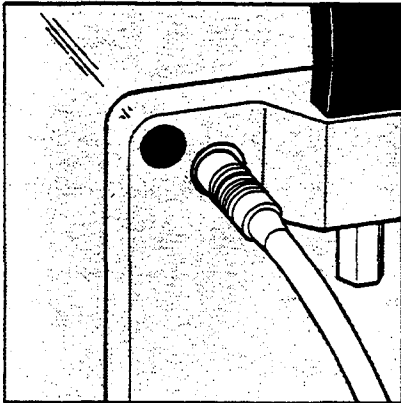
### Attaching the flow sensor:

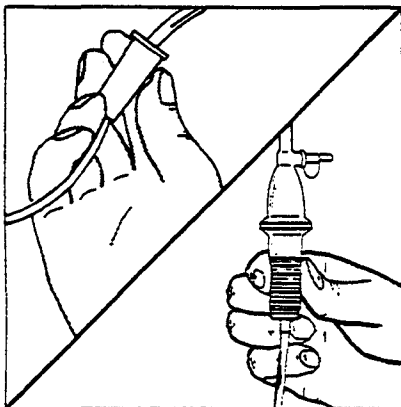Check that the plug of the flow sensor is firmly connected. Tighten screw if necessary.

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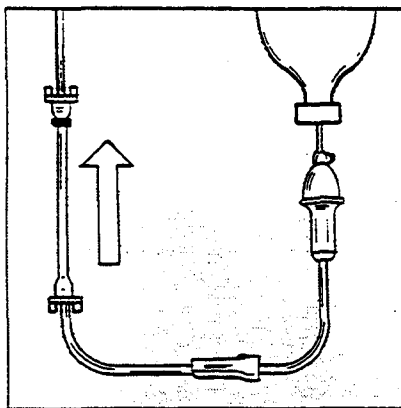
Insert drip chamber of infusion set vertically into infusion bottle. Close roller clamp. Take care that the drip chamber is filled to  $\frac{2}{3}$  only (see fig.).

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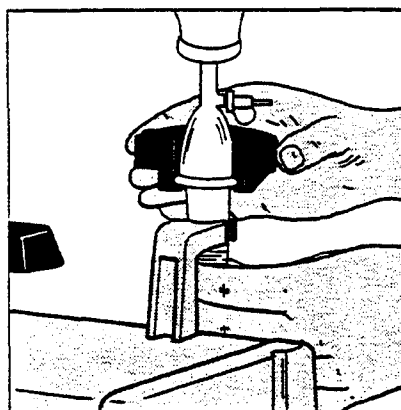Open roller clamp. Prime the infusion line and deaerate. Caution: The pump segment should be filled upside down so the air could be driven out completely. Close roller clamp.

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
Press drip chamber into holder. Slide flow sensor onto drip chamber.

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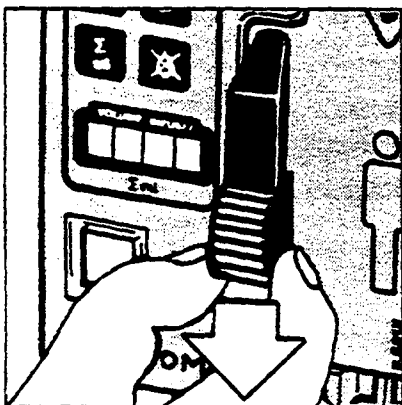
**Insert of infusion set:**  
Unlock pump door.

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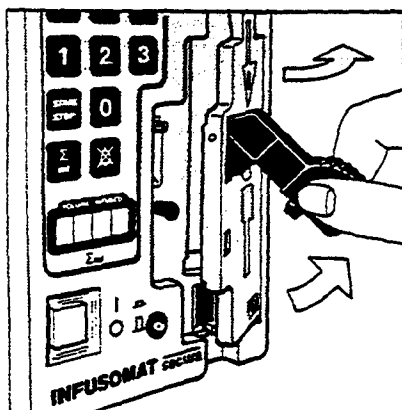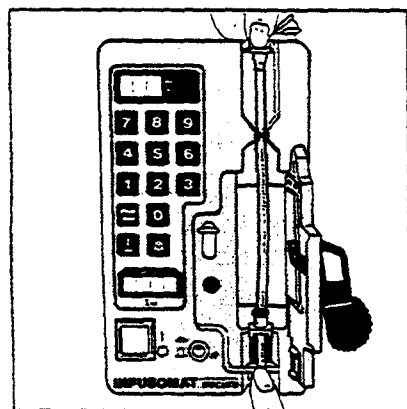
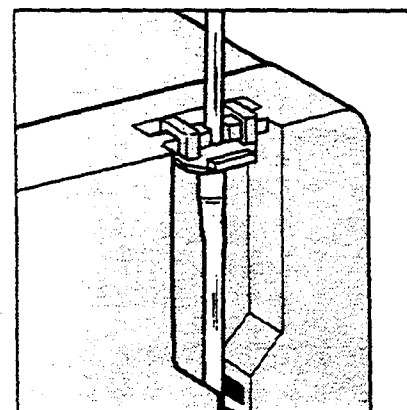
Insert pump tube into  
pumping chamber.  
Pay attention to flow  
direction!

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
Fastening clamp must be  
inserted as illustrated!

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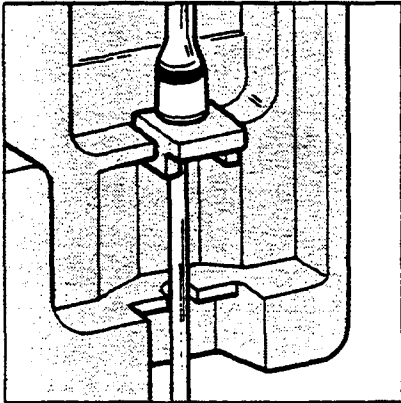
### Important!

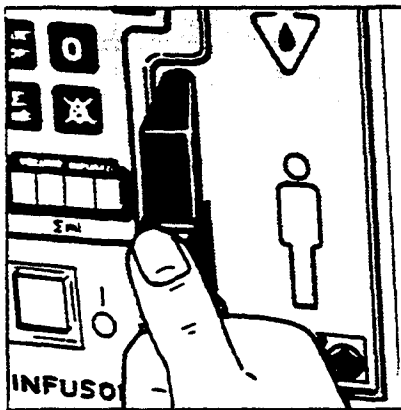Press fastening clamp and PVC-tube tightly into tube guides.

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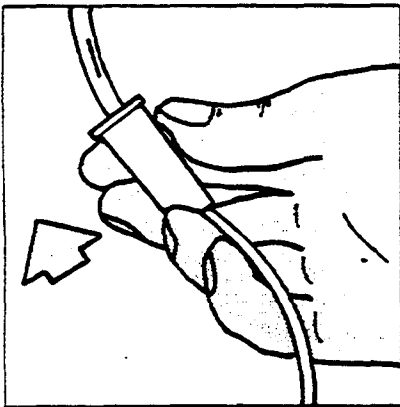
### Important!

The locking devise will click when firmly in place.

The pump segment will position itself properly when the pump door is closed.

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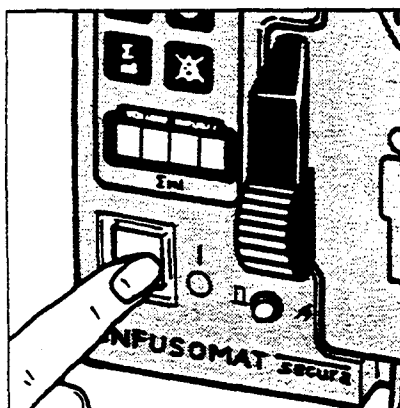
Whilst the roller clamp is open and the unit is not started no continuous dripping should occur.

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
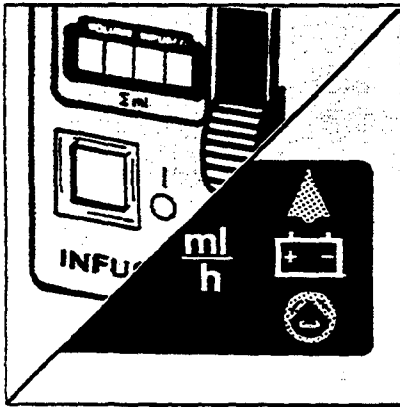
During mains operation, the green indicating lamp of the mains switch and the charging/mains control indicator are illuminated.

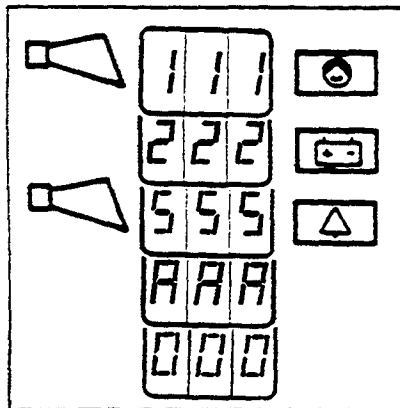During battery operation, the yellow battery control indicator is illuminated. The charging/mains control indicator goes out. Operation continues as for mains operation.

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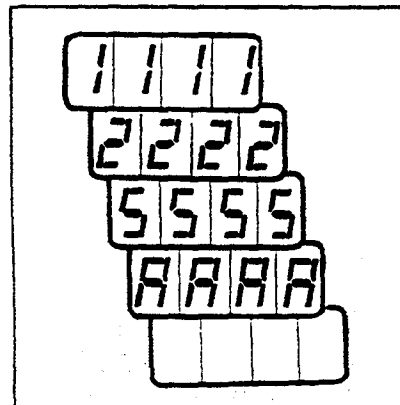
Sequence of internal safety check.

### Important!


Run through internal safety check daily.

Sequence shown in delivery rate display.

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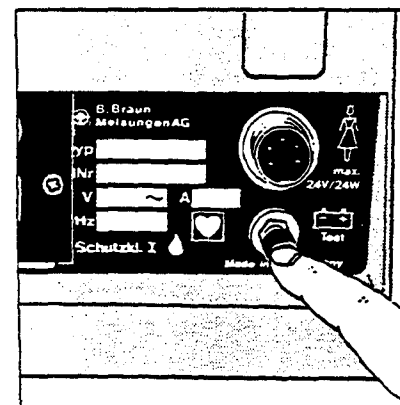
Check the capacity of the batteries.  
(push button for 5 sec max.)

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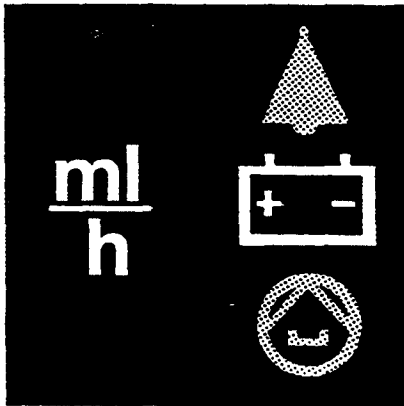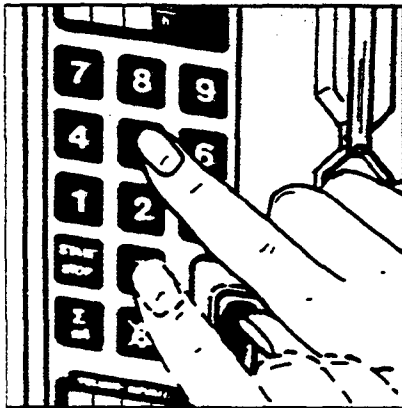
If the yellow control lamp remains lit, there is enough power for short-term battery operation and to actuate the alarm in the case of a power failure. The batteries only have full capacity if they have been recharged for at least 20 h.

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

### Input delivery rate:

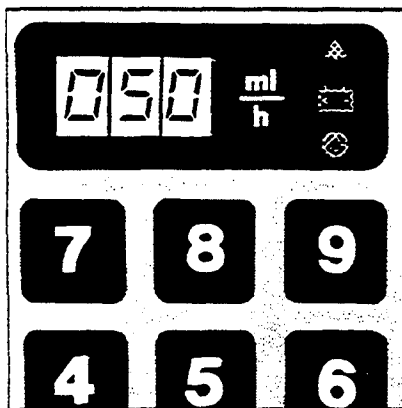Enter required delivery rate by pressing the appropriate digits between 1...999 ml/h  
(e. g. 5 and 0 = 50 ml/h)

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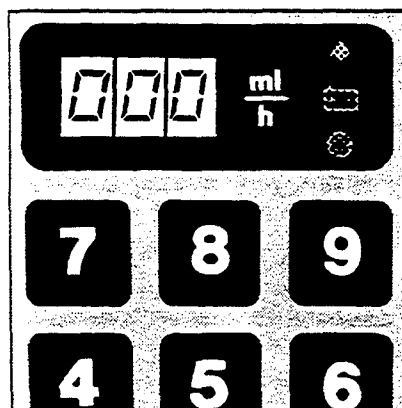
In case of error: Reset to 000 by depressing any numeral key once or up to three times.

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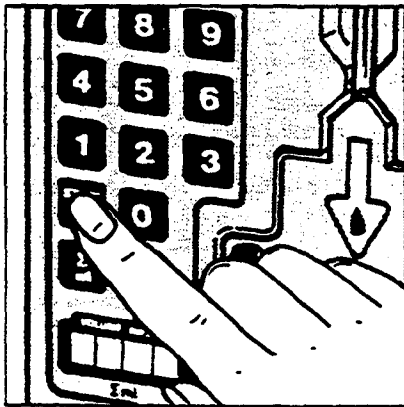
### Starting infusion pump:

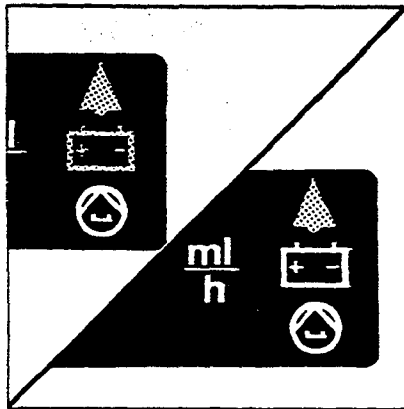Depress start/stop key.  
Pump begins operating at the delivery rate entered.

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
Simultaneously the yellow control lamp is illuminated

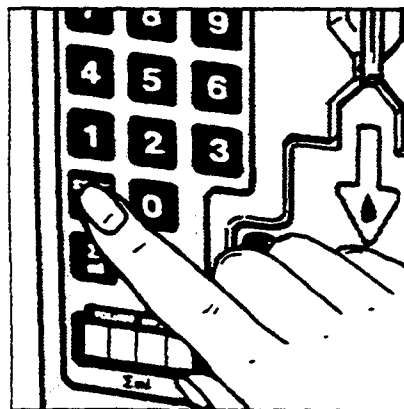With battery operation and/or mains failure, the battery control indicator is also illuminated. The charging/mains control indicator is switched off.

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### Altering the delivery rate or the total delivery quantity:

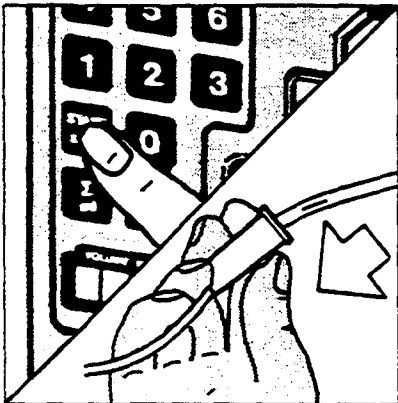
Press Start/Stop button to stop operation.

Enter the new delivery rate and/or quantity.

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
**Replace tube /  
Switching off  
infusion pump:**  
To stop depress start/stop  
key.

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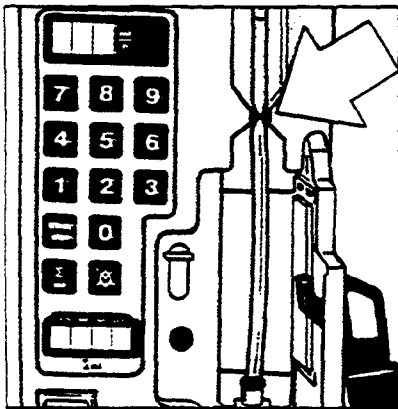
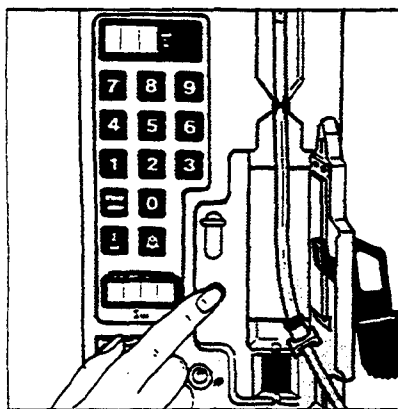
Open the door.  
The tubing is automatically  
clamped off when the pump  
door is opened.

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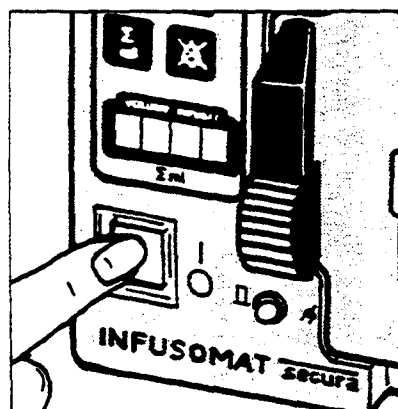Removing the tubing system:  
Push in actuating shaft and  
remove tubing from bottom  
to top.

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
Switch off by pressing  
mains switch.

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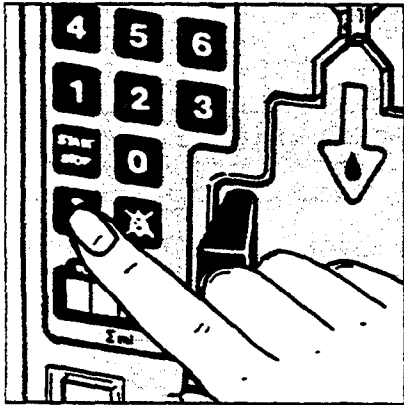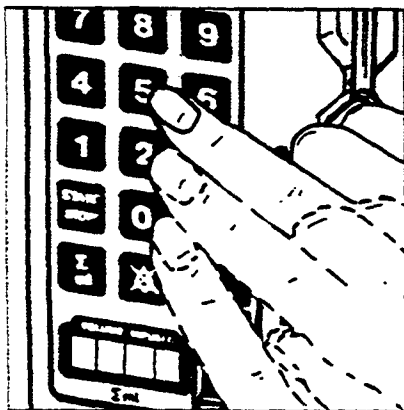
### Volume mode operation


A volume can only be preselected when operation has been stopped.

Input of total quantity to be delivered:  
Depress key  $\Sigma$  ml.

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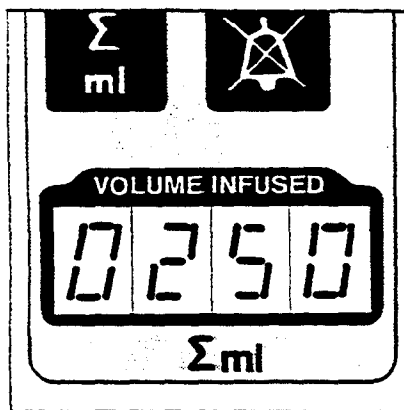
Enter required delivery volume by pressing the appropriate digits between 1...9999 ml (e.g.: 2 and 5 and 0 = 250 ml).

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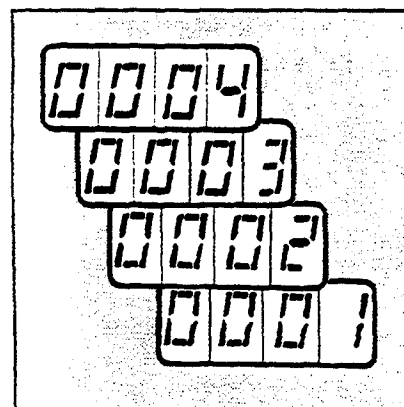
When a volume has been entered, the volume display shows a continuous record of the volume infused. (e.g. 0000 - 0001 - 0002 - 0003 - 0004)

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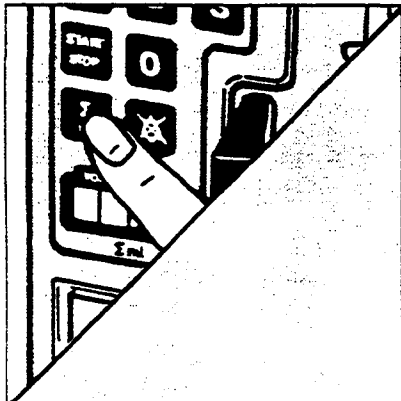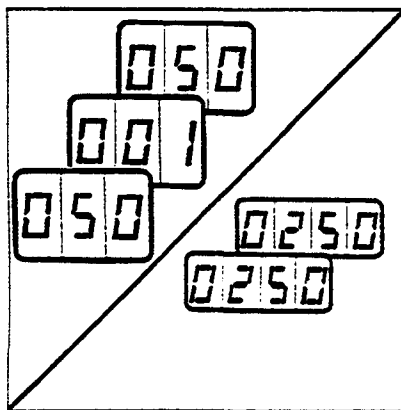
The volume originally entered may be recalled at any time by pressing the button marked  $\Sigma$  ml, e.g. 250 ml

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
### KVO operation

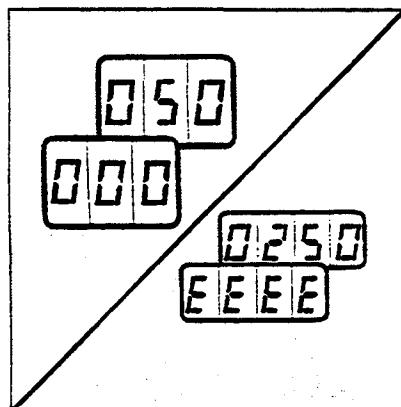Having discharged the preset volume, the unit switches to Keep-vein-open-rate of 1 ml/h.  
Indicator:  
Delivery rate display shows delivery rate and 001 ml/h alternating.  
Volume display flashes preselected volume.

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After 30 minutes, the pump stops and the staff call system is activated, (if connected.)

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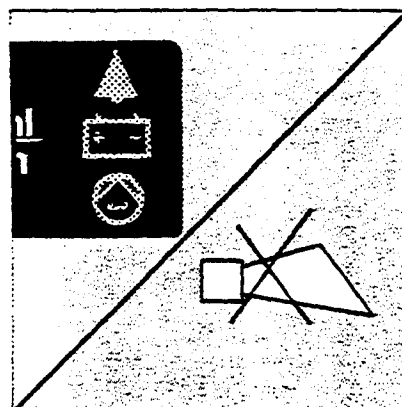
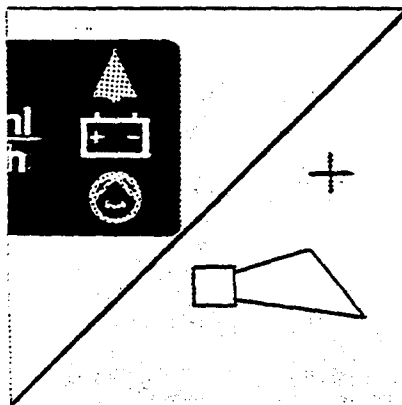
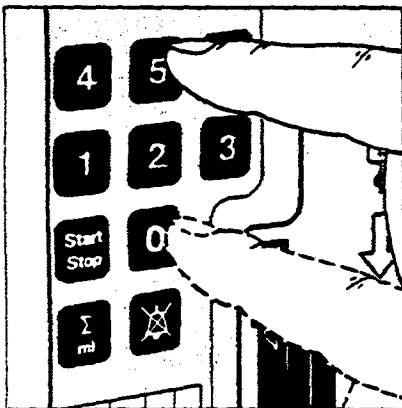
### Battery operation

If the INFUSOMAT® secura is connected to mains the accumulator will be charged whether the unit is switched on or off.

During battery operation the mains switch and charging/mains control indicator are not illuminated.

During battery operation indications are only switched on intermittently. However, at any time, data can be recalled by pressing any key (except the start/stop key).

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Alarm indication during battery operation.

Battery control indicator flashing: battery capacity 10 min., (approx.)

Battery capacity exhausted.  
Pump switched off.

No visual indications = no battery capacity.

Required recharging period at least 20 h.

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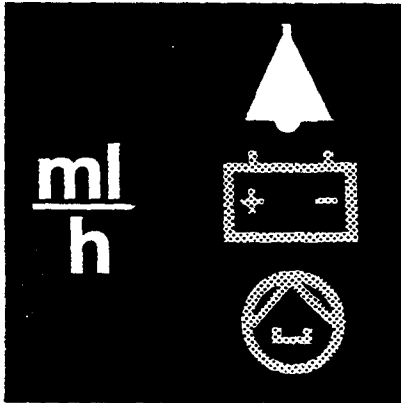
After recharging the battery for 20 h, the battery will have a capacity of at least 3.5 h at the maximum delivery rate.

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
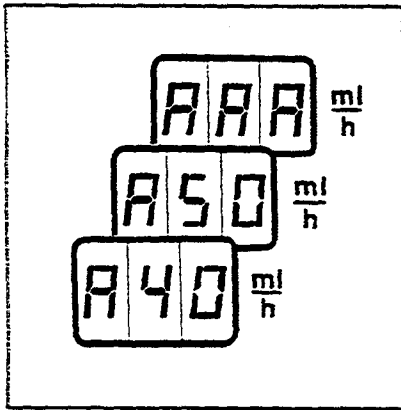
General visual alarm  
indication: red.

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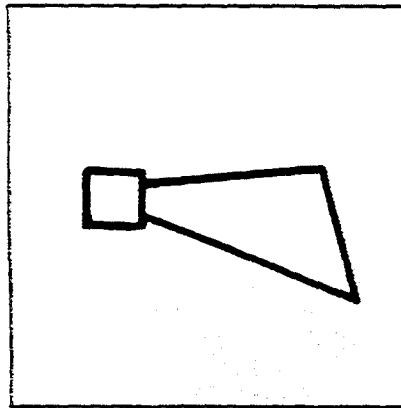
Visual alarms indicated  
in delivery rate display.

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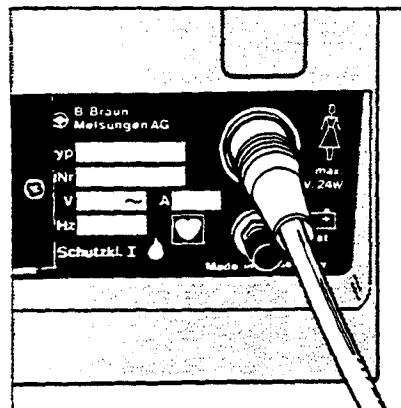Audible alarm system  
(buzzer incorporated)

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
Connection provided for link  
to central remote alarm  
system (staff call system).

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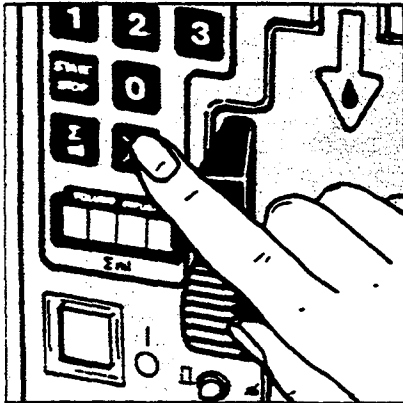### Muting the alarm

By actuating the alarm muting key, the audible alarm can be interrupted for 2 min (e.g. while changing bottle, clearing faults, etc.).

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# Faults and fault clearance

Alarm	Fault	Remedy
red LED activated audible alarm  Delivery rate display shows A 40	infusion container empty	replace infusion container restart unit
	roller clamp closed	open roller clamp restart unit
	occlusion	a. check infusion line for bends. b. check position of cannulae/catheter c. check cannulae/catheter for blockages restart unit
	condensed drip chamber	eliminate condensation by shaking drip chamber restart unit
	Flow sensor tilted	Fix drip chamber in vertical position restart unit
	free-flow	close roller clamp, check correct position of infusion line, if necessary use new infusion line. Open roller clamp (no continuous dripping allowed) restart unit
red LED activated audible alarm  Delivery rate display shows A 50	Air in Line	a. Deaerate infusion line (if necessary use other infusion line) b. Set fluid level in drip chamber restart unit
	incorrect position of infusion line	check position of infusion line according to instructions restart unit
	incorrect closed pump door	close pump door restart unit

The audible alarm can be muted for 2 min.  
by pushing the key 

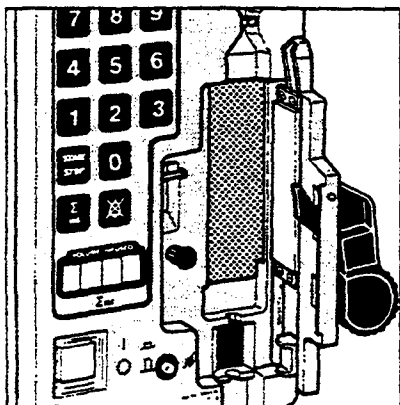
# Faults and fault clearance

Alarm	Fault	Remedy
red LED activated audible alarm  delivery rate display shows AAA  Volume display shows F4F4	flow sensor positioned still on handle	switch off unit place flow sensor on drip chamber switch on unit
	flow sensor disconnected	switch off unit connect flow sensor switch on unit
	Flow sensor defective	switch off unit change flow sensor switch on unit
flashing battery indicator, audible alarm	low battery capacity (capacity still appr. 10 min.)	connect unit to mains
all displays and indicators not illuminated	no capacity disregarded low-capacity- alarm	connect unit to mains. unit should remain connected to mains for the next 20 hours
audible alarm  delivery rate display shows delivery rate and 001 alternating  volume display flashes preselected volume	KVO-operation	preselected volume has been infused.  The INFUSOMAT® <u>secura</u> is operating on KVO (1 ml/h) for max. 30 min.
audible alarm  delivery rate display shows delivery rate and 000 alternating  volume display shows preselected volume and EEEE alternating	end of KVO-operation	INFUSOMAT® <u>secura</u> has stopped after 30 minutes of KVO-operation.  Additional alarm is given on the staff call system, if connected
undefined displays	malfunction of unit	call for service

The audible alarm can be muted for 2 min.  
by pushing the key 

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
### Maintenance

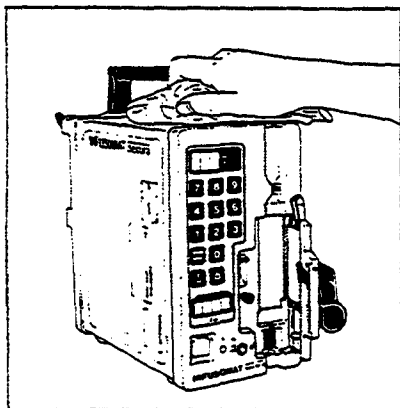The membrane covering the pump assembly and the Air Detector sensors should be cleaned with warm soapy water and checked for damage.

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
### Cleaning

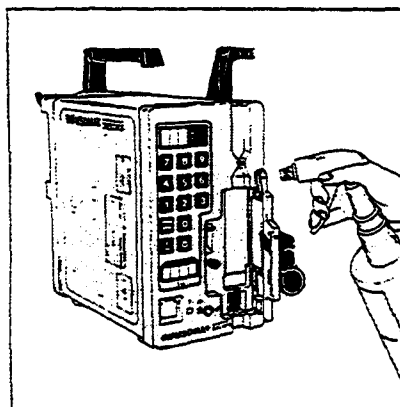Important: Disconnect mains plug and switch off unit. The flow sensor can be removed from unit. Front panel and outer casing are tightly sealed. The unit can be wiped off using a damp cloth and a mild detergent.

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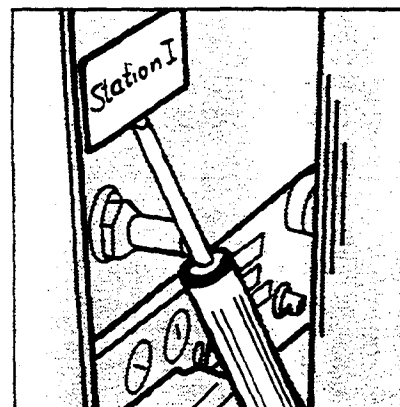
### Disinfection

Important: switch off unit and disconnect mains plug.

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### Identification

You may fix your own identification on the rear panel of the unit (ward, etc.)

Protective cover can be removed by means of a screw driver.

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### **Manufacturers' liability**

The manufacturer considers himself responsible for the effects on safety, reliability and performance of the equipment only if: assembly operations, extensions readjustments, modifications or repairs are carried out by persons authorized by him, and the electrical installation of the relevant room complies with IEC/VDE requirements and the equipment is used in accordance with the instructions for use.

### **Guarantee**

B. Braun Melsungen AG give a guarantee of 12 month from date of delivery for each INFUSOMAT® secura. This guarantee covers the repair or replacement of damaged parts whether as a result of faults in design, production or material. The guarantee is void should the owner or third parties modify or repair the unit.

The following are not covered by this guarantee:

The elimination of faults which can be put down to faulty manipulations, improper handling or normal wear.

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### **Regular servicing**

It is recommended that the unit be serviced regularly after a period of 1000 working hours or at the latest after 6 month.

A complete service handbook is be made available on request but only in conjunction with technical training.

### **Note**

Technical safety inspections are called for after a period of 12 month.

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for the proper condition and the safety of function in accordance with the §6 Abs. 4 of the MedGV and in conjunction of the instructions for use

- check of cleanliness, completeness
- check of damage
- check the membrane covering the peristaltic pump
- check the movement of the pressure plate
- check the position of the glued silicon on the pressure plate
- check the membrane covering the air detector
- check for damage of the flow sensor
- switch on infusion pump (mains supplies)  
self check  
(mains switch, audible alarm, visual operation display, alarm display)
- check of the battery
- functional check of the safety clamp
- check of staff call (if connected)
- check of the accessories
- operating procedure acc. to the instructions for use (operator's manual)  
check of the flow sensor  
(close roller clamp)

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Nennspannung	Nennstrom	Sicherung (5×20 mm nach IEC 127)
220 V ~ 50/60 Hz (Art.-Nr. 871902/0)	0.12 A	T 0.2 A
240 V ~ 50/60 Hz (Art.-Nr. 871904/7)	0.11 A	T 0.2 A
100 V ~ 50/60 Hz (Art.-Nr. 871900/4)	0.26 A	T 0.4 A
120 V ~ 50/60 Hz (Art.-Nr. 871901/2)	0.22 A	T 0.315 A
200 V ~ 50/60 Hz (Art.-Nr. 871905/5)	0.13 A	T 0.25 A

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I

CF (240 V ~ Ord.-No. 871 904/7: Type: B)

B (according to DIN VDE-0871/6.78 and DIN VDE 0875  
part 1/11.84, respectively EWG-Directive 82/499 and  
CISPR 11)

Moisture protection

Duration of operation

drip-proof

100% (continuous)

## Environmental conditions

a. operation

rel. humidity

30% ... 75%

temperature

+10°C ... +40°C

atm. pressure

700 mbar ... 1060 mbar

b. transport and storage

rel. humidity

10% ... 100%

temperature

-40°C ... +70°C

atm. pressure

500 mbar ... 1060 mbar

Accumulator type

nickel-cadmium 4.0 Ah 8×1.2 V (rechargeable)

Accumulator capacity

>3.5 h at max. delivery rate

Recharge time

>20 h

Air detector

technical sensivity

air bubbles ≥0.05 ml

alarm activated

air bubbles ≥0.3 ml

Deviation of preselected

delivery rate (including the  
disposables recommended by  
the manufacturer)

±5%

Automatical alarm activated in  
case of dosage failure

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**Flow sensor** Ord.-No. 381 3236/2

**Original-  
INFUSOMAT®-** Ord.-No. 870 001/0

**LEITUNG ZVD** branch tubing with T-piece for the measuring scale, Luer-Lock connection, length 340 cm, storage volume max. 0,5 ml at ambient temperature

**Original-  
INFUSOMAT®-** Ord.-No. 870 002/8

**LEITUNG N** with monitor section, Luer-Lock connection, length 250 cm, storage volume max. 0,5 ml at ambient temperature

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**Art.-Nr. 870 007/9**

**Original-  
INFUSOMAT®-  
LEITUNG NK with**  
monitor section, with  
injection site,  
Luer-Lock connection,  
length 300 cm,  
storage volume max.  
0,5 ml at ambient  
temperature

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**Art.-Nr. 870 008/7**

**Original-  
INFUSOMAT®-  
LEITUNG K with**  
injection site,  
Luer-Lock connection,  
length 270 cm,  
storage volume max.  
0,5 ml at ambient  
temperature

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**Art.-Nr. 870 012/5**

**Original-  
INFUSOMAT®-  
LEITUNG S black**  
tubing for lightsensitive  
drugs, Luer-Lock  
connection,  
length 250 cm,  
storage volume max.  
0,5 ml at ambient  
temperature

**Ord.-No. 870 012/5**

**Certification of Manufacturer / Importer \_\_\_\_\_**

Herewith we are declaring that the / \_\_\_\_\_ / \_\_\_\_\_

**Infusion-pump INFUSOMAT® secura**

\_\_\_\_\_  
(Device, Type, Model)

the interference suppression meets the requirements of

**“Vfg 1046/1984”.**

\_\_\_\_\_

The marketing of this device was announced to the German Bundespost.  
The German Bundespost is authorized to control the compliance with the requirements.

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This maintenance reference must not be copied, partly or whole, nor made accessible to third parties without our consent, and remains the property of our company.

Service-Manual

INF.secura 999ml/h

Seite  
Page

2

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03/89

B. Braun Melsungen AG

**TECHN**med

Service

## *Diagnostic equipment*

### *General description*

The diagnostic equipment serves for the guarantee of the technical security and it is a technical measuring instrument. In the case of repair it serves to locate faults at assembly level. As far as the diagnosis unit is concerned assemblies are as follows:

Control group : microprocessor circuit board  
Analogue group : analogue circuit board including the pertinent periphery equipment.

This includes voltage controller circuit board, mains transformer, step motor, nurse call system, accutest button, drop light barrier, air trap (ultrasonic sensor).

The mounted testing facilities can be seen in illustration 3 and 4.

The selector switch (see Figure 1) enables all signals from the diagnostic equipment to be switched to the BNC output socket. Measurements can be taken at this point with a multimeter, counter or oscilloscope. The most important signals are shown directly on the diagnostic equipment through LEDs (see Figure 1). The switches and buttons (see Figure 1) are for simulating faults in the assemblies.

The diagnostic equipment has no power supply of its own. This means that the existence of a +6V current supply from the test object is necessary to ensure perfect function.

### Attention:

It must be assured before putting the diagnostic equipment into operation that the supply voltages (6V, 5V UE and 5V ST) are within the given tolerance ranges.

In the diagnostic equipment there is a switch which has been set at the factory to the performance range of the battery used in the Infusomat. On younger units there is a jumper. Switch setting from back wall = 2.5 A/h. (Jumper to front plate). Switch setting on front plate = 4.0 A/h. (Jumper to back wall).

All tests are to be carried out with fitted set and mains connections.

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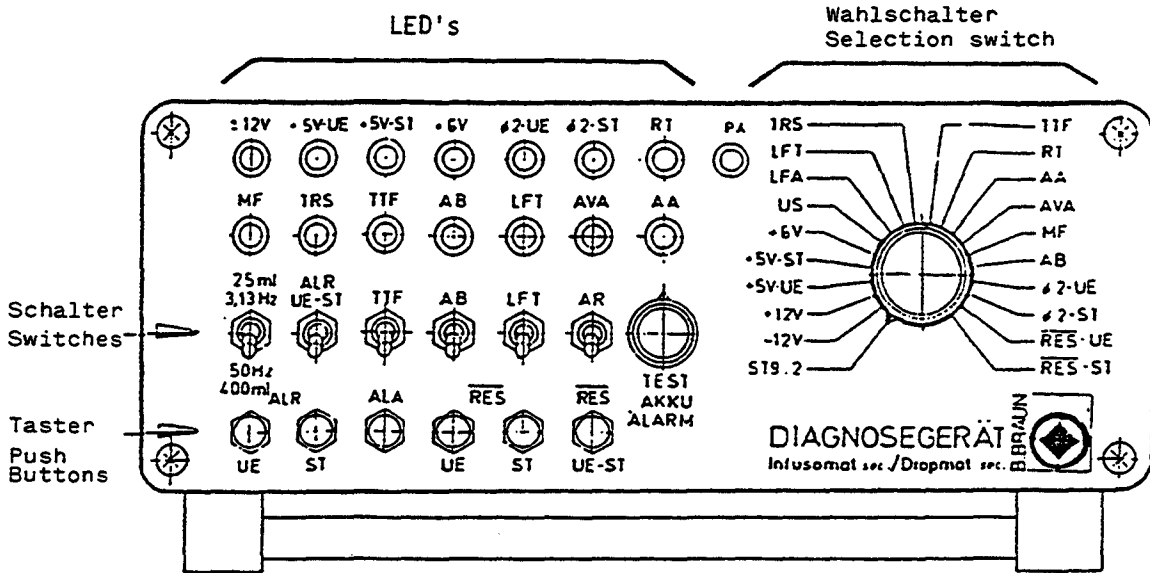
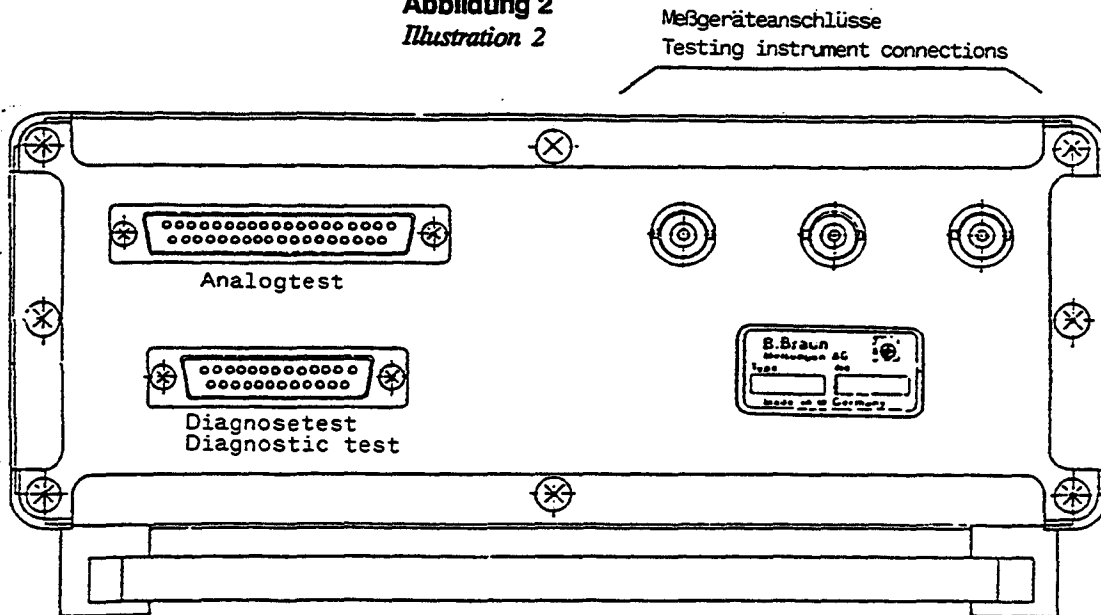
### LEDs

$\pm 12V$	Operating voltage, serial data interface (display is not valid for Infusomat 999ml/h)
+5V UE	Operating voltage, monitoring channel
+5V ST	Operating voltage, control channel
+6V	Operating voltage, analogue circuit board
$\emptyset 2$ UE	System cycle, monitoring channel
$\emptyset 2$ ST	System cycle, control channel
RT	Relay test
PA	Test adapter
MF	Motor frequency
TRS	Drop signal
TTF	Drop carrier frequency
AB	Accumulator operation
LFT	Air signal
AVA	Accumulator prealarm
AA	Accumulator alarm

### Switch

25ml/h (3,13Hz) / 400ml/h (50Hz)

	Change of delivery rate (or motor controlling)
ALR UE ST	Alarm relay monitoring channel (on older units also alarm relay monitoring channel)
TTF	Drop carrier frequency
AB	Accumulator operation
LFT	Air signal
AR	Alarm relay

### Key

ALR UE	Alarm relay, monitoring channel (without function)
ALR ST	Alarm relay, control channel
ALA	Acoustic alarm
RES UE	Reset for monitoring channel
RES ST	Reset for control channel
RES UE ST	Reset for both monitoring and control channel

### Potentiometer Test accu alarm


Testing alarm recognition

### Selection switch

Switched the selected voltages or signals to the connected measuring instruments.

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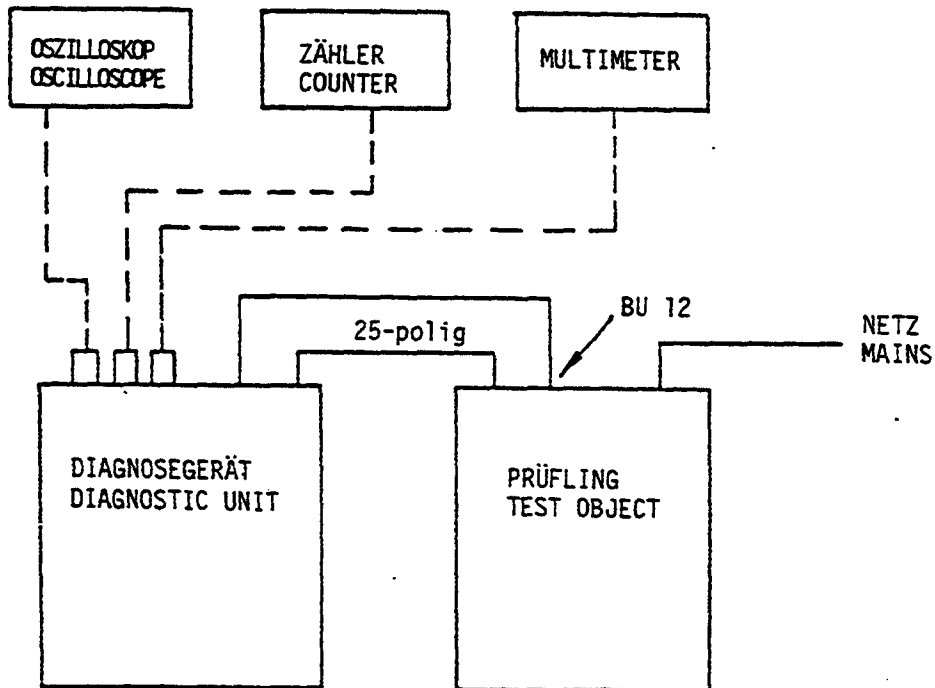*This system is intended for the most part for quick diagnosis and testing the assemblies. The equipment to be tested can remain closed. Signals are displayed during this test which not only comes from the micro-processor circuit board but also from the analogue board assembly.*

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**Operation of the switches and pushbuttons**  
**Switches are activated when in the up position**

Operation sequence, switches, pushbuttons	Meaning/Desired value	Effect on test object
AB (S) (T)	Accumulator operation simulated Level: Low	Microprocessor "sees" accumulator operation <b>Accumulator display comes on</b> (If not, there is a fault on the microprocessor circuit-board)
LFT (S)	Given air signal is ineffective Level with air : High Level without air: Low	1. Test object will not start Rate display shows A 50 (without test adapter) A 90 (with test adapter) (Air compensation is disturbed) 2. Door can be opened during operation without air alarm being given
Give in rate and start unit RES UE (T)	The monitoring channel on the microprocessor circuit board is reset Level when reset is activated: Low	Test object signals alarm Rate display shows 555 Summation display shows 3434
Switch on summation display Give in rate and start unit RES ST (T)	The control channel on the microprocessor circuit board is reset Level when reset is activated: Low	INFUSOMAT-secura only: Test object signals alarm Rate display shows A30 Summation display shows EEEE
RES UE ST (T)	Both channels are reset at the same time Level when reset is activated: Low	Works like an off-on switch

Other switches and pushbuttons are ineffective.

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This maintenance reference must not be copied, partly or  
whole, nor made accessible to third parties without our  
consent, and remains the property of our company.

Service-Manual

INF.secura 999ml/h

Seite  
Page

11

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03/89

B. Braun Melsungen AG

**TECHN**med  
Service

### Fault indication by LEDs

LED Diagnostic unit	Normal	Display If fault is given	Desired value	Cause of fault	To eliminate fault
±12V	Perhaps red (Without meaning)		±5 - 10V	Voltage controller or microprocessor circuit board	Replace µP circuit board
+5V UE	Dark	red	5V ±5%	<ol style="list-style-type: none"> <li>1. Voltage controller Cable BU7 or ST 7</li> <li>2. Fuse F3</li> <li>3. Short circuit on the analogue or microprocessor circuit board</li> <li>4. Flat cable BU3, BU12, ST3</li> </ol>	Change voltage controller circuit board Change fuse on analogue circuit board Change analogue or microprocessor circuit board
+5V ST	Dark	Red	6V ±5%	<ol style="list-style-type: none"> <li>1. Voltage controller cable BU7</li> <li>2. Fuse F4</li> <li>3. Short circuit on the analogue or microprocessor circuit board</li> <li>4. Flat cable BU3, BU12</li> </ol>	Change voltage controller circuit board Change analogue or microprocessor circuit board
+6V	Green	Red	6V ±5%	<ol style="list-style-type: none"> <li>1. Voltage controller cable BU7</li> <li>2. Analogue circuit board</li> <li>3. Flat cable BU3, BU12</li> </ol>	Change voltage controller circuit board
Ø 2 UE	Flashes every second	Does not flash	983,04 KHz		Change microprocessor circuit board
Ø 2	Flashes every second	Does not flash	1228,8 KHz	Microprocessor circuit board	Change
MF	Flashing frequency dependent	Does not flash or stays lit	According to delivery rate original MF of the µP circuit board stretched over a monoflop	Microprocessor circuit board	Change

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### Fault indication by LEDs

LED Diagnostic unit	Normal	Display If fault is given	Desired value	Cause of fault	To eliminate fault
TRS	Light up when drop falls	Does not flash or stays lit	Signal level: Low High	1. Drop lighth barrier 2. Flat cable ST10, ST 7 3. Analogue circuit board 4. Flat cable BU3, BU12 5. Short circuit on microprocessor circuit board	Change
TTF	Flashes every second	Does not flash or stays lit	1200 Hz	Microprocessor circuit board	Change
AB	Dark lights during accu operation	Dark, despite battery operation (Mains plug pulled)	Level: Low by AB	1. Analogue circuit board 2. Flat cable BU3, BU12	Change
LFT	Blinks visually	Display lights continuously or remains dark		<u>Air detection activated:</u> 1. Door not closed 2. Sensors, cable 3. Analogue circuit board 4. Flat cable BU3, BU12	Close door Change

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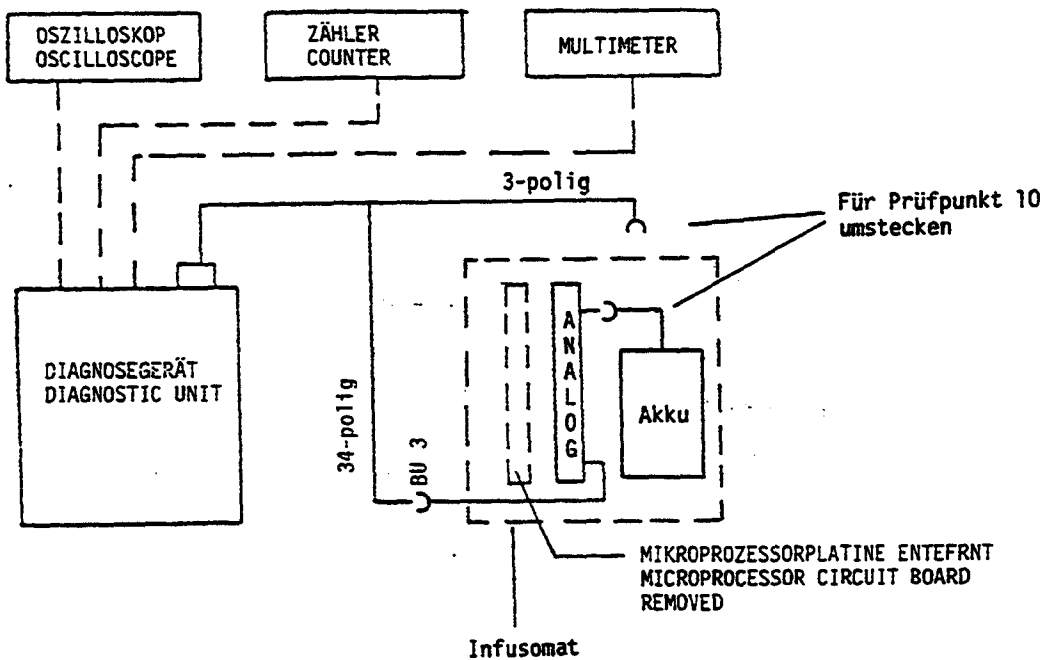
*Analogue circuit boards, voltage controller boards, mains transformer, step motor, nurse call jack, accutest tester, drop light barrier and air trap can be checked with this testing circuit.*

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**Analogue test (red LED  $\pm 12V$  lights)**  
**Operation table for switches, pushbuttons, potentiometers, "TEST ACCU ALARM" and test adapter**

Testing point	Operating sequence for switches, pushbuttons, potentiometers etc.	Meaning	LED Diagnostic unit	Effect on test object
1	ALR UE (T)	Alarm relay 1 is activated		Relay 1 switches audibly
2	ALR ST (T)	Alarm relay 1	RT lights	Relay 1 switches audibly Stepping motor runs
3	ALR UE-ST (S)	Alarm relay 1 is activated	RT lights	Relay 1 switches audibly Stepping motor runs
4	25 ml / 3,13 Hz 400ml / 50 Hz and ALR UE-ST	Switches from one delivery rate / motor speed to another		Stepping motor runs slow Stepping motor runs fast <b>Attention:</b> If motor runs backwards switch unit off and on again!
5	TTF and ALR UE-ST (Push up)	Drop carrier frequency is fed to the analogue circuit board	TRS lights for a short time, as each drop falls	The total drop recognition and evaluation circuit is operated
	TTF (push down)	TTF is turned off	TRS lights	The drop detector does not receive a signal
6	ALA	Acoustic alarm transmitter is activated		Acoustic alarm can be heard
7	LFT (Push down)	Air detection stays active	LFT flickers	<u>No effects</u> Note: For this test the installed set must be free of air !

Note: See "Alignment and adjusting instructions" for a detailed LFT testing instructions !

Switches: (S)

Pushbutton: (T)

**Analogue test (red LED  $\pm 12V$  lights)**  
**Operation table for switches, pushbuttons, potentiometers "TEST ACCU ALARM" and test adapter**

8	Disconnect mains plug AB = down	Battery operation is simulated	AB lights	Test object running from accumulator
9	Plug-in mains plug Disconnect battery plug Plug-in 3-pole testing cable Selection switch in position St 9.2  TEST ACCUMULATOR ALARM! Turn potentiometer from left stop to right stop	Changing accumulator voltage is simulated	AVA lights  AA lights	Analogue circuit board recognizes accumulator pre-alarm between 8.74V and 9.23V Analogue circuit board recognizes accumulator alarm between 8V and 8.38V
10	Nurse call: Bridge socket "Nurse call" Pin 2+4 with check adapter		PA lights	

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### Alarm signals in dosing rate display

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Alarm signals appearing in dosing rate display (ml/h) during switch on test time.  
The used microprocessor technique makes it possible through a service program, to provide indications of faulty functions, and, as a result, contribute to reducing repair times.

ml/h-display	Cause of alarm
AAA	Drop detector is still on handle Drop detector, not illuminated Drop monitor on analogue circuit board
222	Drop monitor defective (Change analogue circuit board)
555 777	REL 1, or relay control system defective
888	Change analogue circuit board
999	EPROM cannot be read correctly Change $\mu$ P circuit board
BBB	Keyboard has been operated during test phase. Short circuit or interruption in keyboard matrix or $\mu$ P circuit board fault
CCC	Change $\mu$ P circuit board
DDD	RAM test, change $\mu$ P circuit board
EEE	$\mu$ PU test, change $\mu$ P circuit board:
FFF	Monitor circuit test, change $\mu$ P circuit board

$\mu$ P = microprocessor

Alarm signals appearing in dosing rate display after start key is activated.  
For expanded alarm indications it is necessary that the test adapter Order No.: 770 07019 be connected to the nurse call socket.

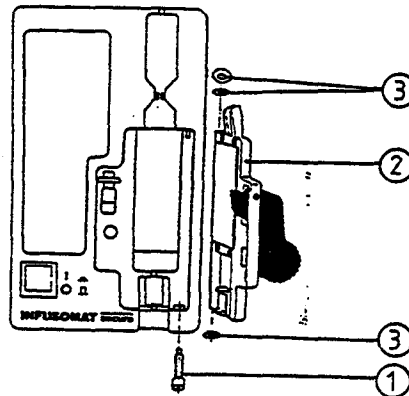
	Infusomat	
	with- out PA	with PA
Air bubble 0.25 ml in system	A 50	A 5.0
Air in starting moment LFT = high	A 50	A 6.0
In starting moment LFT = low (LFT switch wrong on diagnostic tester)	A 50	AB.0
No drops (blocked)	A 40	A 40
Continuous flow	A 40	A 30
Too many drops	A 40	A 10
Too few drops	A 40	A 20
<u>Data interface operation:</u>		
Start not possible	A.B.0	A.B.0

Remarks:  
with test adapter (PA)  
Alarm for ST-channels are indicated with,  
alarms from UE-channels are  
indicated without, decimal points

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
Remove hinge screw ① . Remove pump cover ② . Renew polyamide  
washers ③ . (install two washers at the top and one at the bottom)  
and hinge screw ① .

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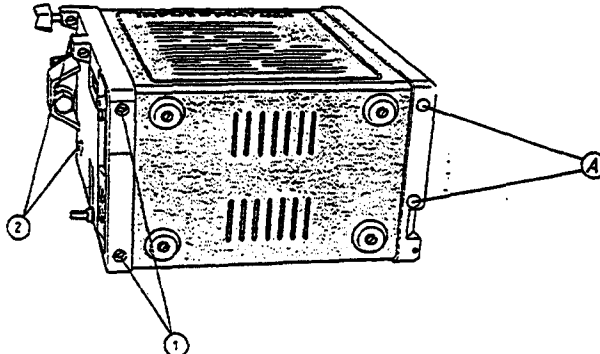Remove the service seal from the bottom of the rear panel. Loosen  
the screws ① . ( 2-3 rotations)


**Attention!** Don't remove the screws ① from the bottom of the front  
panel.

Remove the two black blind plugs ② at the rear panel,  
remove screws. Carefully, move the rear panel away from  
the body, disconnect the ground connection, plug ST7 and  
plug ST4.

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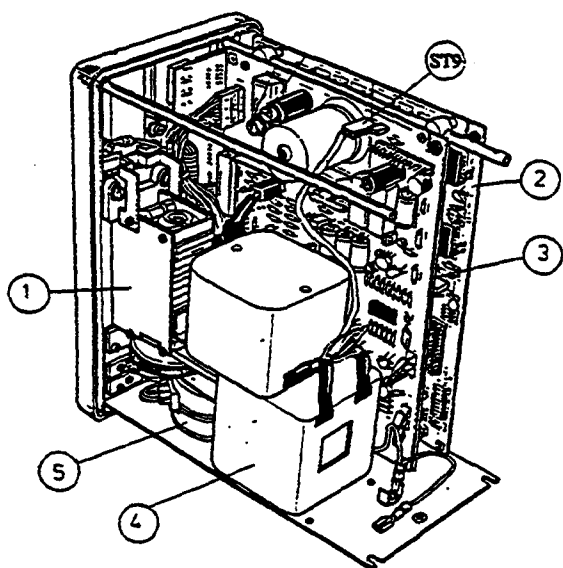*Remove ground connection between the front assembly and the metal cover. Slide front assembly out of the cover.*

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
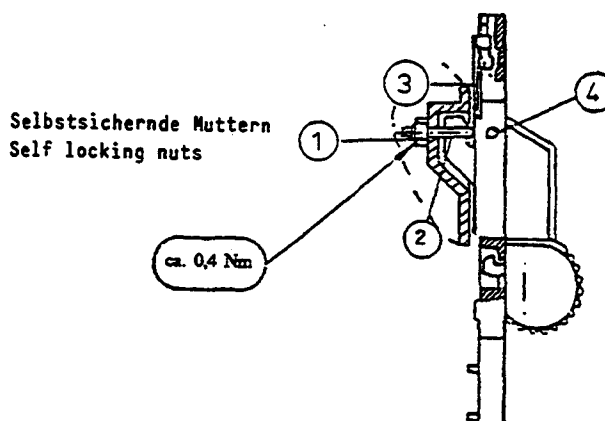
Loosen nuts ① (approx. 2 turns), close pump door, fasten nuts ① with torque wrench to approx. 0,4 Nm. The pump door must be flush, without play. Grease slide faces lightly and check their easy movement. Take counterpressure measurement and carry out air compensation test according to type of unit.

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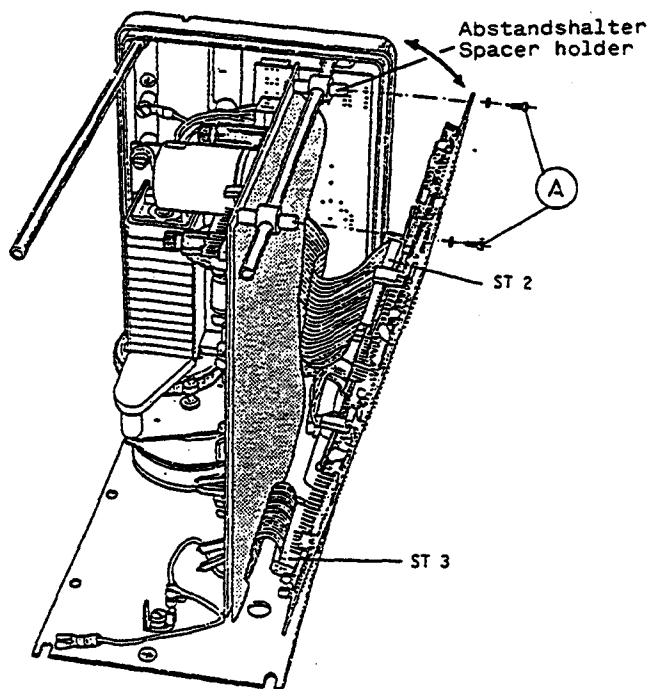
Remove screws ①. Disconnect plug ST2 and ST3. After exchanging the circuit board reconnect plugs ST2 and ST3. The insulating paper and flat cables must be properly placed to prevent a short circuit with the analog circuit board.

**Attention!** Do not tighten screws too tight, if plastic spacer holders are used.

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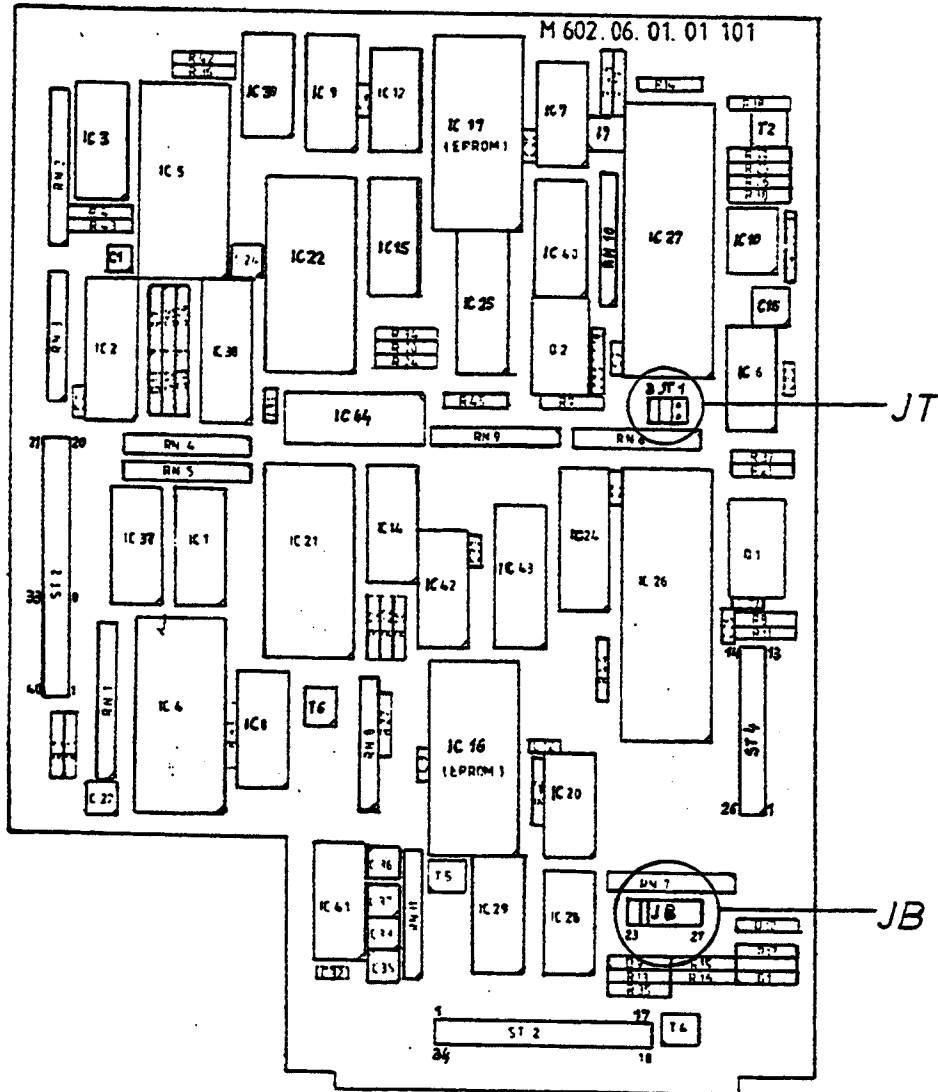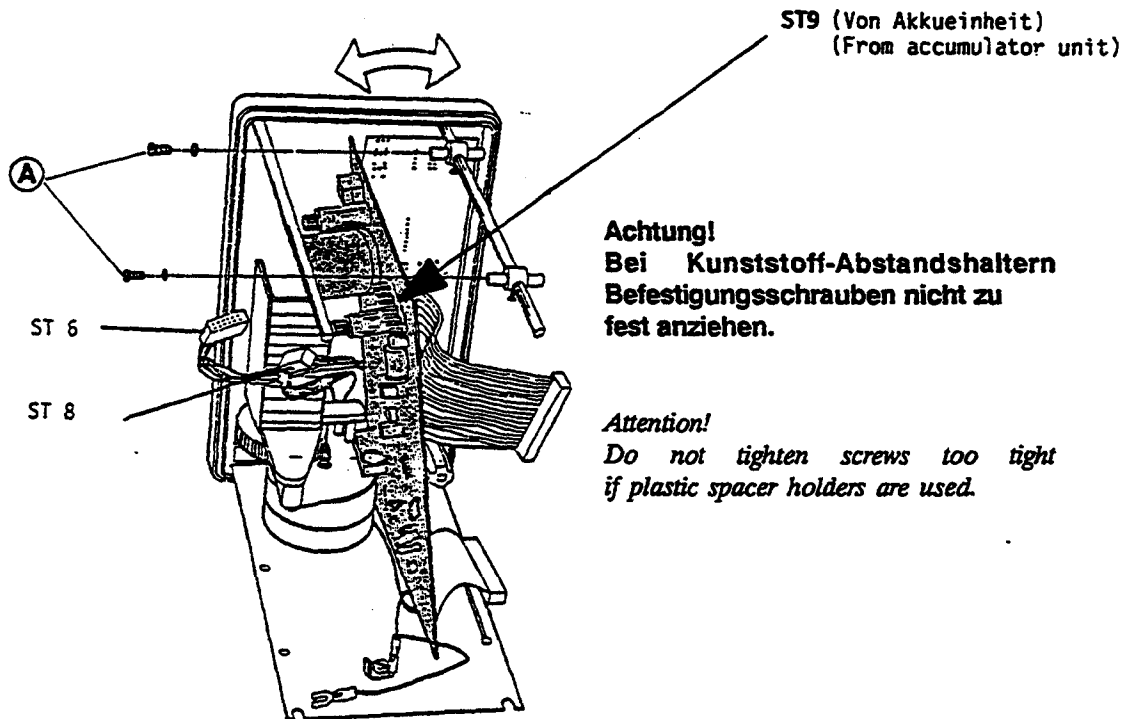
**Attention!**  
 Do not tighten screws too tight  
 if plastic spacer holders are used.

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*Disconnect plugs ST6-ST8-ST9. Remove screws (A). Lift the circuit board and disconnect the flat  
 band cable from microprocessor-circuit board.*

*When assembling the new circuit board, be sure that the insulation paper between the  
 boards is in place.*

**Attention!** *Take care of point 2.3 - 2.6 as well as 2.7.4 adjustment instruction for mains-accu  
 change-over, after exchanging the circuit board testing instruction 2.7.2 "Adjustment  
 and test of the air trap sensors"*

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*Do not remove pump unit, by changing motor and drive gear!  
 (do not maladjust fixing screws)  
 The pump unit can only be replaced if a "measuring equipment for finger pump and pump cover" order no.: 770 137/3 is at present.  
 Without the measuring equipment the change of pump unit is only possible at the factory.*

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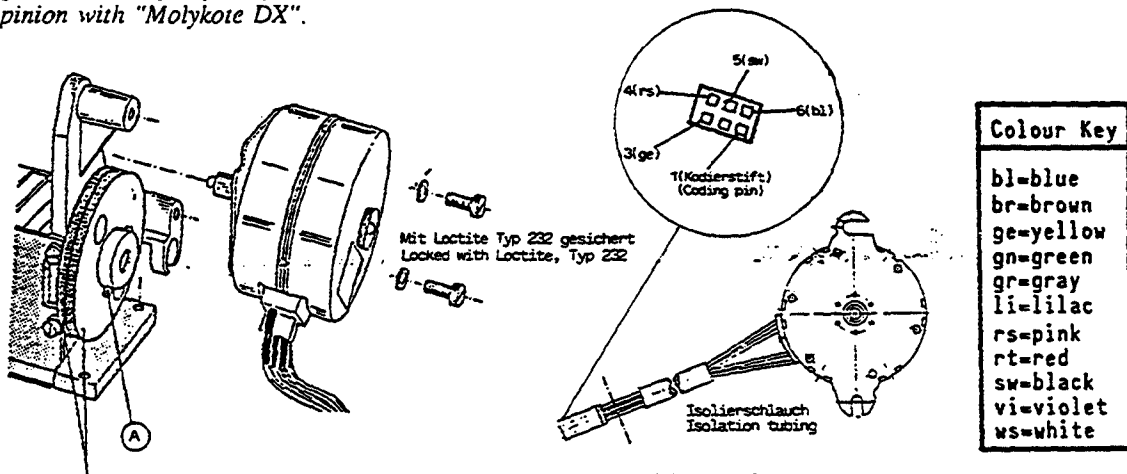
*Remove the screws holding the motor. Check pump head for free movement. Put new motor in place (see illustration) and screw down lightly. The gearwheel set must be tensioned 2 teeth and motor pinion came in contact. Adjust working depth of motor pinion. Screw the motor tightly in place.*

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*Remove motor (see illustration). Drive the pin (A) out of the drive gear, be sure that the primary shaft is properly supported. Remove the drive gear. There is a tension ring between the pump head and the drive gear, leave it in place. Put the new drive gear on the shaft of the pump head, put the fastening pin A in place. Be sure that the tension ring is in place between the drive gear and the pump. Replace the motor (see: motor replacement). Grease the drive gear and pinion with "Molykote DX".*

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
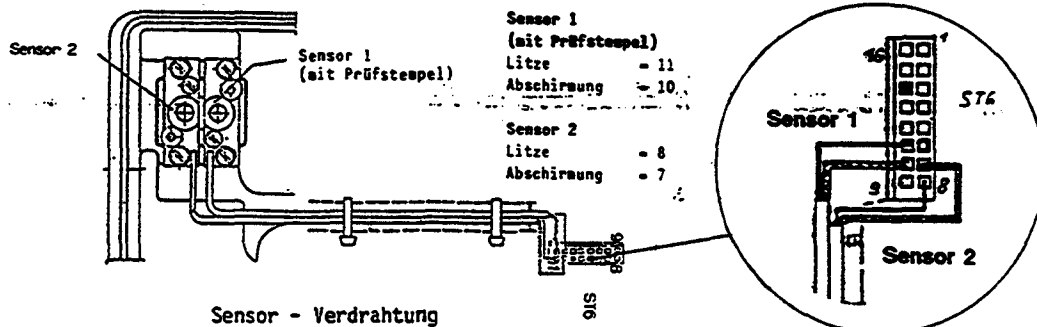
## Ultrasonic sensor replacement


**Attention! For safety reasons both sensors must be replaced at the same time**

*At first remove the battery and the pump motor*

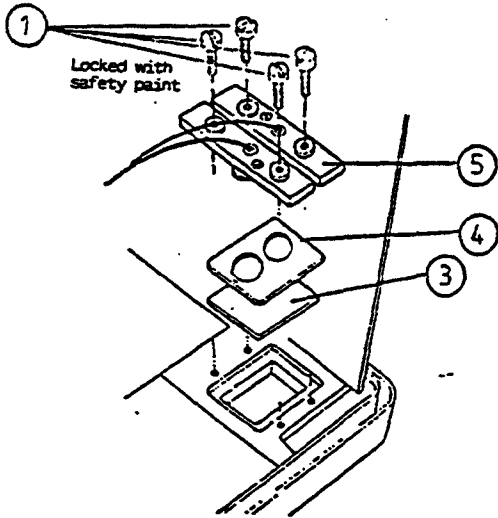Illustration: 1

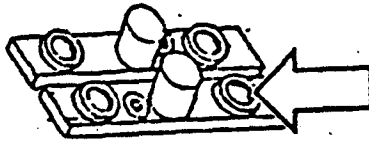1. Remove safety painted screws ①
2. Remove sensors ⑤
3. Remove sensor sealing ④ and membrane ③
4. Loosen and remove contacts from plug ST 6

Fix insulation bushing:

Alternately:

Order-Nr.:

38 33 814/9 0,7mm thick

33 18 201/9 1,0mm thick

Illustration 1


Illustration 2

### New sensor installation:

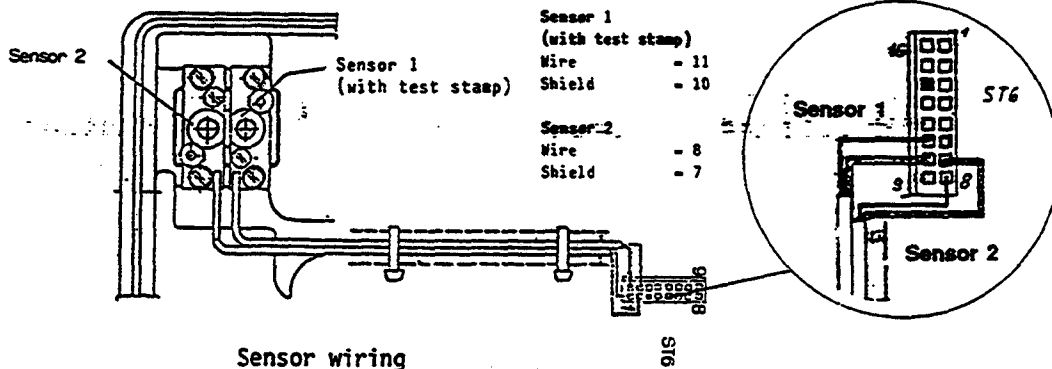Position of membrane ③ and sensor sealing ④. Provide sensors ⑤ with insulation bushings and assemble (sensor 1 - with test stamp - on top). Fasten screws ① with torque wrench to 10Ncm. Fit contacts in plug ST6. Check sensors according to 2.7.2 "Adjusting and testing the air trap sensors." Should the distance between sensor - pump door not be within the moving range this can be corrected by exchanging the insulation bushings (alternately 0,7 or 1 mm thick).


Lock screws ① with safety paint.

After assembly of the unit, test the function according to instructions "Testing the air trap" 2.7.3.

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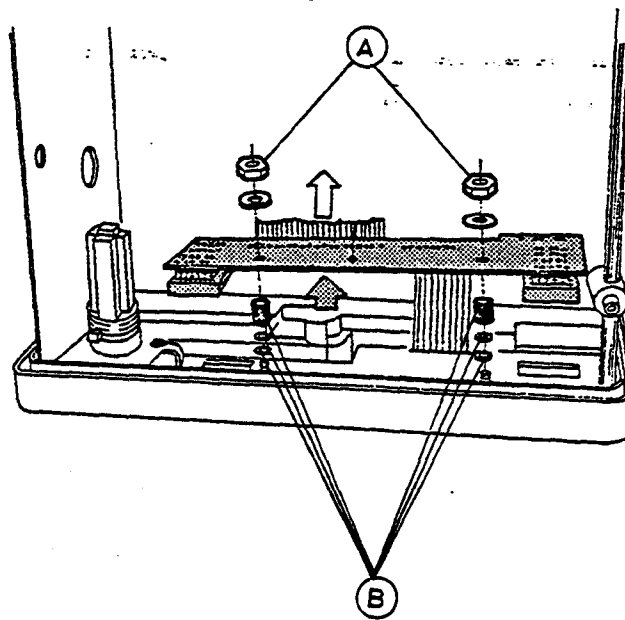Remove "Analog" and "Microprocessor" circuit board. Remove nuts **(A)** and "Display" circuit board.

**Attention!** Do not lose spacers and washers **(B)**.

Remove flat band cable from plug ST1.

Assembling in contrary turn. The circuit board is properly installed when the "Display" indicators lay directly on the window of the keyboard. (The hights difference must be compensate with washers)

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

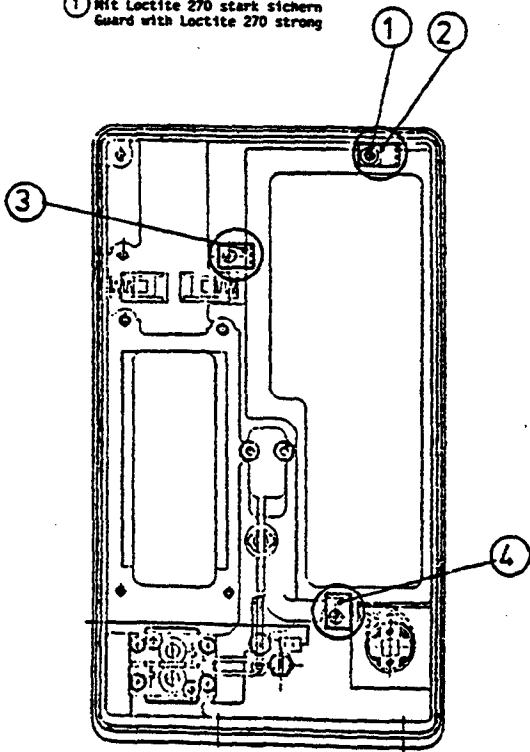
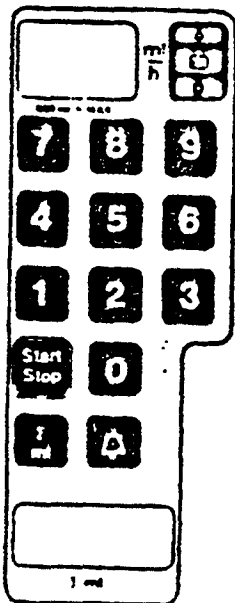
1. Remove "Display" circuit board (see: Display circuit board replacement)
2. Remove the circuit board support shaft ①
3. Remove angle clamp ②
4. Remove angle clamp ③ and ④
5. Press out the keyboard plate aback
6. To renew the seal, remove the adhesive rests with spirit from the supporting surface from front unit
7. Laminate keyboard plate with seal (Remove protective film from handling unit)
8. Remove protective film from the other side of sealing and fit the keyboard plate into place.

Assembling of complete unit in contrary turn (see: "Analog"-Microprocessor" and "Display" circuit board replacement)

Secure the screws with Loctite Type 720

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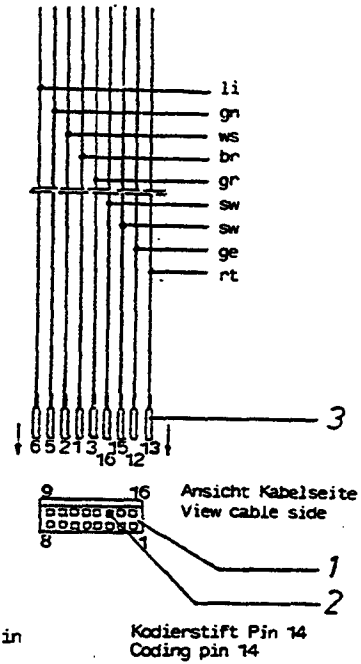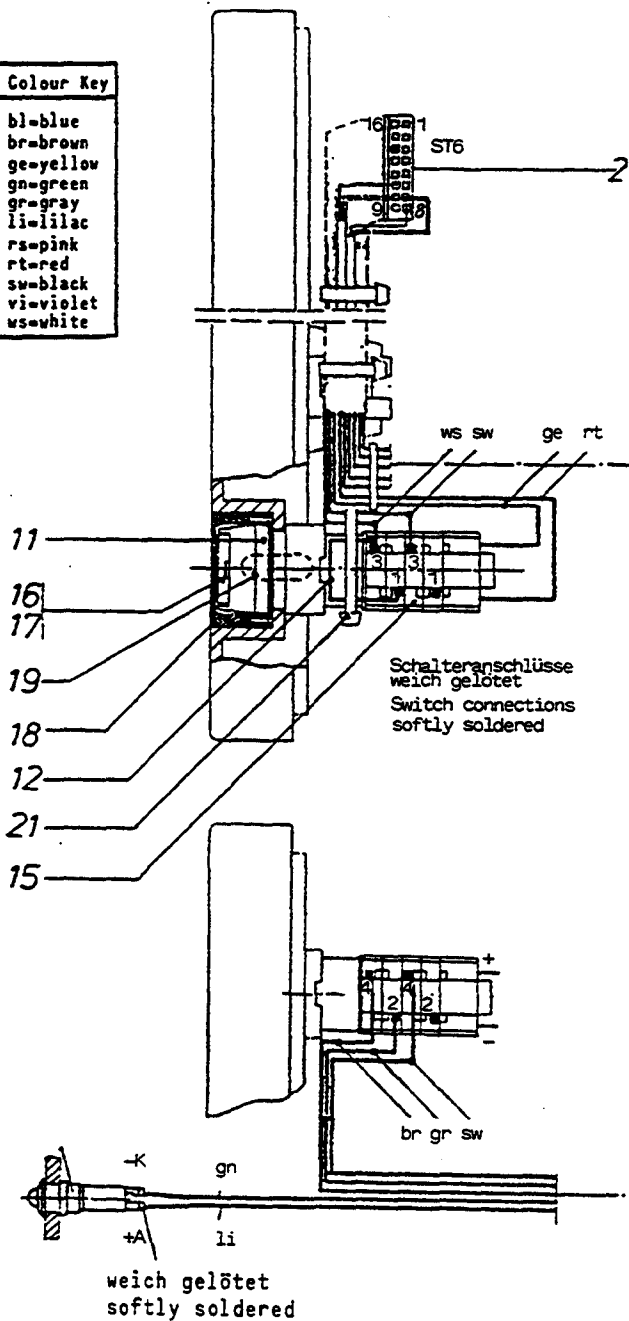Eingerastet in  
Kammer-Nr.  
Engaged in  
chamber No.

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**Attention!** Reuse distance ring 11.  
Assembling in contrary turn.

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Install the new switch so that the numbered rows of contacts 2/4 face outward and 1/3 face inward.

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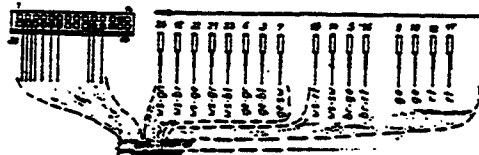
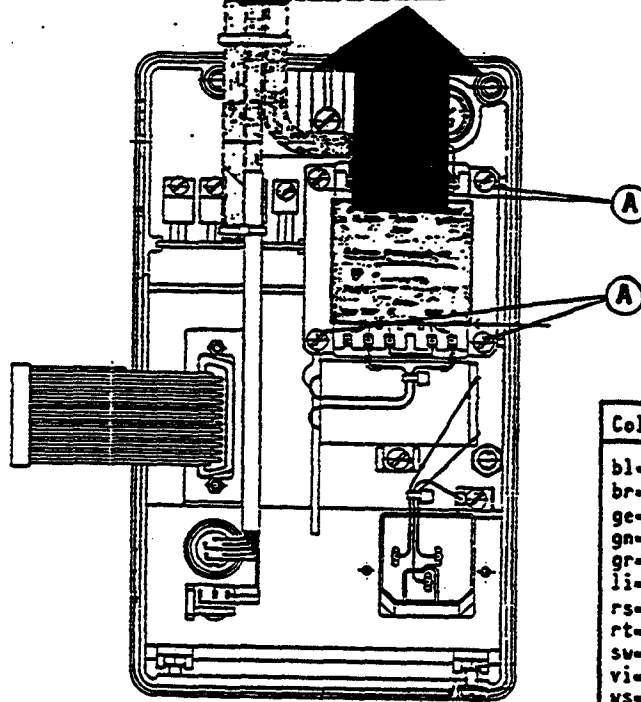
eingesteckt  
 in Kammer-Nr.  
 engaged in  
 chamber No.

Colour Key	
bl	blue
br	brown
ge	yellow
gn	green
gr	gray
li	lilac
rs	pink
rt	red
sw	black
vi	violet
vs	white

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1. Unsolder the connections from the mains filter and the cable harness.
2. Remove screws (A).
3. Install new transformer (secondary side ↑).
4. Fasten screws (A).

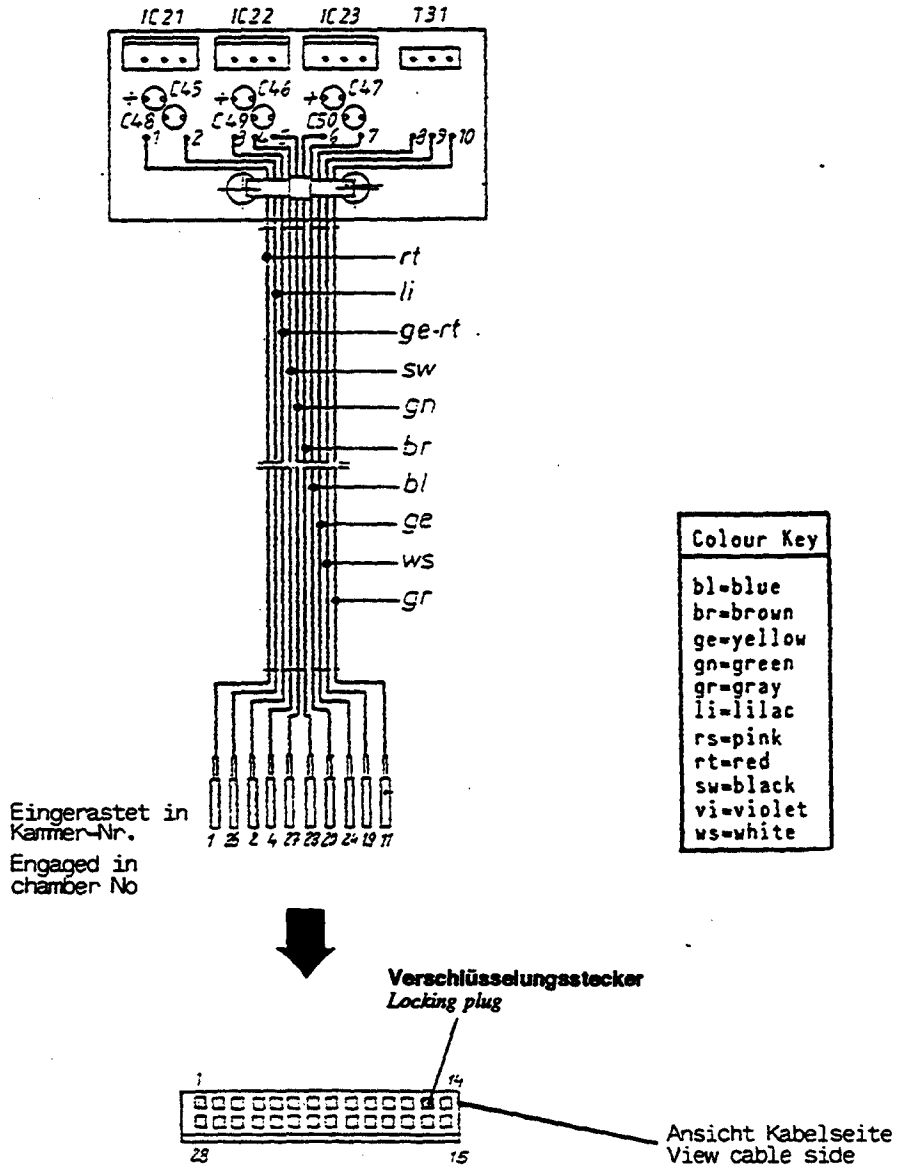
**Attention!**

Insulate flexible cords with shrinkable tubing  
 Connections see circuit diagram M602.07.00.02.103

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*Open cable ties in AMP-plug direction. Remove contacts 2-3-10-20-21-25-26-27-28-29 with AMP tools from plug housing. Remove screws from IC21-IC22-IC23 and T31. Exchange circuit board. Fix ICs with screw and serrated lock washer. Apply heat conductive paste to mica washer and T31 and screw on (use screw with insulation bushing). Build-in new cable tie.*

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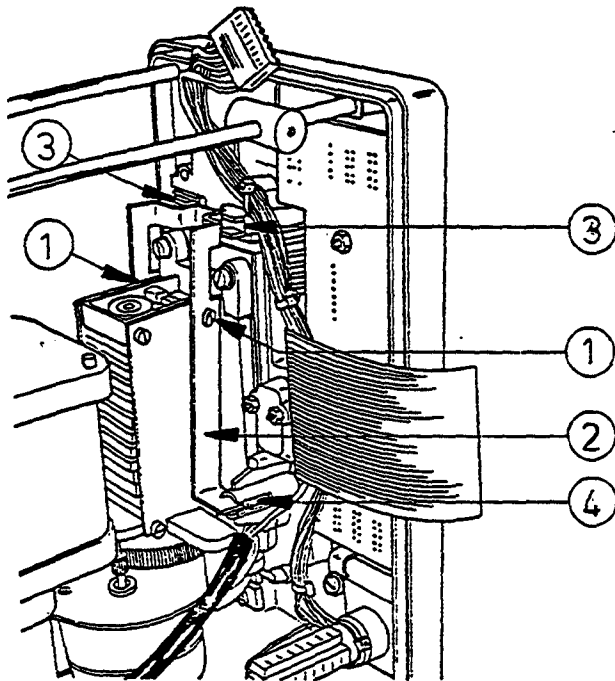
*Remove screws ①. Lift out release lever ②.*  
**Attention! Don't lose the inserted brass bushings!**  
*Lift out safety clamp with spring ③. Exchange pressure spring. Clean guide surface and grease lightly with Molykote paste DX white.*  
*Insert safety clamp with spring ③ in its guide and bring it in correct position.*  
*Insert release lever ② with bushings. The elbow lever must be above the detent pin ④. Insert screws from release lever ② and tighten and secure with Loctite 242.*  
*Check function of safety clamp (see: "Instructions for testing after carrying out repairs".)*

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*Assemble equipment. Pin up all plugs. Inspect equipment in accordance with the published inspection instructions.  
Plump the equipment.*

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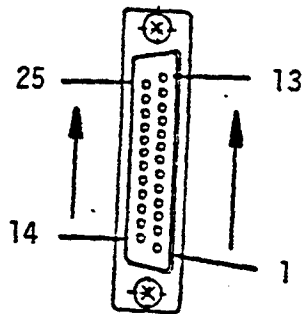
*The diagnostic socket makes it possible to thoroughly test the closed equipment. The important system signals are described below.*

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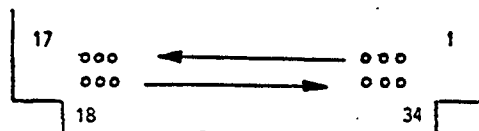
Plug ③ is found on the bottom edge of the microprocessor circuit board and makes possible the measurement of signals that are exchanged between the two functional circuit boards.

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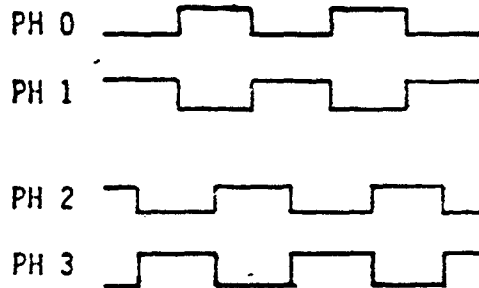
Relais angezogen RT = +5V  
*Relay energized*

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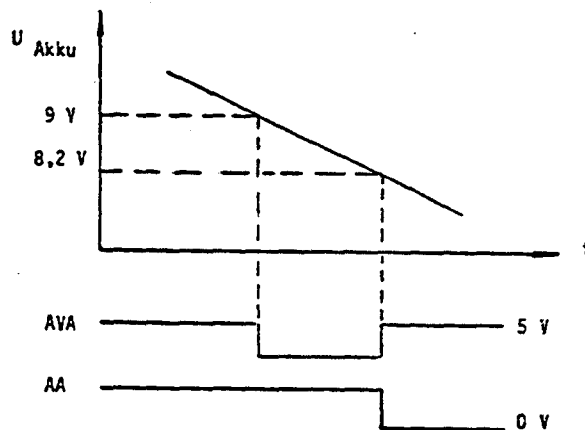
## 2.7 Alignment and adjusting instructions

### 1. To align the air trap oscillator:

- 1.1 Connect jumper JT 1-2 to the uP circuit board.
- 1.2 Switch on unit.
- 1.3 Connect up frequency counter to MP-LFF (R16, analog circuit board).  
Mass connection e.g. upper end of the resistance.
- 1.4 Press keyboard key 2 and balance out with potentiometer P2 (upper poti)  
 $f_{min} = 250\text{kHz}$ .
- 1.5 Press keyboard key 1 and balance out with potentiometer P1 (lower poti)  
 $f_{max} = 450\text{kHz}$ .
- 1.6 Remove jumper JT 1-2.

### 2. Adjusting and testing the air trap sensors:

- 2.1 Undo the 4 fastening screws for the sensors and then do up again lightly (with torque wrench 10 Ncm).
- 2.2 Check the distance between the sensor and pump flap with a thickness gauge (1.5mm easy - 1.7mm tight).
- 2.3 Connect up oscilloscope to MP-LFG (cathode D3, analogue circuit board). Start pump at 4ml/h.  
The signal amplitude should be maximum 0.5V on D3 with pump flap open and without hose.
- 2.4 Fit air-bled infusion line and close pump flap.
- 2.5 Plug jumper JT 1-2 to the uP circuit board. Press keyboard key 3. The two right digits in the quantity display must show a value between min. 18 and max. 28.
- 2.6 Remove jumper JT 1-2.

### 3. To check air trap:

- 3.1 The LFT-signal will be shown by the diagnostic tester.
- 3.2 Fit new air-bled infusion set and close pump flap.
- 3.3 Start unit at 200ml/h.

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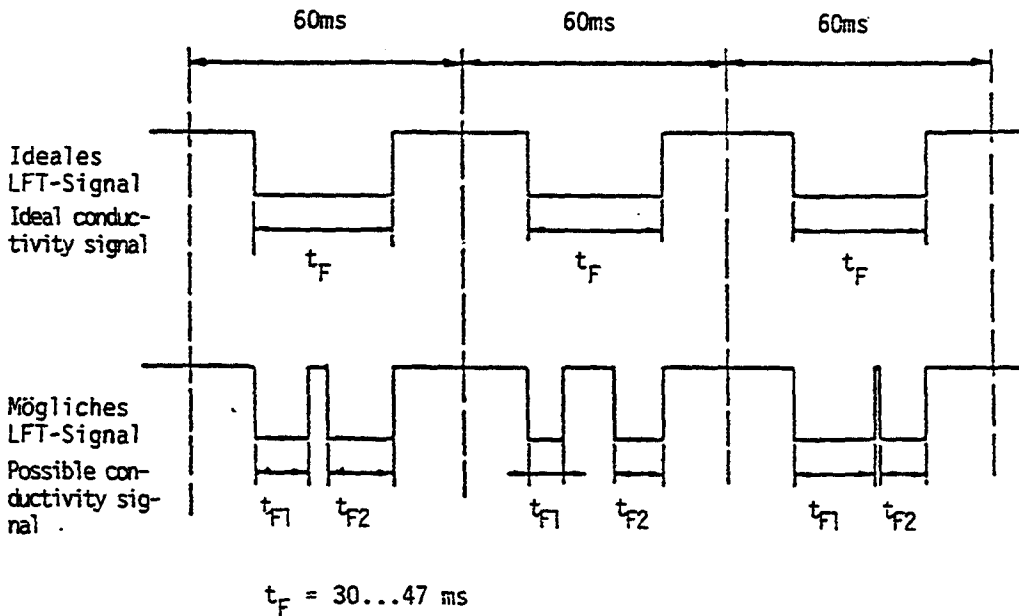
4.6 P3 versiegeln.

3.4 The conductivity signal will be shown as follows on the oscilloscope:

3.5 Start unit at 4ml/h. Open pump flap and remove hose. The time  $t_F$  must amount to 0 ms

3.6 After 1h in operation repeat measurements 3.4 and 3.5

4. Alignment instructions for the mains/accumulator switch-over:

4.1 Check accumulator voltage. Desired value  $> 10V$ .

4.2 Connect unit with control transformer ( $U_A = 192V \pm 1V$ ).

4.3 Set potentiometer P3 on anti-clockwise rotation.

4.4 Switch on unit and wait for switch-on test.

4.5 Turn P3 slowly to the right, until the unit switches on to mains operation.

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*Function of the pump cover closure (play free in closed state without tubing).*

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*Check pressure plate for easy movement. Visual check: Check silicon plate for adhesion.*

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*Check for correct switch on test on the display, according to operating instructions. Test the input keyboard: enter 753.*

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*Battery check: Check the function of the "Akkutest" key in mains operation. Check the mains switch over. Check the battery alarm during the continuous test. Minimum working time when connected to battery = 3.5h. Note: Charging time of battery = 16h.*

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*Check function of call system with multimeter. For plug wiring see circuit diagram.*

<i>In alarm condition:</i>	<i>3 and 5 continuous path</i>	<i>In operating condition:</i>	<i>3 and 5 open</i>
	<i>3 and 1 open</i>		<i>3 and 1 continuous</i>

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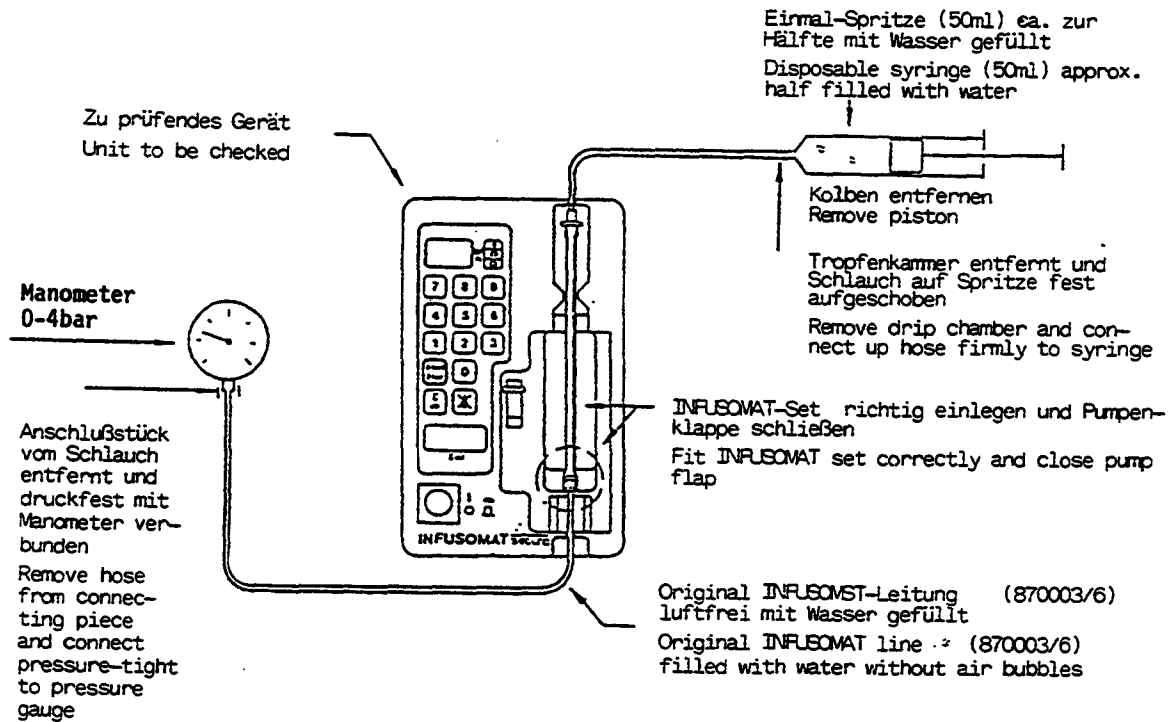
*Check the air monitoring system: Start with open pump cover without tubing installed). Display must show A50 or A60. See instructions for testing carrying out repairs: "Check of air trap".*

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*Checking drop sensor*

- a. Produce a continuous flow in drop chamber. Display A30 or (A3.0)  
 b. Empty bottle, close roller clamp. Start unit at 200ml/h. At approx. max. 10 sec. alarm will appear A40 - (A4.0) or A20 - (A2.0).

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**Attention:** Pressure gauge manometer has to be at minimum 63mm!  
 A new infusion set is required after approx 50 measurements because of decreasing measurement accuracy.

Pressure built-up to 400ml/h. Set at 40ml/h to take measurement. Take at least 2 measurements.

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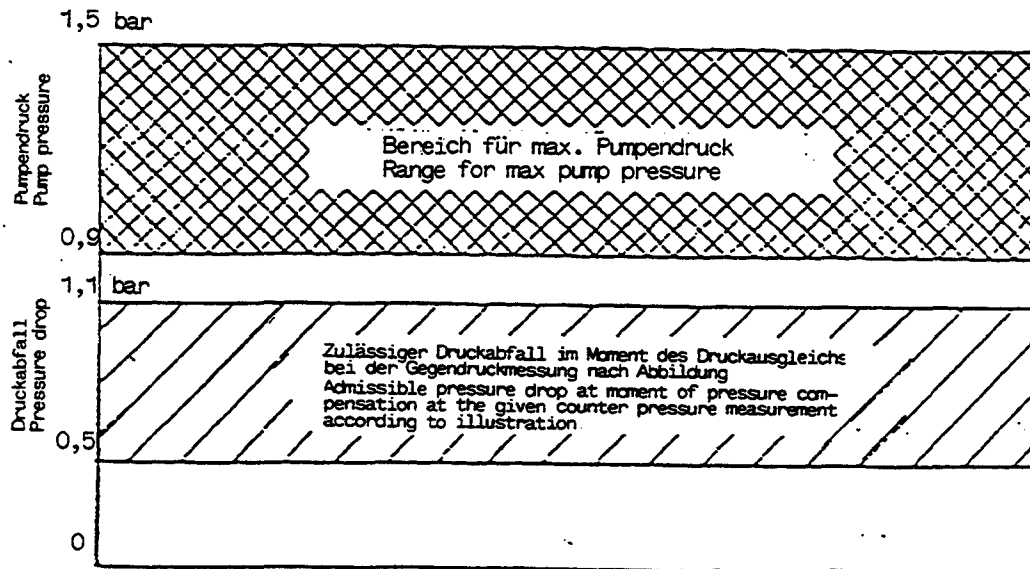
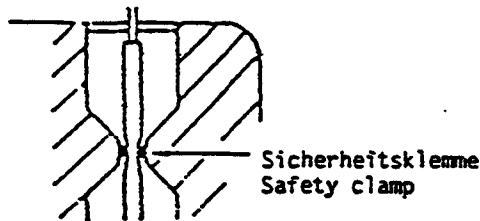
*Check safety clamp device  
 Assembly pressure. The safety clamp has to hold a pressure of 0,4 bar safety. If necessary the test has to be carried out again.*

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*When the release pin is fully actuated (to stop) the safety clamp must have a minimum open width of 5,5 mm. With the pump door open, the maximum open width is 1,0 mm.*

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*Check delivery rate at 250 ml/h. (Only after pump or door repairs have been carried out). With calibrated measuring cylinder 25 ml  $\pm$  5% in 6 minutes.*

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*Check the distance between air sensor and pump cover with gauge 770 084/9. See repair instructions: "Ultrasonics sensor replacement".  
(Check distance with closed playfree cover)*

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*Check pump function with set for at least 24 hours.  
Enter 8888 on volume display.*

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*Protective conductor resistance max. 0,2 Ohm with mains cable  
Measuring points between safety contact and:*

- a. Connection, potential compensation
- b. Upper rivet nut in the housing shell
- c. Bow of door lock
- d. Metal ring of connecting socket "call system"
- e. Metal ring of connecting socket "drop sensor"
- f. Srew on computer interface

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*Measure the insulation voltage of 550V = between the short-circuited mains connection and the housing. The min. insulation voltage is 2 MOhm.*

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*Earth conducting curreing*

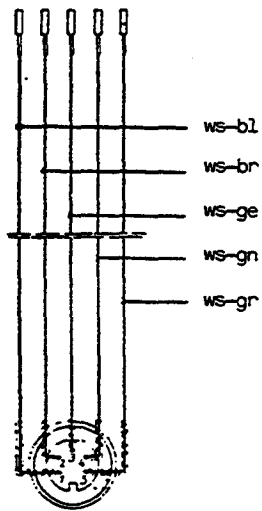
*Measure the earth conducting current in the safety cable of the unit under these conditions:*

- *including the necessary 3 m long mains cable*
  - a. *Mains voltage of unit*  
*Earth conducting current: max. 30  $\mu$ A (0,03 mA)*
  - b. *as a but with reverse polarization*  
*Earth conducting current: max. 30  $\mu$ A (0,03 mA)*
  - c. *Interruption in 0-cable (L and N)*  
*Earth conducting current: max. 60  $\mu$ A (0,06 mA)*
  - d. *Interruption of phase (L and N)*  
*Earth conducting current: 0  $\mu$ A*

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
*Check safety paint  
Affix plumbing seal*

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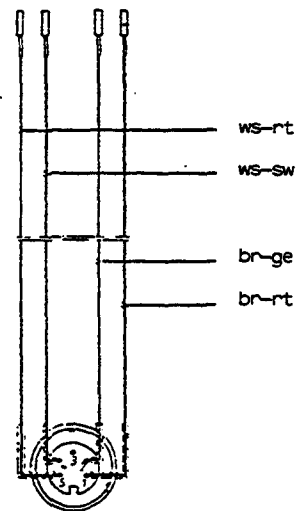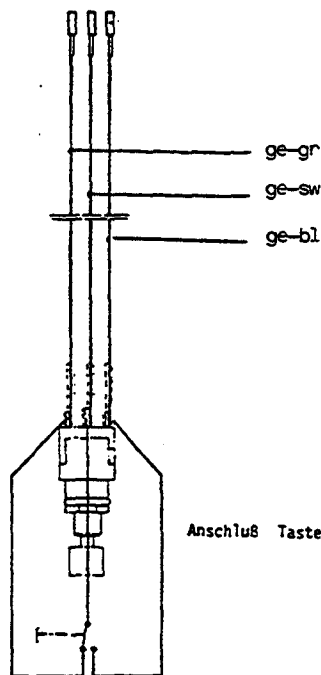
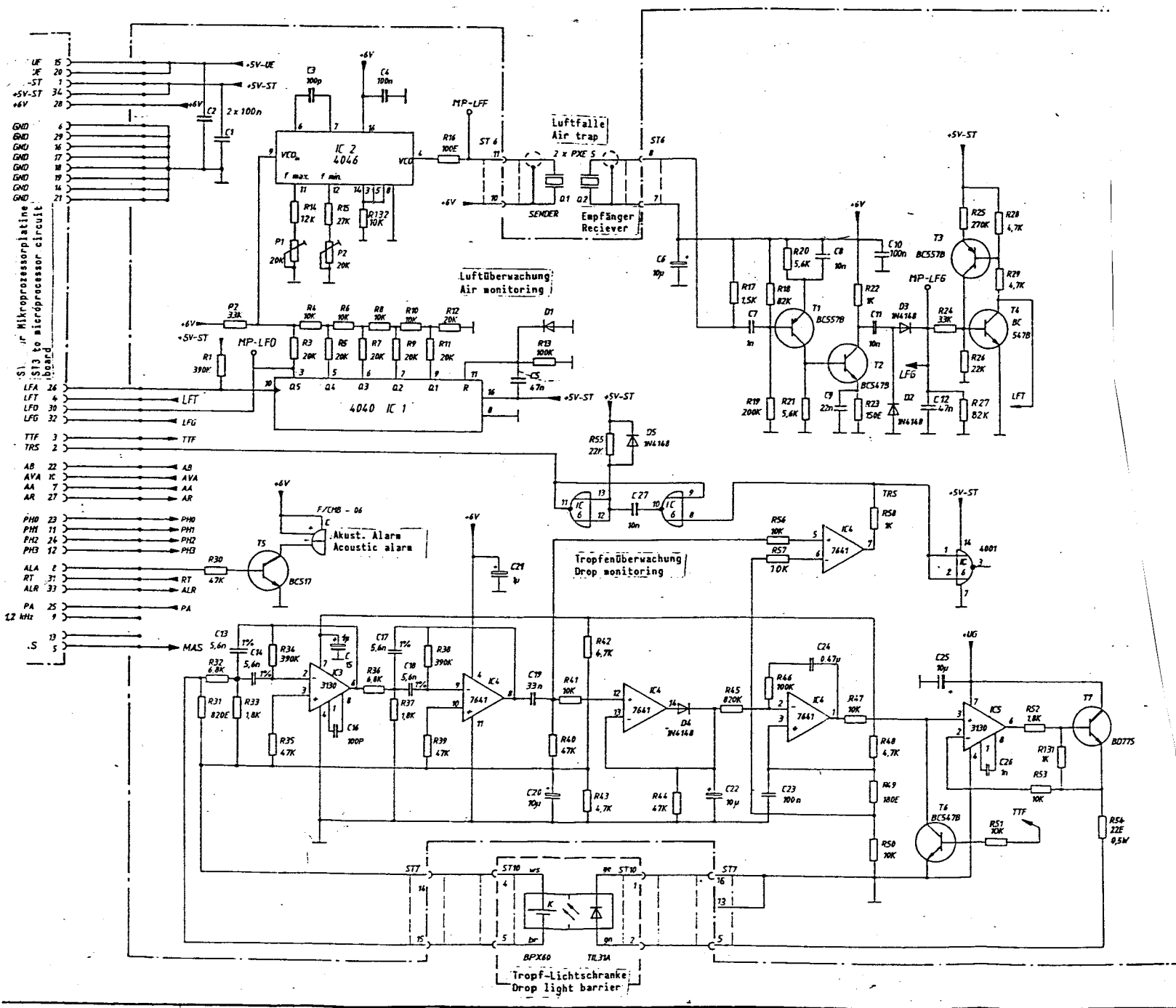
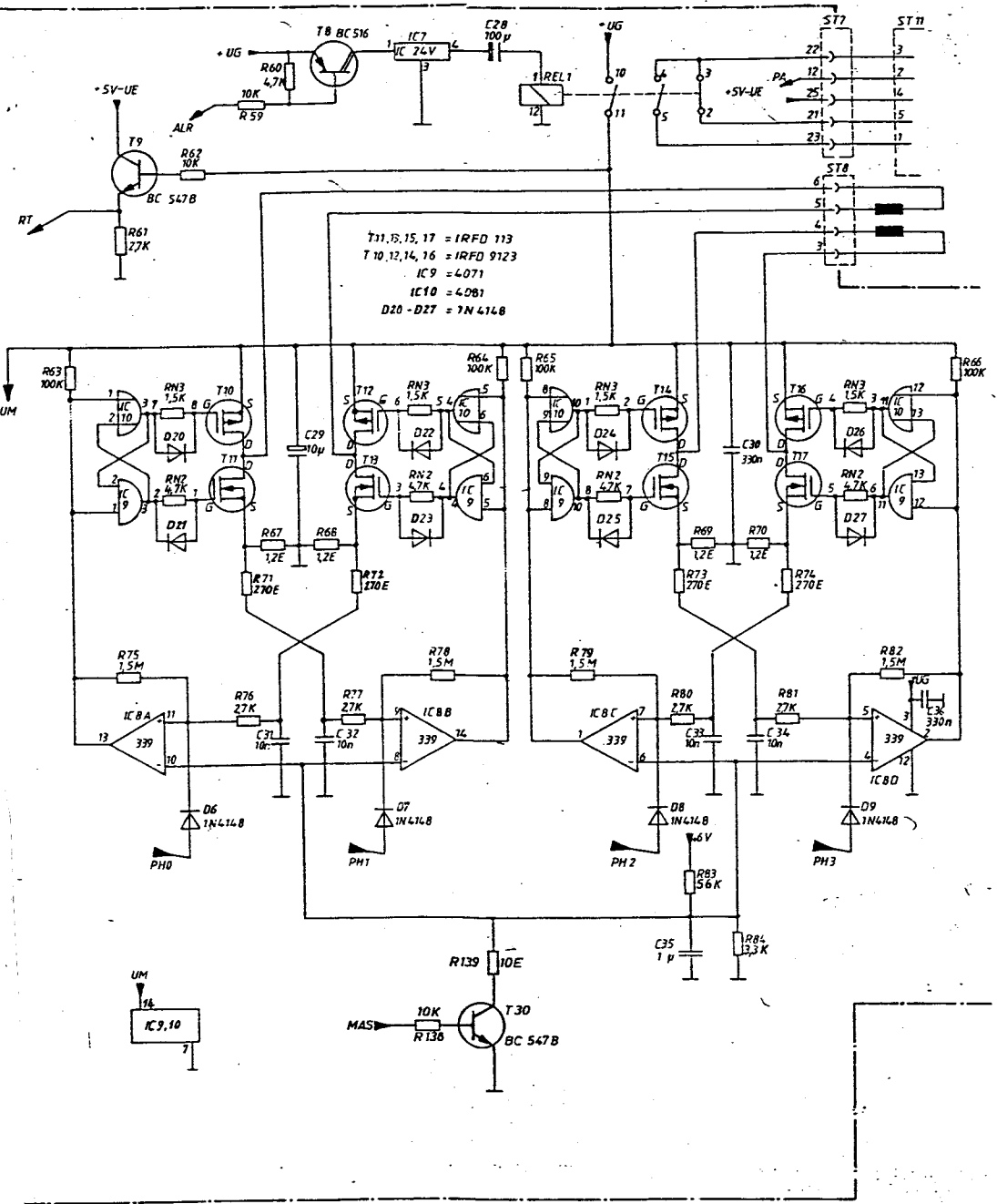
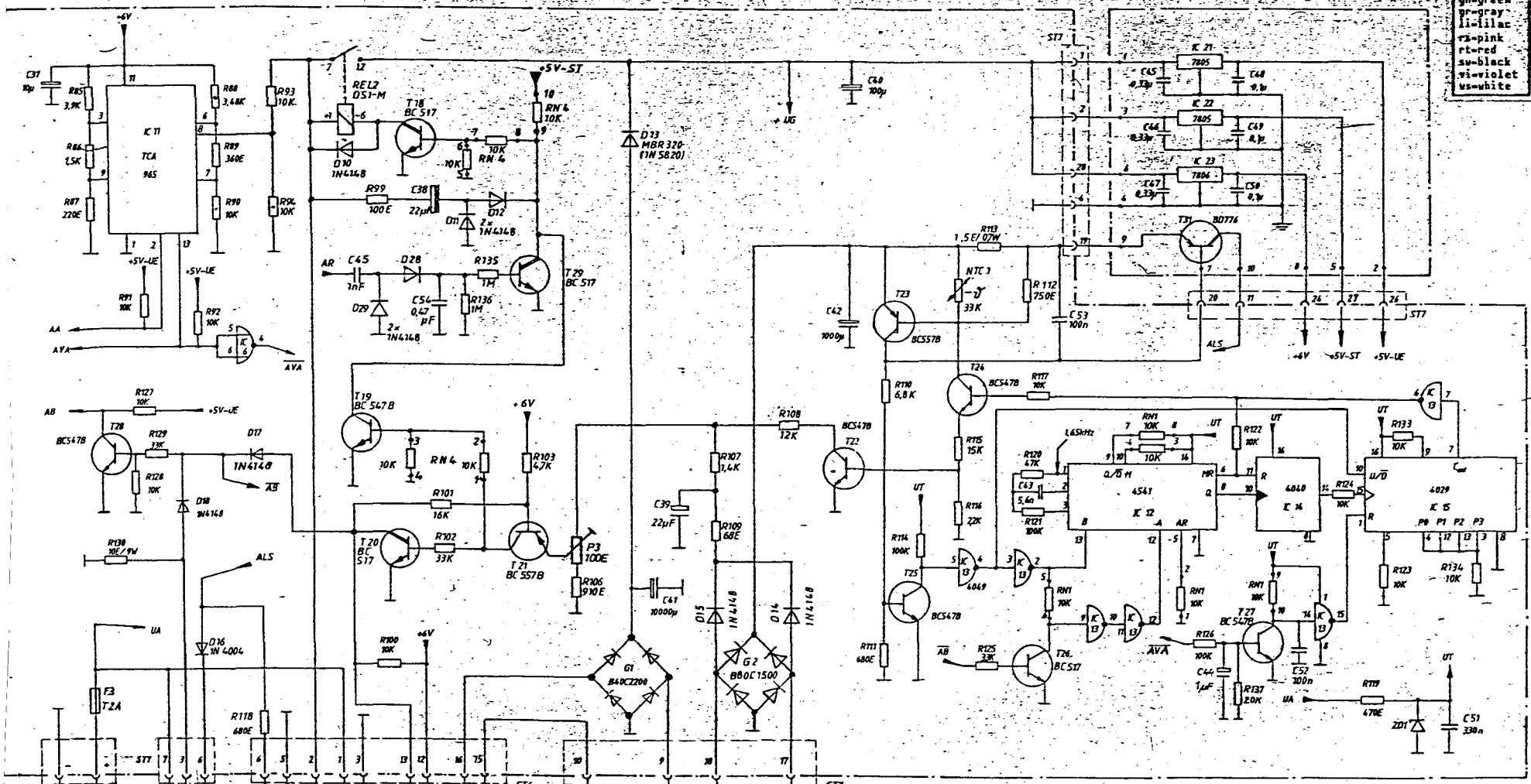
Colour key  
 bl-blue  
 br-brown  
 ge-yellow  
 gn-green  
 gr-gray  
 li-lilac  
 rk-pink  
 rt-red  
 sw-black  
 vi-violet  
 wh-white

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# INSPECTION WITH SERVICE

Manufacturer's recommendation for the INFUSOMAT-secura 999ml/h infusion pump  
Operator:

Fact.No.                      Appliance No.:  
Manufacturer: B. Braun Melsungen AG  
Year of purchase:

The INSPECTION WITH SERVICE is to be carried out according to the check list due reference to the service manual and the operation instructions subject to technical amendment and should be carried out within a period of 12 months.

## CHECK LIST

### Visual controls

- Cleanliness, completeness, damping, moisture attack and other influences  
# clean, if necessary repair
- Sealing membrane - finger pump  
- air trap  
# clean
- Keyboard, mains switch
- Function of pressure plate (if necessary elasticity)  
# clean  
Glue on silicon support
- Pump flap handle (cracking, holder function)  
Pump flap hinge (play-free suspension)  
# if necessary correct
- Safety clamp/free flow stop (function, gap width)  
# clean
- Mains supply (Mains cable and plug connection)
- Staff call (plug connection, if necessary line)
- Drop sensor (line, plug connection, strain relief, elasticity, function)  
# clean
- Support-rubber feet  
Support clamp and holder
- Fuses (2xT0.2A)  
# if necessary exchange

- 
- Function controls  
(all measuring values must be documented)
  - Type shield, labels  
# if necessary renew
  - Switch on unit (operation with test set)                      -- V
    - Mains supply,                      -- V
    - Display (self test, mains)
    - Delivery preset with display indication
    - Accu test key
    - Mains-Accu-Mains switch-over
  - Air sensor
    - Distance membrane/flap 1.5-1.7mm
    - 0.1ml-Air bubble (no alarm)
    - 0.4ml-Air bubble (alarm A50), if necessary repeat measurement
  - Drop sensor
    - Lock simulation (alarm A40)
    - Free-flow simulation (alarm A30)
    - Alarm if drop sensor is not connected
  - Staff call (function or contact pass-through)
  - Alarm suppressing (function)
  - Working pressure of the pump
    - max. 0.9 - 1.5bar                      -- bar
    - min. 0.5 - 1.1bar                      -- bar
    - (start at 400ml/h, check at 40ml/h the lock ability-Occlusion by pressure)
  - Safety clamp/Free-flow stop
    - Pressure check (no free-flow upto 0.4bar)
  - Accu capacity
    - During accu operation (30 min), no alarm should be given (integrate following controls)

- Open up unit
  - Visible inspection inside
  - # if necessary clean
  - Check conformity with service-manual, modifications, etc.
- Test with "Diagnosis unit"
  - Analog part: ALR UE / ALR ST / ALR UE-ST  
25ml - 400ml - ALR UE-ST  
TTF and ALR UE-ST / ALA / LFT / AB / AR  
TEST Accu-Alarm - prealarm 8.74 - 9.23V  
- end alarm 8.00 - 8.38V  
LFT-signal and circuit board control  
# if necessary adjust
- "Diagnosis": Led displays  
Voltage: +12V                      -12V  
                  +5V UE                      +5V ST                      +6V  
Display: 22 UE                      22ST  
                  TPS / TTF / LFT / AB / MF  
Key/switch: AB/ LFT/ RES UE/ RES ST/ RES UE-ST
- Open up pump flap at LFT-Signal
- Set up unit ready for use  
Seal
- Earth protection resistance  $\leq 0.1 \text{ Ohm}$                       -- Ohm  
(look for measuring points)  $\leq 0.2 \text{ Ohm}$  with mains cable -- Ohm
- Isolation resistance  $\gg 2 \text{ MOhm}$                       -- mOhm
- Leakage current  $\leq 0.03 \text{ mA}$                       -- mA  
(at normal conditions)  
if necessary substitute-leakage current  
(according to VDE 0751 part 1)
- Put into operation according to operating instructions
  - Delivery accuracy  $\pm 5\%$                       -- %
  - at approx. 50% of the max. rate
  - (if necessary repeat with new set respectively at 10 or 90% of the max. adjustable delivery rate)
  - Preselection of amount with alarm at the end of infusion
- Peculiarities/remarks  
e.g. - exceptional findings  
- Specifications, Accessories  
- if necessary pump holder, firm standing setting up at the location, ambient conditions, interface operation  
- Manufacturer's data, modifications  
- Unit book  
- Last repairs, if necessary main changes to above mentioned established facts  
- Dangerous deficiencies  
- Directive safety  
- Repairs under item R to be carried out/ carried out  
- To point (x) of check list (e.g. description of faults)

Documentation apart from Check list (i.e. unit book/customer service report)

The next inspection date is scheduled for the  
Unit handed over

The INSPECTION with SERVICE was carried out by  
Date/Signature

The order for this service was placed on

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© 1987 by B. Braun Melsungen AG. Alle Rechte vorbehalten. Nachdruck, Vervielfältigung und Verbreitung, auch auszugsweise, ist ohne schriftliche Genehmigung der B. Braun Melsungen AG.
B Braun Melsungen AG