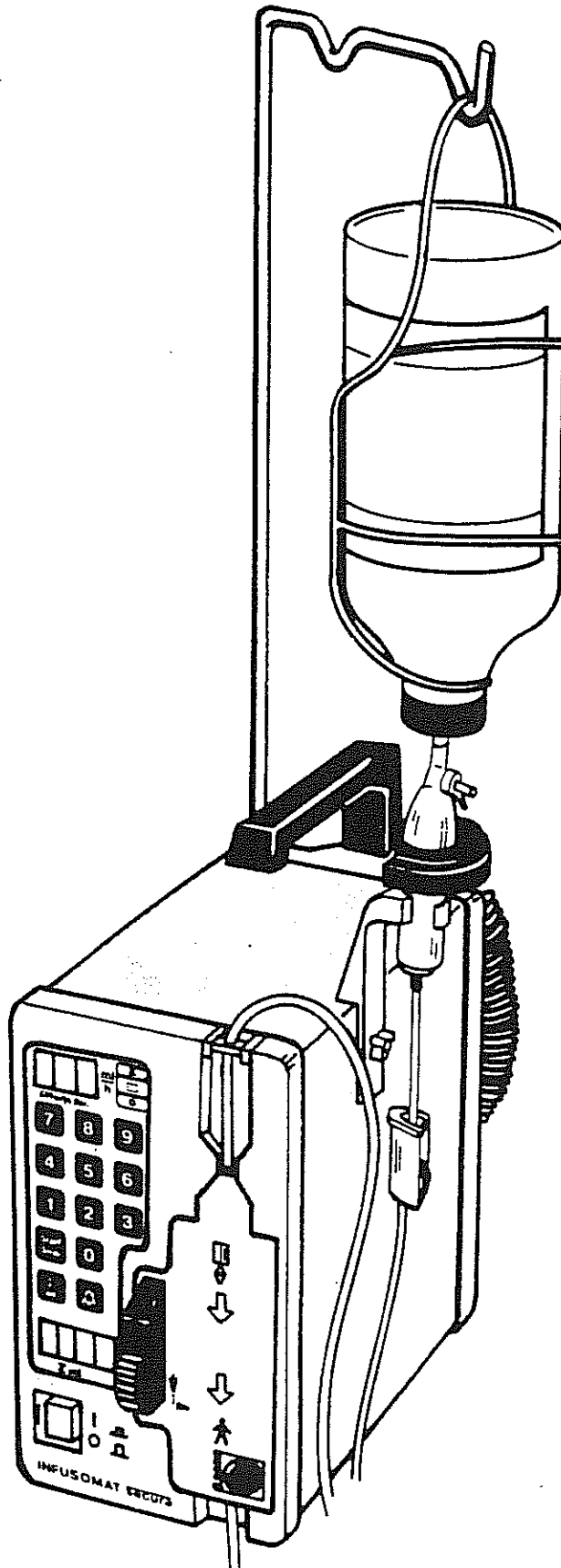
INFUSOMAT[®] **secura**

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Systembeschreibung Equipment Description

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Our drip-proof volumetric infusion pump INFUSOMAT®secura is based on the peristaltic principle (finger pump).

The INFUSOMAT®secura is equipped with a moisture-proof droplet sensor to ensure reliable infusion control as well as with a newly designed air detector system located after the finger pump.

In combination with the B. Braun I.V. set (disposable, pressure-proof, and price-worthy with fastening clamp), we are able to present to you a safe infusion system, operating in accordance with the latest state of technology:

The construction ensures simple cleaning and disinfection of the unit. TÜV-proofed micro-processor electronics integrated into the pump, guarantee safe and nearly maintenance-free operation.

Operational errors are indicated by visual and audible alarms. An electronic automatic control system serves to ascertain defects in the unit, to establish safe operating conditions, and to activate alarms. In case of mains failure, or during transport of patients, the INFUSOMAT®secura switches to accumulator operation.

By means of a mechanical free flow safety clamp, the infusion line is blocked and the infusion interrupted when the pump door is opened.

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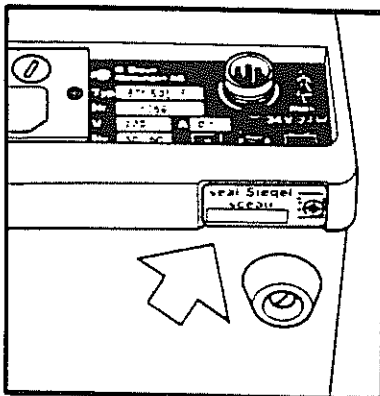
Kindly note all instructions given in these operating instructions

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
The INFUSOMAT®secura has passed the GS-safety test performed by TÜV-Rheinland and thus complies with the requirements for safety for electro-medical apparatus according to the international Standard IEC 601-1.

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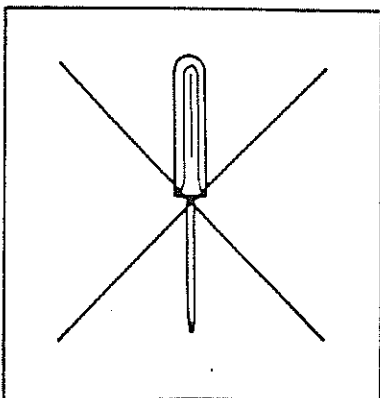
Repairs, alterations and modifications are only to be carried out by personnel explicitly authorized to do so by the manufacturer. Therefore, each unit is sealed with a test seal.

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In case non-authorized personnel carries out repairs or other manipulations, the manufacturer will not guarantee the safety, reliability and performance of the unit.

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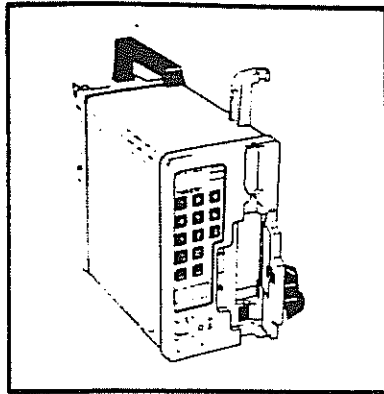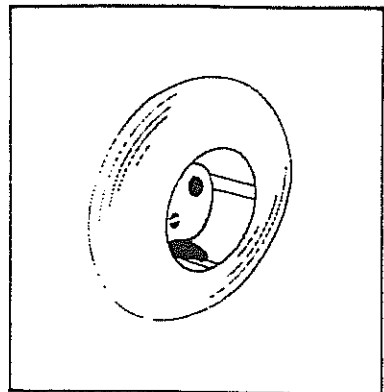
Unpack the unit and check for completeness or damage.

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Important

In case of damage do not start operations. Inform the authorized service station.

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
Electrical installations, in the room where the INFUSOMAT®secura shall be used, have to comply with valid requirements (e.g. VDE 0107). In case of doubt, please contact your electrician.

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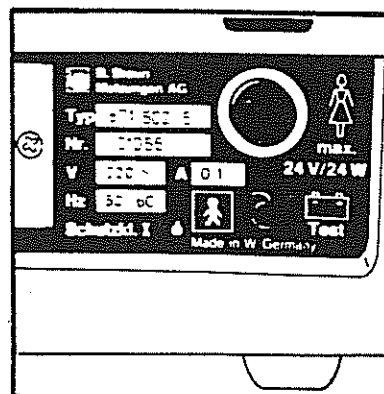The unit must not be operated in explosive danger areas.

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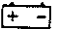
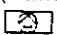
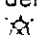
The mains voltage available must correspond to the mains voltage stated on the name plate (e.g. 220 V / 50/60 Hz).

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


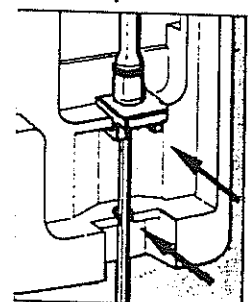
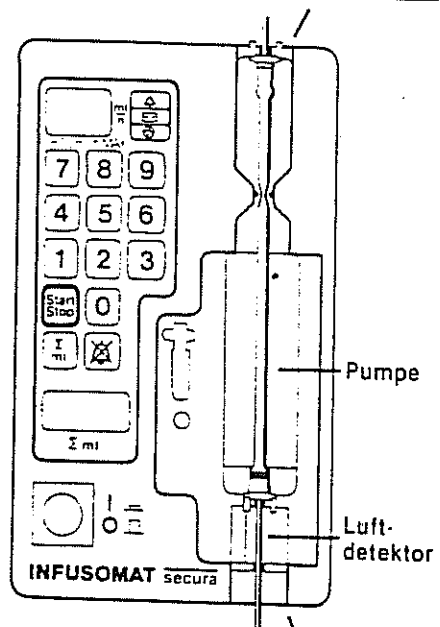
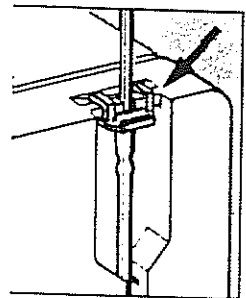
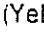
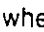
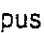
Short operating instruction

INFUSOMAT® secura Short Operating Instruction

(For complete informations please refer to the Operating Instructions Manual)

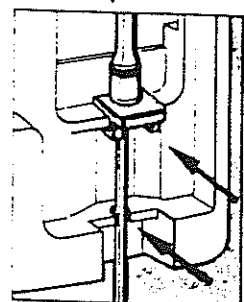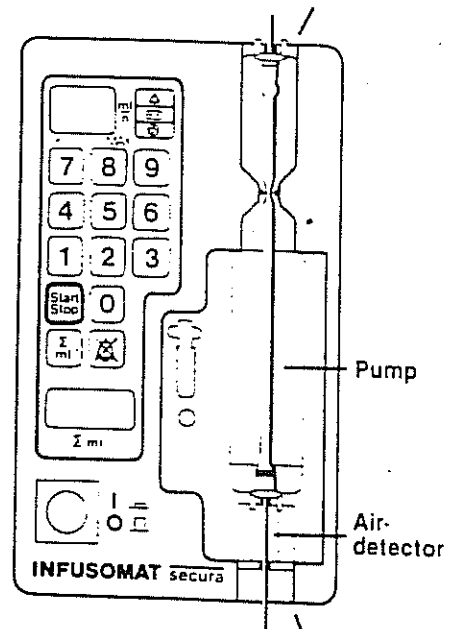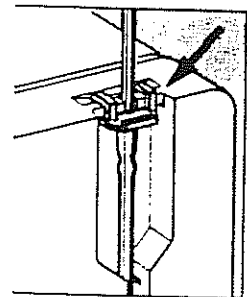1. Fill and deaerate the infusion giving set in the usual manner. (Use **only B. Braun Infusomat giving set.**)
Close roller clamp and make connection to the patient.
2. Place drop detector on drop chamber.
3. Install giving set in accordance with illustrations.
Attention: The reducer adapter must be positioned exactly.
Close the pump door and open the roller clamp.
4. Turn on Mains switch. (There will be about a 5 second delay while the internal safety Monitor program activates.)
5. Check battery condition. Battery test push button  is located on the back panel.
(Yellow Monitor lamp  will light.)
6. Select dosage rate 1 – 600 ml/h.
7. Select dosage to be given (Button Σ ml/0 – 9999 ml).
8. Start equipment with "Start" button. (Pump runs when the yellow operating lamp  illuminates.)
9. To stop equipment (i. e.: to change bottles) push "Stop" button.
10. Alarm tone muting
The alarm tone can be muted for 2 minutes by pushing button .
11. Taking equipment out of operation
Turn off Mains switch. (If possible keep Mains cable plugged in, so that the battery is recharged to full capacity.)


Alarm Indicators

- | | |
|--|---|
| A40 = Drop detector
(i. e.: Bottle empty) | AAA = Drop detector still on
+ handle cradle or
Equipment defect |
| A50 = Air in the system |  |

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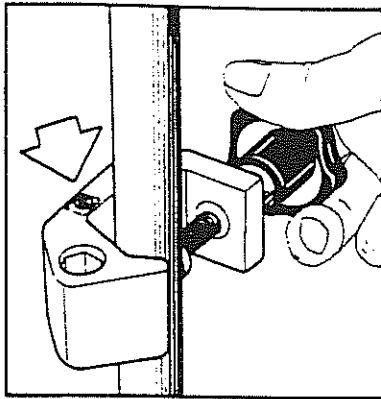
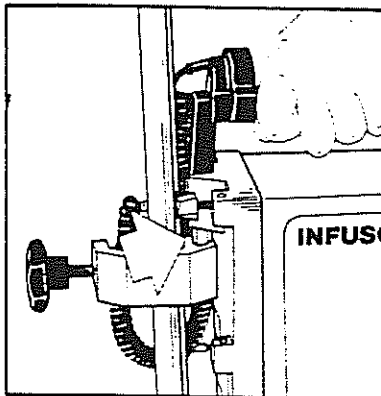
Operation on infusion stand
Tighten universal clamp (with letters showing in upward direction) on infusion stand.

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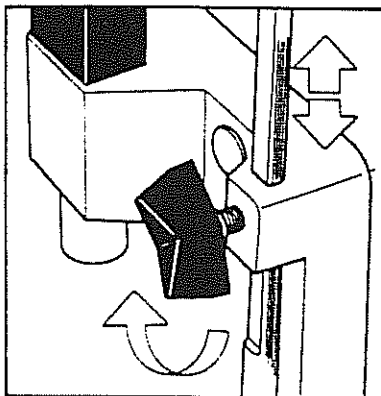
Insert unit into hexagon guide.
During transportation universal clamp may remain at unit without any risk of losing it.

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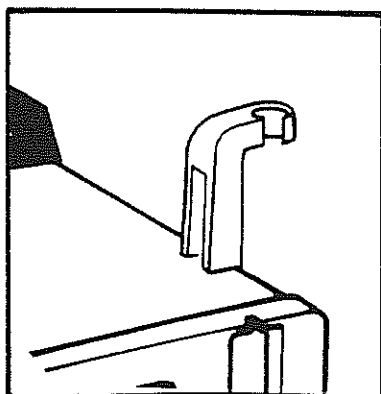Operation as desk instrument and during transportation of patients:
Attach small-sized stand according to the size of infusion bottle to be used.

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
Drop chamber support prevents angular positions and movement of the drop chamber during transportation.

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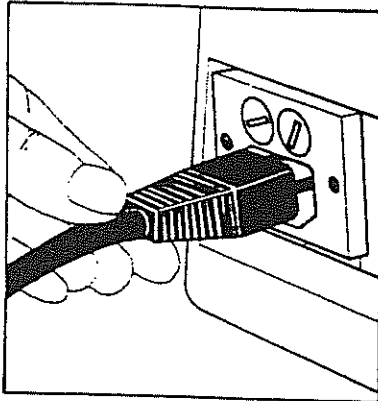
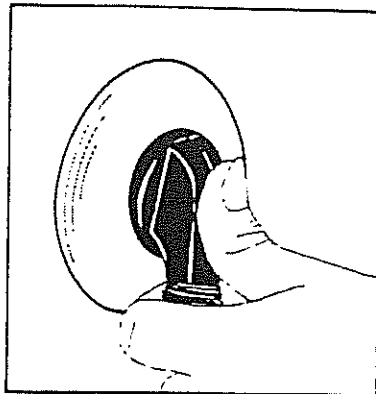
Make mains connection
by means of mains cable:
Insert appliance plug into
appliance socket to the
fixed stop.

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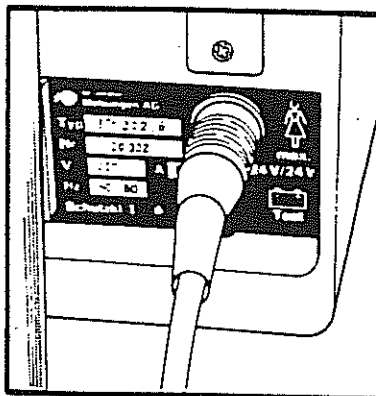Insert mains plug into
mains socket.

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
Connect to the hospital's
nurse call system
(if available)

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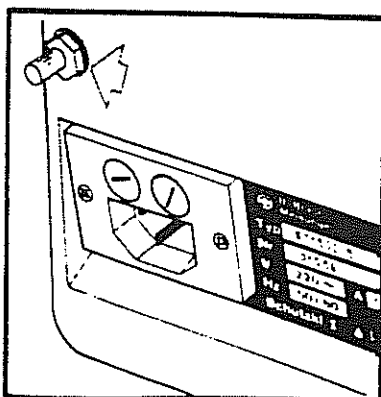Insert special cable
(optional accessory)
into connector.

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
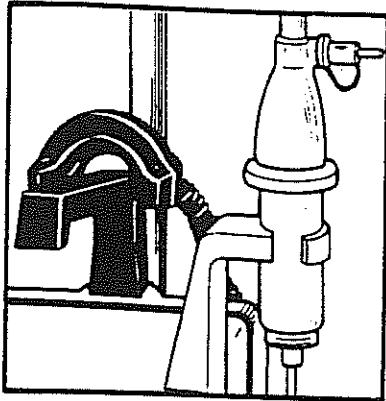
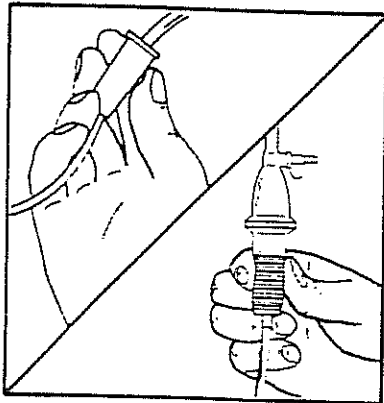
Attachment of droplet
sensor:
Insert drop chamber of
infusion set vertically into
infusion bottle.
Push drop chamber
into holder.

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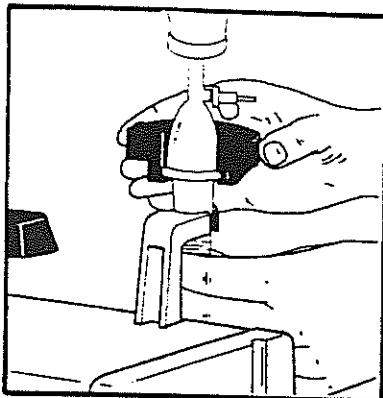
Fill drop chamber as usual
and deaerate.
Shut roller clamp.
Take care that the drop
chamber is filled to 2/3 only
(see ill.).

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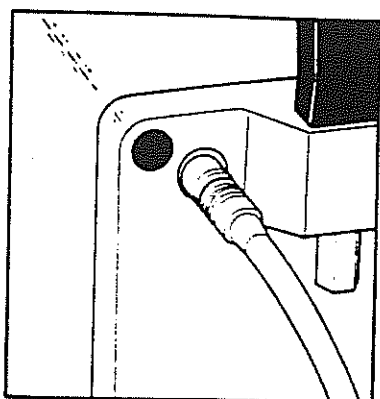Slide droplet sensor onto
drop chamber.

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
Check, if connector of
droplet sensor is connected
correctly.
If necessary, tighten the
connection.

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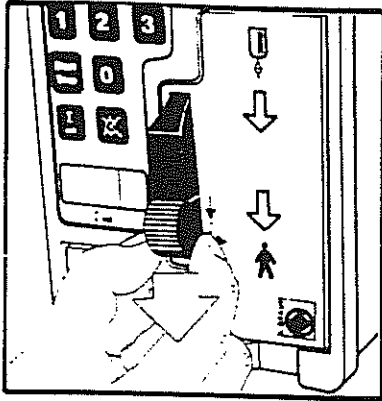
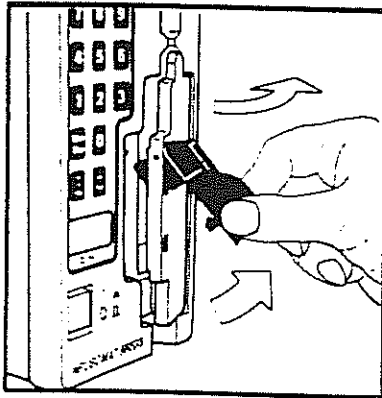
Insertion of infusion set:
Unlock pump door.

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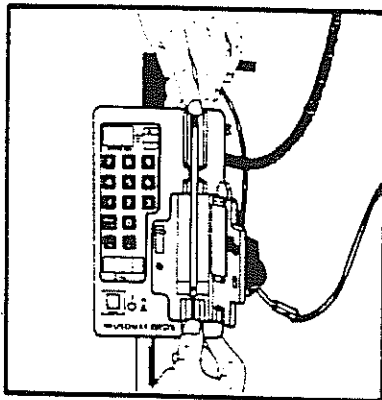
Swing out door interlocking
device.
Open pump door.

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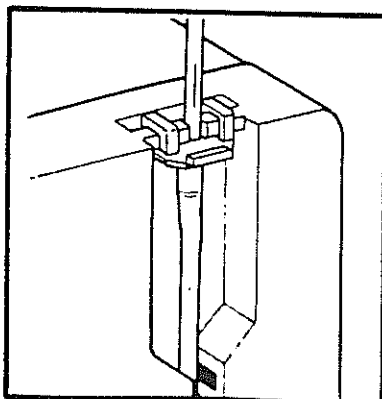Insert pump tube into
pumping chamber.
Pay attention to flow
direction!

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
Fastening clamp must be
inserted as illustrated!

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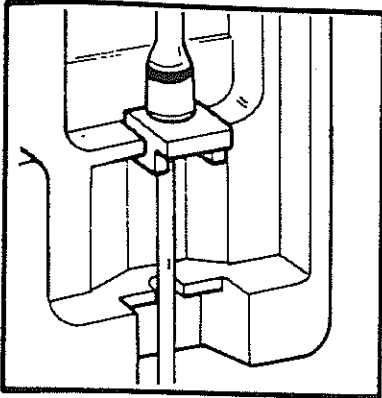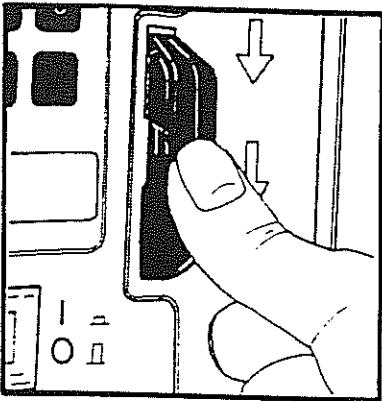
Important!
Press fastening clamp and polyvinylchlorid tube tightly into tube guides.

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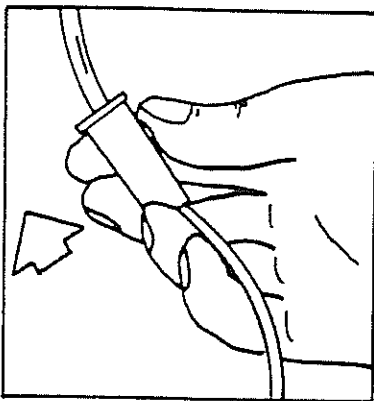
Important
Interlocking devise must snap in audibly.

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
The pump tubing will position itself properly when the pump door is closed.

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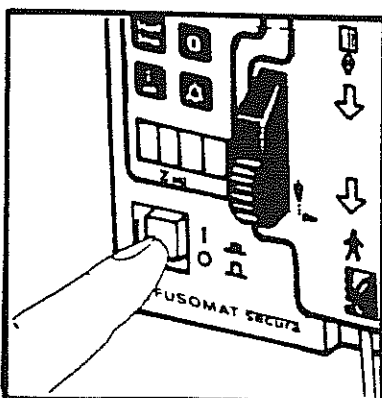
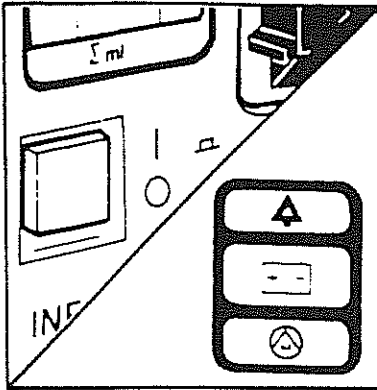
During mains operation, the green indicating lamp of the mains switch is illuminated.

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During accumulator operation, the yellow accumulator control indicator is illuminated. All operating steps are exactly the same as for mains operation.

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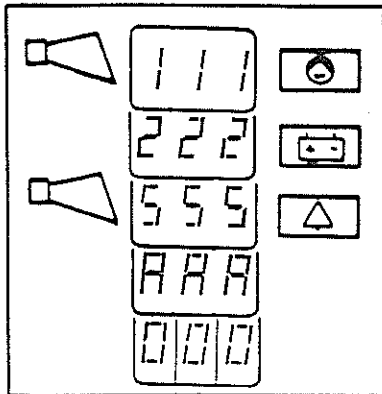Sequence of internal safety control.

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
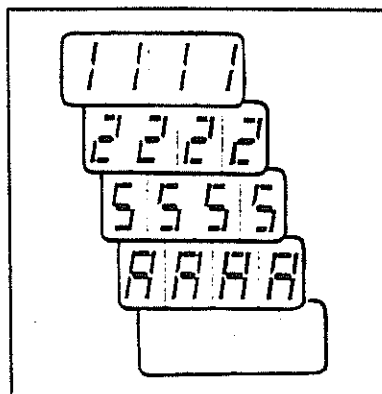
Important

Please monitor the operation of the internal safety control daily.

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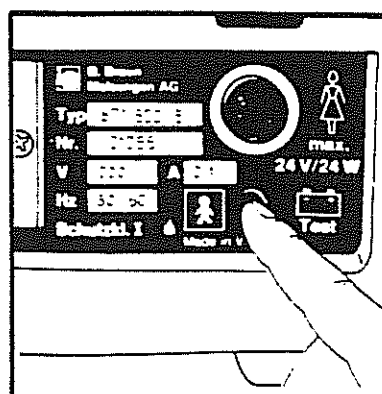
Test the accumulator capacity (actuate for 5 sec. max.)

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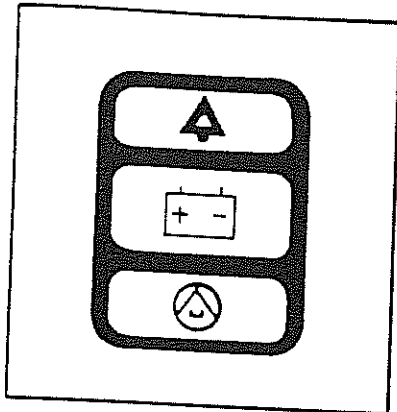
(siehe Seite 22)

Yellow continuous light means that there is sufficient capacity for short-time accumulator operation and alarm release in case of power failure. To attain full capacity, the unit must be charged for at least 20 h.

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
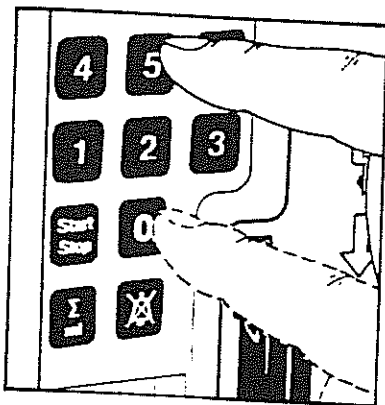
Input volumetric delivery rate:

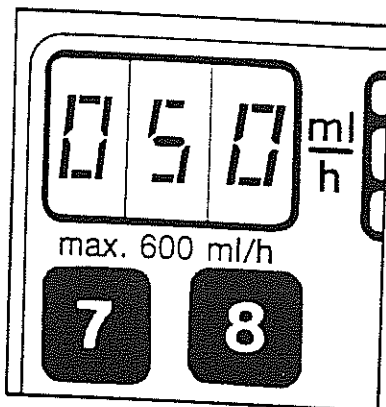Enter on the input keyboard the desired volumetric delivery rate between 1...600 ml
(e.g. 5 and 0 = 50 ml/h).

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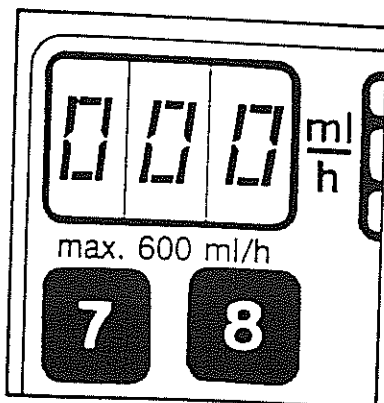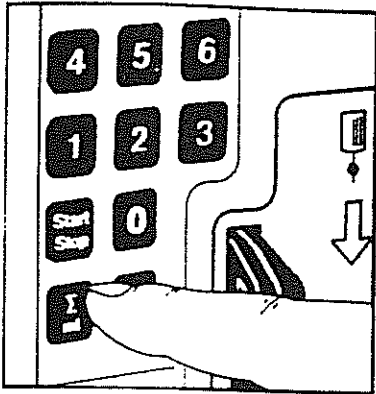
In case of misentry:

Reset to 000 by depressing any numeral key once or up to three times.

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Operational Procedures

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
Taste Σ ml drücken.


Input of total quantity to be delivered:

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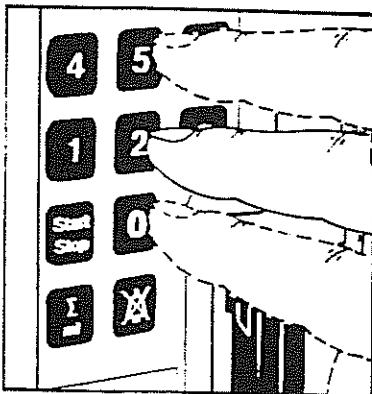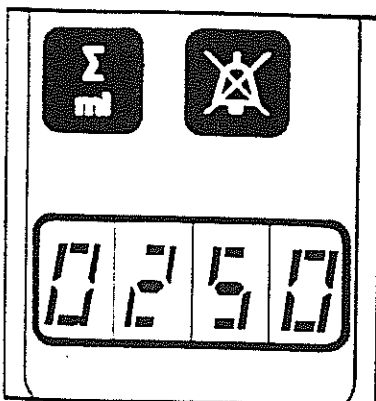
Enter on the input keyboard the desired quantity, between 1...9999 ml, to be delivered (e.g.: 2 and 5 and 0 = 250 ml).

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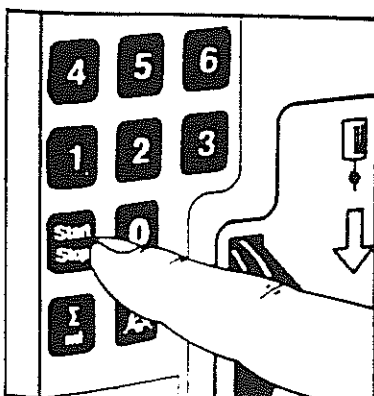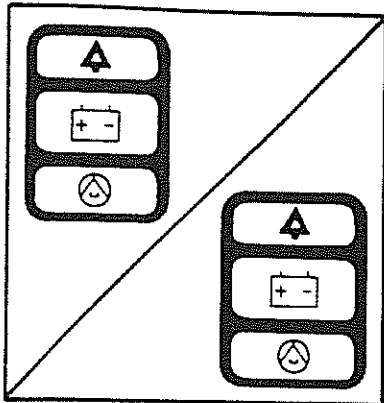
Starting infusion pump:

Depress start key.
Now pump is operating at the selected delivery rate.

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

Simultaneously the yellow control lamp is illuminated.

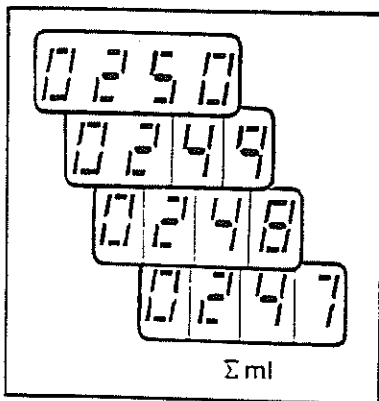During accumulator operation and/or mains failure, the accumulator control indicator is also illuminated.

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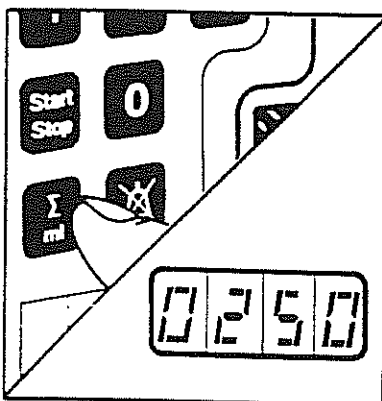
If the total quantity to be
delivered has been entered,
the bottom indicator will
show the remaining quantity
to be infused.
(e.g. 250 – 249 – 248, etc.).

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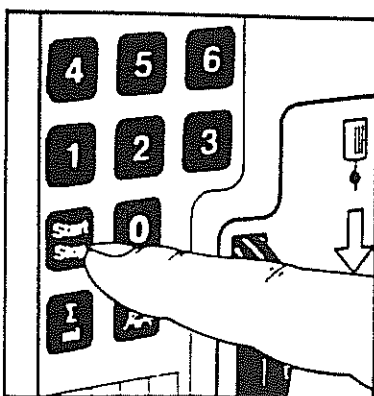The input quantity entered,
can be recalled by
depressing the Σ key at any
time during operation
of the unit.
e.g. 250 ml

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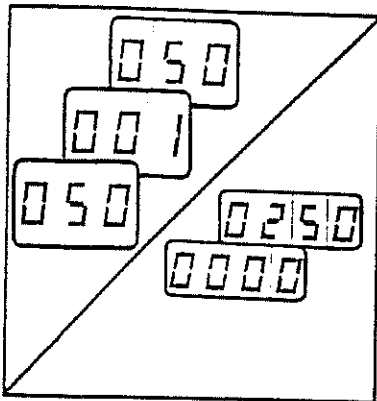
Changing the delivery rate
or the total delivery quantity.

To stop depress the
start/stop key. Enter the
new delivery rate and/or
quantity.

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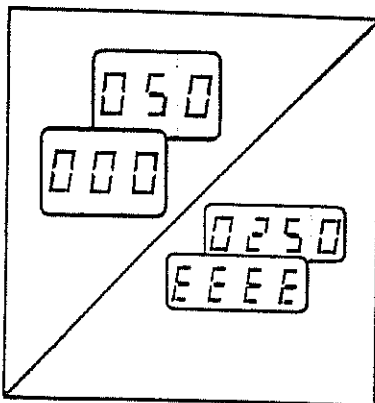
Having discharged the
preset quantity, the unit
switches to Keep-vein-
open-rate of 1 ml/h.
Indicator: Delivery rate
display.

Alternate display of delivery
rate and 001 ml/h.


Indicator: Total quantity
display.

Alternate display of total
quantity to be delivered
and 0000 ml.

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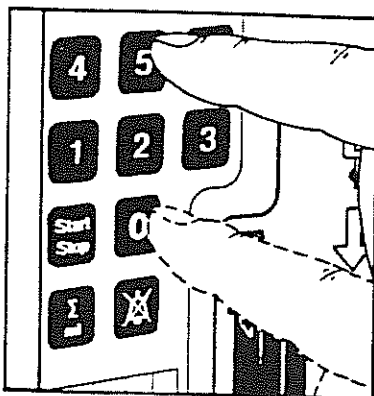After 30 minutes, the pump
stops and a nurse call
system is activated.

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
Indications during
accumulator operation.

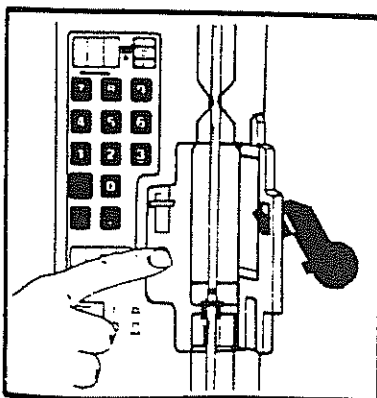During accumulator
operation indications are
only switched intermittently.
However, at any time, data
can be recalled by
depressing any desired key
(except the start and
stop key).

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
The tubing is automatically
clamped off when the pump
door is opened.

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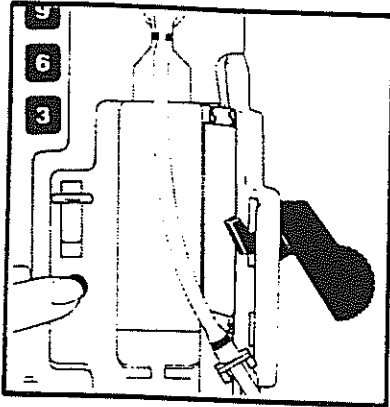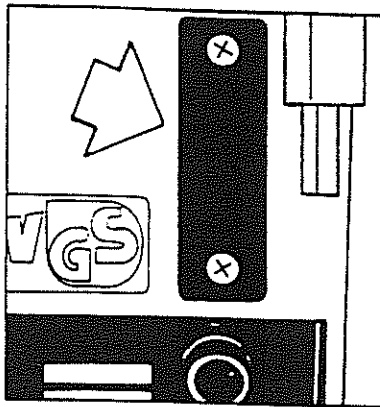
Removing the tubing system: Push in actuating shaft and remove tubing from bottom to top.

Anschluß des INFUSOMAT®secura an externe Rechner.

Connection of INFUSOMAT®secura to external computer:

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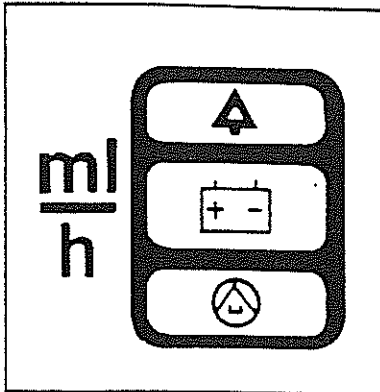
The unit is prepared for connection to external computers. For safety purposes the unit shall be operated only under supervision.

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Please, ask for special information.

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
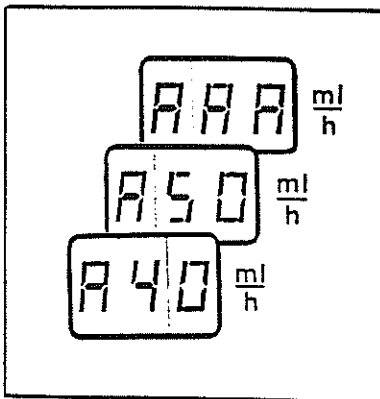
General visual alarm indication: red.

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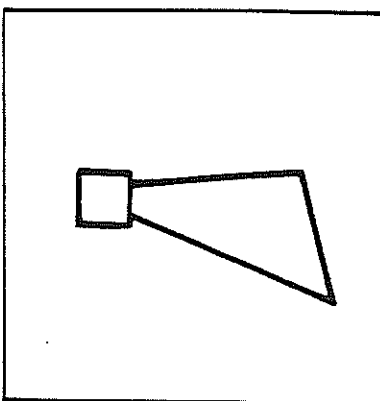
Visual alarm indication on delivery indicator.

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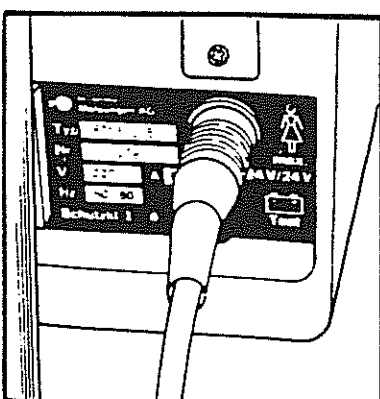
Possibility of connection to central remote alarm system (nurse call system).

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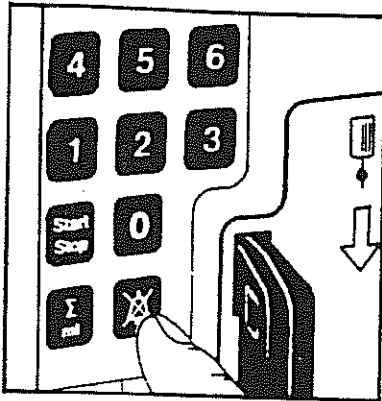
Muting the alarm.

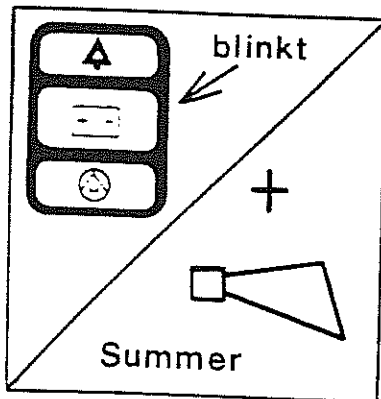By actuating the alarm muting key, the audible alarm can be interrupted for 2 min. (e.g. when changing bottle, clear faults, etc.).

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
Alarm indication during accumulator operation.

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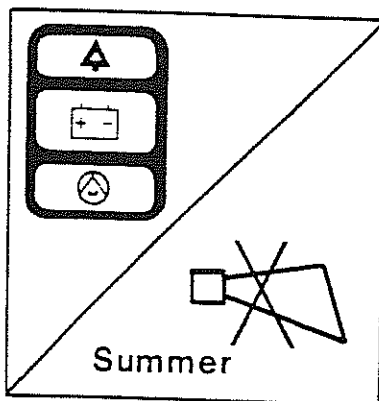Accumulator capacity exhausted Pump switches off.

No visual indications = no accumulator capacity.

Required recharging period at least 20 h.

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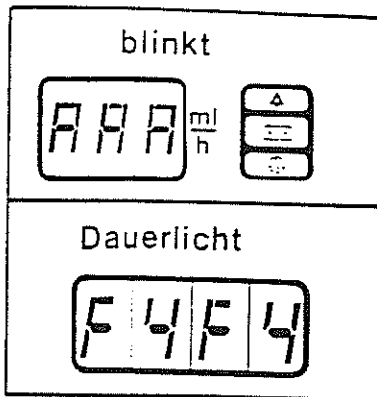
Important

The built-in accumulator must be safety checked every 6 months. To accomplish this check, the accumulator must be charged for a minimum of 20 hours, and then operated in the accumulator mode for 30 minutes. If during that time, no accumulator alarm occurs, then the built-in accumulator is considered to be operationally effective.

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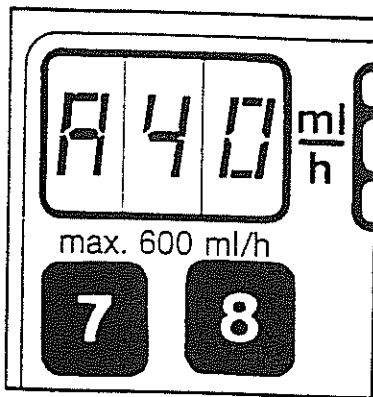
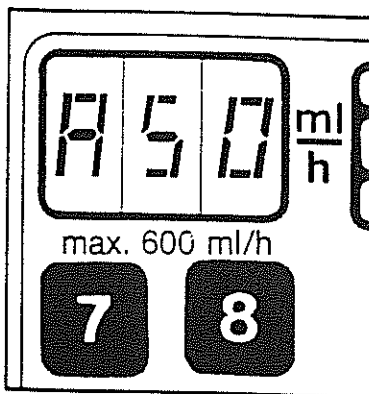
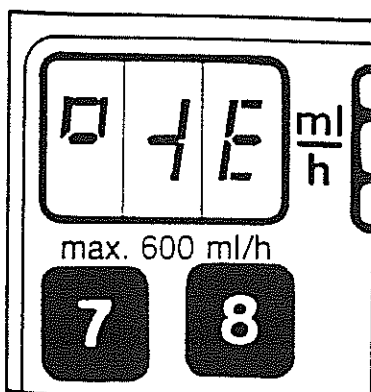
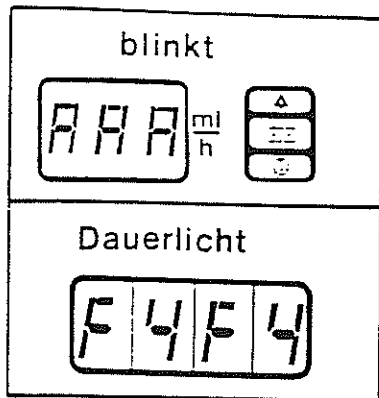
Faults and fault clearance

Fault

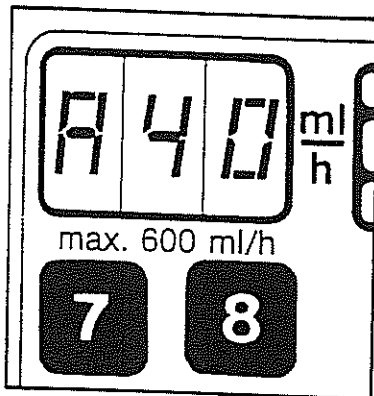
Droplet sensor still on holding device (carrying handle) – not connected.

Clearance

Mount or connect droplet sensor.


Droplet detector defective.

Replace droplet sensor.

Infusion container empty.
Droplet sensor tilted.
Roller clamp closed.
Drop chamber extremely clouded.

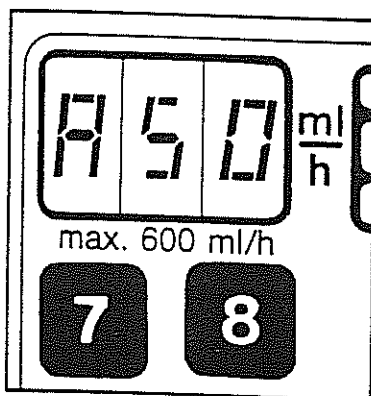Replace infusion container.
Fix drop chamber vertically.
Open roller clamp – remove mist by shaking the drop chamber.


Air in system.
Infusion set not correctly inserted in pump.

Deaerate tube again.
Insert infusion set according to abridged operating instructions.

Pump door not tightly closed.

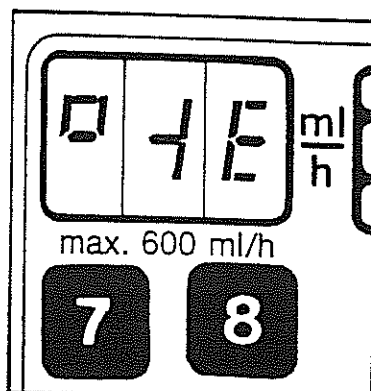Close pump door.


Unit defect (example).

Please bring unit to authorized service station.

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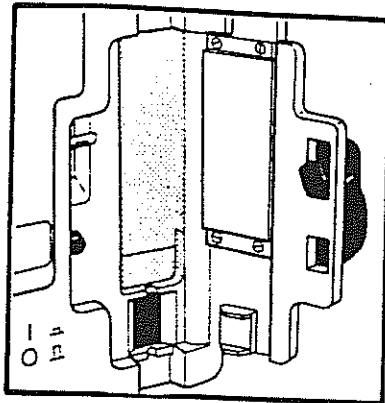
Maintenance

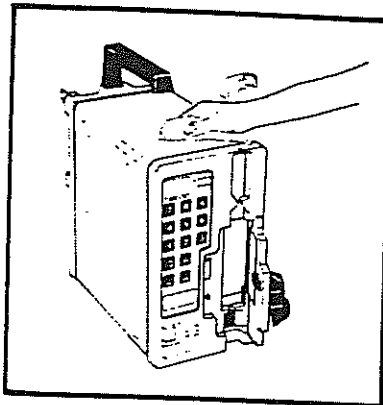The diaphragms covering the pump assembly and the Air Detector sensors should be cleaned with warm soapy water and checked for damage.

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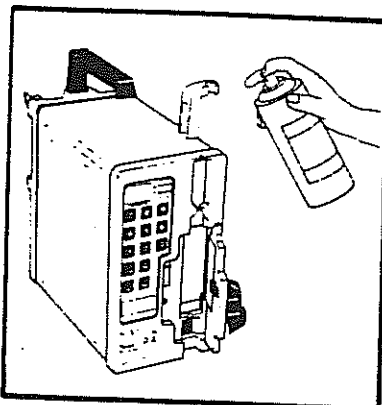
Cleaning

Important: disconnect mains plug and switch off unit.

Droplet sensor can be removed from unit.

Front part and enclosure are tightly sealed. The unit can be wiped off using a damp cloth and a mild detergent.

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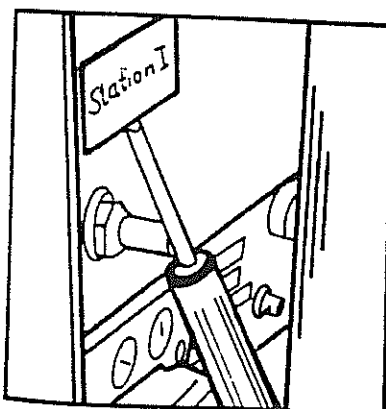
Disinfection

Important: switch off unit and disconnect mains plug.

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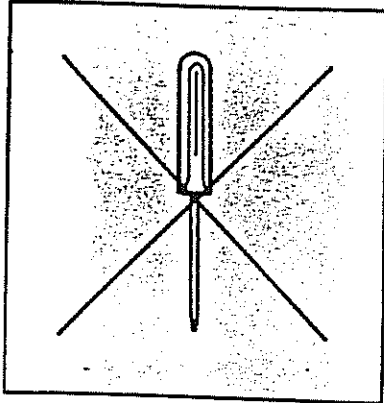
Identification

You may fix your own identification on the rear part of the unit (ward, etc.)

Protective cover can be removed by means of a screw driver.

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Service

According to legal regulations and safety requirements, service work on the INFUSOMAT®secura is only to be performed by us or by persons authorized by us in writing, to perform the service. (Kindly note section: safety instructions).

Warranty

Each INFUSOMAT®secura is guaranteed by B. Braun Melsungen AG as the manufacturer, for 12 months from delivery date. Our warranty includes repair or replacement of defective parts due to constructional, manufacturing or material defects. The purchaser loses his rights under the guarantee, if he or third persons carry out alterations or repair work on the unit and if the seal is broken. The warranty does not include: Repairs of defects caused by faulty manipulations, unqualified handling or fair wear and tear.

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Technical Data

Mains supply	220 V ~ 50/60 Hz	Delivery range:	1...600 ml/h (in 1 ml/h steps)
Power consumption	25 W	Delivery quantify pre-selection:	1...9999 ml (in 1 ml steps)
Protection class	I	Dimensions of enclosure	(length x width x depth) 130 x 270 x 260 (mm)
Type of equipment	BF (acc. to IEC 601-1)	Painted surface	dark green RAL 6010
Radio interference rate	N (acc. to VDE 0875)	Weight	6,8 kg
Moisture protection	drip-proof	We reserve the right to make constructional changes constituting technical improvement	
Duration of operation	100%	Accessories and Order Data	
Appliance fuse	2 x M 0,2 A DIN 41571/T 2	Order Numbers:	
Permissible relative air humidity	30...75%	871 502/5 INFUSOMAT [®] secura continuous infusion pump, volumetrical, delivery rate adjustable. 1...600 ml/h for 220 V 50/60 Hz	
Permissible ambient temperature	+10...+40°C	870 003/6 INFUSOMAT I.V. set Luer-Lok	
Nurse call system	suitable for closed- circuit and open- circuit equipment; max. 1 A/24V	870 160/1 Connecting cable for call system, 3-phase	
Accumulator capacity	sufficient for approx. 3,5 h operating time	Spare Parts	
Accumulator type	cadmium-nickel 4,0 Ah 8 x 1,2 V (rechargeable)	3823542/0 Universal clamp	
recharge time	>20 h	3317301/0 Indicator lamp for mains switch.	
Setting accuracy of delivery rate	± 5%		
Max. overpressure at pump output side	1,3 bar		
Air control after pump	alarm activates at ≥0,3 ml air bubbles		
Earth leakage current	< 10 µA		
Potential equalization conection	post DIN 42801/15 to Infusion stands or beds		
Possible mounting	Ø 10...Ø 40 mm (to wall track systems – see accessories)		

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