Operating manual

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**ACROBAT
swing**

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1 Instructions for safe usage

Dear operator,

Please note that certain operations must only be carried out by personnel with the required qualifications:

- The equipment may only be operated by persons with medical knowledge,
- The equipment may only be cleaned by trained cleaning personnel.
- These operating instructions describe the ACROBAT Swing tripod-mounted, wall-mounted and ceiling-mounted equipment.
The end devices (e.g. OP lights, lamps etc) are supplied by other manufacturers. For using such equipment, please refer to the operating instructions of those manufacturers.
- Please read these operating instructions carefully before using the equipment. You will then be able to exploit all the advantages that the equipment offers, and at the same time, protect yourself and others from injury.
- The equipment is only intended for use for the purposes described in the operating instructions. Any other application can cause injury or death and jeopardize the equipment and other material assets of the user.
- If problems occur that are not adequately described in these operating instructions, please do contact your supplier for your own safety.

Hand-over declaration for the user:

I have read and understood the operating instructions. I am conversant with the safety instructions. I have been adequately instructed in preventive measures for avoiding the dangers and am following these operating rules when using the machine.

Please keep these operating instructions carefully in the vicinity of the equipment so that you can later refer to safety instructions and important information at any time.

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1 Instructions for safe usage

Dear user:

Please note that certain work may only be carried out by suitably qualified personnel.

- The work in chapters 5 and 6 may only be carried out by a hospital technician or a person with comparable qualifications.
- This equipment is state-of-the-art and is operationally safe. Nonetheless, it may present some dangers, especially if it is operated by inadequately trained personnel or if it is used improperly or not used for its intended purpose.
- With the help of these operating instructions, please instruct the persons responsible for operating and cleaning in the working, operation and maintenance and care of this machine.
- The contents of these operating instructions can be changed at any time without prior notice.
- In case of translations in other languages, ONLY the German version of these instructions is legal and binding.
- Unauthorized changes made by the user or modifications to the equipment are not permitted for reasons of safety.

Please keep these operating instructions carefully in the vicinity of the equipment so that you can later refer to safety instructions and important information at any time.

Trademarks:

- ACROBAT Swing® is a registered trademark of ONDAL Industrietechnik GmbH.
- All other trademarks mentioned in these operating instructions are the exclusive property of their respective owners and are acknowledged as such.

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1 Instructions for safe usage

1.1 Explanation of symbols

In the operating instructions, important instructions have been marked with symbols. The pictograms have the following meanings:

Symbols in the operating instructions:

WARNING!

Disregarding this notice is likely to result in serious or even fatal injury.

CAUTION!

Disregarding this notice is likely to result in medium to light injury or damage to property.

HINTS!

Provides usage tips and useful information.

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1 Instructions for safe usage

1.2 General Safety Indications

Please also note the special safety instructions in the following chapters.

Usage conformance with intended application:

- The Acrobat equipment is used for supporting and positioning medical examination lamps, operation-theater lamps and flat screens up to a maximum weight of 6 kg and for supplying the equipment with power. Depending on the transformer version mounted, a 12V and 24 V (without a transformer, 230V); 50/60 Hz supply is available (cannot be optionally adjusted).
- The equipment is suitable for continuous operation.
- The equipment must not be loaded over the maximum permitted load of 3 kg / 6 kg (see chapter 6.2 "Installation of end-devices").

Operating conditions:

- The ACROBAT Swing tripod and wall equipment should only be connected to a properly installed and grounded socket with protected contacts.
- The equipment is not suitable for operation in areas where there is a danger of explosion.
- The equipment is not suitable for usage in combustible mixtures of anaesthetic agents with air or oxygen or laughing gas.
- Operating temperature: +10 to + 40° C
- Relative humidity: 30 - 75%
- Air pressure: 700 - 1060 hPa

ENVIRONMENT:

- For the disposal of this equipment: public laws may contain special specifications with regard to disposal.
To avoid damage to the environment and health, we therefore request you to consult your supplier when you decommission the equipment for disposal as scrap.

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2

Using the ACROBAT Swing Equipment

 **NOTE: Qualifications of the personal:**
The equipment must be operated by medically qualified expert technicians in hospitals and in physician's clinics.

2.1 Components of the equipment Fig.1

Ceiling equipment:	Wall-mounted equipment:	Tripod equipment
• Ceiling bearing (1)	• Power Plug (5)	• Spring arm (9)
• Space tube (2)	• Wall bearing (6)	• Tripod tube (10)
• Extension arm (3)	• Extension arm (7)	• Tripod bse (11)
• Spring arm (4)	• Spring arm (8)	• Rollers (12)
		• Power plug (13)

2.2 Usage

 **NOTE:**
Because of the danger of the tripod mounting tipping over, care must be taken that it should not roll over any objects on the floor or the connecting cable.

1. Move the tripod equipment to the location of usage and **secure the two fastenable rollers (12)**.

WARNING - Electrical shock:

In case of contact with damaged electrical parts, there is a danger of an electrical shock. In case of any damage to the power supply cable or plug, do not connect the equipment to the power supply and immediately inform the service department.

2. Check the **power plug** and the **connection cable (5 / 13)** for damage.
3. The power plug should be inserted into a contact-protected power socket that has been installed and grounded in accordance with regulations.

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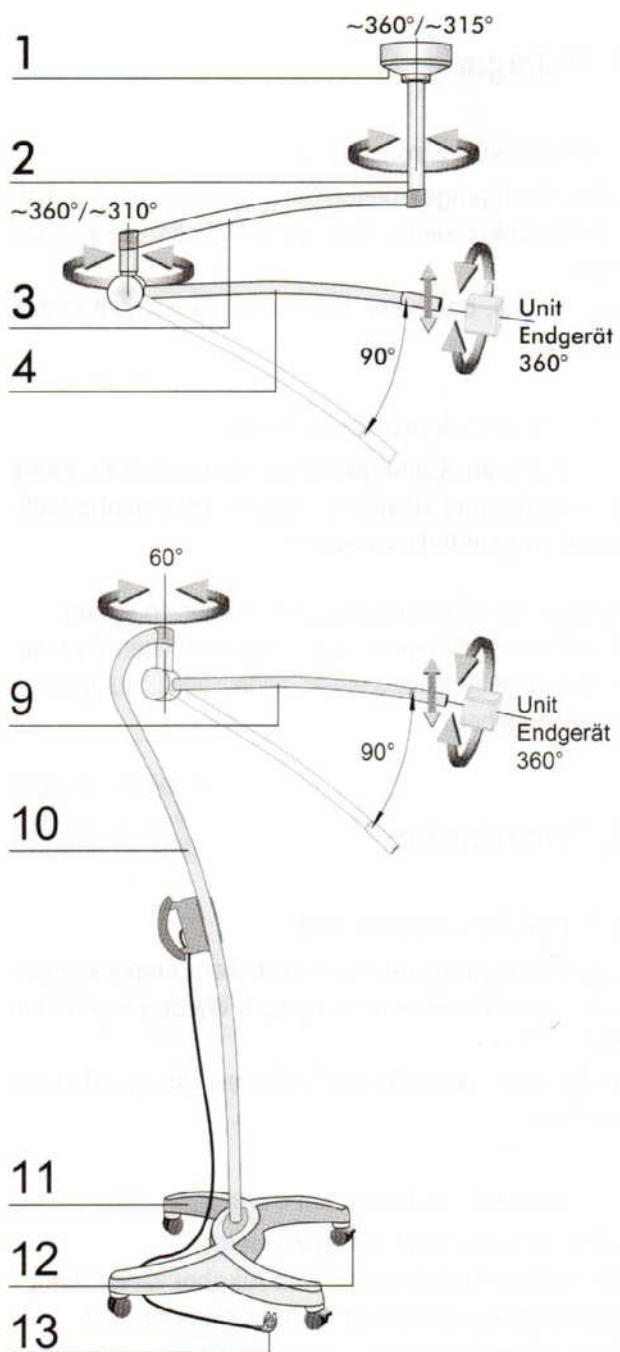
3

Cleaning and disinfection

 **NOTE - Qualifications of the personnel:**
Cleaning and disinfection must only be carried out by trained personnel

3.1 Cleaning

 **WARNING - Electrical shock:**

For all cleaning work, power off the equipment, pull the plug and secure it from being switched on again.
Take care that no cleaning fluid runs into the equipment.

 **CAUTION - damage to the machine:**

To avoid damage to plastic parts, do not use any scouring agents and alkali, acidic or alcohol-containing cleaning agents.

- Pull the power plug and let the end-device cool.
- Wipe the surfaces with a lightly moistened piece of cloth, if required, add a mild solution of soap (washing agent).

3.2 Desinfection

 **WARNING - Danger of electrical shock:**

For all cleaning work, power off the equipment, pull out the plug and secure the equipment from being switched on again.

Ensure that no cleaning fluid flows into the machinery.

 **CAUTION - Damage to the equipment:**

To avoid damage to the housing:

- Do not use any disinfectant containing alcohol,
- Disinfect the equipment only when it is cold,
- Do not use any strongly alkali solutions, acids and pure alcohol.

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Tests and Maintenance

4

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☞ NOTE - Qualifications of the personnel:

Cleaning and disinfection must only be carried out by trained personnel

4.1 Tests

☞ NOTES - Tests:

The user must check all ACROBAT Swing products with regard to the following points:

⚠ WARNING - Electrical shock:

For all testing work, power off the equipment, pull out the plug and secure the equipment from being switched on again accidentally.

Twice a year:

- Damage to paint
- Cracks in plastic parts
- Deformation of the support system

Annually:

- Check the fastening bolt on the lower side of the tripod base and if required, tighten it.
- Check the securing segment according to chapter 5.2, „Greasing the securing segment“, and grease it.

In case of any damage or faults, please inform your supplier.

4.2 Maintenance

☞ NOTE - annual maintenance:

All ACROBAT Swing products should be checked once a year with regard to the following, and maintenance carried out accordingly:

- Function test
- Electrical safety test
- Checking the support system

Your supplier is informed and trained as regards the scope of the maintenance work.

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Maintenance

5

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NOTE - Qualification of the personnel:

The maintenance and care work must be carried out by a hospital technician (or a comparably qualified person) only.

5.1 Adjusting the spring force Fig.2

Like every technical component, springs are subjected to natural wear. Thus, the spring force may give way and reduce after long operation, and has to be re-adjusted.

Adjust the spring force in such a way that, the spring arm with end device comes to a standstill in every desired position.

CAUTION - Destruction of the spring arm:

The spring tension may only be adjusted in the 20° downward position.

1. Pull the end-device about 20 ° C till the **hole 1** is at the top and a screw is visible in it.
2. Insert a slotted-screwdriver in **hole (1)**.

CAUTION - Destruction of the spring arm:

Driving in the brake screw too deeply destroys the spring arm. Tighten the brake screw with care while continually checking the braking force.

If the spring arm drops - the spring force is too low.

- The adjusting screw must be rotated to the left (in the counter-clockwise direction).

If the spring arm goes upward - the spring force is too high:

- the adjusting screw must be rotated to the right (in the clockwise direction).

5.2 Greasing the securing segment

CAUTION - Damage to the equipment:

Lubricant will destroy the plastic spring arm pin.

Do not grease the plastic spring arm pin.

1. Dismantle the end-device as described in chapter 6.1.
2. Check the securing segment for a minimum thickness of 1.5 mm and if required, replace it.
3. Grease the securing segment and the pin of the end-device.
4. Mount the end-device as described in chapter 6.2, „Mount end-device“.

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NOTE: Qualifications of the personnel

The assembly / dismantling must be carried out by a hospital technician (or a comparably qualified person).

6.1 Dismantling the end-device

WARNING - danger of injury

The spring arm is under a high spring tension. If the end device is not dismounted in the uppermost position of the spring arm, it rocks upwards and can result in serious injury.

Dismount the end device only when the spring arm is in the uppermost position.

6.1.2 Variant for end devices with 1-3 kg weight Fig. 3

1. Power off the equipment/ pull out the power plug and secure it from getting switched on again.
2. Push the spring arm in the uppermost position.
3. With a tool, lift the **interlock** (1) on the **plastic ring** (2), so that the **plastic ring** (2) can be rotated.
4. Rotate the **plastic ring** (2) through 180 degrees, till the **securing segment** (3) appears in the **slot** (4).
5. Hold the **end device** (5) firmly and remove the **securing segment** (3) with a narrow screwdriver. Remove the **end device** (5) from the connection.
6. Place the protective cap.

6.1.2 Variant for end devices with 1-6 kg weight (spring arm with brake screw only) Fig. 4

1. Power off the equipment/ pull out the power plug and secure it from being switched on.
2. Bring the spring arm in the uppermost position.
3. Rotate the **brake screw** (1) so that the **sleeve** (2) can be rotated.
4. Rotate the **sleeve** (2) through 180 degrees, till the **securing segment** (3) appears in the **slot** (4).
5. Hold the **end device** (5) firmly and take out the **securing segment** (3) with a narrow slotted-screw screwdriver. Disconnect the **end device** (5).
6. Put on the protective cap.

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Figure / Abbildung 3

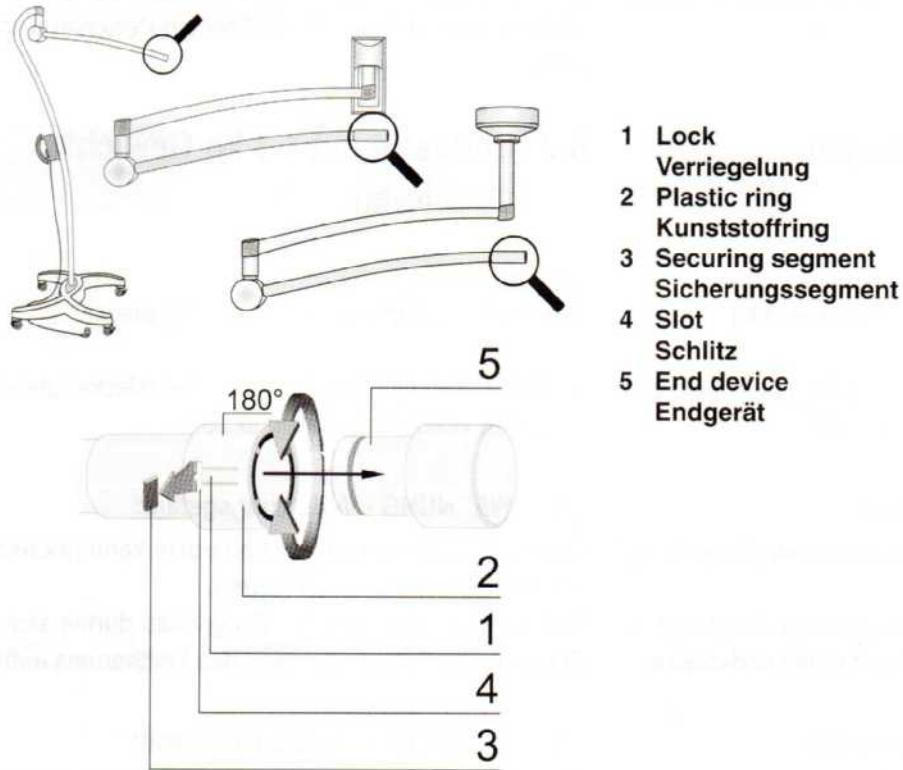
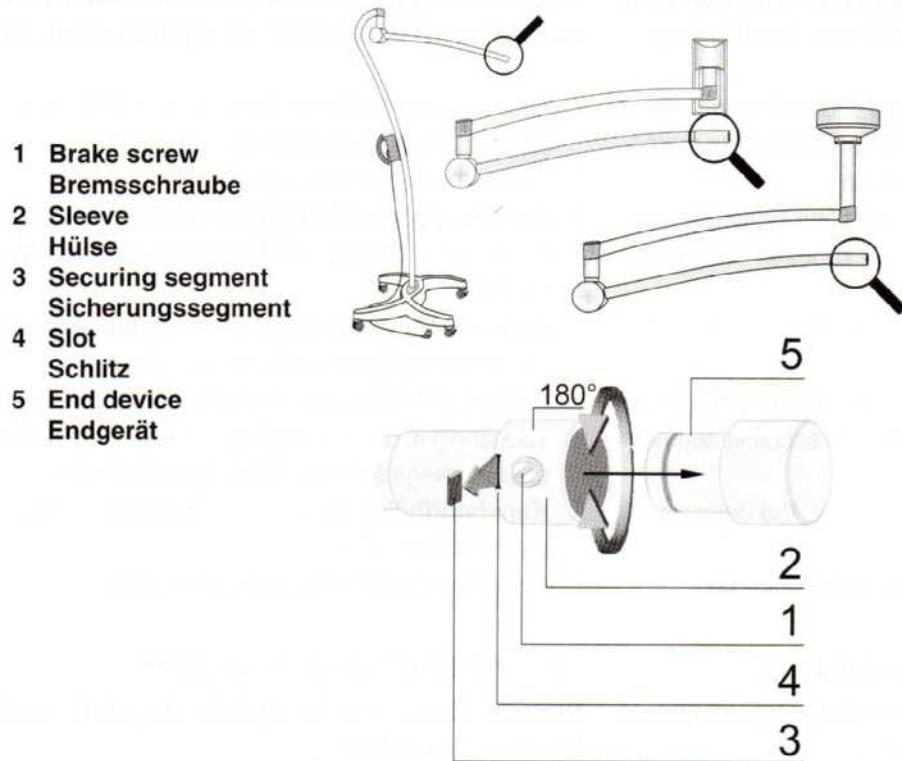
Figure / Abbildung 4

6 Dismantling and mounting for servicing

 **NOTE - Qualifications of the personnel:**
The dismantling / assembly must be carried out by a hospital technician (or a comparably qualified person).

6.2 Mounting the end-device with 1-3 kg weight

 **NOTE:**
The end-device should not weigh more than 3 kg.

1. Power off the equipment / pull out the power plug and secure it from being switched on again.

 **WARNING - Danger of injury:**
The spring arm, when it is pressed downward, can jump up suddenly and cause injury.
Nobody should be present within the swiveling range of the spring arm during the installation of the end-device.

-  **CAUTION - Damage to the device:**
The spring arm will be damaged if it is pulled beyond the lower / upper stop.
Grab the spring arm at the front end (not at the rear end) and slowly pull it downward until it reaches the stop.
2. Grab the spring arm at the front end and slowly pull it downward until it reaches the stop. Take out the protective cap from the spring arm opening.
 3. Check the **black rubber ring** (1) for firm seating, it must be positioned firmly behind the 4 knobs in the **plastic sleeve** (2).
 4. Push the **plastic sleeve** (2) on the spring arm in such a way that the **slots** (3) are covered.
 5. Push in the **end-device** (4) and the **securing segment** (5) completely in the slot, so that the securing segment is guided in the **groove** (6).
 6. Rotate the **plastic sleeve** (2) through 180 degrees, till there is an audible click.
 7. Check the secure seating of the **end-device** (4).

 **CAUTION - Damage to the equipment:**
After the end-device has been installed, perform tilt test in accordance with DIN EN 60601-1.

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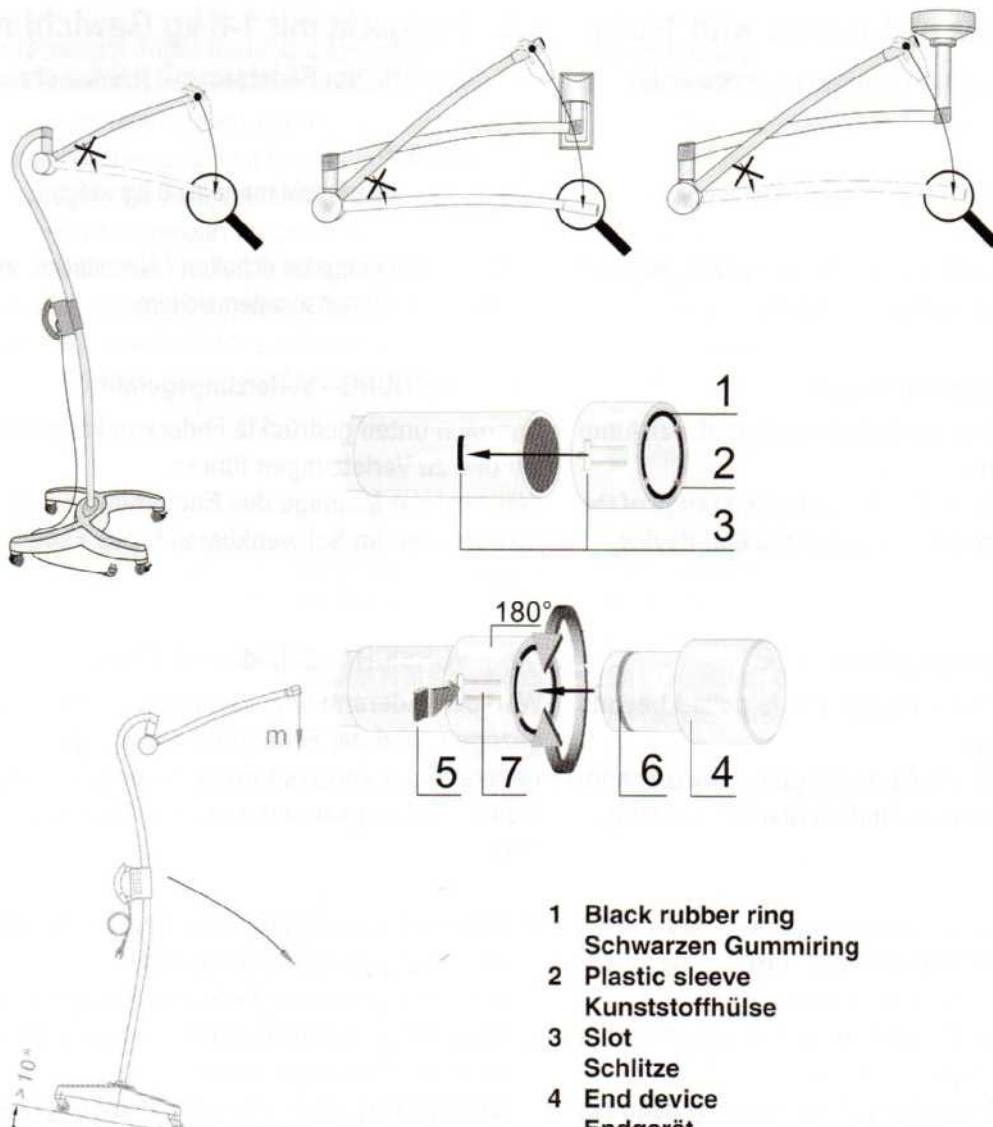
6

Dismantling and mounting for servicing

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NOTE - Qualifications of the personnel:

The dismantling / assembly must be carried out by a hospital technician (or a comparably qualified person).

6.2 Mounting the end-device with 1-6 kg weight (spring arm with brake screw only)

NOTE:

The end-device should not weigh more than 6 kg.

1. Power off the equipment / pull out the power plug and secure it from being switched on again.

WARNING - Danger of injury:

The spring arm, when it is pressed downward, can jump up suddenly and cause injury.

Nobody should be present in the swiveling range of the spring arm during the installation of the end-device.

CAUTION - Damage to the device:

The spring arm will be damaged if it is pulled beyond the lower / upper stop.

Grab the spring arm at the front end (not at the rear end) and slowly pull it downward until it reaches the stop.

2. Grab the spring arm at the front end and slowly pull it downward until it reaches the stop. Take out the protective cap from the spring arm opening.
3. Push on the **sleeve** (1) onto the spring arm in such a way that the **slots** (2) are covered.
4. Push in the **end-device** (3) and the **securing segment** (4) completely in the **slot** (2), so that the securing segment is guided in the **groove** (5).
5. Rotate the **plastic sleeve** (1) through 180 degrees, and tighten the **braking screw** (6) till the **end-device** (3) is braked.
6. Check the secure seating of the **end-device** (3).

CAUTION - Damage to the equipment:

After the end-device has been installed, perform tilt test in accordance with DIN EN 60601-1.

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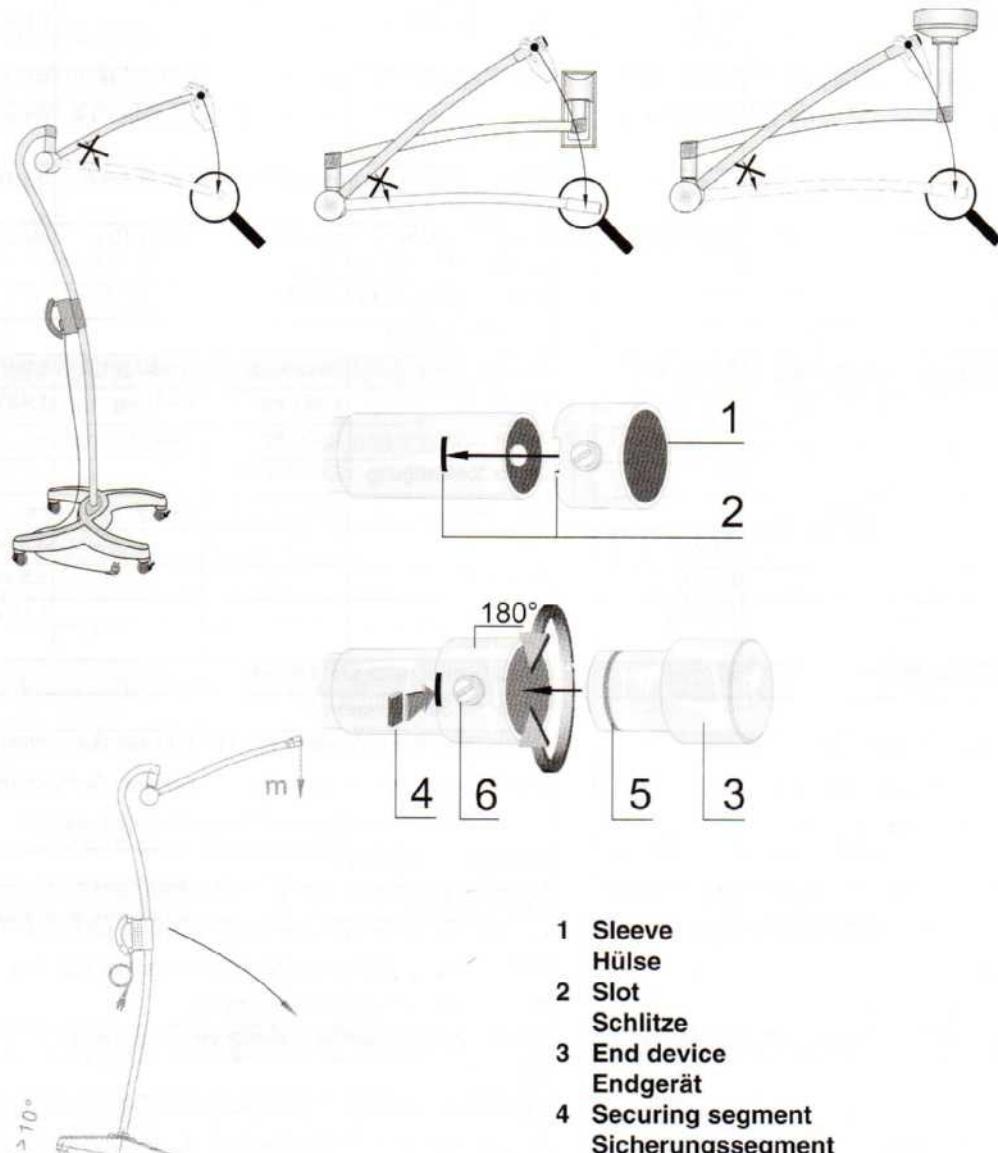
Figure / Abbildung 6

7 Technical Data

Electrical data	Ceiling-unit	Wall-unit	Stativ-unit
Rated voltage, depending on model	120 V / 230 V	120 V / 230 V	120 V / 230 V
Maximum power delivered by the 120 V Version	80 W	80 W	80 W
Maximum power delivered by the 230 V Version	50 W or 80 W	80 W	50 W or 80 W
Nominal frequency	50 / 60 Hz	50 / 60 Hz	50 / 60 Hz
Output current at 12V *	Up to 8,75 A	Up to 8,75 A	Up to 8,75 A
Output current at 24V *	Up to 4,4 A	Up to 4,4 A	Up to 4,4 A
Output current at 230 V *	Up to 10 A	Up to 10 A	Up to 10 A
Type of fuses: T 10 L 250 V			
* depending on version			
Weight in kg	Ceiling-unit	Wall-unit	Stativ-unit
Spring arm	1,431 kg	1,431 kg	1,431 kg
Extension arm	1,630 kg	1,630 kg	---
Ceiling mounting	2,5 - 5,1 kg	---	---
Wall bearing	---	1,8 kg	---
Stativ			3,5 kg
Stativ base			6,5 kg
Stativ base (weighted)			12,0 kg
Classification according to EN 60601-1			
Device of protection class I			
Device is not suitable for usage in combustible mixtures of anaesthetic agents with air or in combustible mixtures of anaesthetic agents with oxygen or laughing gas.			
Device for continuous operation			
EU-conformance			
The product conforms to the specifications in the guideline guideline 93/42/EWG (guidelines on medical products) and the guideline 89/336/EWG (EMV guidelines)			
Classification according to guidelines 93/42/EWG, appendix IX			
Class I			
EU-Conformance			
The Acrobat Swing equipment conforms to the specifications of the guideline 93/42/EWG (medical products guideline) and the guideline 89/336/EWG (EMV guideline).			
The Acrobat Swing equipment is only intended for connecting medical examination lamps, operation lamps and flat-screens with CE-markings and a maximum of 6 kg. self-weight. The person operating the equipment must submit a declaration as per article 12 of guideline 93/42/EWG (medical products guidelines). Another conformance evaluation must be carried out for other end devices.			
Certifications			
UL, VDE, CE			

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© ONDAL Industrietechnik GmbH, 2010