Health o meter[®]
P r o f e s s i o n a l

WelchAllyn[®]

402KLWA

PHYSICIAN BEAM SCALE

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ISO13485:2003
CERTIFIED

CE₀₄₅₉

USER INSTRUCTIONS • INSTRUCCIONES DE USO • MODE D'EMPLOI

PELSTAR, LLC 11800 Austin Ave. Alsip, IL 60803

Distributed by:

WelchAllyn[®]
www.welchallyn.com

Rev. 20101103

PHYSICIAN BEAM SCALE

MODEL: 402KLWA

Assembly & Operating Instructions

Your Scale is a precision instrument and should be handled accordingly. Remove all parts carefully to prevent scratching the finish. In assembling the parts, do not use force since the scale has been assembled at our plant and all parts will fit properly if instructions are followed.

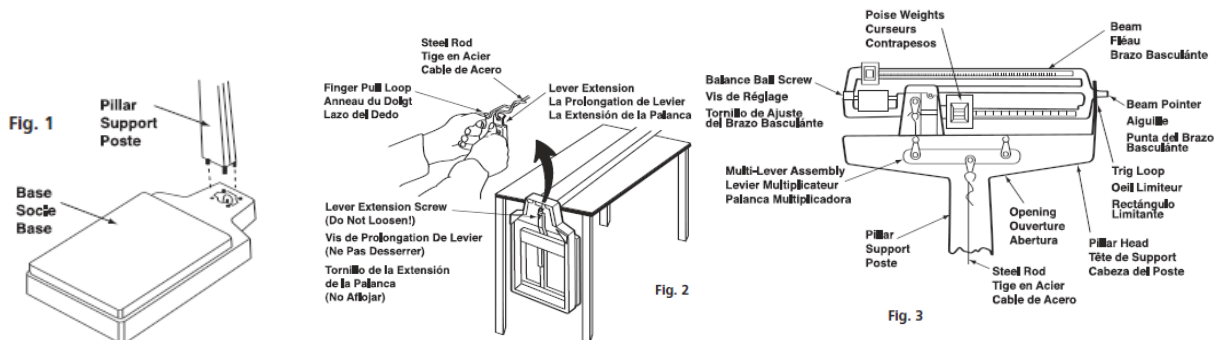
Carton Contents – 1 Pillar/Beam, 1 Base, Counterweight(s) (one 40 kg & one 150 lb) 4 Pillar Nuts and Wrench

Assembly Instructions

1. Insert studs at lower end of pillar into the four holes provided in the base (See Fig 1).
2. Tip scale at angle (See Fig 2) resting top of pillar on any convenient support and using the wrench provided, secure pillar to base with the four nuts (included).
3. Pull the Steel Rod toward the Base by grasping the Finger Pull Loop on the end.
4. Hook the Steel Rod into the Lever Extension by pushing the Lever Extension toward the beam with your hand. While holding this position the hook on the Steel Rod is easily attached to the hole at the end of the Lever Extension. Release the Finger Pull Loop and the scale is ready to use.

Note: The proper position of the multi-lever assembly in order to attach the steel rod to the base is shown in Figure 3. This assembly must be free floating in the pillar head. This can be observed through the opening between the Pillar Head and the Pillar.

5. Stand the scale upright.
6. To increase your scale's capacity by using the included counterweight(s), please refer to section "Using Counterweights".

Operating Instructions

Zero Balance Adjustment

The scale should be checked periodically, particularly when moved, by following the instructions below:

1. Make certain scale is on a hard, level surface. Step on and off scale a few times in order to align all internal parts.
2. Place both poise weights on zero (see Fig 3). The beam pointer should float gently up and down and not touch the top or bottom of the trig-loop.

Note: If the beam pointer is not moving equally centered in the trig-loop opening, turn balance ball screw (See figure 3) to correct.

Conversion (lb to kg / kg to lb)

1. Rotate both the upper and lower poise bars with a gentle twist of the wrist (Figure 4) to the selected unit of measure.
2. Check poise bars to make sure both match in color. For easy identification, pound poise bars have chrome graduations on a black background; kilogram poise bars have black graduations on a chrome background.

Weighing Instructions

1. Set the large and small poises to indicate your approximate weight.
2. Set on the scales. Continue to adjust the small poise until the pointer is centered.
NOTE: It is not necessary for you to wait for the pointer to stop moving in order to read your correct weight. As long as the pointer is moving equally in the trig-loop opening, the pointer would eventually come to rest in the center of the opening. Even when the pointer is not perfectly centered, as long as it is moving, you are within one quarter pound of your correct weight.
3. Add the indicated reading of small poise to the indicated reading of the large poise. The total is your weight.

Weighing Hints

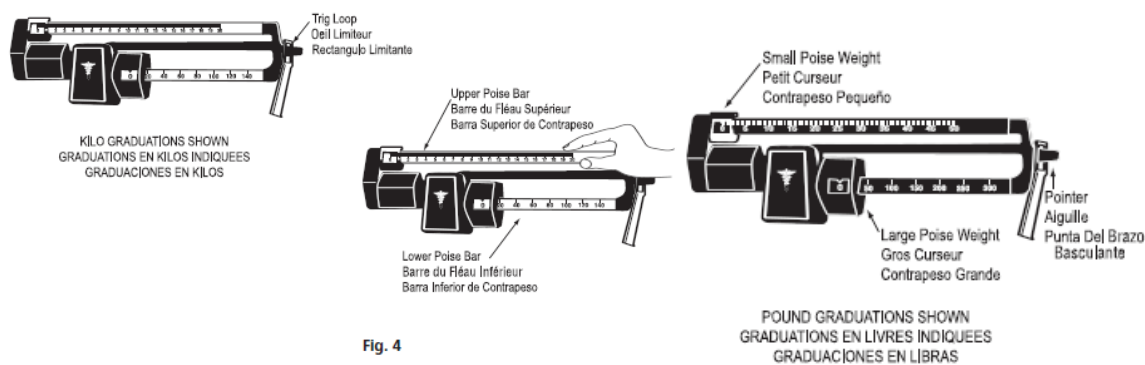
1. Weigh yourself at approximately the same time each day. The foods you eat, liquids you drink and your level of activity will cause your weight to vary.
2. For more realistic results, weigh yourself without clothing. The average weight of clothing, including shoes, is about 0.90-2.26kgs.(2-5 lbs.) for women and 2.26-3.62kgs (5-8 lbs.) for men.

Maintaining The Scale

1. The baked enamel finish, lens and mat can be cleaned with a mild soap and damp soft cloth. **DO NOT USE ABRASIVE OR STRONG CHEMICAL CLEANERS.**
2. All bearings and working parts are permanently lubricated. **DO NOT USE OIL OR LUBRICANTS.**
3. If your scale fails to work properly, do not attempt to dismantle it. This will void your warranty. See warranty information for instructions on having your scale serviced.

Maintenance

Clean scale routinely with anti-bacterial soap.

Using Counterweights

For 150 lb counterweight only:

If the weight of the patient is between 350 lbs and 424 lbs

1. Place the large (lower) poise weight in the 250 lb notch.
2. Hang 150 lb counterweight in the notch on lower poise bar between 50 lbs and 100 lbs.
3. Weigh the patient normally using the small and large poise weights.
4. To arrive at the patient's total weight, add the weights registered on the small and large poises and add 88 lbs.
Refer to the enclosed quick reference guide to arrive at the total weight

If the weight of the patient is between 425 lbs and 500 lbs

1. Place the large (lower) poise weight in the 250 lb notch.
2. Hang 150 lb counterweight in the notch on lower poise bar between 150 lbs and 200 lbs.
3. Weigh the patient normally using the small and large poise weights.
4. To arrive at the patient's total weight, add the weights registered on the small and large poises and add 150 lbs.
Refer to the enclosed quick reference guide to arrive at the total weight.

For 40 kg counterweight only:

If the weight of the patient is between 160 kg and 180kg

1. Place the large (lower) poise weight in the 120 kg notch (this will weigh 160 kg with the addition of the 40 kg counterweight).
2. Hang the 40 kilogram counterweight in the notch on lower poise bar between 60 kg and 80 kg.
3. Weigh the patient normally using the small and large poise weights.
4. To arrive at the patient's total weight, add 160 kg to the weight registered by the small poise weight. Example: for a patient weighing 172 kg, the large poise weight will be on 120 kg, and the small poise weight on the upper bar will be on 12 kg.

If the weight of the patient is over 180 kg

1. Place the large (lower) poise weight in the 150 kg notch (this will weigh 180 kg with the addition of the 40 kg counterweight).
2. Hang the 40 kg counterweight in the notch on lower poise bar between 60 kg and 80 kg.
3. Weigh the patient normally using the small and large poise weights.
4. To arrive at the patient's total weight, add 180 kg to the weight registered by the small poise weight.

CUSTOMER SERVICE INFORMATION:

For further information, please contact Welch Allyn Customer Service at one of the numbers listed below:

European Call Center - (+353) 46-90-67790

France - (+33) 1-55-69-58-49

Germany - (+49) 695-098-5132

Italy - (+39) 026-968-2425

Netherlands- (+31) 202-061-360

Spain - (+34) 917-499-357

Sweden - (+46) 85-853-6551

United Kingdom - (+44) 207-365-6780

Latin America - (+1) 305-669-9003

Middle East - (+49) 7477-927186

South Africa - (+27) 11-777-7555

Australia - (+61) 2-9638-3000

China - (+86) 21-6327-9631

Troubleshooting Guide

INCORRECT WEIGHT

The accuracy of the 402KLWA scale when calibrated at the factory is within plus or minus 0.1kg or ¼ pound. This means that two properly calibrated scales can vary 0.1kg or ¼ pound from each other and still be calibrated correctly. For the most accurate weight, use the scale on a hard level surface and stand in the center of the platform with the weight distributed as evenly as possible. If the error seems excessive, check the following:

1. **Zero Balance of the Beam**

The beam must be balanced so the pointer comes to rest in the center of the trig-loop when both poises are set at zero. Make certain that the large poise is firmly against the shoulder of the beam. Adjust the balance by turning the screw at the left end of the beam.

2. **Beam Does Not Move Freely**

Check if the beam pointer is touching the side of the trig-loop in its range of travel. Contact Customer Service Support at one of the numbers listed above for instruction on realigning the beam.

3. **Platform Rocks Excessively or Touches Base at Any Corner**

When you push down at any corner of the platform you should not feel any significant rocking. If you do, please contact Customer Service Support at one of the numbers listed above on leveling the platform.

4. **Beam Does Not Move at All during Weighing Process**

This may be caused by the poises being set to a higher weight than the person's actual weight. Reset the weights to a lower weight. An improperly connected Steelyard Rod (the rod that runs inside the pillar) or a disconnected rod which can occur if the scale is transported may also be the cause. See Figure 2.

5. **Scale Out of Calibration**

This can happen if an adjustment is jolted out of place during transit or by wear in the notches in the beam after a long period of use. Lifting the large poise over the notches during movement will preserve the accuracy of the scale and lengthen the life of the beam. The accuracy should be determined with accurate test weights or the scale can be compared to a scale known to accurate, such as a commercial scale with a Weights & Measures sticker affixed. The scale can be recalibrated by loosening the lock screw at the end of the long lever in the bottom of the base with a 5/16" hexagon socket wrench and moving it outward (longer) in the elongated slot to decrease the indicated weight or inward (shorter) to increase the indicated weight. One sixteenth inch movement of the screw will change the indicated weight by approximately 0.1kg per 45.35 kgs or ¼ pound per 100 pounds.

402KLWA PARTS FOR MAINTENANCE

Figure No.*	Part No.	Description	Qty.
Field replaceable parts			
1	400BASE	COMPLETE BASE ASSEMBLY	1
1	400COVER	PLATFORM COVER	1
3	400BEAM	COMPLETE BEAM ASSEMBLY	1

10-YEAR LIMITED WARRANTY

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of ten (10) years. If product fails to function properly, return the product, freight prepaid and properly packed to Customer Service (see “To Get Warranty Service”, below, for instructions). If the authorized Customer Service provider determines that a defect of material or in workmanship exists, the customer’s sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Authorized dealers selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call the Customer Service Center nearest you for assistance--please see page 3 of this manual. **Before returning a product, you must obtain authorization from Customer Service.** Customer Service personnel will give you a Service Notification number. Please note this number on the outside of your shipping box. Returns without a Service Notification number will not be accepted for delivery. **Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send it with shipping and insurance prepaid to the address you are given by Customer Service.** Write, “Attention SN#_____”, and the shipping address, on the shipping label, following Customer Service instructions.

PRODUCT LIFE: 10 YEARS / 100,000 CYCLES

PELSTAR, LLC

11800 South Austin Avenue – B • Alsip, IL 60803 • USA

www.homscases.com

Health o meter® Professional products are manufactured, designed and owned by Pelstar, LLC and are distributed by Welch Allyn outside of the US and Canada. Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. We reserve the right to improve, enhance or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2009