Health o meter[®]
P r o f e s s i o n a l

WelchAllyn[®]

349KLXWA

PHYSICIANS REMOTE READ DIGITAL SCALE

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USER INSTRUCTIONS • INSTRUCCIONES DE USO • MODE D'EMPLOI • 用户说明

Rev.20101116

PHYSICIANS REMOTE READ DIGITAL SCALE MODEL: 349KLXWA

OPERATING INSTRUCTIONS

USING THE SCALE LB/KG: Pressing this button will change the unit of measure displayed. This scale weighs in kilogram (KG) or in pounds (LB) **.
ZERO: Pressing this button will reset the display to zero: 0.0

Preparing Scale for use

1. Remove the plastic film from display.
2. Plug the coiled, "phone" cord into the bottom left of the remote head.
3. Place scale on the floor on a vibration-free, level surface.
4. Press "ON", Wait for the display to show zero in the units of measure last used (to change the units of measure, see section "Using the Scale: LB/KG" above). You are now ready to begin weighing.
5. Always allow the display to return to Zero before stepping on the platform.

Special Features

AUTO SHUT-OFF: Scale will automatically turn off after approximately 2 minutes of inactivity

DEACTIVATING AUTO SHUT-OFF: While the scale is off, press and hold the "LB/KG" button. While still holding "LB/KG" press "ON". "A_of" will display. The scales will remain until manually turned off.

Understanding Messages

"0000": the weight capacity has been exceeded

"-3.0": minus weight, to adjust press ZERO

"bAtt": Blinking means the battery is low

POWER SUPPLY

Your new scale operates on one 9V battery or 120V-240VAC, 50-60Hz, 9V DC Adapter

BATTERY USE: Battery compartment is located on the underside of the remote display head. Connect battery +/- accordingly.

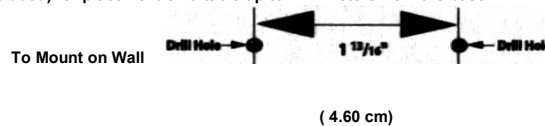
ADAPTER USE: First plug into bottom right of remote head, then plug into wall outlet.

 **WARNING:** IF THE SCALE IS BEING CONNECTED TO ANOTHER MEDICAL DEVICE, THE POWER ADAPTER MUST NOT BE USED. DISCONNECT THE POWER ADAPTER FROM THE SCALE.

 **WARNING:** Use only Battery power when the scale is connected to another medical device such as the Welch Allyn Spot LXi vital signs monitor. This ensures proper electrical isolation.


REMOTE HEAD MOUNTING

The remote head is attached to the scale by a coiled "phone" cord. The remote head can be mounted to a wall up to 2.74 meters from the base, or it can be mounted on the display bracket (included) for placement on a table up to 2.74 meters from the base.

1. Use the above diagram to locate the position of the screws at the desired height.
2. Drill two holes 4.60 cm apart.
3. Screw the two screws into the wall, let the heads protrude 0.3175 cm from the wall. Extending the heads further could damage the scale.
4. Align the holes on the back of the remote head with the screws. The remote head should slide onto the screw heads, firmly and securely.

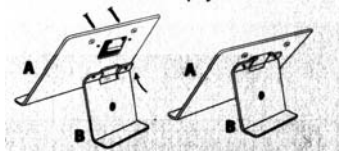To Mount on Display Bracket

1. Hold Part A (Stand) with the knobs facing away from you. Slide the center tab of Part B (Support) up and under the tab of Part A on the back of the Stand. Be sure to align the two parts, so that the tab on the Support can slide beneath the tab on the rear of the Stand. The holes on both parts of the brackets should line up.
2. Turn the bracket over so Part A is facing you and insert the two screws (included) into the holes and through the openings in Part B.
3. Using a Phillips head screwdriver, tighten the screws to secure Part B to Part A.
4. Align the holes in the back of the remote display head with the protruding knobs on the bracket. The remote head should slide down onto the knobs, firmly and securely. The display bracket is now ready for use.

SPECIFICATIONS

Dimensions:

Remote Head: 12.7 cm x 7.62 cm x 3.175 cm (5" x 3" x 1.25") Platform Size: 30.48 cm x 30.48 cm x 3.175 cm (12" x 12" x 2.25")

Power Source: 9 Volt battery or 120V- 240V AC, 50-60Hz, 9V DC Adapter

Model: 349KLXWA

Capacity: 180 kg / 400 lb

Resolution: 0.1 kg / 0.2 lb

** After being shut off, the scale will remember the last unit of measure selected. To prevent the unit of measure from being changed, please contact Customer service at one of the numbers below to deactivate the LB/KG button.

SERVICE

CUSTOMER SERVICE INFORMATION:

For further information or telephone support, please contact Welch Allyn Customer Service at one of the numbers listed below:

European Call Center - (+353) 46-90-67790

France - (+33) 1-55-69-58-49

Germany - (+49) 695-098-5132

Italy - (+39) 026-968-2425

Netherlands- (+31) 202-061-360

Spain - (+34) 917-499-357

Sweden - (+46) 85-853-6551

United Kingdom - (+44) 207-365-6780

Latin America - (+1) 305-669-9003

Middle East - (+49) 7477-927186

South Africa - (+27) 11-777-7555

Australia - (+61) 2-9638-3000

China - (+86) 21-6327-9631

MAINTENANCE AND CLEANING

Before first use and after periods of non-use, check the scale for proper operation.

Disconnect the scale from the power source, if plugged in.

1. This scale contains electronic parts. Use it carefully. Do not drop, kick or jump on it.
2. Do not store near chemicals, heaters or radiators.
3. Do not store scale on its side.
4. To clean the scale, wipe the outside of the scale with a soft cloth and mild detergent. Do not use abrasive cleaners. Do not use isopropyl alcohol, other solutions or sanitizing wipes to clean the display surface.
5. Never place the scale in the water. Do not spray it with liquid.
This scale is an extremely sensitive weighing device. Items such as clothing or towels placed or dropped on the scale may cause it to activate. In order to prevent run down of the battery, do not store or leave anything on the scale.

CALIBRATION

This scale has been factory calibrated and does not require calibration prior to use.

Note: Calibration weight must be 200 lb.

ACTION	DISPLAY SHOWS
Press and hold the LB/KG and ZERO buttons at the same time, then press the ON/OFF button to turn on the scale. "U 10" followed by a number indicates you can calibrate.	"CAL" then "XXXXX"
Press the ZERO button. "C-200" (for lb mode) will flash on the display and then the display will return to "0".	"C-200"
Place 200 lbs on the scale and when calibration number stabilizes, press the ZERO button. END will flash on the display and then OFF will flash display the scale will turn off.	"END" then "OFF"

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting Customer Service personnel.

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	<ol style="list-style-type: none"> 1. Battery problem 2. Bad Power Supply or Not Plugged In 3. Electrical Outlet not functioning 	<ol style="list-style-type: none"> 1. Check / Replace the batteries 2. Check / Replace Power Supply Adapter 3. Use a different outlet
"bAtt" shows in display	The batteries are low	Replace batteries according to instructions
Questionable weight or the scale does not zero	<ol style="list-style-type: none"> 1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level surface 4. Scale is out of calibration 	<ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Ask the patient to step off the scale. Zero the scale and begin weighing process again. 3. Place scale on level surface 4. Check weight with known weight value
"OVER" shows in display	Overload. The load on the scale exceeds scale capacity (180 kg / 400 lbs)	Remove weight, turn scale off, turn scale on again and try again
"UNDER" shows in display	Negative weight, indicating a tare was present. Load on the scale is below initial zero point	Clear scale of any weight and re-zero scale by pressing ZERO button

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Customer Service (see "To Get Warranty Service", below, for instructions). If the authorized Customer Service provider determines that a defect of material or in workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Authorized dealers selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call the Customer Service Center nearest you for assistance--please see page 2 of this manual. **Before returning a product, you must obtain authorization from Customer Service.** Customer Service personnel will give you a Service Notification number. Please note this number on the outside of your shipping box. Returns without a Service Notification number will not be accepted for delivery. **Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send it with shipping and insurance prepaid to the address you are given by Customer Service.** Write, "Attention SN# _____", and the shipping address, on the shipping label, following Customer Service instructions.

PRODUCT LIFE: 10 YEARS / 100,000 CYCLES

11800 South Austin Avenue – B • Alsip, IL 60803 • USA

www.homscases.com

Health o meter® Professional products are manufactured, designed and owned by Pelstar, LLC and are distributed by Welch Allyn outside of the US and Canada. Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. We reserve the right to improve, enhance or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2009