Health o meter[®]
P r o f e s s i o n a l

WelchAllyn[®]

160KGWA

LARGE DIAL, LARGE PLATFORM SCALE

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MODEL: 160KGWA

OPERATING THE SCALE

1. Place the scale on a hard and flat surface for best accuracy. Accuracy may be affected by very soft, deep pile carpet or rugs.
2. Step on and off the scale a few times before attempting to weigh yourself. This will align the internal mechanism.
3. Adjust the pointer to zero, if necessary, using the zero adjust knob at the bottom of the scale.
Note: As you use the scale over a period of time you may find that the pointer drifts slightly from zero. The best time to check the zero adjustment is after weighing when you step off the scale, if the pointer does not return exactly to zero, adjust the zero adjust knob.
4. Step on the scale with weight equally distributed on both feet, stand still.

WEIGHING HINTS

1. Weigh yourself at approximately the same time each day. The foods you eat, liquids you drink and your level of activity will cause your weight to vary.
2. For more realistic results, weigh yourself without clothing. The average weight of clothing, including shoes, is about 0.9-2.26 kg for women and 2.26-3.62 kg for men.

MAINTAINING THE SCALE

1. The baked enamel finish, lens and mat can be cleaned with a mild soap and damp soft cloth. **DO NOT USE ABRASIVE OR STRONG CHEMICAL CLEANERS.**
2. All bearings and working parts are permanently lubricated, **DO NOT USE OIL OR LUBRICANTS.**
3. If your scale fails to work properly, do not attempt to dismantle it. This will void your warranty. See warranty information for instructions on having your scale serviced.

CUSTOMER SERVICE INFORMATION:

For further information, please contact Welch Allyn Customer Service at one of the numbers listed below:

European Call Center - (+353) 46-90-67790

France - (+33) 1-55-69-58-49

Germany - (+49) 695-098-5132

Italy - (+39) 026-968-2425

Netherlands- (+31) 202-061-360

Spain - (+34) 917-499-357

Sweden - (+46) 85-853-6551

United Kingdom - (+44) 207-365-6780

Latin America - (+1) 305-669-9003

Middle East - (+49) 7477-927186

South Africa - (+27) 11-777-7555

Australia - (+61) 2-9638-3000

China - (+86) 21-6327-9631

1-YEAR LIMITED WARRANTY

What does the Warranty Cover?

This Health o meter[®] Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Customer Service (see "To Get Warranty Service", below, for instructions). If the authorized Customer Service provider determines that a defect of material or in workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Authorized dealers selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call the Customer Service Center nearest you for assistance--please see page 1 of this manual. **Before returning a product, you must obtain authorization from Customer Service.** Customer Service personnel will give you a Service Notification number. Please note this number on the outside of your shipping box. Returns without a Service Notification number will not be accepted for delivery. **Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send it with shipping and insurance prepaid to the address you are given by Customer Service.** Write, "Attention SN# _____", and the shipping address, on the shipping label, following Customer Service instructions.

PRODUCT LIFE: 10 YEARS / 100,000 CYCLES

PELSTAR, LLC

11800 South Austin Avenue – B • Alsip, IL 60803 • USA

www.homscales.com

Health o meter[®] Professional products are manufactured, designed and owned by Pelstar, LLC and are distributed by Welch Allyn outside of the US and Canada. Health o meter[®] is a registered trademark of Sunbeam Products, Inc. used under license. We reserve the right to improve, enhance or modify Health o meter[®] Professional product features or specifications without notice.

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© Pelstar, LLC 2009