
Neu ab:  
New since: 03.2007  
Nouveau depuis:  
Nuevo desde:

# ORTHOPHOS Plus / Plus Ceph

Schaltunterlagen  
Wiring References  
Schémas de circuits  
Colección de esquemas

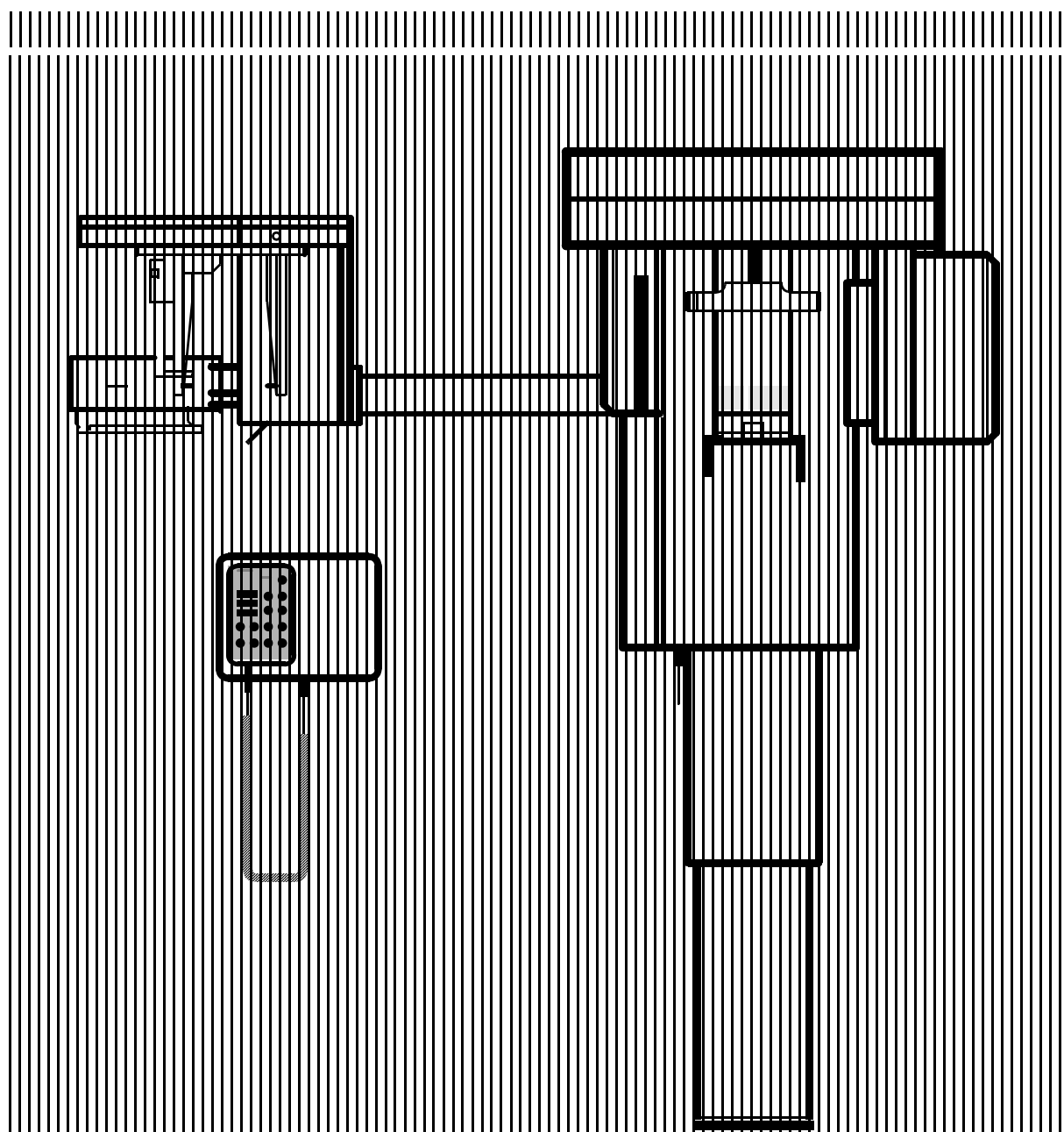**! ACHTUNG**

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**! ATTENTION**

Interference with electromedical devices by radio telephones:  
To guarantee the operational safety of electromedical devices, the operation of mobile radio telephones in the medical practice or hospital area is prohibited.

**! ATTENTION**

When opening the equipment:  
Please observe the safety measures for handling PC boards. Touch a ground point to remove any personal electrostatic charge before touching the components.

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Modification compared with last edition:  
10.2004

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## Legend

### **A** Foil Button

SR	Return button <b>R</b>
SA	Anomaly button <b>A</b> + -
SL	Light localizer ON / OFF
S↑	Display height adjustment UP
S↓	Display height adjustment DOWN
S←	Forehead support towards column
S→	Forehead support away from column
	LED, indication help messages
V1	LED, cooldown period
V2	LED, tube position Pan/Ceph
V3	LED, diaphragm setting
V4	LED, forehead support position
V5	LED, height adjustment
V6	LED, Cassette not ready for exposure
V7	LED, Cssette holder not in position
V8-11	LED, anomaly

### **D4** Multitimer PC-Board

	<b>LED - Displays</b>
V3	Operating indication
V8	Memory display
V9	Test run display
V10	Ready LED
V11	Large adult
V12	Adult
V13	Adolescent
V14	Child
V15	Radiation indication
V16-22	7-segment display
V23	AES function

#### Buttons on Multitimer

S8	Memory
S9	Test run
S10	Reverse
S11	Large adult
S12	Adult
S13	Adolescent
S14	Child
S15	Release button
S23	AES button
S24	Pan. s - cephalometer program
S25	Pan. s+cephalometer program
S26	KV -
S27	KV +
S28	Film density compensation

### **D6** DC / AC Converter

V15	LED, supply voltage +15V
V16	LED, bridge short-circuit
V17-20	LED, DC/AC converter bridge active

### **DX1** Control Board

V100	LED, reset active
V101-103	LED, Test LEDs
V400	LED, kV controller
V614	LED, VH/mA switchover
V1106	LED, supply voltage for digital unit +5V
V1900	LED, supply voltage for Multitimer +9,5V
V4020	LED, HV pulse stand
V4021	LED, Ceph position
V4022	LED, HV pulse Ceph
V4023	LED, HV correction switch
V4025	LED, VH/mA controller active

### **DX5** Distributor Board

### **DX7** Display Board

V1-V3	7-segment display height adjustment
V4-V6	7-segment display forehead support

### **DX8** Film Control Board

R26	Potentiometer Offset
R27	Potentiometer Gain

### **DX31** Power Supply Board

F1	Fuse for Transformer <b>T</b> +38V
F2	Fuse for ~28V
F3	Fuse for ~28V
F4	Fuse for ~17V
F5	Fuse for ~17V
F6	Fuse for ~28V
F7	Fuse for ~13,5V
V3	LED, demonstration mode
S1	Demonstration mode switch position 2 : no radiation PTC
R1	PTC

### **DX32** Supply Board

V10	LED, +24V
V11	LED, +9,5V
V12	LED, +15V
V13	LED, - 15V
V14	LED, +16V
V31	LED, +30V
V40	LED, +38V
V52	LED, +30V switched on
V53	LED, +24V switched on
V54	LED, relay K1 switched on
V56	LED,+5V dig. extension

### **H1** X-Ray Head

H1.S1	Thermoswitch
S2	Diaphragm locked
S3	Diaphragm switch 1
S4	Diaphragm switch 2

### **M** Memory Board

### **T** Transformer

### **V** Light Barriers

V2	Forehead support, zero position
V3	Actuator 1 ON position
V4	Actuator 2 ON position
V5	Rotation start position
V6	Height adjustment pulses
V7	Ceph position
V8	Height adjustment Ceph pulses
V9	Reference point for film cassette
V10	Cassette inserted
V11	Panorama position
V12	Cephalometer position

### **Motors M**

M1	Motor for rotation
M2	Actuator 1
M3	Actuator 2
M4	Motor for cassette movement
M5	Motor for height adjustment
M6	Motor for forehead support

### **Switches S**

S1	Main switch
SHV	Height adjustment devise

### **Other Components**

F1/F2	Main fuses
K2-13	Terminal/coupling at/in unit
L1, L2, . .	Leads
H1/H2	Light Localizer
R1	Forehead potentiometer

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**Sirona Dental Systems GmbH**
Sirona Dental Systems LLC